

1. This document was created to support maximum accessibility for all learners. If you would like to print a hard copy of this document, please follow the general instructions below to print multiple slides on a single page or in black and white.
2. This handout is for reference only. Non-essential images have been removed for your convenience. Any links included in the handout are current at the time of the live webinar, but are subject to change and may not be current at a later date.
3. Copyright: Images used in this course are used in compliance with copyright laws and where required, permission has been secured to use the images in this course. All use of these images outside of this course may be in violation of copyright laws and is strictly prohibited.
4. Social Workers: For additional information regarding standards and indicators for cultural competence, please review the NASW resource: [Standards and Indicators for Cultural Competence in Social Work Practice](#)
5. Need Help? Select the “Help” option in the member dashboard to access FAQs or contact us.

How to print Handouts

On a Mac

- Open PDF in Preview
- Click File
- Click Print
- Click dropdown menu on the right “preview”
- Click layout
- Choose # of pages per sheet from dropdown menu
- Checkmark Black & White if wanted.

On a PC

- Open PDF
- Click Print
- Choose # of pages per sheet from dropdown menu
- Choose Black and White from “Color” dropdown

No part of the materials available through the continued.com site may be copied, photocopied, reproduced, translated or reduced to any electronic medium or machine-readable form, in whole or in part, without prior written consent of continued.com, LLC. Any other reproduction in any form without such written permission is prohibited. All materials contained on this site are protected by United States copyright law and may not be reproduced, distributed, transmitted, displayed, published or broadcast without the prior written permission of continued.com, LLC. Users must not access or use for any commercial purposes any part of the site or any services or materials available through the site.

Genie 2 Fitting Focus Series:

Oticon Real First Fit

Learner Outcomes:

After this course, participants will be able to:

- . . . explain the process of pairing and connecting Oticon Real instruments using Genie 2 software.
- . . . list the required initial fitting steps for Oticon Real.
- . . . find the MoreSound Intelligence and Sudden Sound Stabilizer tools in the Genie 2 fitting screen and describe the controls available on these screens.

Getting ready before the fitting

Fitting software and wireless programmer

Fitting Software

Genie 2 2023.2



Oticon Genie Updater



USB / DVD media

Programmer

Noahlink Wireless



Single device for hassle free programming

Wireless programming & firmware updates

**Real sound.
Real benefits**



The Oticon Real family



miniRITER



miniRITET



miniBTET




miniBTET

Stay charged at home or on the go



Feedback predictions

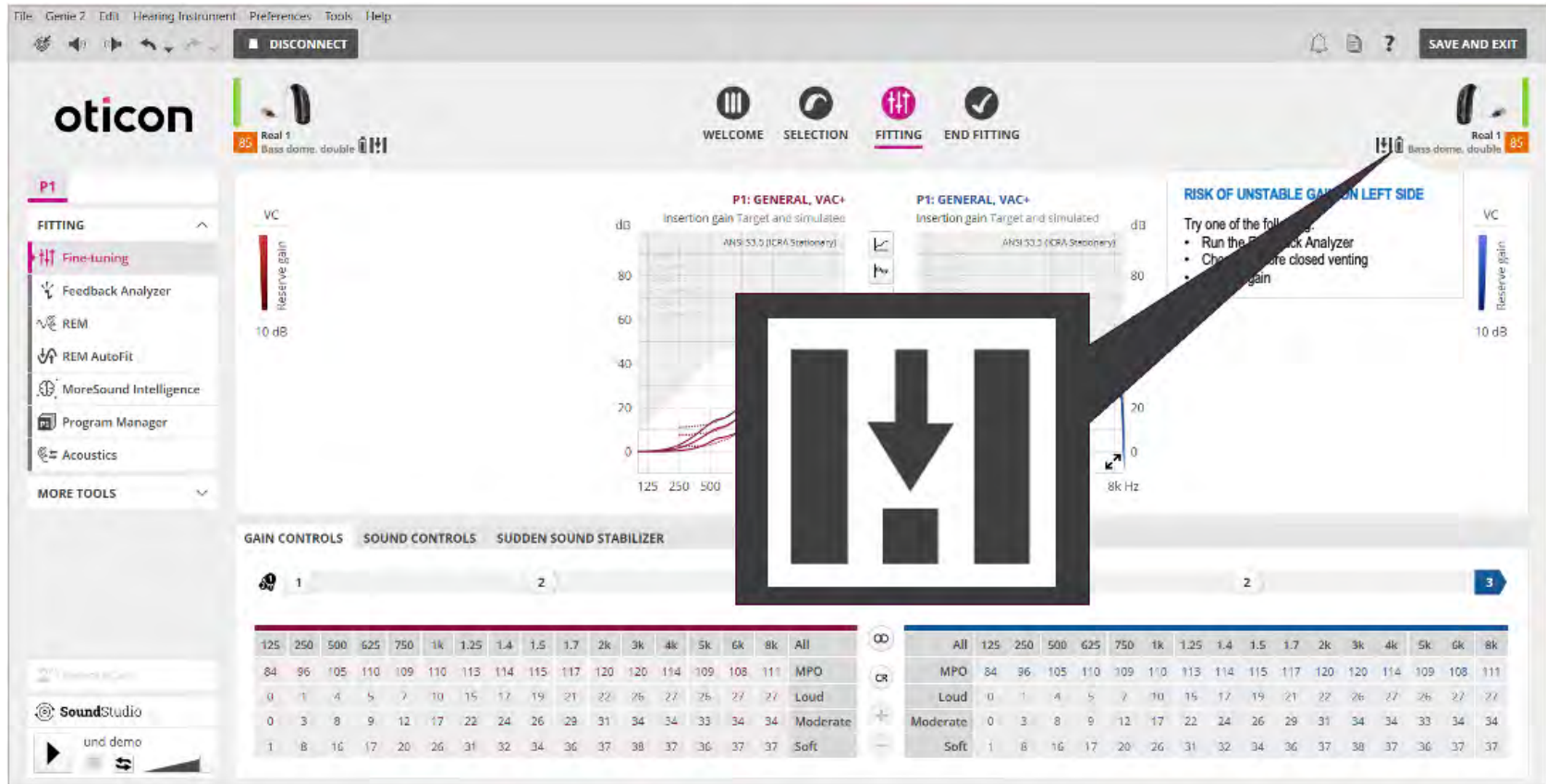

Voice notifications

For **Low Battery** warning

Available in **9 languages**

One **voice** per language

Choose from **3 levels**

For all **Polaris R** hearing aids



DISCONNECT

SAVE AND EXIT

oticon

60 Real 1
Bass dome, double

CLIENT



WELCOME



SELECTION



FITTING



END FITTING

Real 1
Bass dome, double 60

END FITTING

OPERATE PUSHBUTTON

AUDIBLE INDICATORS

CONNECTCLIP

VISUAL INDICATORS

STANDARD

CALL CONTROLS

Adjust the general settings.

✓ Save and Exit

Buttons and Indicators

Batteries

Accessories

Generate Report

LINKS

Technical Data

SoundStudio

Voice notification

Language ⓘ

English (US)

Level

Normal

Tonal signal

Frequency

Medium

Level

Normal

Start-up

✓ Jingle ⓘ

Shutdown

✓ Shutdown beep

Volume control indicators

Voice notification

Language ⓘ

English (US)

Level

Normal

Tonal signal

Frequency

Medium

Level

Normal

Start-up

✓ Jingle ⓘ

Shutdown

✓ Shutdown beep

Volume control indicators




DISCONNECT



SAVE AND EXIT

oticon

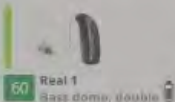

CLIENT



WELCOME




SELECTION



FITTING



END FITTING

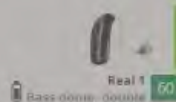

END FITTING

✓ Save and Exit

Buttons and Indicators

Batteries

Accessories

Generate Report

LINKS

Technical Data

SoundStudio

OPERATE PUSHBUTTON

AUDIBLE INDICATORS

CONNECTCLIP

VISUAL INDICATORS

STANDARD

CALL CONTROLS

Adjust the general settings.

Voice notification

Language ⓘ

English (US)

Level

Normal

Chinese

Danish

English (UK)

English (US)

French

German

Italian

Japanese

Spanish

Level

Normal

Voice notification

Language ⓘ

English (US)

Level

Normal

Tonal signal

Frequency

Medium

Level

Normal

Start-up

✓ Jingle ⓘ

Shutdown

✓ Shutdown beep

Shutdown

✓ Shutdown beep

Volume control indicators

Volume control indicators

Genie 2 | 2023.1 [Real 1, Oticon]

FileGenie 2EditHearing InstrumentPreferencesToolsHelp

DISCONNECT

SAVE AND EXIT

oticon

Real 1

Bass dome, double

CLIENT

WELCOME

SELECTION

FITTING

END FITTING

Real 1

Bass dome, double

END FITTING

✓ Save and Exit

Buttons and Indicators

Batteries

Accessories

Generate Report

LINKS

Technical Data

SoundStudio

OPERATE PUSHBUTTON

AUDIBLE INDICATORS

CONNECTCLIP

VISUAL INDICATORS

STANDARD

CALL CONTROLS

Adjust the general settings.

Voice notification

LanguageEnglish (US)LevelNormal

Tonal signal

FrequencyMediumLevelNormal

Start-up

✓Jingle

Shutdown

✓Shutdown beep

Volume control indicators

Voice notification

LanguageEnglish (US)LevelNormal

Tonal signal

FrequencyMediumLevelNormal

Start-up

✓Jingle

Shutdown

✓Shutdown beep

Volume control indicators




DISCONNECT



SAVE AND EXIT

oticon

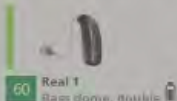

CLIENT



WELCOME




SELECTION



FITTING



END FITTING

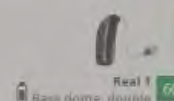

END FITTING

✓ Save and Exit

Buttons and Indicators

Batteries

Accessories

Generate Report

LINKS

Technical Data

OPERATE PUSHBUTTON

AUDIBLE INDICATORS

CONNECTCLIP

VISUAL INDICATORS

STANDARD

CALL CONTROLS

Adjust the general settings.

Battery low indicator

✓ Pre-warning

Voice



Tonal



Battery low indicator

✓ Pre-warning

Voice



Tonal



✓ Out of battery



✓ Out of battery



Program change indicator

✓ Program change (beep)



Program change indicator

✓ Program change (beep)



Tinnitus volume control indicators



Tinnitus volume control indicators



SoundStudio

miniRITE R

End Fitting/Batteries

The screenshot displays the 'End Fitting/Batteries' screen in the Oticon miniRITE R software. The interface is divided into several sections:

- Top Bar:** Includes the Oticon logo, a 'DISCONNECT' button, and a 'SAVE AND EXIT' button.
- Navigation Tabs:** WELCOME, SELECTION, FITTING, and **END FITTING** (highlighted).
- Left Sidebar:** Contains 'END FITTING' options: Save and Exit, Buttons and Indicators, **Batteries** (highlighted), Accessories, and Generate Report. Below this are 'LINKS' for Motivational Tool, Technical Data, and Instruction Videos.
- Main Content Area:**
 - CURRENT BATTERY LEVEL:** Shows 100% for both hearing aids.
 - BATTERY HEALTH:** Shows 'The battery is in good condition.' with a green checkmark icon for both.
 - BATTERY REPLACEMENT AND RESET:** Includes a 'WATCH VIDEO' button and instructions: 'After replacing batteries, place the instruments in the charger until the instrument LED turns orange.'
 - REPLACEMENT HISTORY:** Shows steps for replacing batteries, selecting new batteries, and filling in the serial number.

Current battery level

Battery health

Battery replacement

miniRITE R

End Fitting/Batteries

SERIAL NUMBER

Enter new battery serial number:

LEFT
70 digits

Where do I find the number?

DESC	BATTERY, 312+ LI-ION	
SER	(11) 190606	QTY (30) 1
	(15) 200615	
→ SN	(21) 2011000315	
REF	(240) 205490	LOT (10) 201902

Look at the back of the battery package

CANCEL

Genie 2 Fitting Focus Series:

Oticon Real First Fit

