



An Optimized Tour through Connexx Fitting Software with IX

Danielle Robertson, AuD
Sr. Clinical Education Specialist

Erin Downard, AuD
Clinical Education Specialist



An Optimized Tour through Connexx Fitting Software with IX



Webinar FAQs

- **Where to get the handout:**

- Download from Pending Courses
- Click on link from Chat

- **How to earn CEUs:**

- After the course, go to your Pending Courses on AudiologyOnline and pass the multiple-choice exam
- Must have a paid membership to earn CEUs

- **For assistance or tech support:**

- Call 800-753-2160
- customerexperience@continued.com

An Optimized Tour through Connexx Fitting Software with IX

1. Upon completion of this course, participants will be able to perform a First Fit with any Signia hearing aid.
2. Upon completion of this course, participants will be able to describe how to view Signia Assistant changes within Connexx.
3. Upon completion of this course, participants will be able to identify what's new in the software for the IX platform.





IX

Integrated
Xperience

Introducing Signia's Integrated Xperience: Unleash the Power of Conversation

Danielle Robertson, AuD
Sr. Clinical Education Specialist

Be
Brilliant™





A woman with dark hair tied back, wearing a blue and white striped turtleneck, is smiling and looking upwards and to the right. She is standing in front of a large, vibrant, abstract mural featuring various colors like purple, green, yellow, and pink, with some black and white striped patterns. The mural appears to be a collage of different images and textures.

Signia Integrated Xperience

Unleash the Power
of Conversation



Be
Brilliant™

Group Conversation in Noise is Difficult...



Group conversations in noise are the biggest unmet need for people with hearing Loss

Can lead to frustration, isolation and ultimately withdrawal

They miss out on the moment, we miss out on their contribution to the conversation




Wearers Need a New Approach that Puts Conversation First

Adapts to and enhances
all conversation partners in real time

Preserves
the surrounding sounds
keeping them comfortable



Built on Augmented Focus™ Split Processing

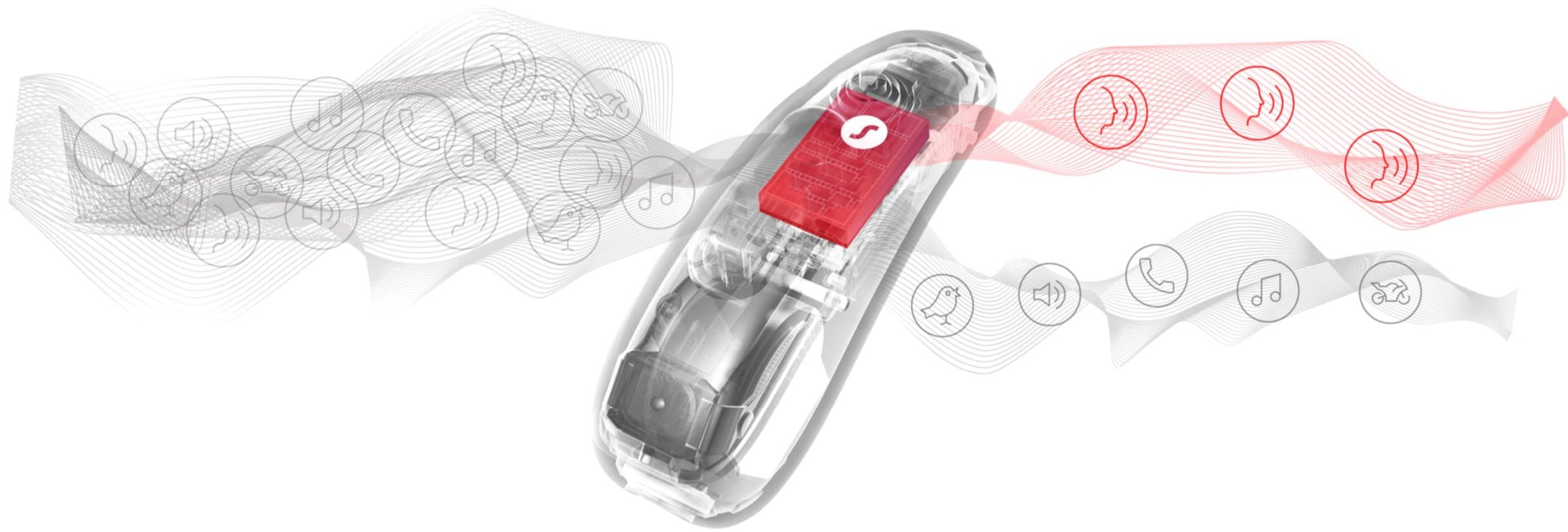



Introducing Signia Integrated Xperience with RealTime Conversation Enhancement


A breakthrough approach to address the long-standing challenges faced by hearing aid wearers



Don't Just Take our Word for It, See It in Action...






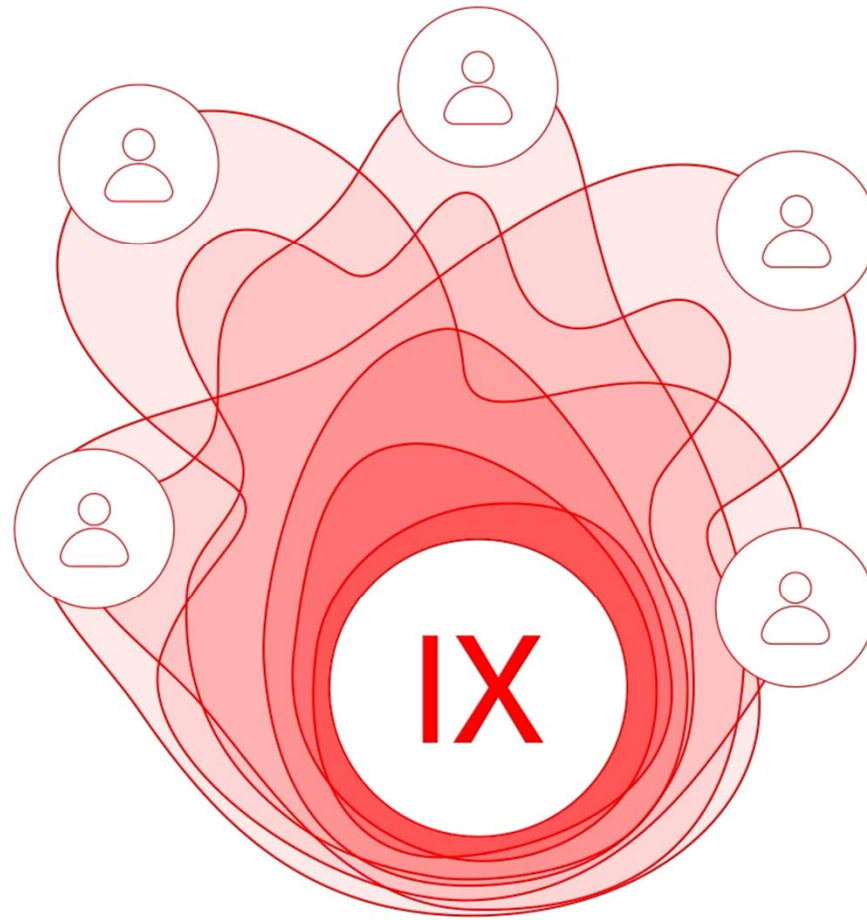

Signia IX with RealTime Conversation Enhancement

A revolutionary three stage process that puts conversation first:

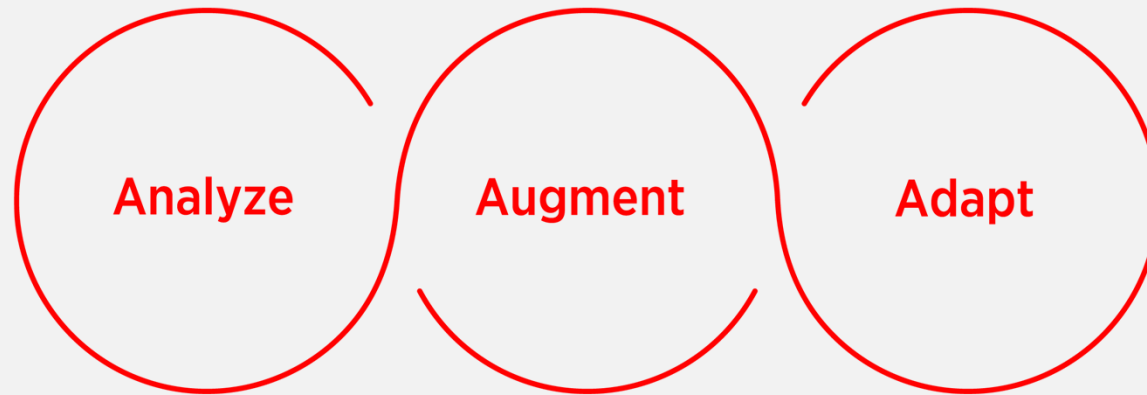


The world's first hearing aid with a multi-stream architecture capable of accurately pinpointing multiple moving speakers in real-time conversation



Adapt: A Live Auditory Space

The result is a live auditory space that can keep up with dynamic conversations in challenging acoustic environments.

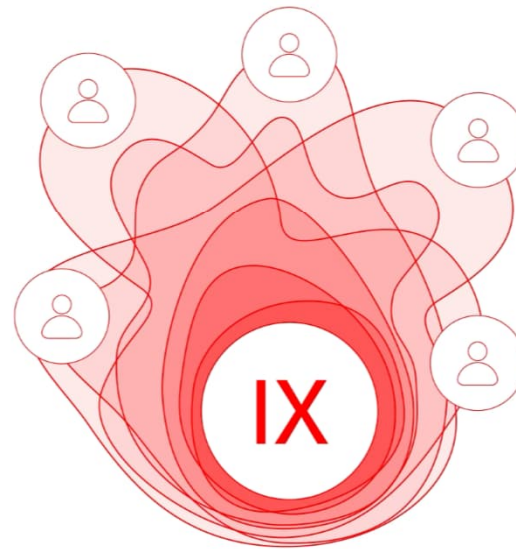

RealTime Conversation Enhancement Benefits

A comfortable, immersive conversational experience without distractions


Access to multiple speakers with speech that pops above the surrounding sound

Keep up and contribute to the conversation



Unleash the Power of Conversation With Integrated Xperience

Access to group conversation

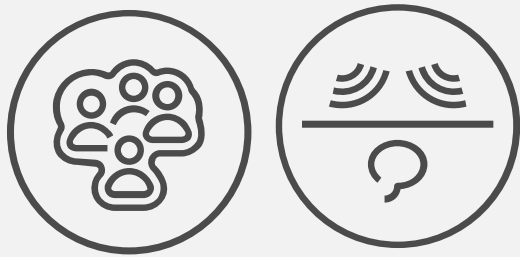

RealTime Conversation Enhancement
&
Split Processing

Contribute comfortably



Own Voice Processing 2.0

Fine-tuning in the moment



Signia Assistant




Data Insights (previously Data Logging)

Signia Assistant history is always available.

Signia Assistant icons:

 if new Signia Assistant changes are available

 if only history is available

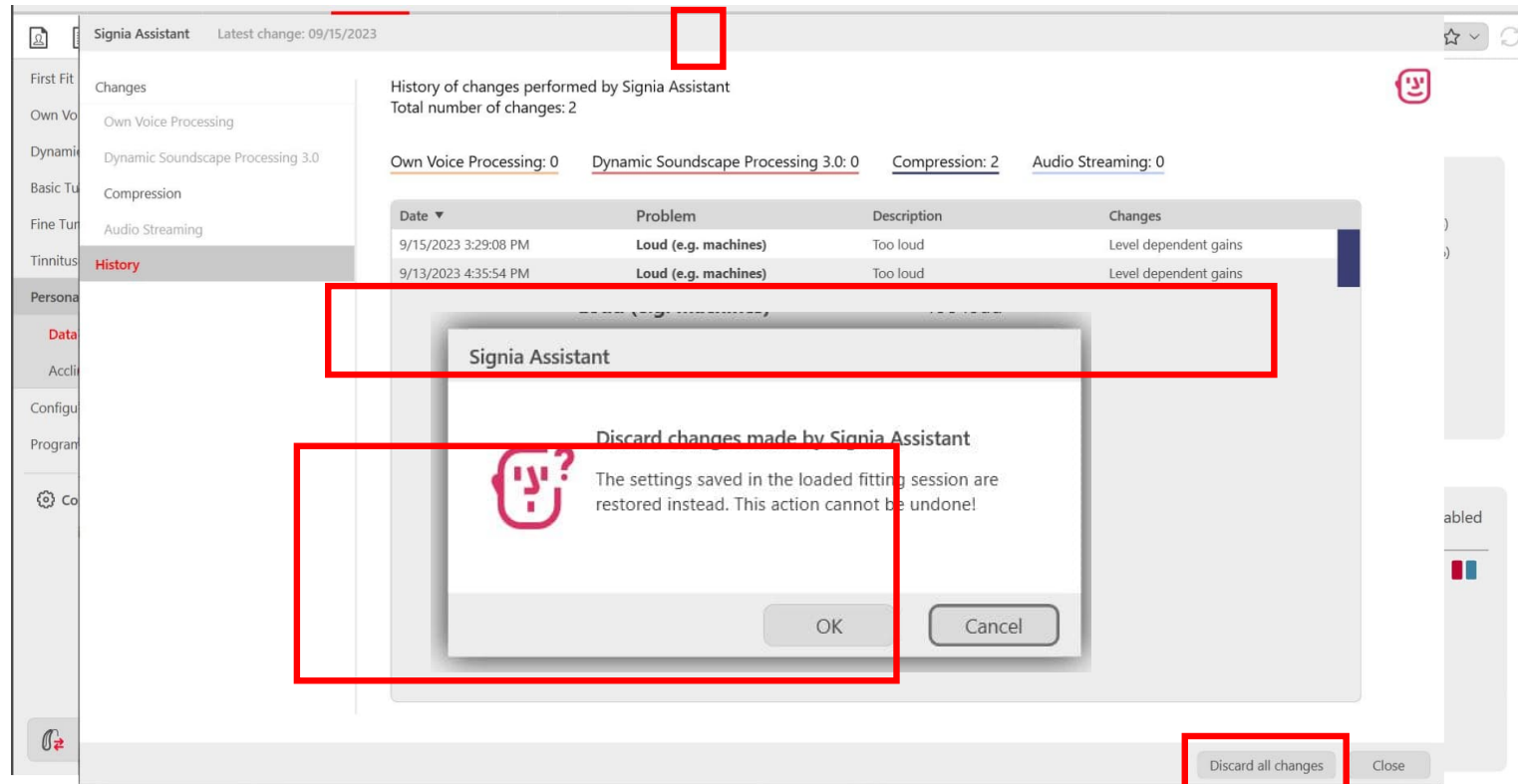


The screenshot shows the Signia Assistant software interface. The top bar indicates the latest change was on 09/15/2023. The left sidebar lists various settings categories, with 'History' selected. The main panel displays the 'History of changes performed by Signia Assistant' with a total of 2 changes. Below this, a table lists the changes:

Date	Problem	Description	Changes
9/15/2023 3:29:08 PM	Loud (e.g. machines)	Too loud	Level dependent gains
9/13/2023 4:35:54 PM	Loud (e.g. machines)	Too loud	Level dependent gains

A warning dialog box is overlaid on the screen, titled 'Signia Assistant' and 'Discard changes made by Signia Assistant'. The message states: 'The settings saved in the loaded fitting session are restored instead. This action cannot be undone!'. The dialog has 'OK' and 'Cancel' buttons. A red box highlights the 'Discard all changes' button at the bottom right of the interface.

What's new in Connexx 9.11

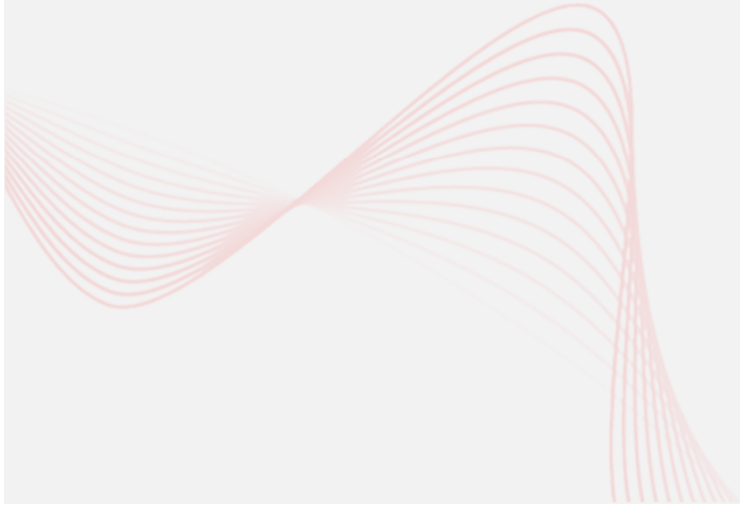

Further Personalization via DSP 3.0

RealTime Conversation Enhancement is a part of **Dynamic Soundscape Processing 3.0**.

This means that it is affected adjusting the **DSP slider in Connexx** and by DSP changes made via the **Signia Assistant**.



What's New in Connexx 9.11?


- Increase in the number of processing channels and frequency shaping handles for 5 IX and 3 IX performance levels
- Improved AutoFit workflow and ease-of-use:
 - Displaying workflow steps and progress
 - Adding helpful visualization (e.g., correct tube calibration position on ears)
 - Additional textual hints for each step of the AutoFit process

Overview of handle (channel) numbers

	700	500	300
AX	20 (48)	16 (34)	12 (34)
IX	20 (48)	18 (36)	16 (32)




7 IX

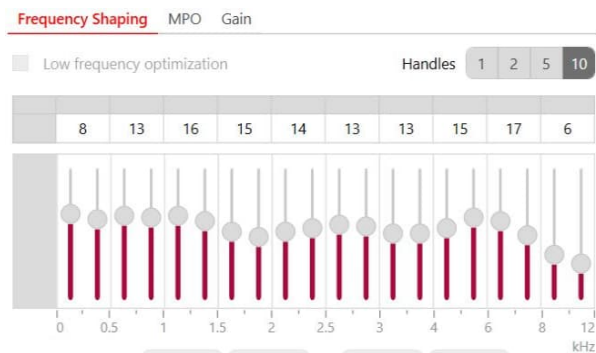

5 IX



3 IX

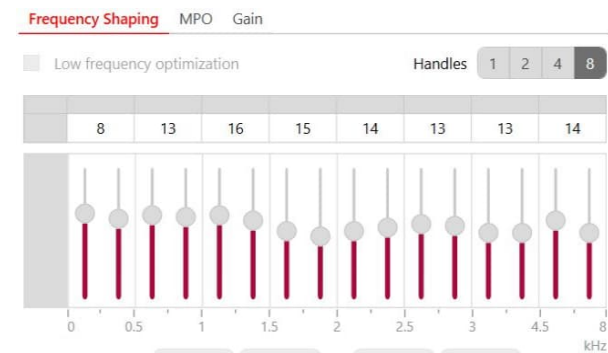


Unleash the Power of Conversation with Signia Integrated Xperience

A decade of research and breakthrough technologies come together to drive unique wearer benefits

- Hear multiple talkers even as they move with RealTime Conversation Enhancement technology
- Contribute comfortably with Own Voice Processing 2.0
- Get help when you need it, where you need it with Signia Assistant

Integrating seamlessly to unleash the power of conversation

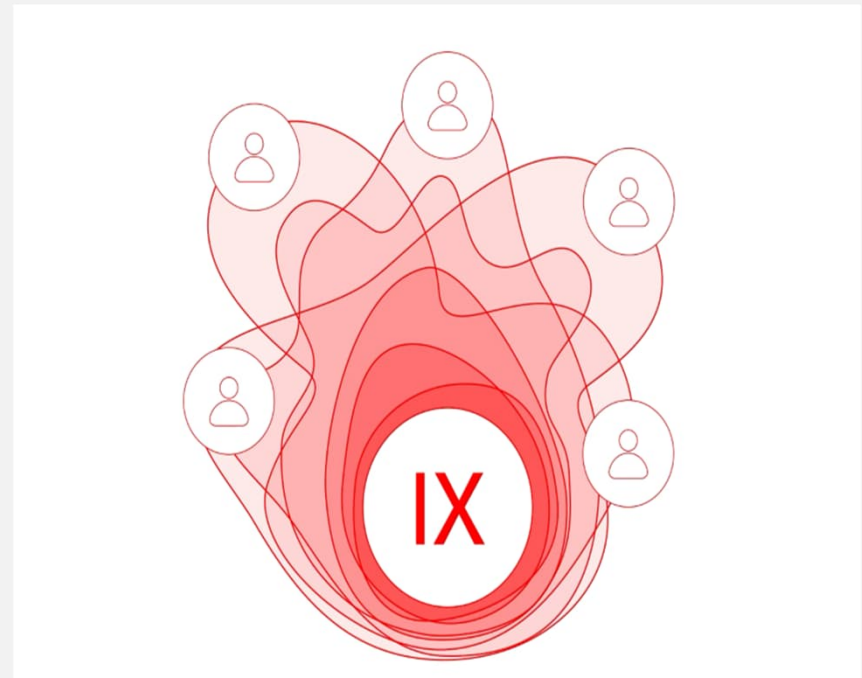

CEU Courses

How to Personalize Your Patient's Hearing Solution

Course# 38297 0.5 CEU

Learn how to maximize Signia AX technology to personalize your patient's prescription fitting. This course will focus on fitting adjustments in Connexx 9.8 to meet unique wearer needs and preferences.

A Guided Tour of Connexx 9.9

Course# 38469 1 CEU

Discover how Signia's Connexx 9.9 software brings fitting tools to enhance patient experiences while remaining intuitive for the professional. This course is a deep-dive tour providing details on how to perform a first fit as well as strategies for addressing common patient concerns.

Quick Tips

Signia QuickTips: What's New in Connexx

Course# 37919

Connexx is the fitting software for all Signia hearing aids. Learn about the new features in Connexx which will enhance the fittings of your AX patients.

Signia QuickTips: NEW Customizable Navigation in the Connexx Fitting Software

Course# 37920

Connexx now adapts to the Hearing Care Professional. Learn how to customize the fitting software to best match your fitting workflow.

Browse all Signia AO courses at <https://www.audiologyonline.com/ce/signia/events>



Unleash the Power of Conversation with Integrated Xperience!

Thank you for joining today!

IX

Integrated
Xperience

Be
Brilliant™



signia