

1. This document was created to support maximum accessibility for all learners. If you would like to print a hard copy of this document, please follow the general instructions below to print multiple slides on a single page or in black and white.
2. This handout is for reference only. Non-essential images have been removed for your convenience. Any links included in the handout are current at the time of the live webinar, but are subject to change and may not be current at a later date.
3. Copyright: Images used in this course are used in compliance with copyright laws and where required, permission has been secured to use the images in this course. All use of these images outside of this course may be in violation of copyright laws and is strictly prohibited.
4. Social Workers: For additional information regarding standards and indicators for cultural competence, please review the NASW resource: [Standards and Indicators for Cultural Competence in Social Work Practice](#)
5. Need Help? Select the “Help” option in the member dashboard to access FAQs or contact us.

How to Print Handouts

- On a Mac
 - Open PDF in Preview
 - Click File
 - Click Print
 - Click dropdown menu on the right “preview”
 - Click layout
 - Choose # of pages per sheet from dropdown menu
 - Checkmark Black & White if wanted.
- On a PC
 - Open PDF
 - Click Print
 - Choose # of pages per sheet from dropdown menu
 - Choose Black and White from “Color” dropdown

No part of the materials available through the continued.com site may be copied, photocopied, reproduced, translated or reduced to any electronic medium or machine-readable form, in whole or in part, without prior written consent of continued.com, LLC. Any other reproduction in any form without such written permission is prohibited. All materials contained on this site are protected by United States copyright law and may not be reproduced, distributed, transmitted, displayed, published or broadcast without the prior written permission of continued.com, LLC. Users must not access or use for any commercial purposes any part of the site or any services or materials available through the site.



Introducing **Beltone Serene**



miniRIE

Beltone **Achieve**[™]
Previous generation




microRIE

Beltone **Serene**[™]




1

Beltone Serene **microRIE Circuit**

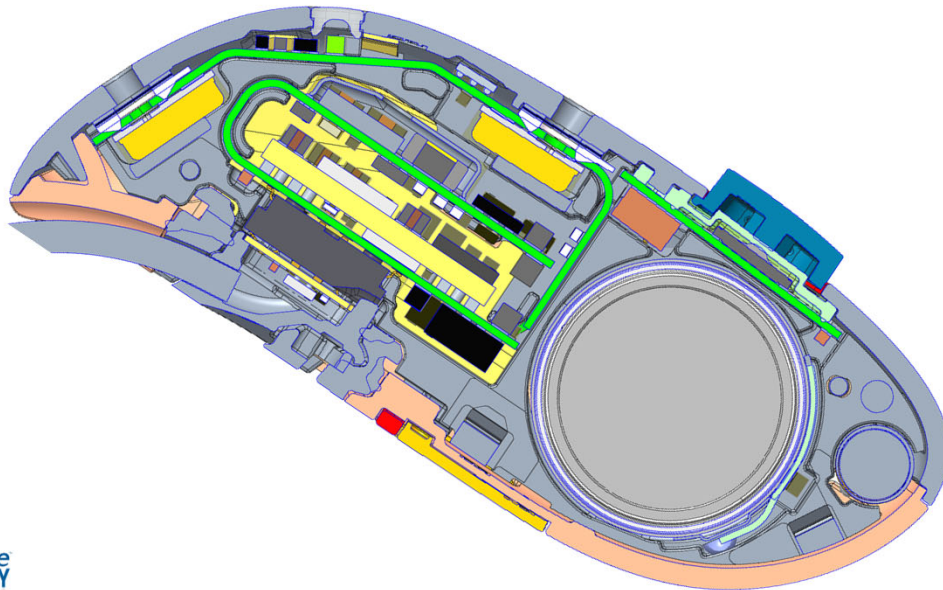


2

Beltone Serene

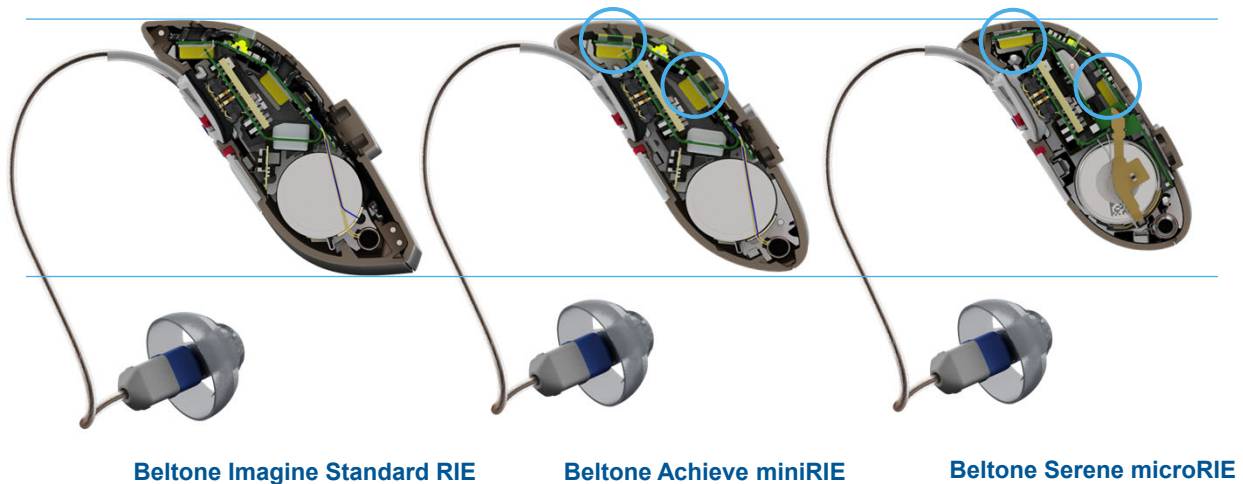

3

A New Generation of Hearing

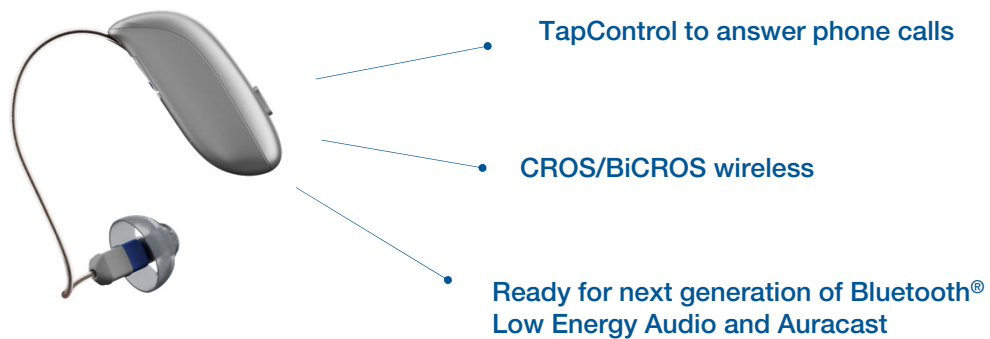

4

Beltone Serene Styles

TapControl



microRIE (62S)

Rechargeable



Standard RIE

312 battery (63) & 13 battery (64)

In technology levels 17, 9, 6



5

Beltone Serene Rechargeable microRIE



microRIE SER 62S

- Lithium ion/induction rechargeability
- All day battery on 1 charge
- All-Weatherproof (IP 68)
- Push to assign, LED Light, Push Button
- 2.4 GHz and E2E communication
- MFi and Hands-free calls
- ASHA

Plus

- TapControl
- Compatible with new TV-Streamer+
- Bluetooth Low Energy Audio (Auracast ready)



6

Beltone Serene **RIE** Charger Options



Premium Charger

- Integrated on-board battery
- Works as charging case
- Provides up to 3 charges
- LED battery status indicator



Standard Charger

- Charges while connected to power cable
- Single LED indicator when connected to power source



Desktop Charger

- Charges while connected to power cable
- Simple, easy to use design

7

All-day Power

30 hr

Maximum for the
microRIE on one
charge

~24 hr

Typical use time on
one charge

20 hr

When streaming audio
50% of the time

Ultra fast charging

Charge time

3 hr

1 hr

24 hr (all day)

8 hr (business day)



8

Beltone Serene Replaceable Battery RIEs



- Push Button/Multi Toggle button
- All-Weatherproof (IP 68)
- Push to assign
- 2.4 GHz and E2E communication
- MFi and Hands-free calls
- ASHA

Plus

- Compatible with new TV-Streamer+
- Bluetooth Low Energy Audio (Auracast ready)



9

Eight Stylish Colors




IP68 protection rating in & out, including receivers



10

SureFit 3 Receivers **Fitting Ranges**

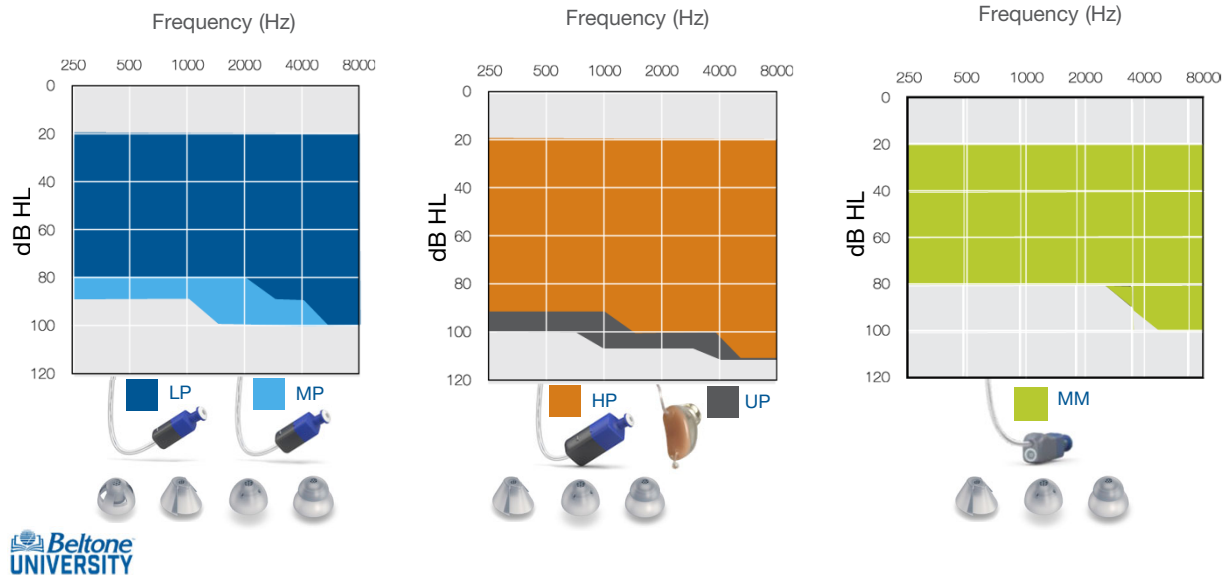

11

Beltone Solus Max 1.17



12

First Time User -Onboarding

- Goal: focus on comfort across all input levels & most comfort in noise
 - Gain level% set to 70%
 - Increased noise reduction
- Replaces Comfort User profile

 Prescribed First Fit ⓘ

Target Rule
BAFA ✓

Experience Level ⓘ
First Time User - Onboarding ✓
Focus on comfort across all input levels & most comfort in noise

Gain Level (%) ⓘ
70% ✓

☐ Recalculate Target Curves Only
Check to keep existing gain curves. Best to view new target curves while preserving fine tuning.



13

New Hear in Noise Program

Goal: simplify programming



14

Hear in Noise Directionality Options




Hear in Noise icon in the HearMax app



Restaurant





Ultra Focus & Ultra Focus 2

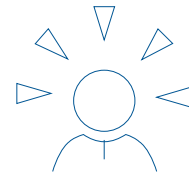

Hear in Noise


15

TapControl

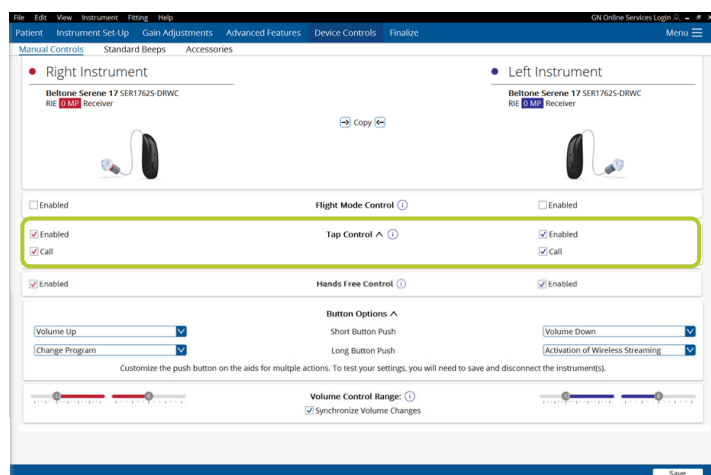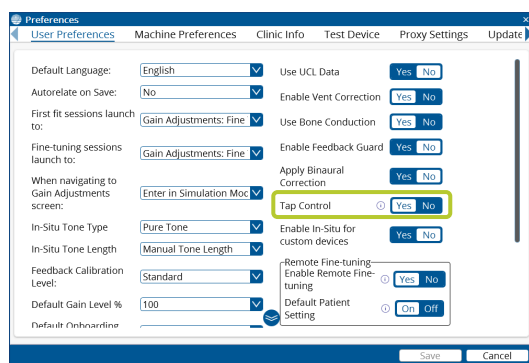


16

TapControl Settings

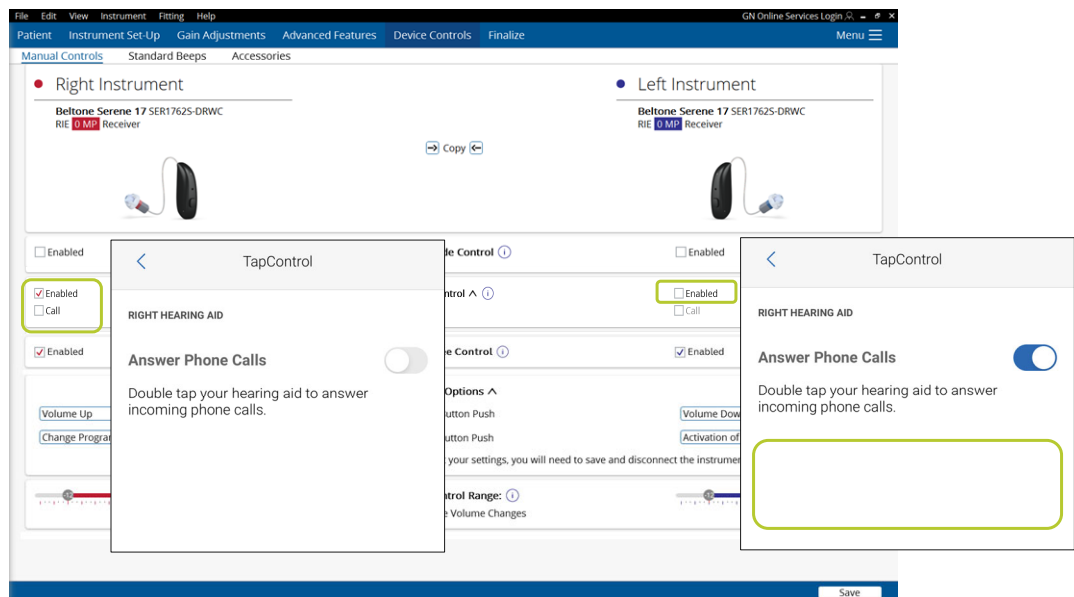

17

Disabling Hands-Free

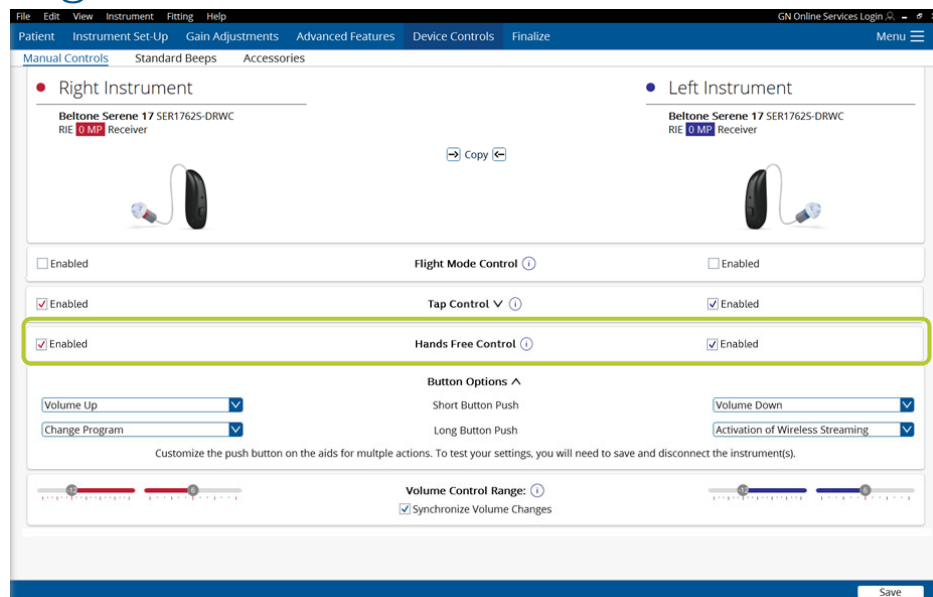

18

Over the Air Updates

Menu X

Patient

Speech Scores

Patient Consent

RECD | REUG

Instruments

Express Fit

Reset to Initial Fit

Reconfigure

Test Device

Accessory Updater

Fitting

Beltone Solus Max Guide

Notifications

Firmware Update

Check for Updates

Other

Sound Player



Preferences

User Preferences Machine Preferences Clinic Info Test Device Proxy Settings Updates

Enable Web Updates ☒

Check for updates on a scheduled interval ☐ 1 Week

Always download updates when available ☐

Enable Firmware Download

Save Cancel

19

Learning Summary



Beltone Serene is offered in 3 RIE style using our SureFit 3 receivers and domes



TapControl on Bluetooth Low Energy Audio are the new technology features of Beltone Serene.



In Beltone Solus Max 1.17 we have our new user profile and our new Hear in noise program.



20