


HOW TO FIND MANUFACTURERS' ANSI TEST SETTINGS (Feb. 2024)


Phonak (Target v9.0)

No client is necessary. Open the Target module, then click the **Trials & tools** icon at the bottom. Then select **Measurement settings** on the left menu.

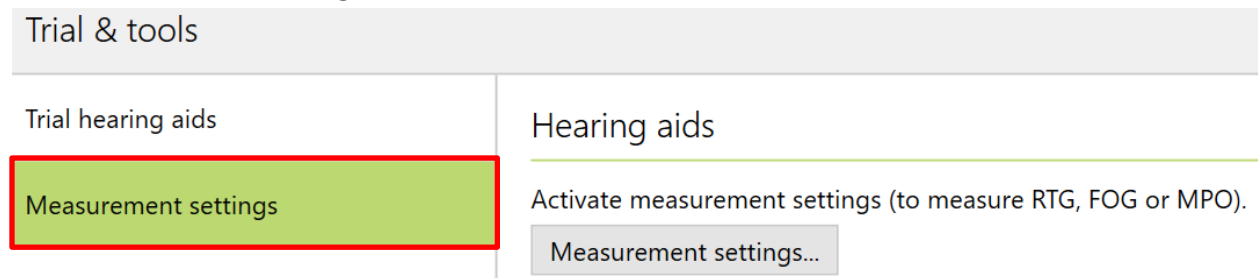


Unitron (TrueFit v5.5)

No client is necessary. Select **Test Mode** under the **Hearing Instrument** menu at the top.

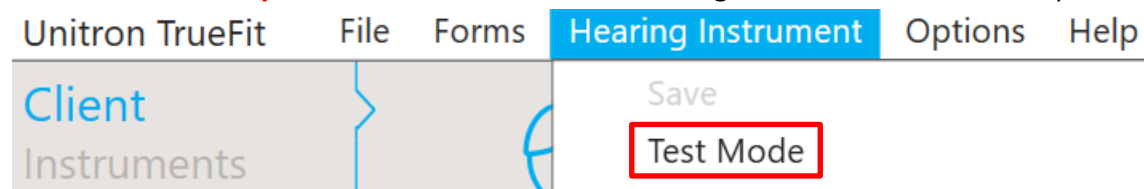


Oticon Genie 2 (v2023.2)

After loading the client, select **Technical Measurements** under the **Tools** menu.

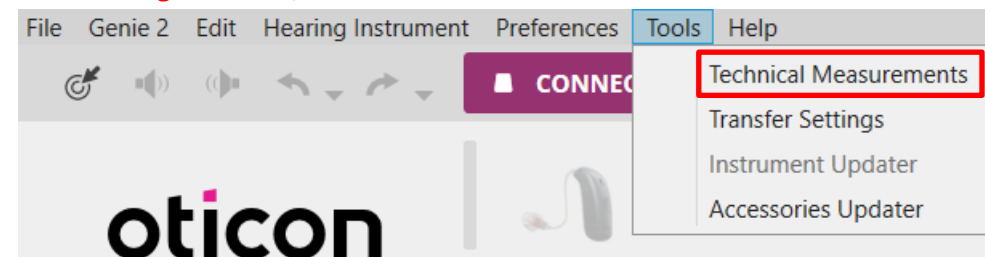

Signia (Connexx 9.11)


After loading the client, click the **Fitting** menu on the top of the screen. Then, select **Test Settings.**

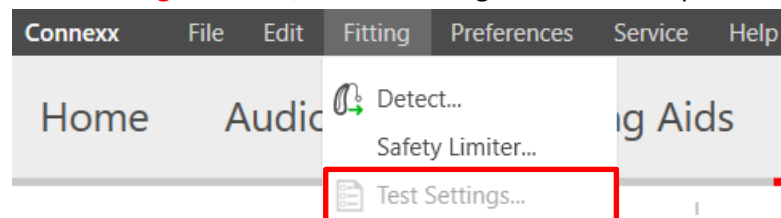

HOW TO FIND MANUFACTURERS' ANSI TEST SETTINGS (Feb. 2024)


ReSound (Smart Fit 1.17)

After loading the client, select **Test Device** under the **Instrument** menu.

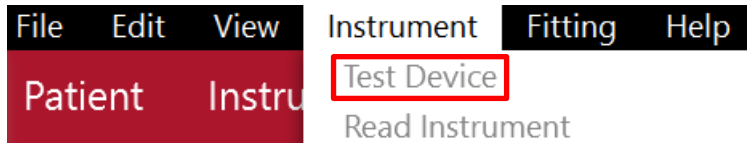

Starkey (Inspire X 2023.1)

After loading the client, click the **Advanced Tools** menu on the left side of the screen. Then, select "Hearing Aid Test."

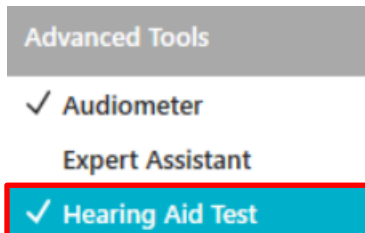

Widex (COMPASS GPS 4.5)

After loading the client, click the **Quality assurance** option on the left side of the screen.

