



signia

Quick Tips

AUDIOLOGYonline



Integrated Xperience Performance Level Differentiation

Dani M. Rood, AuD



Integrated Xperience Performance Level Differentiation

- Thank you for joining us to learn more about the performance level offerings for the Signia Integrated Xperience platform and their technology differences.
- For assistance with this course, contact AudiologyOnline (1-800-753-2160)
- Please visit the AudiologyOnline website for other live and recorded events from Signia.

<http://www.audiologyonline.com/ce/signia>

Dani M. Rood, AuD



audiologyonline | 



Group Conversation in Noise is Difficult...

Group conversations in noise are the biggest unmet need for people with hearing loss


Can lead to frustration, isolation and ultimately withdrawal

They miss out on the moment, we miss out on their contribution to the conversation



Signia IX with RealTime Conversation Enhancement


A revolutionary three stage process that puts conversation first:

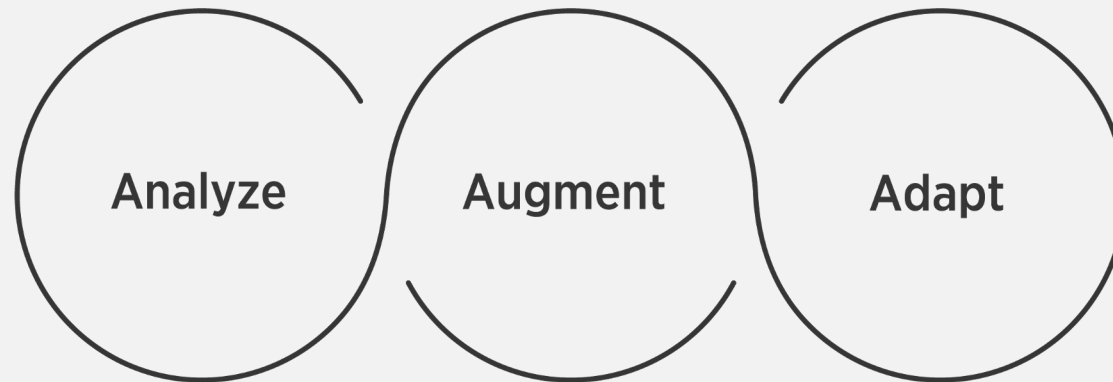

The world's first hearing aid with a multi-stream architecture capable of accurately pinpointing multiple moving speakers in real-time conversation



7IX: Real Time Conversation Enhancement

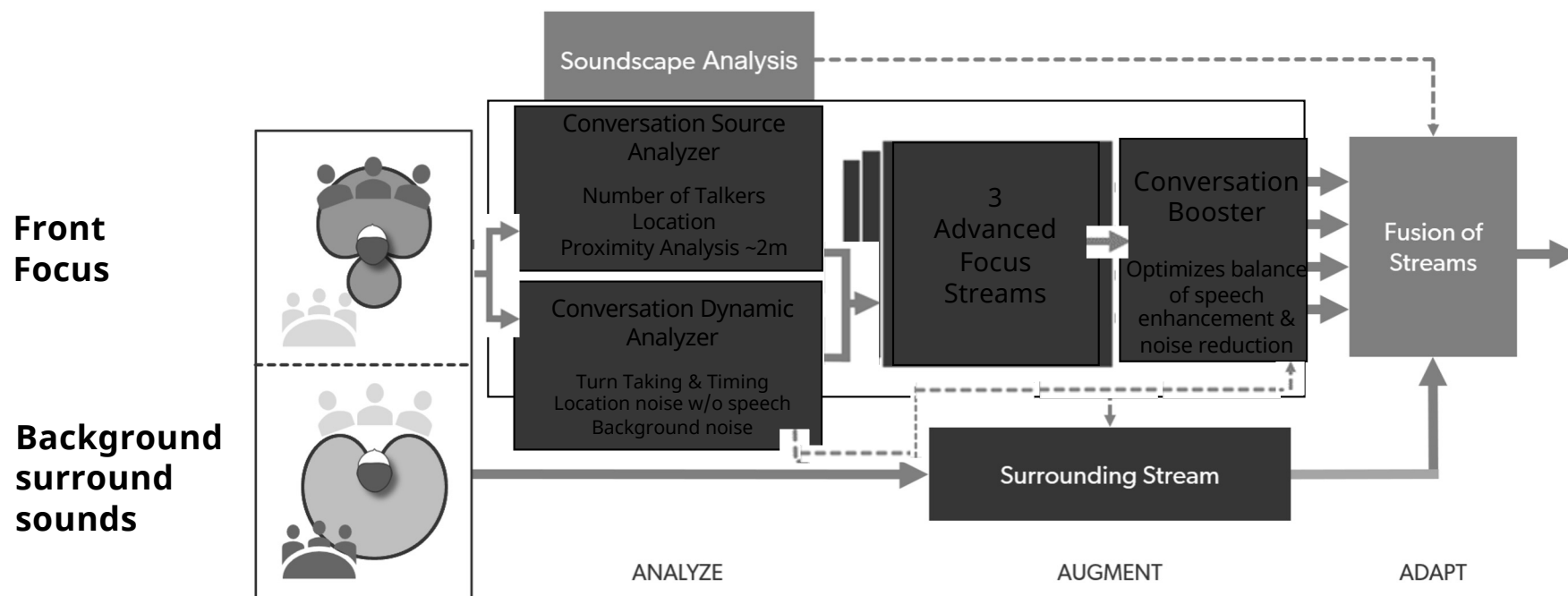

Unleash the Power of Conversation

Signia Integrated Xperience (IX) with RealTime Conversation Enhancement.



Premium 7 IX

48 Channels




Real-time enhancement of all your conversation partners, creating a live and dynamic auditory space to help you stay in the conversation.



Star ratings indicative to benefits of features available.



5IX: Real Time Conversation Enhancement

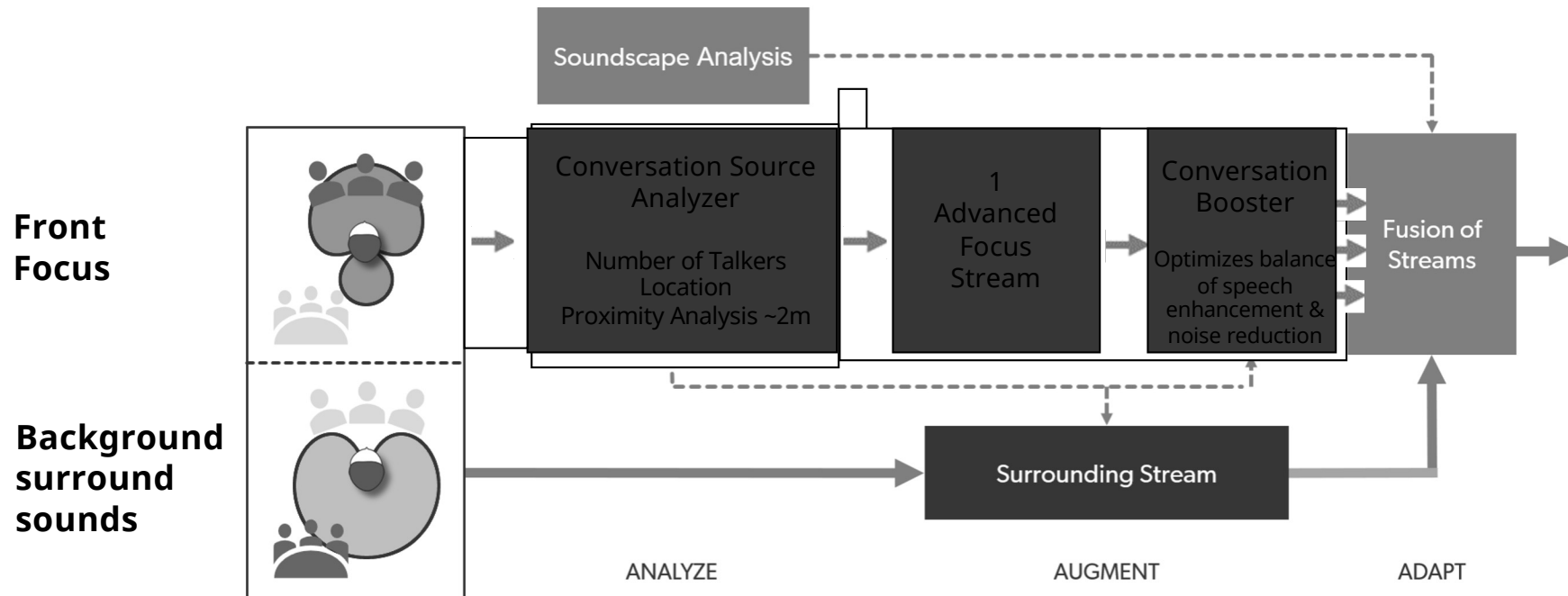

Unleash the Power of Conversation

Signia Integrated Xperience (IX) with RealTime Conversation Enhancement.



Advanced 5 IX

36 Channels




Directed enhancement of conversation partners within your area of focus while keeping you connected to your surroundings.

★ Star ratings indicative to benefits of features available.



3IX: Real Time Conversation Enhancement

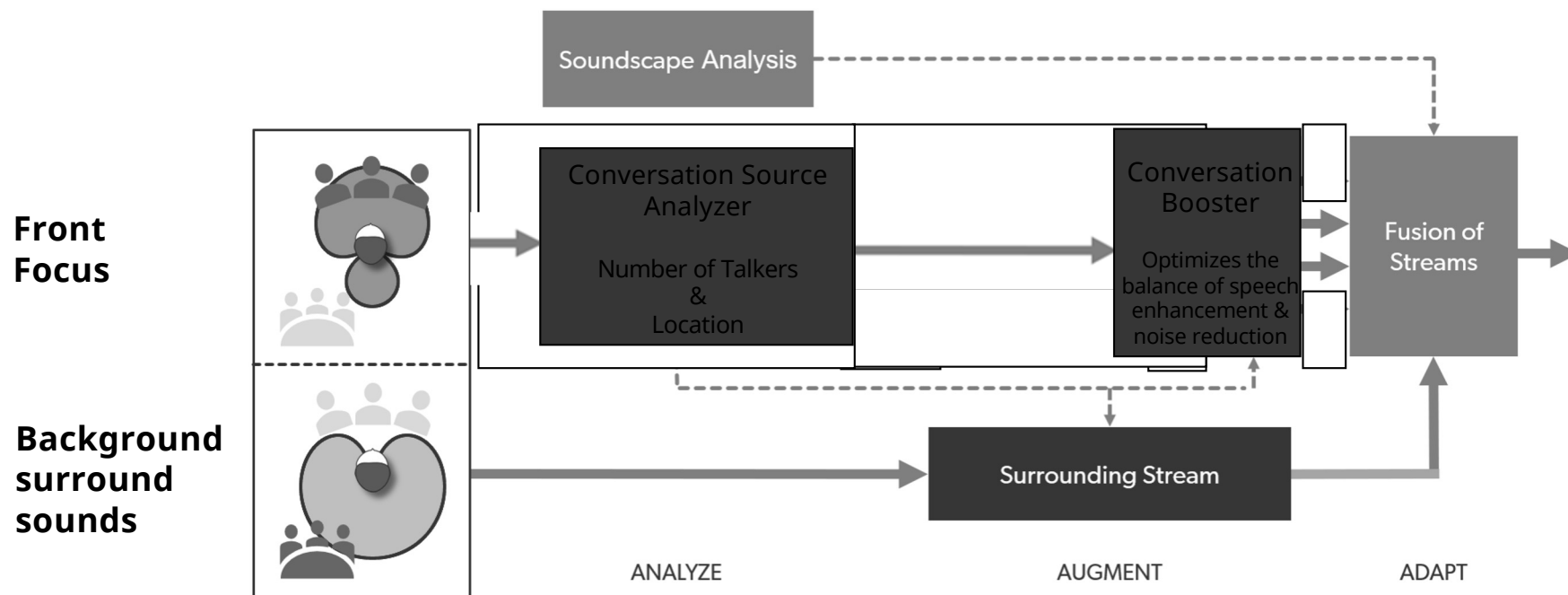

Unleash the Power of Conversation

Signia Integrated Xperience (IX) with RealTime Conversation Enhancement.



Standard 3 IX

32 Channels



General enhancement of conversation speech over the background noise to create an easier listening experience.

★ Star ratings indicative to benefits of features available.



- [Voiceover] Conversations have the power



14

Unleash the Power of Conversation

Signia Integrated Experience (IX) with RealTime Conversation Enhancement

95% of wearers
have better speech
reception with
RealTime Conversation
Enhancement than
without.*

Premium 7 IX

48 Channels

Real-time enhancement of all your conversation partners, creating a live and dynamic auditory space to help you stay in the conversation.

Advanced 5 IX

36 Channels

Directed enhancement of conversation partners within your area of focus while keeping you connected to your surroundings.

Standard 3 IX

32 Channels

General enhancement of conversation speech over the background noise to create an easier listening experience.

*See rating a indicator to benefits of features available.

Scan QR code to
watch a video about
Integrated Experience
technology in vivo

*Hearsting.com/2015/05/01/signia-ix-for-hearing-loss

Signia Integrated Xperience (IX) with RealTime Conversation Enhancement*

A breakthrough innovation that meets the expectations of hearing aid wearers

Conversation Management




	7	5	3
Real Time Conversation Enhancement*	✓	✓	✓
Augmented Focus	✓	✓	✓
Dynamic "SoundShape" Processing 3.0	✓	✓	✓
Speech-Definer	✓	✓	✓
Motion Sensor	✓	✓	✓
6.0 Wireless 4.0	✓	✓	✓
Speech & Noise Management	✓	✓	✓
TriBax 360	✓	✓	✓
Binaural Directivity	✓	✓	✓
CROS/B/CROS Compatibility	✓	✓	✓
Advanced Focus Streams	✓	✓	✓
SpeedFocus	✓	✓	✓
Conversation Dynamics Analyzer	✓	✓	✓
Spatial SpeechFocus	✓	✓	✓

Listening Comfort




	7	5	3
Own Voice Processing 2.0	✓	✓	✓
Feedback Cancellation	✓	✓	✓
eMusicScreen™	✓	✓	✓
SoundSmoothing™	✓	✓	✓
Extended Dynamic Range	✓	✓	✓
Timbre Noise Therapy Manager	✓	✓	✓
Timbre Noise Therapy	✓	✓	✓
HD Music	✓	✓	✓
Reverberant Room Program	✓	✓	✓
Auto EchoShield	✓	✓	✓
Extended Bandwidth	✓	✓	✓

Wearer Empowerment




	7	5	3
Telecoil - Phone/Audio Loop Systems	✓	✓	✓
Direct Streaming for iOS® & Android™**	✓	✓	✓
Adaptive Streaming Volume	✓	✓	✓
Handset for iOS	✓	✓	✓
Call Control for iOS	✓	✓	✓
Bluetooth® Streaming Accessories	✓	✓	✓
Remote Control Option	✓	✓	✓
Up to 6 Custom Programs	✓	✓	✓
IP68	✓	✓	✓
Signia App for iOS and Android	✓	✓	✓
Signia Assistant	✓	✓	✓
My WellBeing	✓	✓	✓
TelCare	✓	✓	✓
Directional Hearing	✓	✓	✓




Apple, the Apple logo, and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. Apple products are marks of Apple Inc. Android, Google Play and the Google logo are trademarks of Google Inc.

*Real Time Conversation Enhancer smartest new feature of the IX. **iOS App Store only for download. Signia is registered trademark of the SDA, A/S. (S1-13026)

**Be
Brilliant™**





signia

Thank You!

SigniaUSA
800-766-4500

aUDIOLOGYonline