



# Quick Tips: Introducing Styletto IX

AUDIOLOGYonline

# Signia QT: Introducing Styletto IX

- Thank you for joining us to learn more about Styletto IX.
- For assistance with this course, contact AudiologyOnline (1-800-753-2160)
- Please visit the AudiologyOnline website for other live and recorded events from Signia.

<http://www.audiologyonline.com/ce/signia>

Dr. Dianna Theokas, Au.D.



audiologyonline | 

# Styletto IX A Conversation Starter




Be  
Brilliant™





# Styletto IX - A Conversation Starter

- Many older adults still have very little awareness of modern hearing aids
- Start the conversation with Styletto IX and redefine what a hearing aid can be
- Styletto IX and its modern, super-slim and sleek design can open the door to the unique benefits of Signia IX

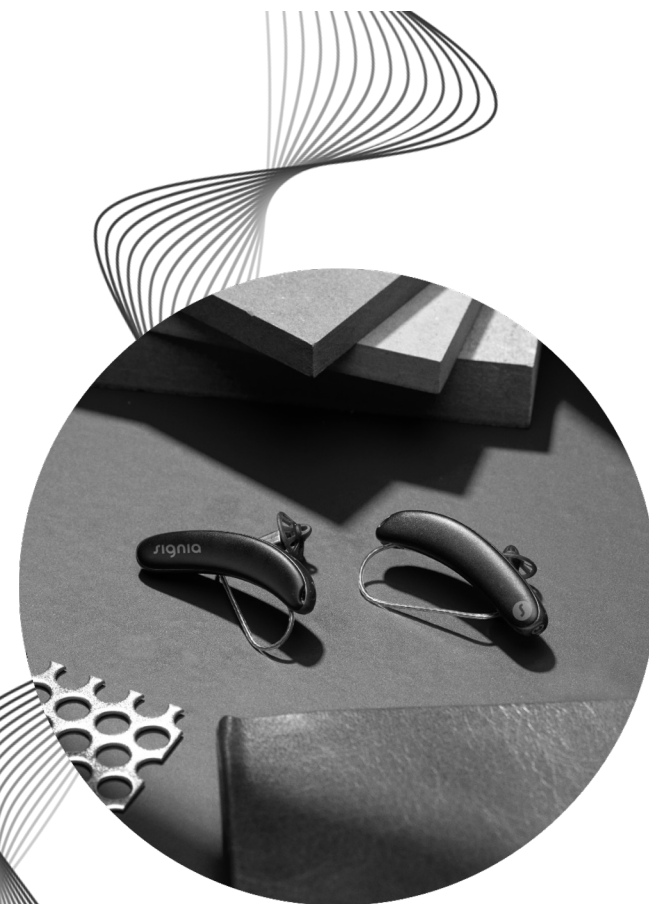

## Who Could Benefit... Meet Sylvia

- A bit taken aback that she has a hearing loss
- Feels she isn't "bad enough" for a hearing aid
- Likes to look good - doubts hearing aids will fit on her ears with her glasses



audiologyonline |



# Re-imagining Hearing Aids

- The industry is awakening to user needs, with attempts at making traditional hearing aids slimmer
- But it is about more than this
- Patients need devices designed from the ground up to redefine what a hearing aid can be



audiologyonline |



# Styletto With the Full Power of Signia Integrated Xperience

Hear multiple talkers  
even as they move

**RealTime Conversation  
Enhancement technology**

---

Contribute comfortably  
**Own Voice Processing 2.0**

---

Get help when you need it,  
where you need it  
**Signia Assist**

audiologyonline | 



# More Reasons to Offer Styletto

## **Article:**

*The Appeal of Signia's Innovative Form Factors and Their Role in the Adoption of Hearing Aids*

## **Results:**

Non-buyer rate decreased 21% when Styletto and Silk form factors are among the choices of devices to try.



<https://canadianaudiologist.ca/issue/volume-10-issue-6-2023/the-appeal-of-signias-innovative-form-factors-and-their-role-in-the-adoption-of-hearing-aids/>

## The Subtle and Sophisticated Styletto IX




Styletto IX

# Styletto IX – Ensuring Best performance



## **IP 68 rating**

Can be immersed in under 1m of water for 60-min  
No dust ingress after 2 h in a dust chamber

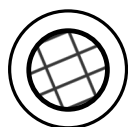

## **Hydrophobic & Oleophobic coating**

Water and oil protection



## **Patented anti-capillary design**

I – 6: dust tight, no ingress of dust after 2h  
P – 8: still functional after immersion in 1 meter of water for 1h

**audiologyonline** |



# Styletto IX Ready For a Full Day



20 hours of runtime

17 hours of battery runtime, including 5 hours of streaming

Get up to an additional 5 hours of runtime with a 30-minute fast charge



\*These values are the minimal guaranteed runtime of a fresh cell  
20% capacity degradation expected after 2 years

audiologyonline



# Stylecto IX – Acoustical Performance With EarWear 3.0



## miniReceiver 3.0:

Lengths: 0,1,2,3

Power: S, M, P

## Eartips 3.0:

Open: All sizes (XS, S)

Tulip: All sizes (M,L)

Round: All sizes (S,M,L)

## Sleeves 3.0:

Vented: XS, S, M

Closed: XS, S, M

## Custom earmolds 3.0

Compatible: Yes

| S<br>Receiver                         | M<br>Receiver                            | P<br>Receiver                         |
|---------------------------------------|------------------------------------------|---------------------------------------|
| 46dB / 110 dB SPL<br>(2 ccm coupler)  | 60 dB / 119 dB<br>SPL<br>(2 ccm coupler) | 65 dB / 122 dB SPL<br>(2 ccm coupler) |
| 56 dB / 120 dB SPL<br>(Ear simulator) | 70 dB / 129 dB<br>SPL<br>(Ear simulator) | 75 dB / 131 dB SPL<br>(Ear simulator) |

Options for EarWear 3.0 are limited due to size of the charger.

## Styletto IX – Seven Chic Colors



## Stylecto IX – Seven Chic Colors



# Styletto IX – Performance Levels and CROS



# Styletto IX Charger Details



## Styletto IX Charger

- Integrated power bank holds five additional charges
- LEDs to indicate charging status of hearing aids and of charger
- Can be charged wirelessly (Qi) or via USB-C cable




## Styletto Dry&Clean Charger

- Same product for Styletto IX and Styletto AX
- Charges, cleans and dries the hearing aids to ensure peak performance
- UV cleaning cycle takes 15 minutes
- LEDs indicate status

# Styletto IX - Seamless Connectivity




LE Audio Ready\*

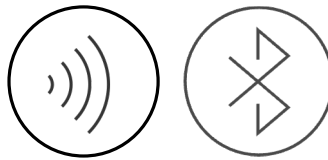

\*Future firmware update required

# Signia App

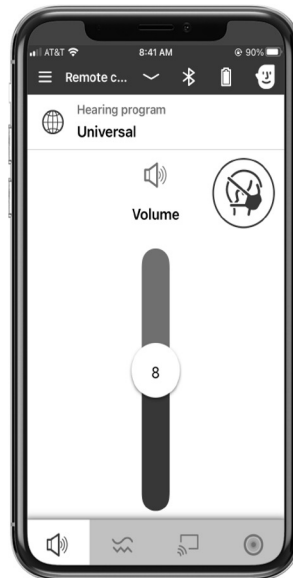

audiologyonline | 

# Styletto IX Accessories

## StreamLine™ TV

Direct high-quality streaming of TV sound.  
Perfect and clear TV signal directly into the hearing aids.



## StreamLine™ Mic

Stream audio from any Bluetooth device  
Remote mic


## miniPocket™

Discrete convenient remote control.

## Styletto IX – Wireless Programming



Wireless programming via Noahlink Wireless

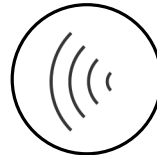

# A Conversation Starter - The Ultra-Slim Styletto IX

In summary:

- Expand your portfolio
- Uniquely slim super-sleek design
- With no compromise,  
and with fully-featured IX technology
- Enabling more people to  
unleash the power of conversation



AUDIOLOGYonline |





signia

**Thank You!**

**SigniaUSA**  
**800-766-4500**

**aUDIOLOGYonline**