

Phonak Target Software: Flexibility meets expertise


Erika Porter, AuD

Webinar FAQs

- **How to ask a question:**
 - Click Q&A, type your question and Send.
- **Resources Tab:**
 - Download the course handout
 - Link to course information page
 - View presenter information
- **How to view captions:**
 - Click Show Caption CC Button
- **How to earn CEUs:**
 - After the webinar, go to your Pending Courses and pass the multiple-choice exam.
- **For assistance or tech support:**
 - Call 800-866-2160
 - Use the Q&A window

Phonak Target Software: Flexibility Meets Expertise

Erika Porter, AuD

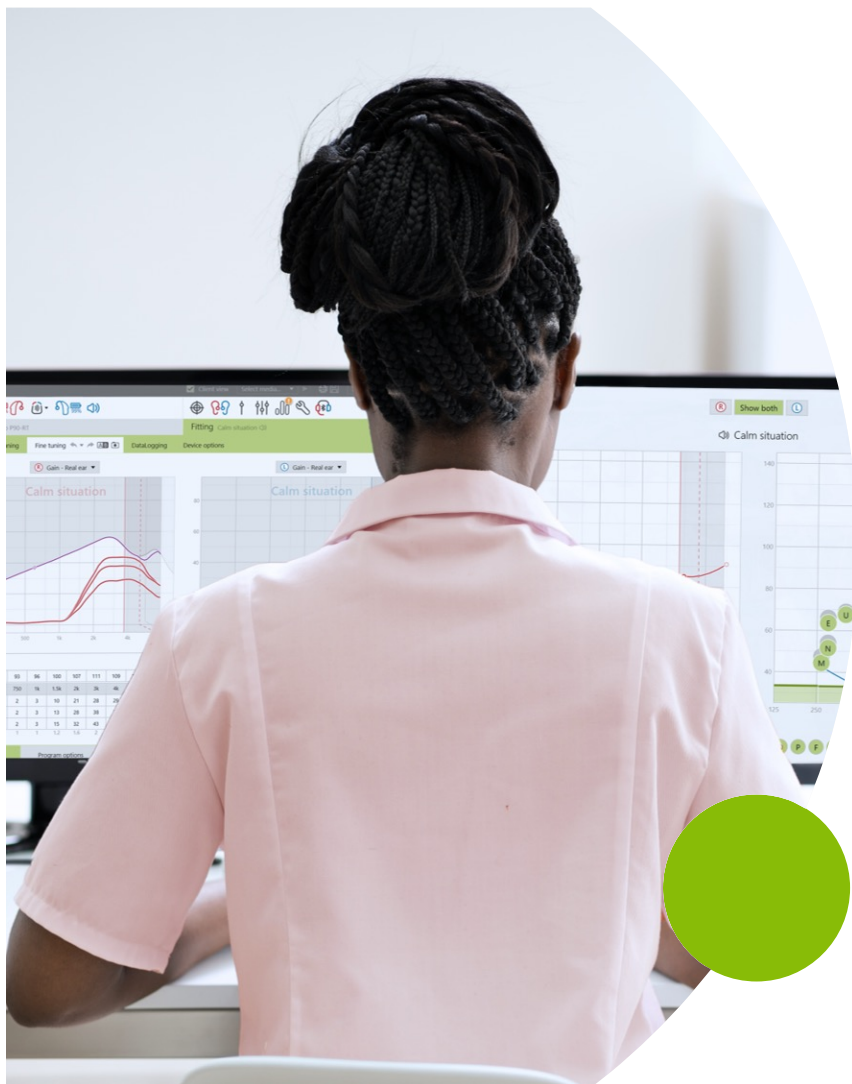




Learning Objectives


After this course, participants will be able to

1. Describe the controls that are available to program and adjust Phonak hearing aids.
2. Explain how to adjust Phonak hearing aid settings based upon patient needs.
3. Identify situations when adjustments will be beneficial.

Target Updates – Download via Target

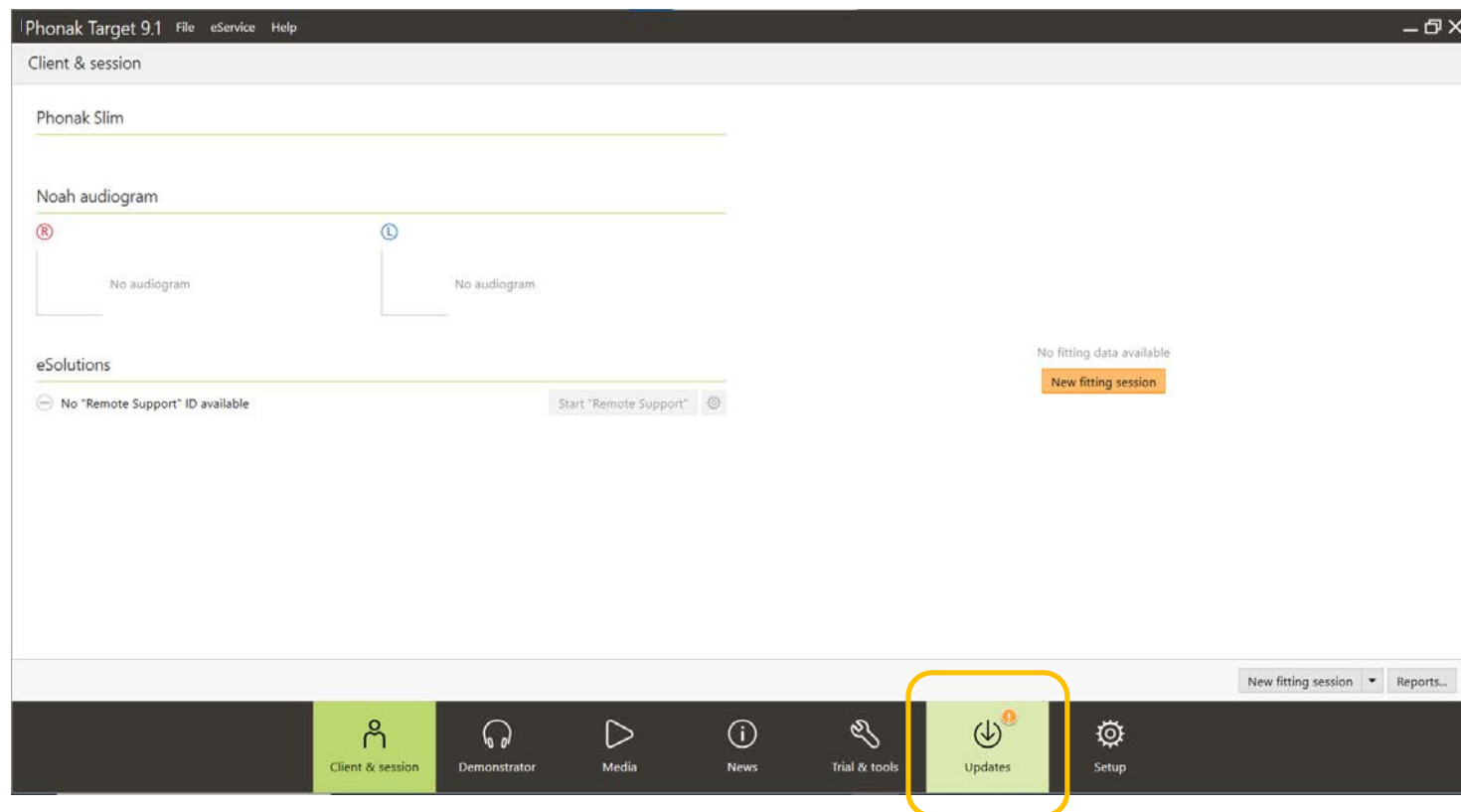

Target Software and Updates – Download via eStore

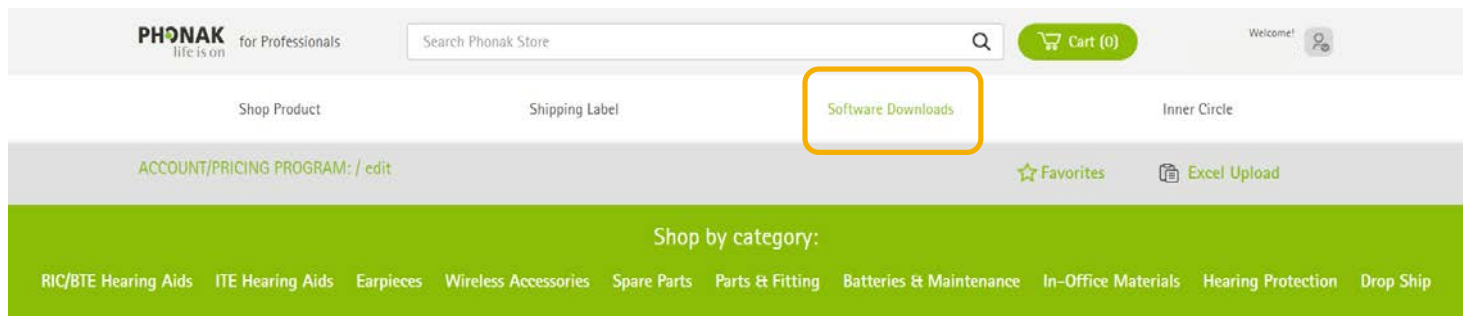

Phonak Audéo™ Lumity

Conversations shine with Lumity.

- SmartSpeech™ Technology featuring StereoZoom 2.0 and Speech Sensor
- Universal connectivity with hands-free calls
- Activity data tracking and instant personalization
- All-rechargeable RIC models including R, RT and Life

[Order Now](#)



Target Software and Updates – Download via eStore

Software Downloads

FITTING SOFTWARE	VERSION	RELEASE DATE	SIZE	LINK
 Phonak Target (full installer) Phonak's latest fitting software supporting the most recently launched products. Phonak Target supports platforms from 2010 (e.g. Ambra) all the way to the latest Phonak Lumity.	9.1.0.1862	26 February 2024	822 MB	Download 
Phonak Target Media Phonak Target media includes instructional videos and data sheets for the products.	9.1.2273	26 February 2024	1.1 GB	Download 
Phonak Target Sounds Phonak Target Sounds includes a variety of sound samples to demonstrate to your patients and the Benefit Demonstrator	5.0.2273	26 February 2024	1.2 GB	Download 
UTILITY SOFTWARE	VERSION	RELEASE DATE	SIZE	LINK
Sonova Firmware Update (Charger Go & TV Connector) A firmware update utility for TV connector, partner mic, and Charger GO More info 	3.1	n/a	80.8 MB	Download 
Roger Upgrader A firmware update utility for Roger Devices More info 	1.28	n/a	96.1 MB	Download 

System requirements

Operating system	- Windows 11, Home/Pro/Enterprise/ Education - Windows 10, Home/Pro/Enterprise/ Education
Processor	Intel Core or higher performance
RAM	4 GB or more
Hard disk space	3 GB or more
Screen resolution	1280 x 1024 pixels or more
Graphic card	16 Million (24bit) screen colors or more
Drive	DVD/ USB
Serial COM port	Only if RS-232 HI-PRO is used
USB ports	Bluetooth adaptor
One for each purpose	- Accessory programming - HI-PRO if used via USB port - Noahlink Wireless

Programming interfaces	Noahlink Wireless/ iCube II/ NOAHlink/ RS-232 HI-PRO/ HI-PRO USB/ HI-PRO 2
Noahlink driver	Latest version available
Noahlink Wireless driver	Latest version available
Internet connection	Recommended
Sound card	Stereo or surround 5.1
Playback system	20 Hz – 14 kHz (+/- 5 dB), 90 dB
NOAH version	Latest version (NOAH 4.4 or higher) Please check the NOAH limitations for Windows 64 bit operating systems on http://www.himsa.com
TargetMatch	- Noah Version 4.4.2280 or higher - Natus® Otosuite 4.81.00 or higher - Auditdata Primus version 4.1 until 5.2 - Interacoustics Affinity Suite version 2.19 until 2.24 - Signia Unity version 5.9 until 6.2

The Bluetooth® wordmark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Sonova AG is under license.

Live programming via Target

