


Phonak Target

Quick First Fit Guide

Before starting, ensure the patient's hearing loss information is entered into Target.

Connect the instruments

Click **[Connect]** to start the fitting. The connected hearing aids will appear in the dashboard.

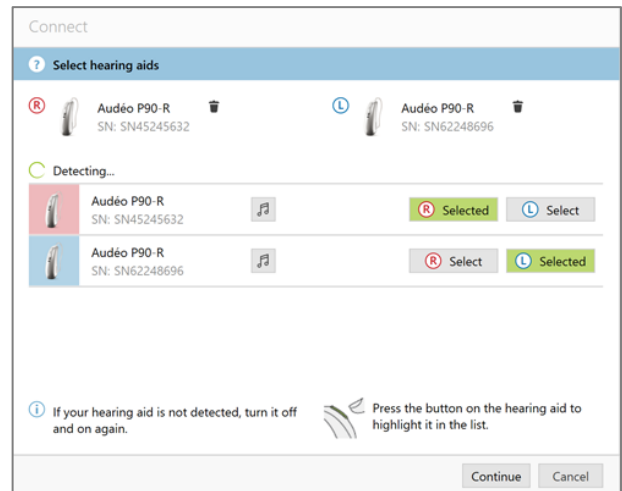


For all new fittings, a suggested patient experience level (based upon fitting session information available) will be suggested.

First time user = 80% target gain

Experienced user = 90% target gain


Long Term user = 100% target gain

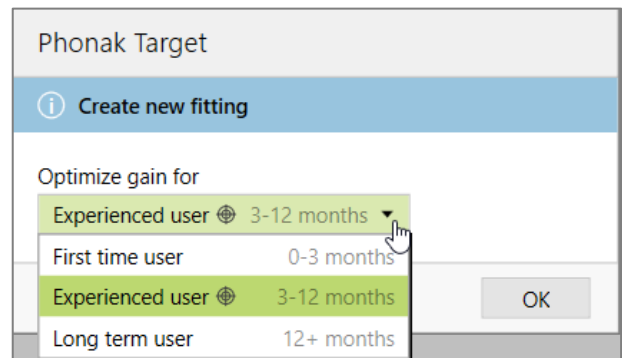

Check acoustic parameters


Ensure the correct receivers and domes are selected.

Run Feedback test/apply overtuning if needed.

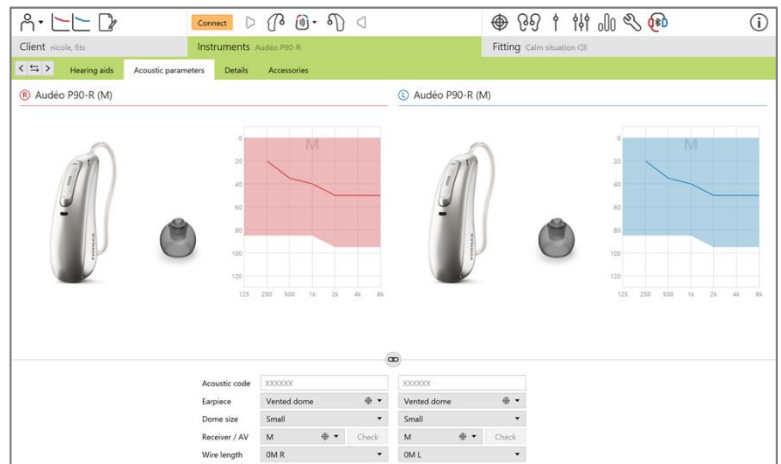


Global Tuning

Verify desired fitting formula and gain level have been selected.

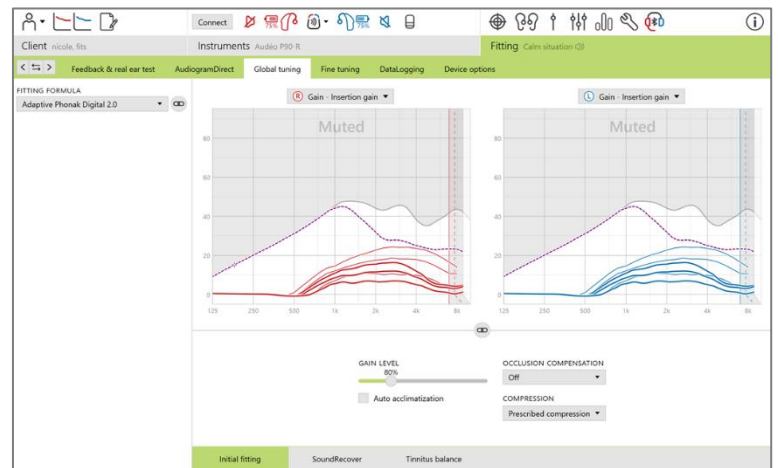


Device Options

Configure the hearing aid options such as the multifunction button behavior and Tap Control.

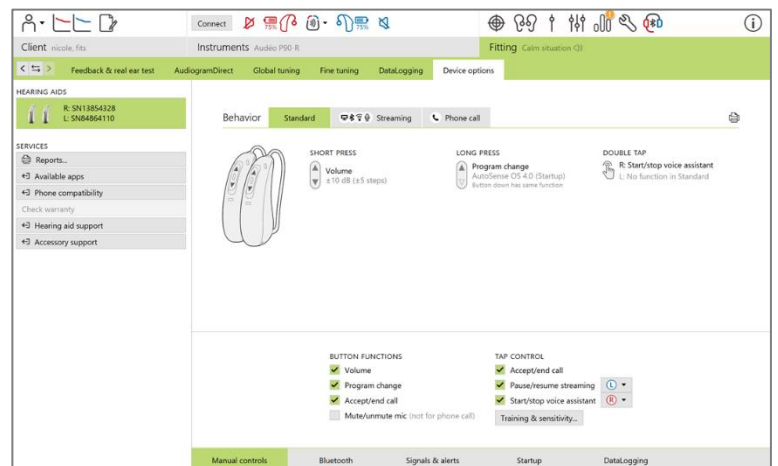

Bluetooth Options

Configure the **[Bluetooth side]** and **[Bluetooth name]** under Device options to select the phone ear and hearing aid name for easy identification when pairing with a new device.

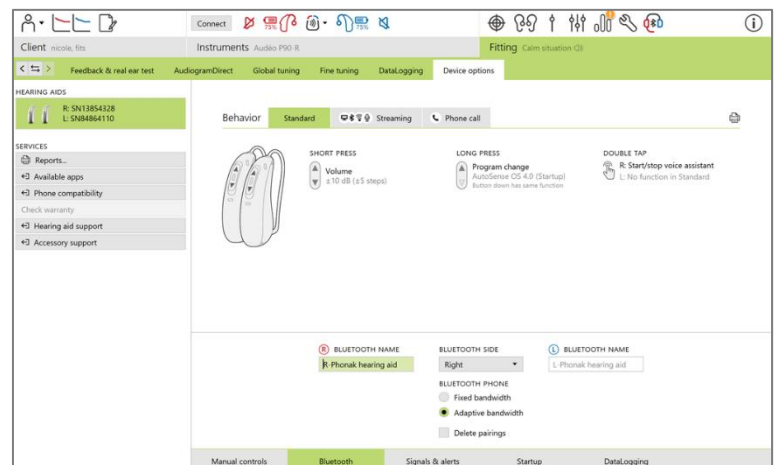

Finish the fitting session

Click **[Save & close session]** in the top right corner of the screen. Select the items to save.

After saving, Target will guide you to the start screen.

