



Quick Tips

AUDIOLOGYonline



Signia QuickTips: Signia's CROS/BiCROS Options

Amanda Reed, Au.D., FAAA




Signia QuickTips: Signia's CROS/BiCROS Options

- Thank you for joining us to learn more about Signia's CROS/BiCROS Options.
- For assistance with this course, contact AudiologyOnline (1-800-753-2160)
- Please visit the AudiologyOnline website for other live and recorded events from Signia.

<http://www.audiologyonline.com/ce/signia>


Presenter Name

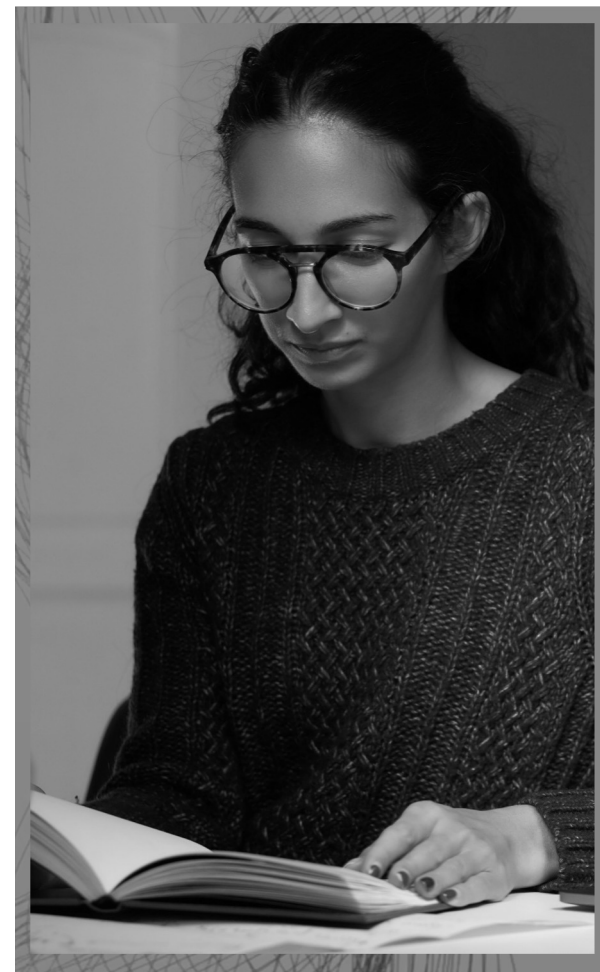

audiologyonline | 

CROS vs. BICROS


CROS

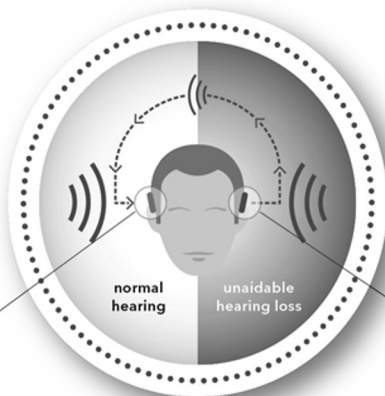

Hearing instrument receives the sound from the unaidable side and presents it to the better ear so the signal can be heard.

CROS Pure
wirelessly transmits sounds.

BiCROS

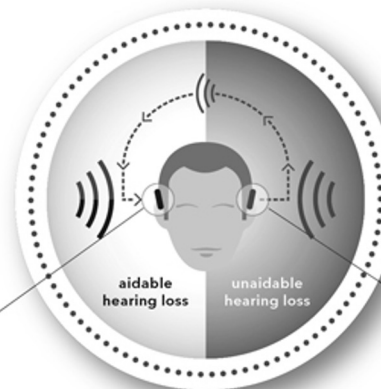

Hearing instrument receives the sound from the unaidable side, mixes it with its own input and **amplifies** the mixed signal.

CROS Pure
wirelessly transmits sounds.

CROS Pure Charge&Go IX

CROS Pure Charge&Go IX

- Compatible with all performance levels 7/5/3

CROS battery runtime

- Up to 29 hours without streaming



Styletto IX – Performance levels and CROS



CROS Pure 312 AX



-  Increased reliability
-  Better audio quality
-  60% lower latency

Insio Charge&Go AX ITC/ITE

- Full AX Feature set including Augmented Focus
- Built-in Li Ion cell for contactless charging
- e2e Wireless 4.0
- Bluetooth connectivity for streaming audio
 - Made for iPhone and Android ASHA
- Signia Assistant compatible
- IP 68 rated

NEW! CROS compatible for all AX performance levels (7/5/3/2/1)
with **Pure 312 AX CROS** or **Pure C&G AX CROS**

7AX

5AX

3AX

2AX

1AX



audioLOGYonline




CROS Pure Charge&Go X & CROS Pure 312 X Compatible with Motion X Family



Pairing Motion Charge&Go P X and SP X
with CROS Pure Charge&Go X results in
using two different chargers


CROS IX Highlights

Directional pattern skewed toward the unaidable ear for improved speech understanding in noise

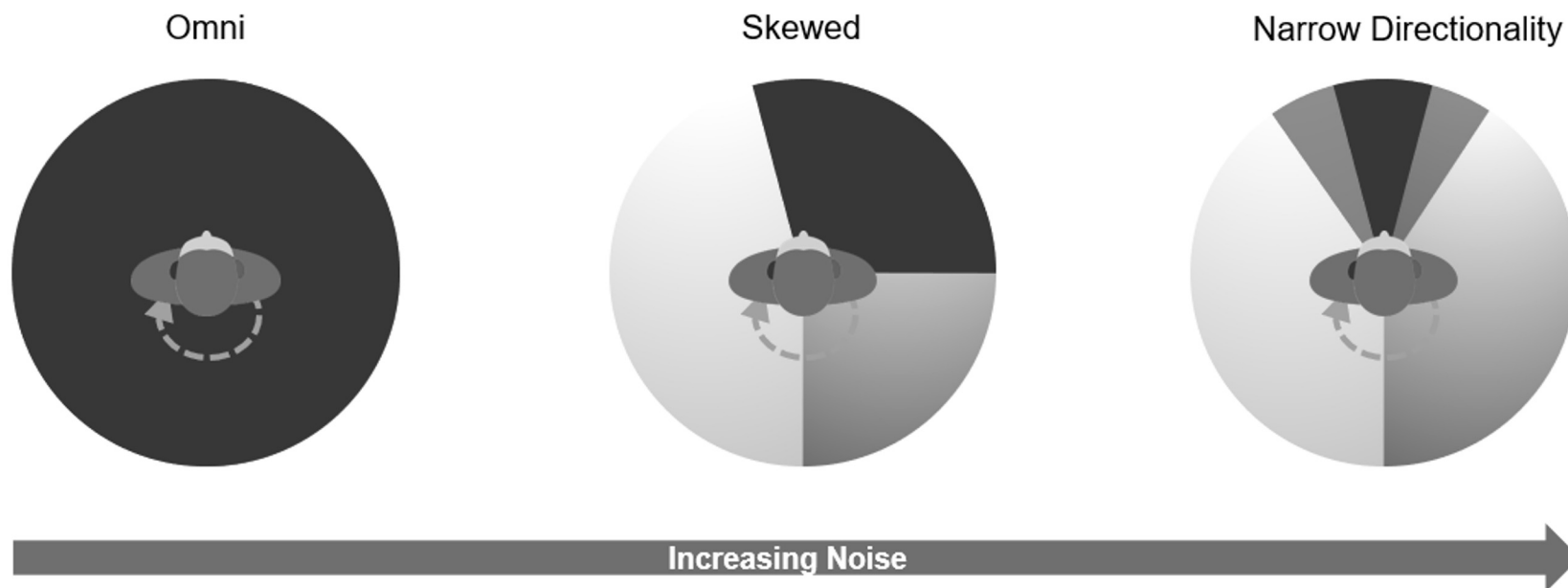


Compatible with Signia App

- All CROS are compatible with Signia app.

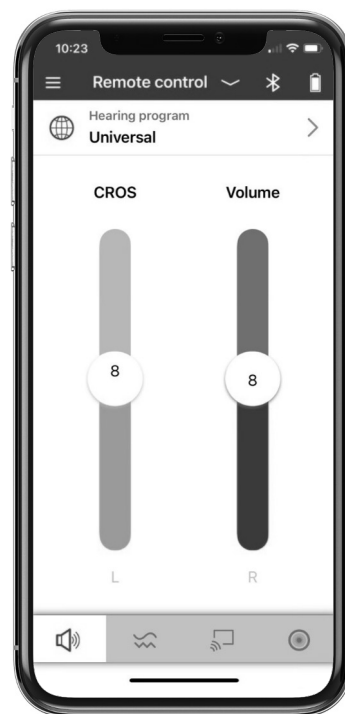

Directional Hearing Functionality with CROS

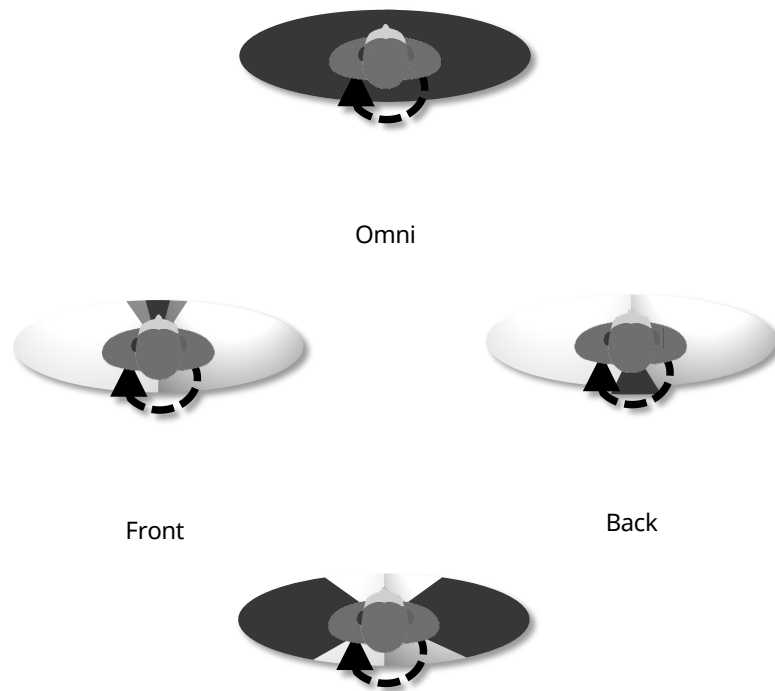


Omni

Front

Back

Shows as Left or Right = Omni

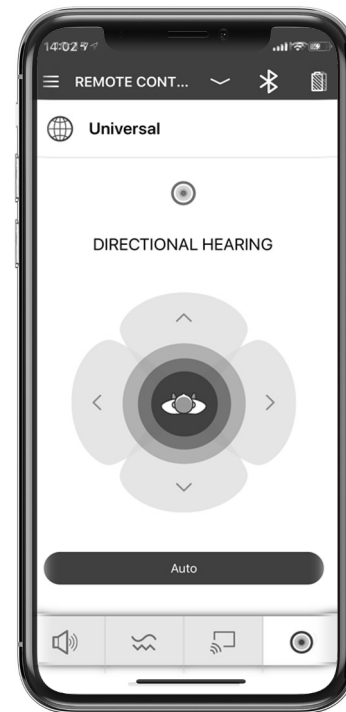

audiologyonline |




Additional CROS Features

- Li-ion rechargeability or battery-operated options
- HandsFree with CallControl (on the receiver side) for iOS (IX and AX technology)
- CROS mixing-level adjustment to balance spatial benefit vs. noise level via Signia App and/or on rocker – BiCROS
- Compatibility with the Signia App
 - My WellBeing
 - TeleHealth via portal

CROS Considerations

- The CROS transmitter microphone is muted:
 - When streaming
 - TV program
 - Phone acoustically
 - Tcoil is active (on receiver hearing aid)
 - Pure tinnitus noise program
 - Apple's Live Listen which is available in the iOS native app
- Streaming requires the CROS transmitter to be powered on and in range of the receiver.
 - The transmitter can be removed once streaming is initiated. If the stream is interrupted the transmitter again must be on and in range to re-initiate the streaming.

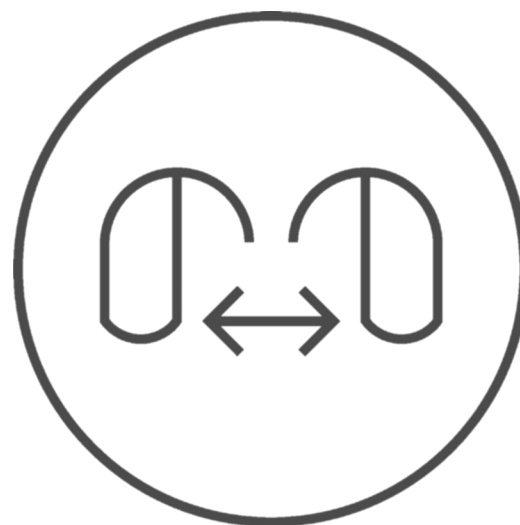

CROS Silk Charge&Go IX



- **CROS Silk Charge&Go IX**
 - Compatible with all performance levels 7/5/3
- **CROS battery runtime**
 - Up to 28 hours



Insio IX

7IX

5IX

3IX

CROS



Insio CIC IX



Insio IIC IX

Insio IX IIC / CIC – CROS Compatibility



Compatible with CROS Silk Charge&Go IX

Must have e2e wireless 4.0

Non-wireless devices are not compatible with CROS



CROS Silk Charge&Go IX battery runtime:

Up to 28 hours

Note: This scenario would result in the Insio IX IIC/CIC using a size 10 zinc-air battery and the CROS Silk C&G IX would use the Silk Charger.



audioLOGYonline |





signia

Thank You!

SigniaUSA
800-766-4500

aUDIOLOGYonline