

Manufacturer according to FDA:

ReSound Government Services

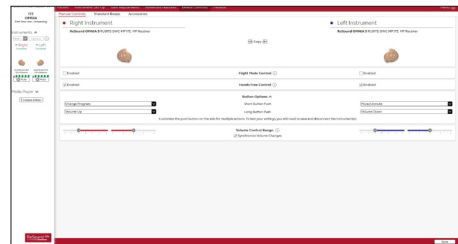
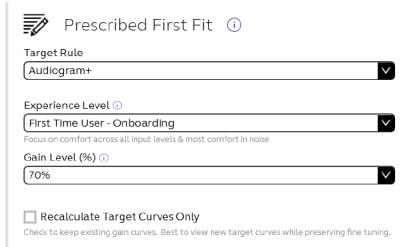
8001 E Bloomington Freeway
Bloomington, MN 55420
USA
1-800-392-9932
gs.resound.com

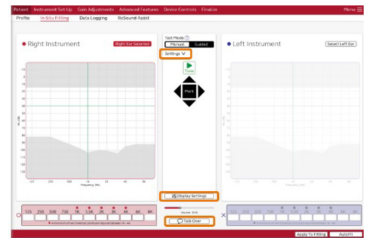
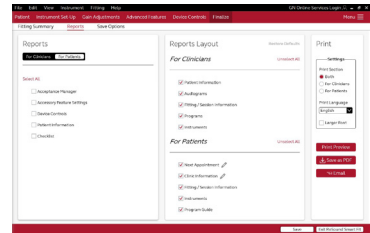
ReSound Smart Fit 1.18

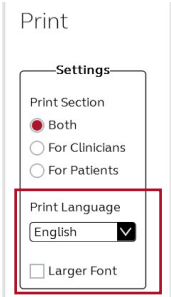
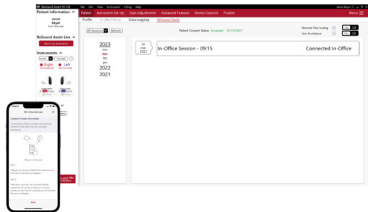
We continuously strive to improve the fitting experience for you and your patients. Based on your feedback, ReSound Smart Fit™ now offers more accurate fittings and a faster workflow.

What's new?

- Support for all ReSound Nexia styles;
 - Rechargeable Micro RIE 60S
 - 312 RIE 61
 - 13 RIE 62
 - Rechargeable Customs ITC/ITE
 - Non-wireless CIC
 - Rechargeable Standard BTE 77
 - Rechargeable Power BTE 88
- Hands-free enable/disable feature
- 'Hear In Noise' program
- User Profile, 'First Time User - Onboarding'
- Improved AutoREM experience
- Improved In-Situ Fitting experience
- New onboarding report
- Consent in App only

What's new	Feature	Benefit	Available For	Image
Hands-free enable/disable feature	Enable or disable hands-free feature under Device Controls	Provides option for patients who work in secure environments where use of devices that have 2-way communication is prohibited	ReSound Nexia & ReSound OMNIA	
'Hear in Noise' Program	Hear in Noise program describes the general program function replacing Front Focus, Ultra Focus, Restaurant program names regardless of Directional settings selected in Advanced Features.	Consistent and simple offering of listening programs, with one superior program specifically intended for hearing in noise.	All ReSound hearing aids in ReSound Smart Fit	
'First Time User - Onboarding' Patient Profile	User profile named 'First Time User - Onboarding' replaces the existing 'Comfort User' profile.	Give your patients a smooth start to hearing – and an alternative if the 'First Time User' profile is perceived as too aggressive by your patient. Includes UI updates to better guide you to which profile might suit your patient best for a great first fit.	All ReSound hearing aids in ReSound Smart Fit	

What's new	Feature	Benefit	Available For	Image
Improved AutoREM	New fine-tuning options to support existing functionality: • Match Target option: Finetune gain to target for 3 inputs • Quick Run option: Existing option to run the soft and loud input for recording and observation purposes • Mix and Match: Choose to Target Match or Quick Run for soft and loud inputs based on your needs and patient tolerance	Easily find AutoREM and the finetuning options with fewer clicks. Allows you more control over how you want to run AutoREM – by either using existing functionality (Quick Run), get more accurate low and high intensity results (Match Target) or even mix and match what type of curves to run.	All ReSound hearing aids in ReSound Smart Fit	
Improved In-Situ Fitting	Improved process flow with fewer clicks to get through testing	More seamless and easy navigation with no compromise on functionality.	All ReSound hearing aids in ReSound Smart Fit	
Onboarding reports - Modernized the report offering for clinicians and patients	Easier access to the Patient Onboarding Reports and a single place for customizations	Reduce navigation time, enabling more time for patient care	All ReSound hearing aids in ReSound Smart Fit	

What's new	Feature	Benefit	Available For	Image
Onboarding report - Larger font size and different language options	Easier access to view with a larger font or select preferred language when printing report	Improve ease of reading with larger font sizes and ability to change to patient's preferred language.	All ReSound hearing aids in ReSound Smart Fit	
Onboarding report - Downloading and emailing report options	Download reports in PDF files for your record or email those reports to your patients	Ability to share digitally to save on printing and save for fitting records	All ReSound hearing aids in ReSound Smart Fit	
Consent in App only – Removing Patient Consent menu from Fitting Software	Moving all consent management from Fitting Software to the ReSound Smart 3D app	Simplify workflow/fitting flow and alleviate responsibility of storing and documenting consent.	All ReSound hearing aids in ReSound Smart Fit	

If you have any questions about ReSound Smart Fit 1.18 and the new updates, please contact your ReSound sales representative.