

1. This document was created to support maximum accessibility for all learners. If you would like to print a hard copy of this document, please follow the general instructions below to print multiple slides on a single page or in black and white.
2. This handout is for reference only. Non-essential images have been removed for your convenience. Any links included in the handout are current at the time of the live webinar, but are subject to change and may not be current at a later date.
3. Copyright: Images used in this course are used in compliance with copyright laws and where required, permission has been secured to use the images in this course. All use of these images outside of this course may be in violation of copyright laws and is strictly prohibited.
4. Social Workers: For additional information regarding standards and indicators for cultural competence, please review the NASW resource: [Standards and Indicators for Cultural Competence in Social Work Practice](#)
5. Need Help? Select the “Help” option in the member dashboard to access FAQs or contact us.

How to print Handouts

On a Mac

- Open PDF in Preview
- Click File
- Click Print
- Click dropdown menu on the right “preview”
- Click layout
- Choose # of pages per sheet from dropdown menu
- Checkmark Black & White if wanted.

On a PC

- Open PDF
- Click Print
- Choose # of pages per sheet from dropdown menu
- Choose Black and White from “Color” dropdown

No part of the materials available through the continued.com site may be copied, photocopied, reproduced, translated or reduced to any electronic medium or machine-readable form, in whole or in part, without prior written consent of continued.com, LLC. Any other reproduction in any form without such written permission is prohibited. All materials contained on this site are protected by United States copyright law and may not be reproduced, distributed, transmitted, displayed, published or broadcast without the prior written permission of continued.com, LLC. Users must not access or use for any commercial purposes any part of the site or any services or materials available through the site.

A solid red square is located in the bottom left corner of the page.

Spring 2024: Hearing Care Solutions Launches New Platform and Form Factors

Jennifer Gehlen, AuD
Sr. Program Manager, WSA

Webinar FAQs

Where to get the handout:

Download from Pending Courses

How to earn CEUs:

After the webinar, go to your Pending Courses and pass the multiple-choice exam.

For assistance or tech support:

Call 800-753-2160

customerservice@audiologyonline.com

Learning Objectives

1. After this course, participants will be able to describe how the new HCS 13 platform provides more speech support in multi-talker situations.
2. After this course, participants will be able to list the form factors on the new HCS 13 platform.
3. After this course, participants will be able to describe the features of the new HCS 13 platform and their benefits to wearers.





Spring 2024: Hearing Care Solutions Launches
New Platform and Form Factors

Introducing: New HCS Private Label Hearing Instruments

NEW! HCS 713 RIC LI
HCS 513 RIC LI



NEW! HCS 713 RIC LI T
HCS 513 RIC LI T



Additional Focus for Multiple Speakers in Conversation



The foundation of the platform: 2 processors

Two Separate processors:

One for the focus – speech

- To enhance speech signals from the front hemisphere



One for the surrounding sounds

- Processes environmental sounds to preserve environmental awareness without masking speech

HCS 13 RIC LI and HCS 13 RIC LI T



HCS 13 RIC LI

- 28h without streaming
- 24h including 5h streaming

- Tech levels 713 & 513
- Rechargeable
- Bluetooth connectivity
- Rocker switch control
- CROS-compatible*

*Compatible with Signia CROS Pure C&G IX



HCS 13 RIC LI T

- Telecoil* 
- 39h without streaming
- 36h including 5h streaming

Colors



Sandy Brown



Beige



Black



Dark Brown



Granite

EarWear 3.0



EarWear 3.0

miniReceivers, Tips & Sleeves

miniReceiver 3.0 4 power levels

S



M



P



HP*



S (110/46)
M (119/60)
P (122/65)
HP (131/75)

miniReceiver 3.0 6 lengths



0



1



2



3



4



5



Open EarTips 3.0



Round One EarTips 3.0



Tulip EarTips 3.0



Sleeves 3.0

Measure to Top of Tragus

How to select correct miniReceiver 3.0 length.mp4



Length 0 – Good Choice!

How to select correct miniReceiver 3.0 length.




Length 1 – May be too long

▶ How to select correct miniReceiver 3.0 length.



EarWear 3.0

Eartip and Sleeve Installation

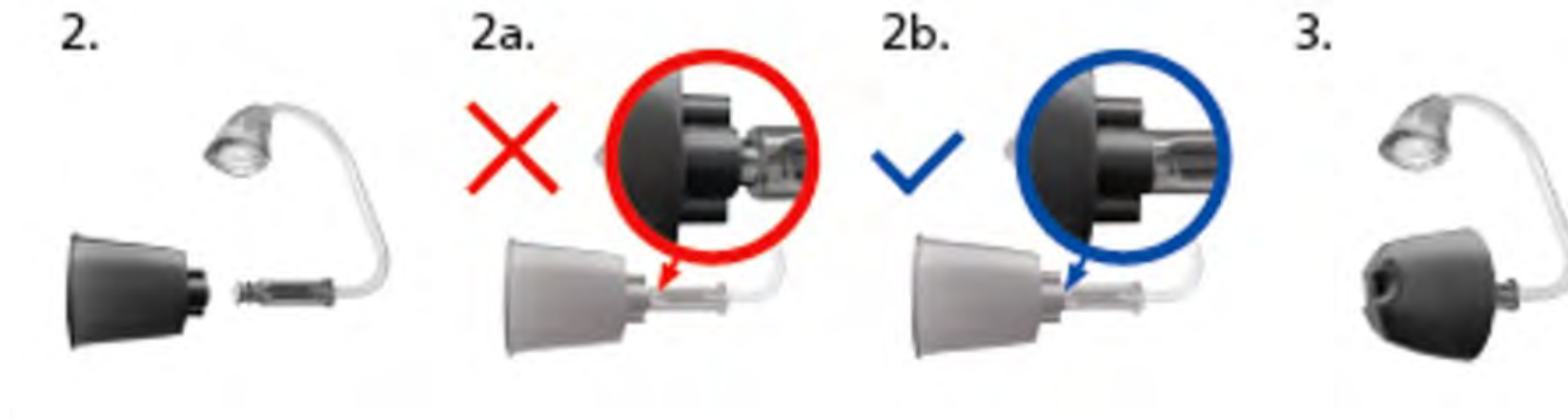

EarWear 3.0 Additional Parts

Item	Description
Measurement Gauge 3.0 	Ensure perfect match for new miniReceiver 6 lengths (0-5) and Thin Tube 3.0 in 5 lengths (0-4)
Wax Guard 3.0 NanoCare + CeruSTOP 3.0 XL 	Matchbook design containing 8 wax guards
Concha Locks 3.0 	Now sizes S/M/P miniReceiver Concha locks 10 pieces per bag
Removal Tool 3.0 for Custom earmold 	Tool to remove miniReceiver/Thin Tube from Earmold 3.0 Short/Long
Handling Tool 3.0 for custom earmold 	Tool to press out the miniReceiver from Earmold 3.0 Short

Accessories



- StreamLine Mic™
- StreamLine TV™
 - Compatible with Bluetooth®-enabled aids
- miniPocket™
 - Also compatible with non-Bluetooth®

Additional HCS Private Label Hearing Instrument Portfolio

700 technology levels

HCS 712 RIC LI
HCS 712 RIC LI T
HCS 12 CROS RIC LI

HCS 712 RIC 312
HCS 712 CROS RIC

HCS 711 IF CIC

HCS 711SR LI

HCS 711BTE LI/ P/ SP

500 technology levels

HCS 512 RIC LI
HCS 512 RIC LI T
HCS 12 CROS RIC LI

HCS 512 RIC 312
HCS 512 CROS RIC

HCS 511 SR LI

HCS 511 BTE LI/ P/ SP

Let's Do a quick First Fit in Connexx Fitting Software

