



Positioning our Custom Portfolio

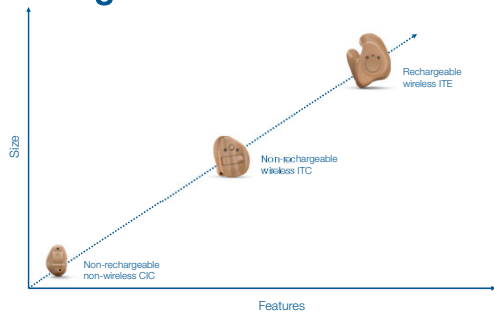

The Size Balance

Not included

- Rechargeability
- Most ear-to-ear communication
- Hands-free calls

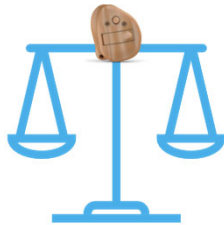

Included


- Hybrid C6
- Redesigned antenna layout

Smaller Size with Connectivity

Optimizing components



Antenna placement

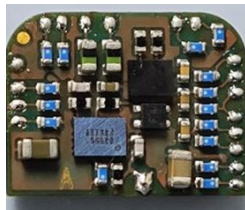



Beltone Serene Wireless ITC

- Technology levels: 17, 9 and 6
- Available in 3 shell styles with 312 battery



ITC Half shell Full shell

- Four receiver powers
- Offered in 6 colors
- IP 68 rated
- Push button synchronization





Manual Controls

Manual Controls Standard Beeps Accessories

Right Instrument

Beltone Serene 17 SERBTIC-DW LP ITC LP Receiver



☐ Enabled ☐ Enabled

Flight Mode Control  Bluetooth LE Audio 

Button Options 

Volume Up  Short Button Push

Change Program  Long Button Push

Customize the push button on the aids for multiple actions. To test your settings, you will need to save and disconnect the instrument.

Volume Control Range:  

☒ Synchronize Volume Changes

Left Instrument

Beltone Serene 17 SERBTIC-DW LP ITC LP Receiver



☐ Enabled ☐ Enabled

Volume Down 

Volume Up 

Volume Down 

Change Program 

Activation of Wireless Streaming

Minimum Volume

Mute/Unmute



Feature Set

- Smart Gain Pro
- Sound Cleaner Pro
- Feedback Eraser 2
- Impulse Noise Reduction
- Silencer
- Wind Noise Reduction
- Mixing Point Frequency
- Tinnitus Breaker Pro
- Sound Shifter
- Time Constants
- Speech Spotter Pro
- Smart Beam
- Remote Care


Plus:

- Bluetooth LE Audio and Auracast
- MFi and Audio Streaming for Hearing Aids (Android)
- Connectivity to our accessories
- Manual control synchronization




Speech Spotter Pro

- Moves from omni to directional
- Has adjustable beamwidth

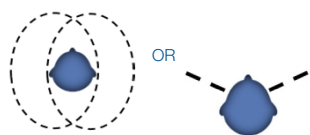


Belmonte
UNIVERSITY

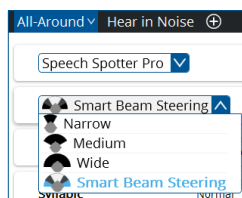


Smart Beam


- Always directional
- Has adjustable beamwidth

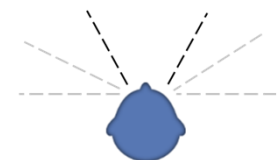

Belmonte
UNIVERSITY

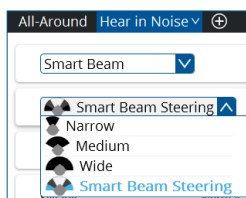

Venting for Customs

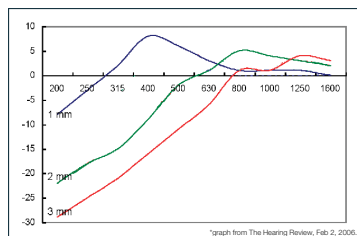

Avg Thresholds 500 - 4000 Hz	Recommended vent diameter
< 20 dB	Max Open Vent (MOV)
20-30 dB	3.5-3.1 mm
30-40 dB	3.1-2.5 mm
40-50 dB	2.5-1.8 mm
50+ dB	1.8-1.2 mm

Custom Shell: No vent

Custom Shell: 0.6-1.2 mm vent

Custom shell: 1.3 - 2.1 mm vent

Custom shell: 2.2 - 3.0 mm vent

Custom shell: 3.1 - 4.5 mm vent

Belmonte
UNIVERSITY

Beltone's New Accessories



Multi-Mic+



TV-Streamer+

Beltone
UNIVERSITY

Patient Population

- Beltone Amaze ITC users
- Patients that want batteries and custom
- Patients that want custom and directional benefits in noise
- Patients that want custom and connectivity

Beltone
UNIVERSITY



Beltone Serene Custom Portfolio

Smallest & near-invisible



Completely-in-Canal (CIC)
Non-Wireless
10A

Small & connected



In-The-Canal (ITC)
Wireless
312

Full-features & rechargeable



ITC / In-the-Ear (ITE)
Wireless
Rechargeable

Beltone
UNIVERSITY

Custom Characteristics

	OC	ITC	ITC / BE
Sound	Directionality	Natural Prime Effect	Speech Spotter Pro
	Speech Intelligibility	Omnidirectional mic	Smart Beam
	Discreteness	Nearly invisible	Visible, mid size
Design	Colors		
	LED		✓
	Push Button Call pick up (iOS/LE Audio)	✓	✓
Connectivity	Battery	25A	Rechargeable
	Bluetooth LE Audio/Auracast	✓	✓
	MFi/Android streaming (ASHA)	✓	✓
	Hands-free calls (iOS/LE Audio)		✓

Learning Outcomes

①

The main feature set of our new Beltone Serene wireless ITC hearing aid, is essentially the same as our other Beltone Serene hearing aids.

②

Patient benefits includes wireless streaming, Auracast and improved battery life.

③

This product is for patients that was a more discreet hearing aid, but with the latest connectivity options.
