

1. This document was created to support maximum accessibility for all learners. If you would like to print a hard copy of this document, please follow the general instructions below to print multiple slides on a single page or in black and white.
2. This handout is for reference only. Non-essential images have been removed for your convenience. Any links included in the handout are current at the time of the live webinar, but are subject to change and may not be current at a later date.
3. Copyright: Images used in this course are used in compliance with copyright laws and where required, permission has been secured to use the images in this course. All use of these images outside of this course may be in violation of copyright laws and is strictly prohibited.
4. Social Workers: For additional information regarding standards and indicators for cultural competence, please review the NASW resource: [Standards and Indicators for Cultural Competence in Social Work Practice](#)
5. Need Help? Select the “Help” option in the member dashboard to access FAQs or contact us.

How to print Handouts

On a Mac

- Open PDF in Preview
- Click File
- Click Print
- Click dropdown menu on the right “preview”
- Click layout
- Choose # of pages per sheet from dropdown menu
- Checkmark Black & White if wanted.

On a PC

- Open PDF
- Click Print
- Choose # of pages per sheet from dropdown menu
- Choose Black and White from “Color” dropdown

No part of the materials available through the continued.com site may be copied, photocopied, reproduced, translated or reduced to any electronic medium or machine-readable form, in whole or in part, without prior written consent of continued.com, LLC. Any other reproduction in any form without such written permission is prohibited. All materials contained on this site are protected by United States copyright law and may not be reproduced, distributed, transmitted, displayed, published or broadcast without the prior written permission of continued.com, LLC. Users must not access or use for any commercial purposes any part of the site or any services or materials available through the site.

A solid red square is located in the bottom left corner of the page.

CONNEXX Made Simple: A Tour of the Signia Fitting Software

Fall 2024



Be
Brilliant™

CONNEXX Made Simple: A Tour of the Signia Fitting Software



audiologyonline

Webinar FAQs

- **How to earn CEUs:**
 - After the course, go to your Pending Courses on AudiologyOnline and pass the multiple-choice exam
 - Must have a paid membership to earn CEUs
- **For assistance or tech support:**
 - Call 800-753-2160
 - customerexperience@continued.com



Learner Objectives


Upon completion of this course attendees will be able to:

- 1) Describe and perform the simple steps to complete a First Fit with a Signia hearing aid.
- 2) List the steps and benefits of running Own Voice Processing 2.0.
- 3) Discuss how the flexibility of the Dynamic Soundscape Processing 3.0 can enrich the Real Time Conversation Enhancement adjustments.



Signia IX with RealTime Conversation Enhancement

Revolutionary three stage process

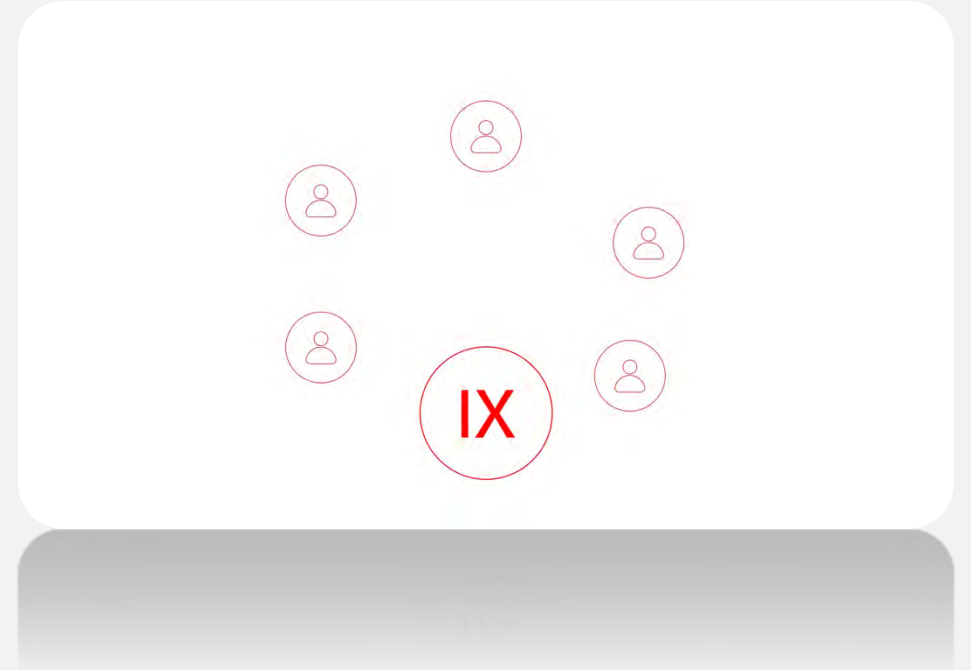

A live auditory space

Immersed in surrounding sound

Smoothly enhancing multiple speakers



Expanding the Signia Portfolio



Pure
Charge&Go IX



Pure
Charge&Go T IX



CROS Pure
Charge&Go IX



Pure
312 AX



CROS Pure
312 AX



Styletto
IX



CROS Styletto
IX



Intuis 4



Silk
Charge&Go
IX



CROS Silk
Charge&Go
IX



Insio IIC/CIC
IX



Insio
Charge&Go
AX



Motion
Charge&Go
SP X



Motion
Charge&Go
P X




Motion
Charge&Go
X




Active
X

Connexx 9.12

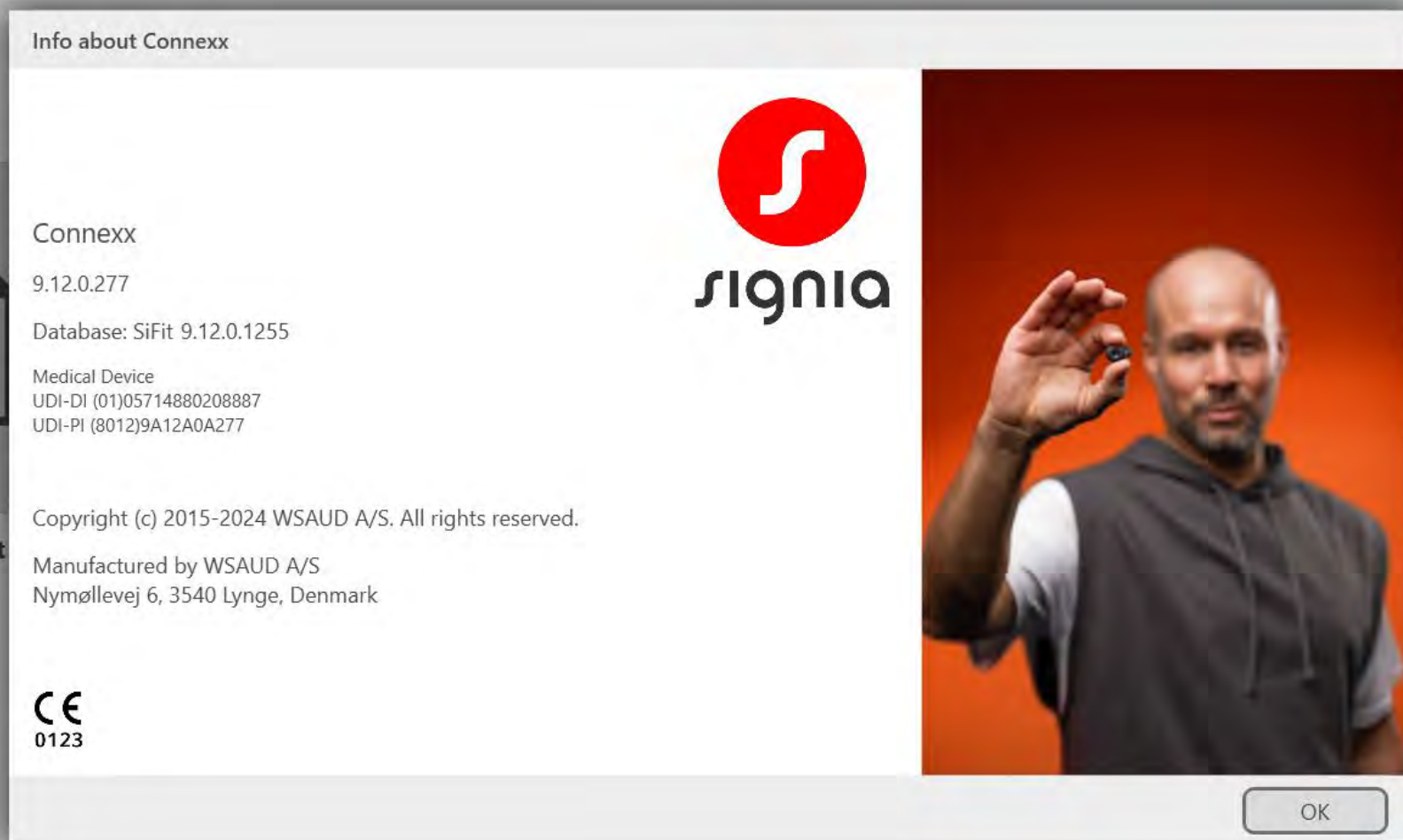

Connexx 9.12

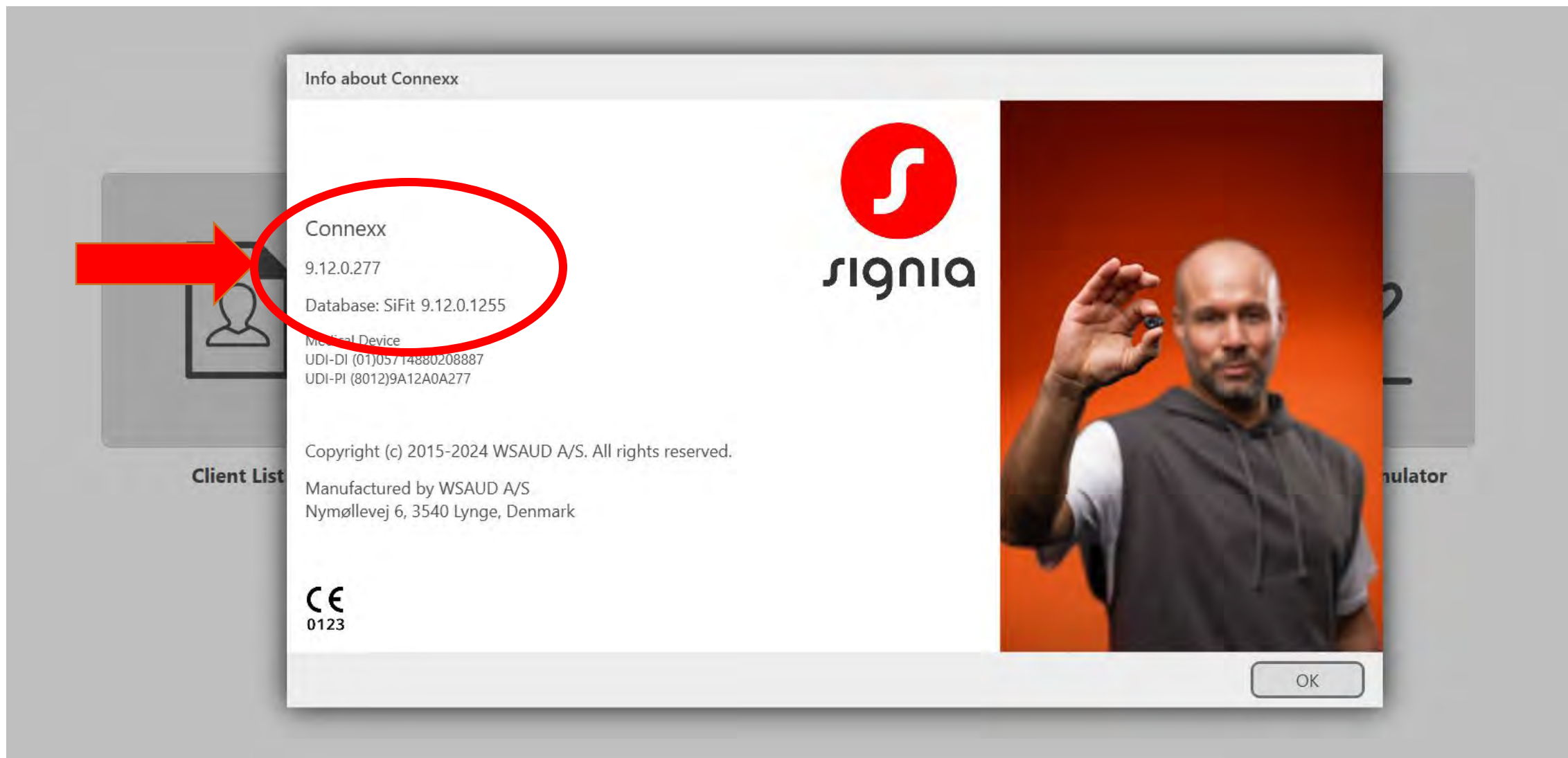

audiologyonline

Thank You

SigniaUSA

800-766-4500



signia

**Be
Brilliant™**