

Genie 2 Fitting Focus Series:

Oticon Intent First Fit

Karen Toole, Au.D.
National Education & Training Specialist

oticon
life-changing technology

Learner Outcomes:

After this course, participants will be able to:

- . . . explain the process of pairing and connecting Oticon Intent instruments using Genie 2 software.
- . . . list the required initial fitting steps for Oticon Intent.
- . . . find the MoreSound Intelligence and Sudden Sound Stabilizer tools in the Genie 2 fitting screen and describe the controls available on these screens.

Fitting Oticon Intent

Programming devices



Noahlink Wireless/Noahlink Wireless 2

are the only programming devices for
programming and firmware updates

First steps

Attach speaker units to hearing instruments


"Wake up" hearing aids
to deactivate battery protection mode



Wireless programming
with Noahlink Wireless



Discreet and packed with **rechargeable power**

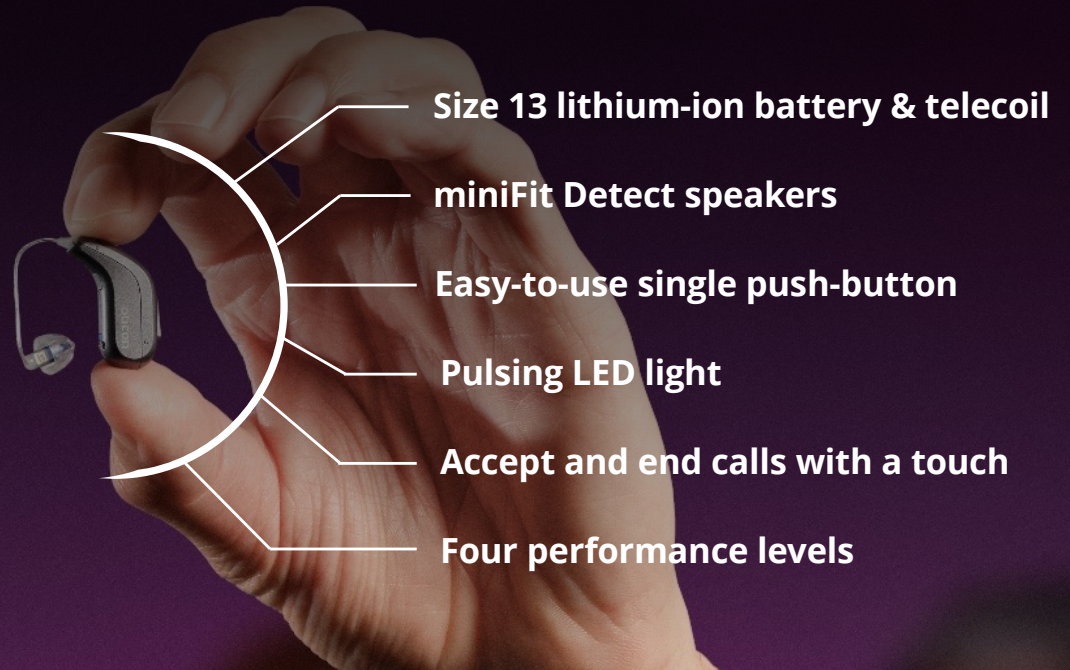

C044
Silver



C091
Silver
Grey



C092
Steel
Grey



C063
Diamond
Black



C093
Chestnut
Brown



C094
Terracotta



C090
Chroma
Beige



C116
Honey
Beige

NEW



C115
Sky
Blue

NEW

Recommending an Oticon charger

A user-centered choice

SmartCharger miniRITE



Desktop Charger



Genie 2 Fitting Focus Series:

Oticon Intent First Fit

Karen Toole, Au.D.
National Education & Training Specialist

oticon
life-changing technology