

Charging options for ReSound hearing aids

ReSound GN

How to match ReSound RIE and BTE hearing aids with the correct charger

Each charger type is offered with different inserts – known as the **charging form factor (CFF)** – which relate to a specific ReSound hearing aid collection or style.

The CFF is indicated with a number (see chart below) and is shown inside the charger bay and on the charger packaging.

ReSound offers rechargeable solutions with inductive charging and long battery life to keep your patients on the go.



	Premium charger	Desktop charger	Custom charger*
ReSound Nexia™ Micro RIE 60S	8 22978800	8 23366400	
ReSound OMNIA™ Mini RIE 60	06 22405700	06 23366300	-
ReSound OMNIA™ and ReSound ONE™ RIE 61	1 21569800	1 23366200	-
ReSound OMNIA™ and ReSound ONE™ BTE	-	70 22816900	-
ReSound Nexia™, ReSound OMNIA™ and ReSound ONE™ Power/High Power BTE 88	-	80 22817000	-
ReSound Nexia™ Customs, ReSound OMNIA™ Customs and Customs by ReSound	-	-	C 22465400

* Custom charger includes custom-made inserts. **We recommend** reminding patients to bring their custom-made charger along with their custom hearing aids to all follow-up appointments.

Charger features



	ReSound Nexia			ReSound OMNIA	
	Premium charger	Desktop charger	Custom charger	Premium charger	Desktop charger
	Micro RIE	Micro RIE BTE/Power BTE	Customs	Mini RIE/RIE	Mini RIE/RIE
Inductive charging	•	•	•	•	•
Portable with onboard power bank	•			•	
Portable without power bank			•		
USB-C connector and cable	•	•	•	•	•
LED on the back	Shows the power bank battery status	Shows when the charger is connected to a power supply	Shows when the charger is connected to a power supply	Shows the power bank battery status	Shows when the charger is connected to a power supply
LED on the front	Shows battery level of the hearing aids*		Shows the hearing aids charging	Shows battery level of the hearing aids*	

*LED lights are visible when lid is open

How to tell if the hearing aids are charging

Place the hearing aids in the charger and let the light indicator on the hearing aids guide you. For rechargeable Customs, the LED on the front of the charger will be your guidance.

You know that charging has started when the hearing aid lights blink slowly.

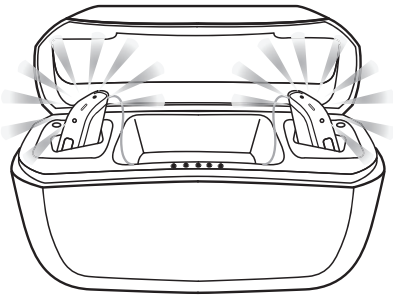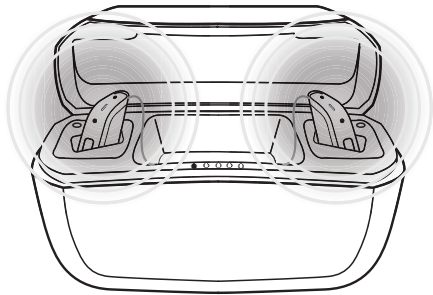

When fully charged, the hearing aid lights remain constant, not blinking.

The lights turn off when the hearing aids are removed from the charger.

Premium charger illustrated. The process is the same for the Desktop charger. For more information, please see the individual user guide for each charger.

Manufacturer according to FDA:

ReSound Government Services
8001 E Bloomington Freeway
Bloomington, MN 55420
USA
1-800-392-9932
gs.resound.com