



Signia Quick Tips

Introducing Intuis 4 BTE:

Samantha Powers, AuD




Signia Intuis 4 Behind the Ear (BTE) Hearing Aids

- Thank you for joining us to learn more about Signia Intuis 4 BTE.
- For assistance with this course, contact AudiologyOnline (1-800-753-2160)
- Please visit the AudiologyOnline website for other live and recorded events from Signia.

<http://www.audiologyonline.com/ce/signia>

Samantha Powers, AuD

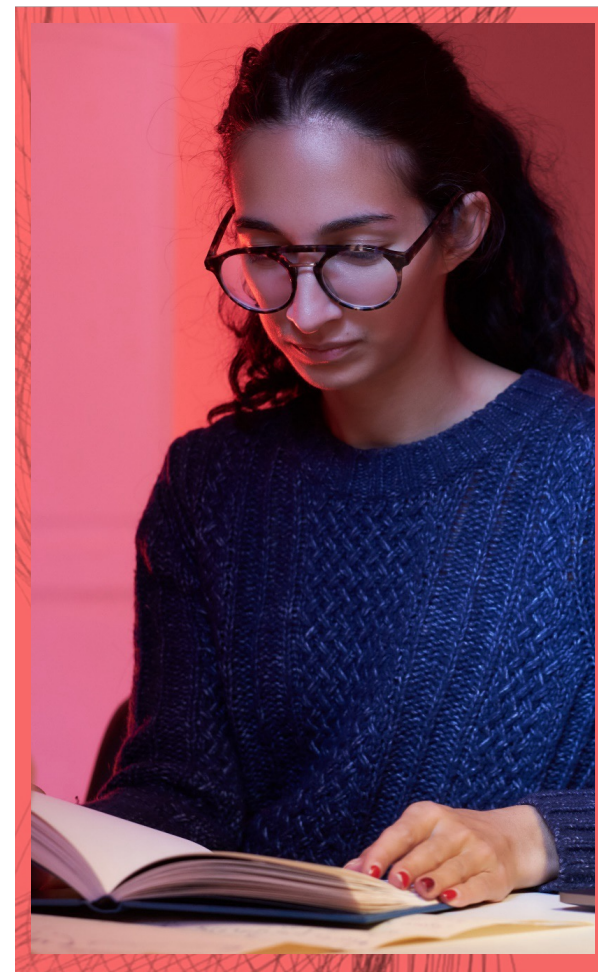

Intuis 4 BTEs



Be
Brilliant™

Advanced technology for more of your customers

Intuis 4 BTEs feature advanced technology that delivers brilliant sound and speech clarity, plus the convenience of enhanced connectivity and Bluetooth streaming.

Suitable for all levels of hearing loss, the ergonomic design and easy-to-use controls of Intuis 4 make daily operation straightforward and comfortable.

The Signia App provides a suite of controls and communication options.



Equipped with Bluetooth



Signia app ready



4.7

4.5

4.3

4.2


4.1

sDemo

audiologyonline |



Built upon split processing platform

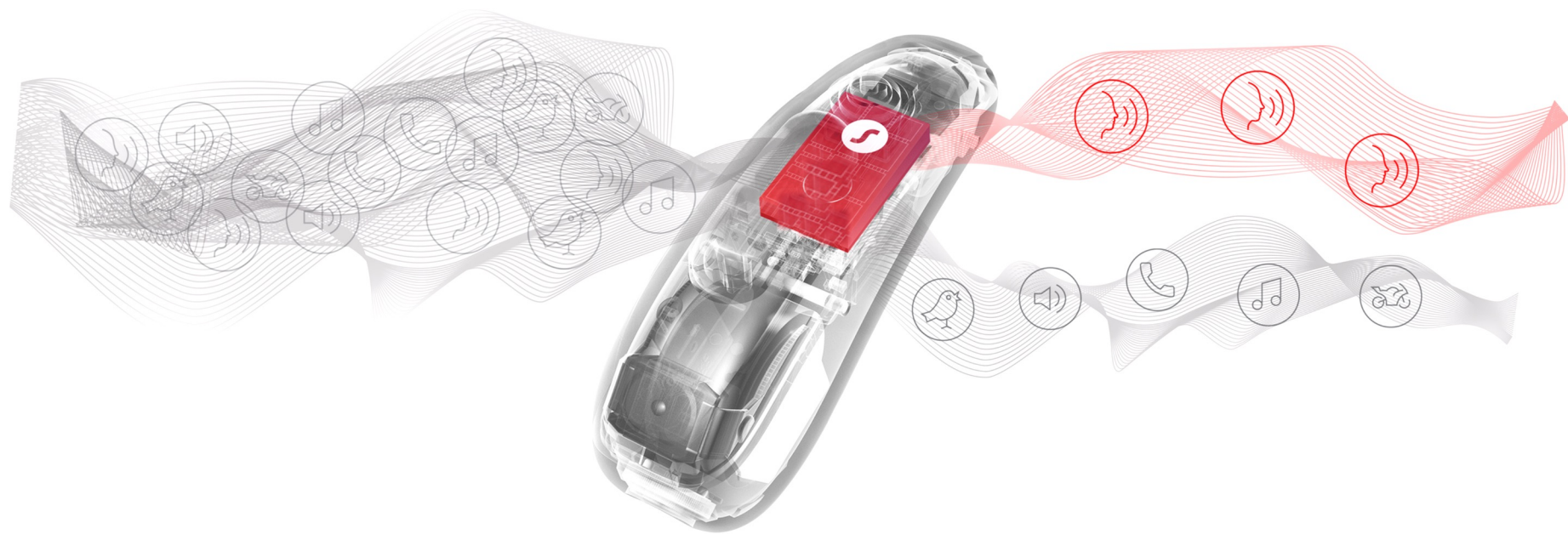

No compromise in how speech and ambient sound are processed

Intuis M 4

Intuis M 4.7 / 4.5 / 4.3 / 4.2 / 4.1 / sDemo

Battery size 13

Battery runtime (without streaming) = up to 142 h
Battery runtime (incl. 20 h streaming) = up to 129 h



Intuis P 4

Intuis P 4.7 / 4.5 / 4.3 / 4.2 / 4.1 /
sDemo

Battery size 13

Battery runtime (without streaming) = up to 148 h
Battery runtime (incl. 20 h streaming) = up to 115 h



Intuis SP 4

Intuis SP 4.7 / 4.5 / 4.3 / 4.2 / 4.1 / sDemo

Battery size 675

Battery runtime (without streaming) = up to 277 h
Battery runtime (incl. 20 h streaming) = up to 217 h




EarWear 3.0 compatibility

Intuis M 4 and Intuis P 4 are compatible with:

ThinTube 3.0

ThinTube 3.0 P

Both can be used with **Eartips 3.0, Sleeves 3.0, & Earmold 3.0**

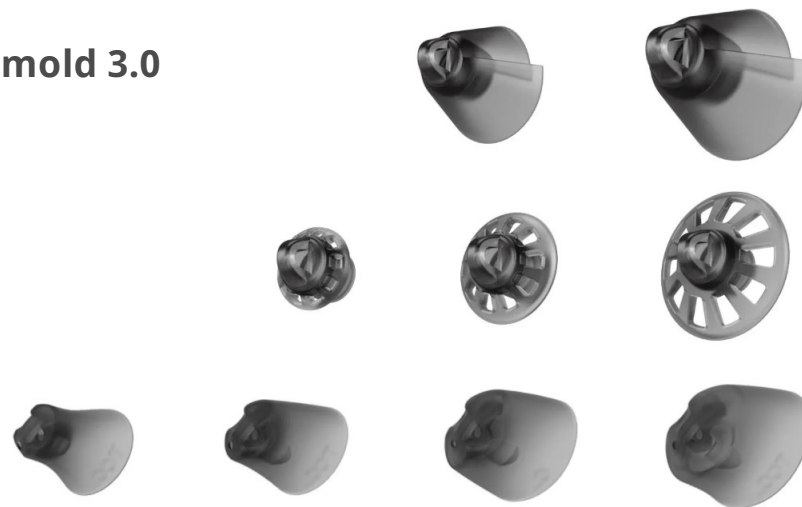


Intuis M 4



Size 13 battery

Battery runtime (without streaming)	up to 142 h
Battery runtime (incl. 20 h streaming)	up to 129 h

Intuis M 4 | Fitting Range

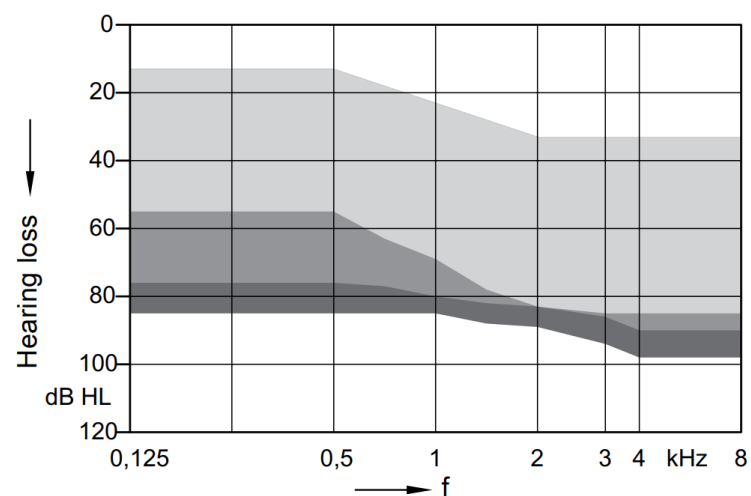

ThinTube 3.0 + Eartip 3.0 Open
ThinTube 3.0 P + Sleeve 3.0 Power
Earhook + Mold

Intuis P 4



Size 13 battery


Battery runtime (without streaming)

up to 148 h

Battery runtime (incl. 20 h streaming)

up to 115 h

Intuis P 4 | Fitting Range

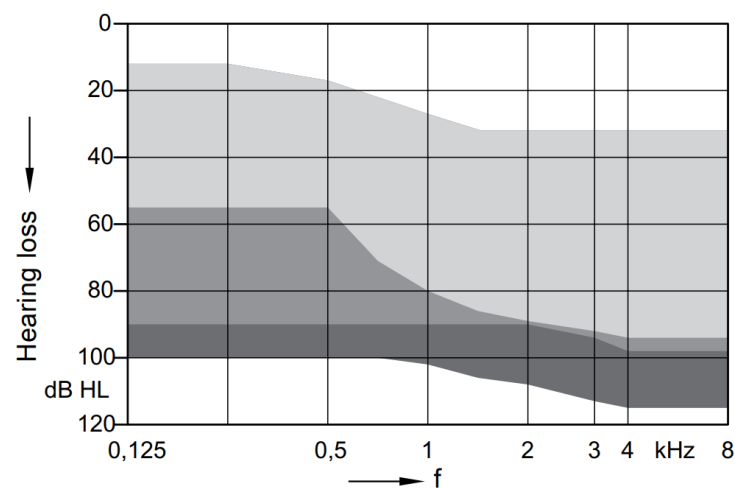

ThinTube 3.0 + Eartip 3.0 Open
ThinTube 3.0 P + Sleeve 3.0 Power
Earhook + Mold

Intuis SP 4



Size 675 battery


Battery runtime (without streaming)

up to 277 h

Battery runtime (incl. 60 h streaming)

up to 217 h

Intuis SP 4 | Fitting Range

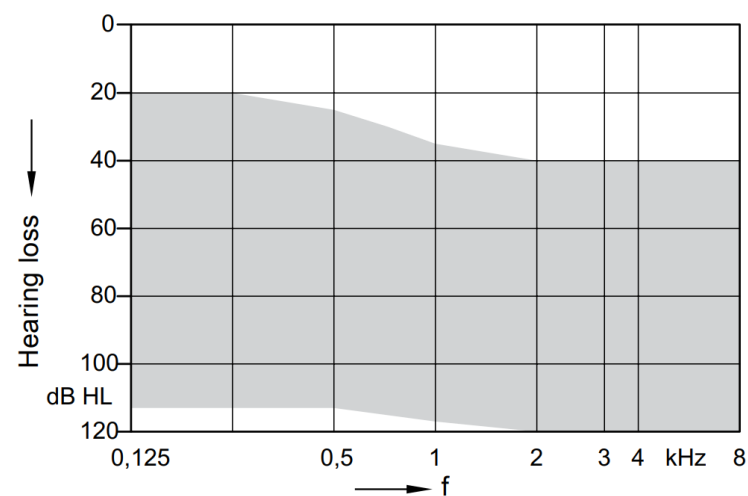

Earhook + Mold

Available in six attractive colors



Black



Graphite



Dark
Champagne



Silver Hero



Sandy Brown



Beige

audiologyonline |



Connectivity that empowers wearers

Signia Intuis 4 comes packed with features such as Bluetooth streaming for Android and iOS devices, and an integrated telecoil.



**Direct Bluetooth
streaming for iOS®
and Android™ (ASHA)**



Integrated telecoil



**Compatible with
StreamLine™ TV**




**Compatible with
StreamLine™ Mic**

audiologyonline | 

Ready for any challenge

Intuis 4 is rated IP68 for water and dust resistance

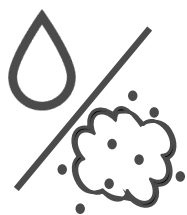

Intuis 4 offers a range of tinnitus therapy options*

Intuis 4 is CROS compatible


Signia Intuis 4 is compatible with the CROS transmitters below*



Signia CROS Pure
Charge&Go AX transmitter



Signia CROS Pure 312AX
transmitter

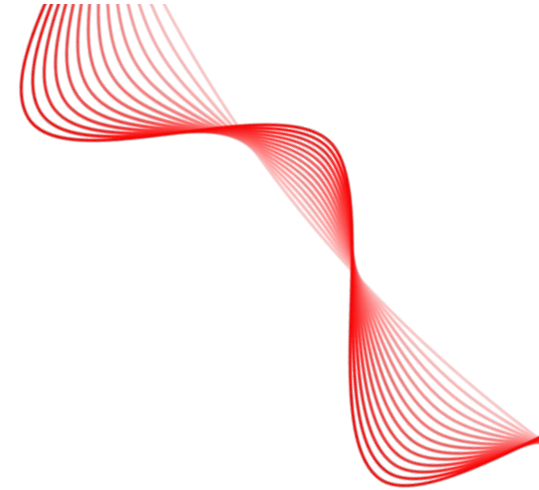

Wireless programming

Signia Intuis 4 is programmable with
Noahlink Wireless



Tamper-proof components for enhanced safety

Earhook

Secured with a pin that must be removed in order to unscrew earhook

Rocker switch

Cannot be removed without removing the hearing aid shell

Battery door


When desired, hearing care professionals can quickly and easily convert all Intuis 4 battery doors to be tamper-proof



Tamper-proof battery door conversion

Intuis M 4 and Intuis P 4 battery doors can be made tamper-proof by inserting an additional pin into the battery door.


See next slide for video demonstration.

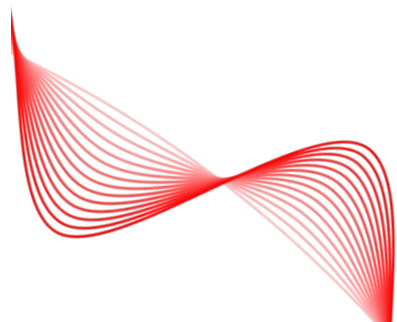

Tamper-proof battery door conversion

Intuis SP 4 standard battery doors can be easily removed and replaced with tamper-proof battery doors.

See next slide for video demonstration.

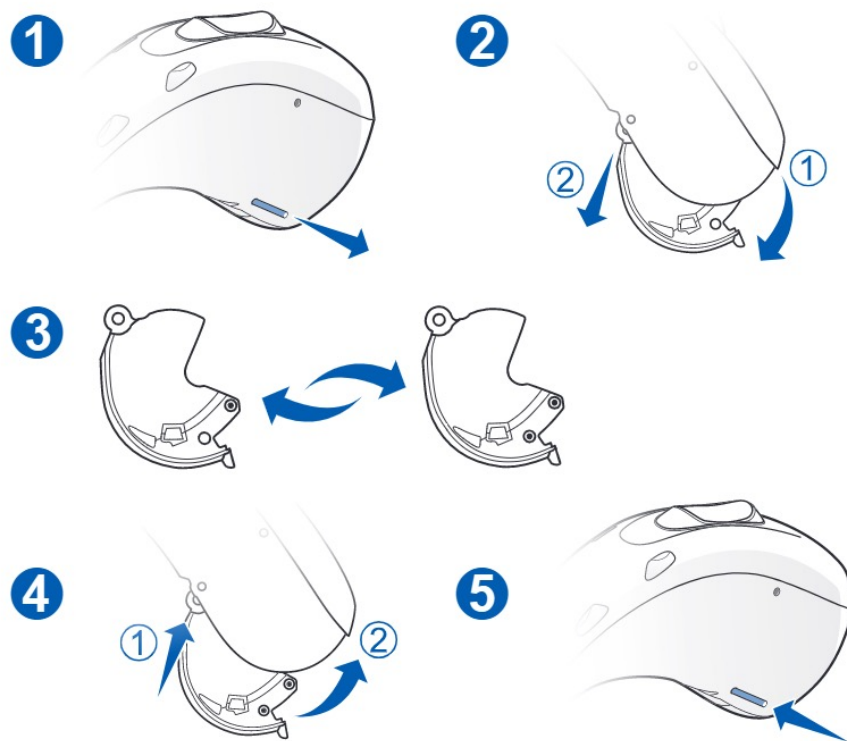

Opening tamper-proof battery doors

Turning off the hearing aid:



Tamper-proof battery doors can be easily opened a few millimeters to turn off the hearing aid.

Changing the battery:

A higher force is required to fully open tamper-proof battery doors compared to the standard battery door.

For easier opening of tamper-proof battery doors:

1. Place a small cloth or tissue over the battery compartment grip.
2. Open the door with your thumb as shown in the illustration.

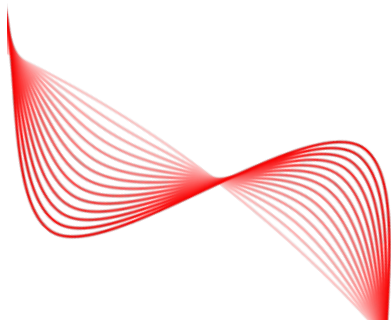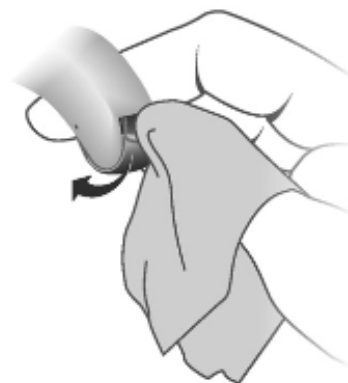



Thank You!

SigniaUSA
800-766-4500

