



signia

Quick Tips: Active Pro IX

AUDIOLOGYonline

Quick Tips: Signia Active Pro IX

- Thank you for joining us to learn more about the Signia Active Pro IX and Active IX
- For assistance with this course, contact AudiologyOnline (1-800-753-2160)
- Please visit the AudiologyOnline website for other live and recorded events from Signia.

<http://www.audiologyonline.com/ce/signia>

Allie Baeten, AuD
Clinical Education Specialist



Active Pro IX
It's a MODERN
Hearing Aid



Reconnect to the conversation with Signia Active Pro IX

A full prescription-hearing aid in a familiar consumer-friendly design

An easy-to-insert modern earbud design with premium hearing enhancement



A game changing **modern** prescription hearing aid



Reconnect with the conversation

RealTime Conversation Enhancement – a breakthrough technology that enhances what people are saying even when they move



Modern earbud design

Instant fit iconic earbud design with premium technology makes Active Pro IX a hearing aid unlike any other



All-day power

Full day wear time up to 34 h and convenient portable charging on the go with Qi wireless charging case



Bluetooth connectivity

Bluetooth streaming for iPhone and Android - Hands-free calling with CallControl to answer/end calls with a simple tap



Water & Dust resistant

IP68 certified





Active Pro IX & Active IX

Reconnect with the conversation



Active Pro IX & Active IX is ready for a full day

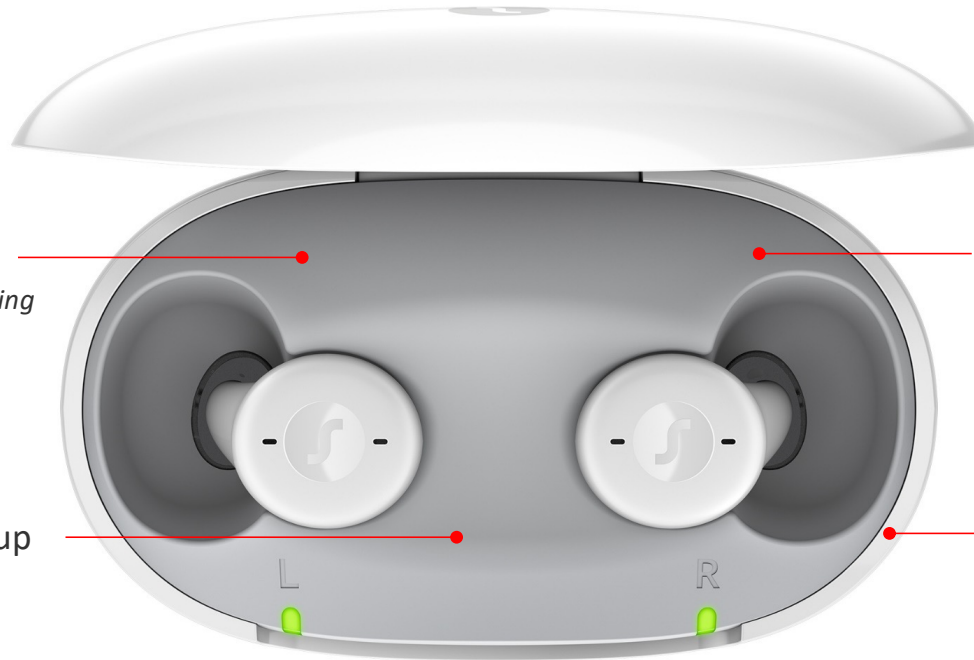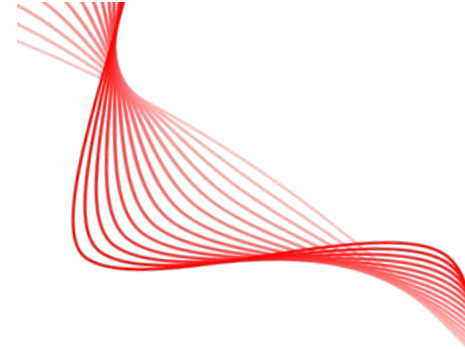

Up to 34 h of wear time*
Up to 29 h wear time with 5 h streaming

Charger holds up to 3 full charges

Quick 30-min charge provides up to 8 h of wear time

Charge on the go with Qi wireless charging

**Li-ion cell after 3 years of use under normal operating condition = up to 27 h of runtime*



Android and iOS connectivity



Stream music, phone calls, and enjoy
the full features of the Signia app

Made for iPhone (MFi)

Android ASHA

Android LE Audio



"Made for iPhone" and "MFi" are trademarks of Apple Inc., registered in the U.S. and other countries.

"Made for Android" and "ASHA" (Audio Streaming for Hearing Aids) are trademarks of Google LLC.



HandsFree with CallControl



Calls can be answered or ended by tapping directly on the hearing aid.

Wearer's own voice is picked up by the hearing aid microphone.

Convenient HandsFree calling with iPhone and Android LE Audio compatible devices



Performance Levels, Colors, and EarWear 3.0

Active Pro IX / Active IX

Ideal for mild to moderate hearing loss

47 dB / 116 dB SPL

(2 cc coupler)

EarWear 3.0

Sleeves 3.0 *recommended*

Eartips 3.0 *compatible*

Wax Guard 3.0 miniReceiver Signia
(#10993607 Waxguard 3.0 Nanocare Signia)

Active Pro IX

Active IX

CROS

sDemo



Black*



Rose Gold



Snow White



Brown



Mocha**



Beige**

*sDemo available in Black

**Mocha and Beige available December 2, 2024



CROS Active IX



Convenient rechargeability

Battery runtime up to 35 h



Enhanced clarity and transparency

e2e Wireless 4.0 delivers a boosted dynamic range between the hearing aid and CROS for enhanced clarity and transparency



Speech enhancement

Speech stabilizer preserves the acoustic cues, even in noise, important for speech understanding



Bluetooth Connectivity

Made for iPhone, Android ASHA, Bluetooth LE Audio



HandsFree with CallControl


The hearing aids stream the call audio and pick up the wearer's voice to deliver to the conversation partner. Tap to answer or end a call.




Water, sweat, and dust resistant

IP68 certified

Traditional CROS transmitter in noise

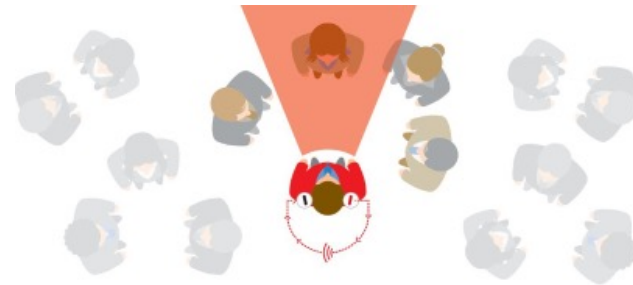

Signia CROS transmitter in noise

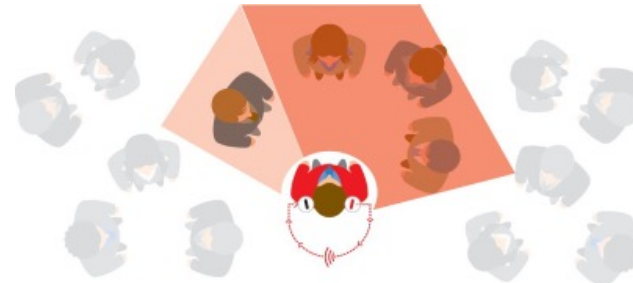

Unique Signia **skewed directionality** improves speech understanding in noise by skewing directional pattern toward unaided ear



Accessible familiar design



Note: Photo shows a development version without correct laser markings

Easy to handle modern instant-fit ITC design

- Great choice for wearers who prioritize ease of use, including those with limited vision or dexterity
- Rechargeable for easy all-day wear

New design elements to identify left vs right

- Easy tactile identification
 - A small raised bump on the inner part of the **left** device
- Easy visual identification
 - Prominently visible **R** (right) and **L** (left) labels on the inner side of the devices



Improved ease of use design

New design element for EarWear alignment

- Easy visual identification of proper alignment of EarWear 3.0 Sleeves
- HCPs and Patients can align their Sleeve 3.0 with confidence for a secure & comfortable fit



Note: Photo shows a development version without correct laser markings



Wearer Empowerment

Active Pro IX and Active IX



Instant fit
modern earbud



Bluetooth
connectivity*



HandsFree with
CallControl



Convenient
rechargeability



IP68



CROS
compatible



Signia
app



Signia Assistant



TeleCare



RealTime Conversation Enhancement

Active Pro IX



	Active Pro IX	Active IX
RealTime Conversation Enhancement	...	-
Conversation Dynamics Analyzer	✓	-
Conversation Source Analyzer	✓	-
Advanced Dynamic Focus Streams	3	-
Conversation Booster	...	-



Conversation Management & Listening Comfort

Active Pro IX and Active IX



	Active Pro IX	Active IX
Channels/Handles	48/20	16/8
e2e Wireless 4.0	✓	✓
Augmented Focus	✓	✓
Speech Stabilizer	✓	✓
Extended Dynamic Range	✓	✓
Feedback Cancellation	✓	✓
Programs	6	4
Dynamic Soundscape Processing 3.0	●●●●●	●○○○○
Automatic Directionality	●●●●●	●○○○○
TruEar 360	●●●●●	●○○○○
Speech & Noise Management	●●●●●	●○○○○
SoundSmoothing	●●●●●	●○○○○
eWindScreen	●●●●●	●○○○○



Conversation Management & Listening Comfort

Active Pro IX and Active IX



	Active Pro IX	Active IX
Channels/Handles	48/20	16/8
RealTime Conversation Enhancement	√	—
Conversation Dynamics Analyzer	√	—
Conversation Source Analyzer	√	—
Advanced Dynamic Focus Streams	3	—
Conversation Booster	√	—
Spatial Speech Focus	√	—
Speech Focus	√	—
Motion Sensor	√	—
Binaural Directionality	√	—
Adaptive Directionality	√	—
Extended Bandwidth	√	—
Auto EchoShield	√	—
Reverberant Room Program	√	—
Directional Hearing	√	—
My WellBeing	√	—
Tinnitus Noise Therapy Manager	√	—
Notch Therapy	√	—
Static Therapy Signals (channels/presets)	20/5	—
Ocean Waves Therapy Signals	4	—



Active Pro IX & Active IX accessories



Active Pro IX and Active IX – Wireless programming



Wireless programming
via NoahLink Wireless



It is a fully featured prescription hearing aid

- The worlds only instant-fit rechargeable ITC with the power of the Signia Integrated Xperience platform
- Empowering patients to contribute to dynamic group conversations in noise with RealTime Conversation Enhancement

..In a uniquely desirable design





signia

Thank You!

SigniaUSA
800-766-4500

audiologyonline