

6 easy steps to fit ReSound Nexia

A ReSound Nexia™ binaural fitting is depicted.



1. Launch ReSound Smart Fit™ fitting software and put the devices in discovery mode by removing them from the charger or closing the battery door for non-rechargeable models. Click **Connect to ReSound Smart Fit** in the upper right corner of the Smart Launcher screen.
2. The hearing aids will appear in the **Select Instruments** section. Use the **Push to Assign** feature to assign to the respective right and left ear placement. Click **Continue** in the lower right of the screen. Select appropriate physical properties and click **Continue** to proceed through connection process.

3. Calibrate the hearing aids on the patient's ears to activate DFS Ultra III. Click **Continue** in the lower right of the screen.
Note: Devices will automatically unmute once calibrated.

4. Navigate to **Patient > Profile** screen, customize the Prescribed First Fit selections as appropriate:

- a. Select Patient **Experience Level** based on previous experience with amplification and to maximize acceptance of hearing aids.


- b. Click **Fit Patient** in the lower right, or navigate to **Gain Adjustments**.

5. Fine-tune instruments using **Gain Adjustments**:

- a. Add or remove programs from this screen.

- b. Once programming is set, navigate to **Device Controls** to customize the manual push button, demonstrate beeps or pair accessories.

6. Click **Save** at bottom right of screen. Devices will reboot and play the start-up melody upon exiting the software.

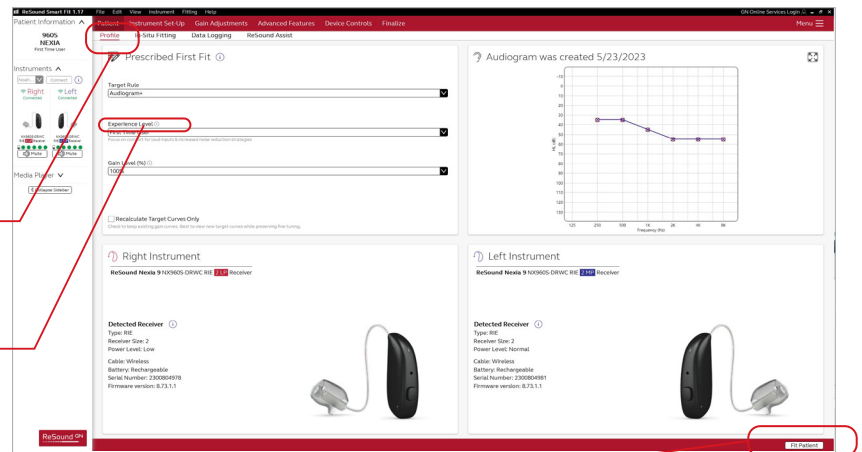

Manufacturer according to FDA:

GN ReSound North America
8001 E Bloomington Freeway
Bloomington, MN 55420
USA
1-800-248-4327
pro.resound.com

ReSound Government Services
8001 E Bloomington Freeway
Bloomington, MN 55420
USA
1-800-392-9932
gs.resound.com

Manufacturer according to Health Canada:

ReSound Canada
2 East Beaver Creek Road, Building 3
Richmond Hill, ON L4B 2N3
Canada
1-888-737-6863
pro.resound.com