

1. This document was created to support maximum accessibility for all learners. If you would like to print a hard copy of this document, please follow the general instructions below to print multiple slides on a single page or in black and white.
2. This handout is for reference only. Non-essential images have been removed for your convenience. Any links included in the handout are current at the time of the live webinar, but are subject to change and may not be current at a later date.
3. Copyright: Images used in this course are used in compliance with copyright laws and where required, permission has been secured to use the images in this course. All use of these images outside of this course may be in violation of copyright laws and is strictly prohibited.
4. Social Workers: For additional information regarding standards and indicators for cultural competence, please review the NASW resource: [Standards and Indicators for Cultural Competence in Social Work Practice](#)
5. Need Help? Select the “Help” option in the member dashboard to access FAQs or contact us.

How to Print Handouts

- On a Mac
 - Open PDF in Preview
 - Click File
 - Click Print
 - Click dropdown menu on the right “preview”
 - Click layout
 - Choose # of pages per sheet from dropdown menu
 - Checkmark Black & White if wanted.
- On a PC
 - Open PDF
 - Click Print
 - Choose # of pages per sheet from dropdown menu
 - Choose Black and White from “Color” dropdown

No part of the materials available through the continued.com site may be copied, photocopied, reproduced, translated or reduced to any electronic medium or machine-readable form, in whole or in part, without prior written consent of continued.com, LLC. Any other reproduction in any form without such written permission is prohibited. All materials contained on this site are protected by United States copyright law and may not be reproduced, distributed, transmitted, displayed, published or broadcast without the prior written permission of continued.com, LLC. Users must not access or use for any commercial purposes any part of the site or any services or materials available through the site.



Introducing Beltone Envision

Presenter:
Laura Schachtel, Au.D.
Product Training Development
Manager





Webinar FAQs

- **How to ask a question:**
 - Use Q&A to submit your question
- **Resources Tab:**
 - Download the course handout
 - View Presenter Information
 - Link to Course Details
- **How to view captions:**
 - Click CC button (or ... More button)
 - Click Show/Hide Subtitles
- **How to earn CEUs:**
 - After the webinar, go to your Pending Courses page and pass the multiple-choice exam
- **For assistance or tech support:**
 - Call 800-753-2160
 - Email customerservice@AudiologyOnline.com
 - Use the Q&A

1

Learning Outcomes

1

Describe the updated feature set of our Beltone Envision RIEs.

2

Explain the process for changing our RIE microphone filters.

3

List the connectivity options of Beltone Envision.

2

Risks/ Limitations





- The applicability and accuracy of the course content may vary by the clinical, geographical, and cultural context; you should evaluate the course accordingly.
- As healthcare is rapidly evolving, new findings may change the validity of the information presented in this course. Providers should always consult the latest research and guidelines from professional associations.
- The interventions discussed in this course may be limited in generalizability, requiring practitioners to consider specific individual and population factors.
- The course may not cover all possible risk factors, diagnostic methods, or treatment options, and complex cases may require additional considerations.
- Healthcare providers should apply cultural competence and sensitivity to ensure effective and respectful care based on their patients' diverse needs.
- Providers should always evaluate the limitations of diagnostic and screening tools, considering their potential for false results as well as any ethical implications.
- It is the responsibility of course participants to critically evaluate the evidence from multiple sources to guide professional practice.

3

Our Updated Product Portfolio



Beltone Setup
Max 2.1™



360 Chip



SureFit 3



Replaceable
microphone filters



Bluetooth LE Audio
& Auracast



True Noise
Reduction

PREMIUM



Beltone Envision

ESSENTIAL



Beltone Commence

4

World’s Smallest AI Hearing Aid*

DNN –Tech level 17
True Focus and
True Noise Reduction



ENV62S-DRWC
microRIE
Rechargeable



ENV63-DRW
Standard RIE
312, 13 battery



ENV64-DRW
Standard RIE
312, 13 battery

Tech level 17
Clear Focus

Updated style

Technology level 17 – 9 – 6 – 4

5

Exchangeable
Microphone Filters



6

2

New Colors

Sparkling Silver Sand Gold Bronze Warm Grey Espresso Graphite Deep Black Navy Blue Red

All-weatherproof IP68, including receivers

7

SureFit 3 Receivers

M&RIE LP MP HP UP

8

microRIE ENV 62S

Beltone Envision Rechargeable microRIE

- True Focus and True Noise Reduction
- Lithium-ion battery with all day power
- Compatible with TV-Streamer+ and Multi-Mic+
- Compatible with Beltone Envision RIE CROS Transmitter
- Bluetooth Low Energy Audio
- Auracast and compatible with Auracast Assistant
- TapControl
- Push to assign, LED Light, Push Button
- 2.4 GHz and E2E communication
- MFi, ASHA and Hands-free calls

9

Charger Options



Premium Charger

- Integrated on-board battery
- Works as charging case
- Provides up to 3 charges
- LED battery status indicator



Standard Charger

- Charges while connected to power cable
- Single LED indicator when connected to power source



Desktop Charger

- Charges while connected to power cable
- Simple, easy to use design



10

All-day Battery

Hours of use







11

Beltone Envision Replaceable Battery RIEs



63 RIE
using 312
battery



64 RIE
using 13
battery



Updated housing style
Clear Focus

Compatible with TV-Streamer+ and Multi-Mic+

Compatible with Beltone Envision RIE CROS Transmitter

- Bluetooth Low Energy Audio

Auracast compatible with Auracast Assistant

Push Button/Multi Toggle Button

Push to assign

- 2.4 GHz and E2E communication
- MFI, ASHA and Hands-free calls



12

4

ENV162S CROS Transmitter




- Need to use this transmitter when fitting with any Envision RIE
- Only available in the microRIE style
- Has TapControl
- Same colors
- Transmitter's only feature is Balance
 - Patient's can control their Balance from the push button or Beltone HearMax app

 Beltone UNIVERSITY

13

Hear in Noise Program

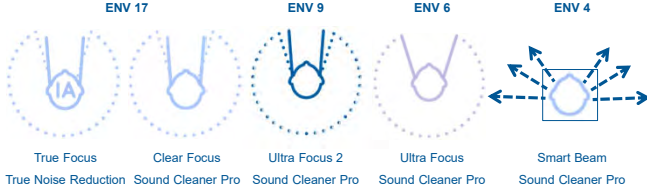


ENV 17	ENV 9	ENV 6	ENV 4	ENV 4
True Focus	Clear Focus	Ultra Focus 2	Ultra Focus	Smart Beam
True Noise Reduction	Sound Cleaner Pro	Sound Cleaner Pro	Sound Cleaner Pro	Sound Cleaner Pro

 Beltone UNIVERSITY

14


Clear Focus (upgraded beamformer)

- Full low-frequency directional response
 - Maximizes the effectiveness of our noise reduction system

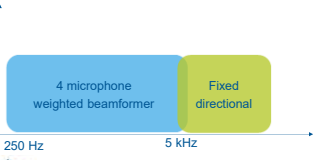


 Beltone UNIVERSITY


15

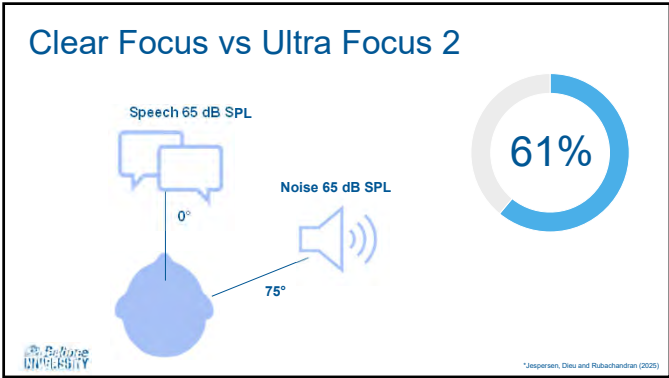

16

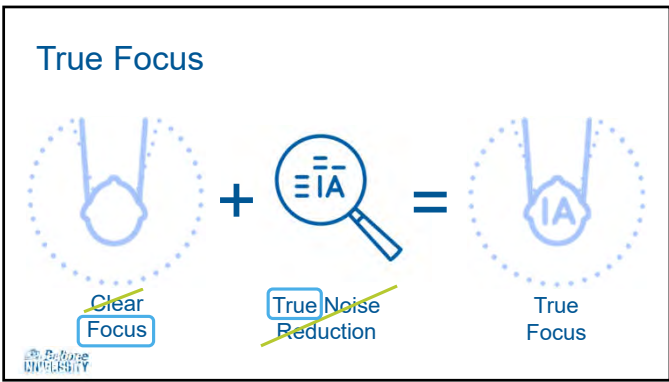


17

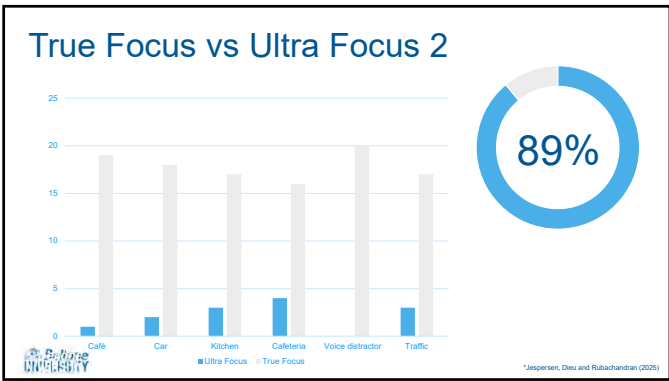

18

True Noise Reduction


- Uses DNN training to decrease distracting noise and enhance speech
 - Better ability to handle fast-changing, novel or ambiguous sounds
- Three levels of reduction
 - First 6 bands lower frequencies
 - The other 11 bands mid and high frequencies

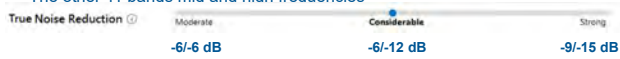

19

Other Feature Enhancements

In all technology levels

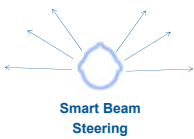

Additional option in 9, 6, 4 level



Wind Noise Reduction



Impulse Noise Reduction



Silencer



Sound Cleaner

20

Summary



Beltona Envision RIEs are now offered in 4 technology levels all running on our 360 chip. Our 17 level RIEs offer the new features of True Focus, Clear Focus and True Noise Reduction.



To replace our new RIE microphone filters you remove the old and insert the new.



Beltona Envision offers a lot of connectivity options; Beltona accessories, Bluetooth LE Audio, Auracast and hands-free calls.

21
