

Charging options for Beltone hearing aids

Beltone offers rechargeable solutions with inductive charging and all-day battery life to keep your clients on the go.



Premium charger



Standard charger



Desktop charger



Custom charger

	Premium charger	Standard charger	Desktop charger	Custom charger
microRIE ¹ 	8	8	8	-
miniRIE Beltone Commence 	9	9	9	-
miniRIE Beltone Achieve 	6	6	6	-
BTE ² 	-	-	70	-
PBTE ³ 	-	-	80	-
ITE/ITC ⁴ 	-	-	-	C

¹ microRIE: Beltone Envision & Beltone Serene

² BTE: Beltone Serene & Beltone Achieve


³ PBTE: Beltone Serene & Beltone Achieve

⁴ ITE/ITC: Beltone Serene & Beltone Achieve

How to match Beltone hearing aids with the correct charger

Each charger type is offered with different inserts – known as the charging form factor (CFF) – which relate to a specific Beltone hearing aid collection or style.

The CFF is indicated with a number and is shown inside the charger and on the charger packaging.

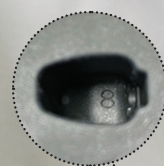

Number inside charger



Number on packaging

CFF numbers are not shown on all charger packaging yet, please refer the number inside the charger:

- Beltone Achieve RIE have CFF 1 shown inside the charger
- Beltone Amaze RIE & Beltone Rely RIE have CFF 4 shown inside the charger



Charger features

	Beltone Envision, Beltone Serene & Beltone Achieve				Beltone Commence, Beltone Amaze & Beltone Rely	
	Premium charger	Standard charger	Desktop charger	Custom charger	Premium charger	Desktop charger
	microRIE/miniRIE/RIE	microRIE/miniRIE/RIE	microRIE/miniRIE/RIE BTE/Power BTE	ITE/ITC	miniRIE/RIE	miniRIE/RIE
Inductive charging	•	•	•	•	•	•
Portable with onboard power bank	•				•	
Portable without power bank		•		•		
USB-C connector and cable	•	•	•	•	•	•
LED on the back	Shows the power bank battery status	Lights up when the charger is connected to the main power supply	Shows when the charger is connected to a power supply	Shows when the charger is connected to a power supply	Shows the power bank battery status	Shows when the charger is connected to a power supply
LED on the front	Shows the hearing aids charging			Shows the hearing aids charging	Shows the hearing aids charging	

How to tell if the hearing aids are charging

Place the hearing aids in the charger and let the lights indicator on the hearing aids guide you. It is the same process whichever charger is used.

You know that charging has started when the hearing aid lights blink slowly.

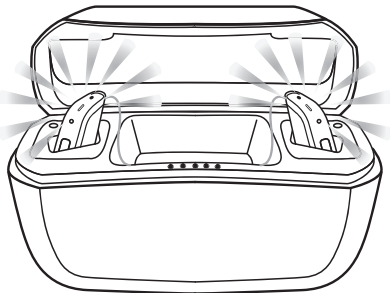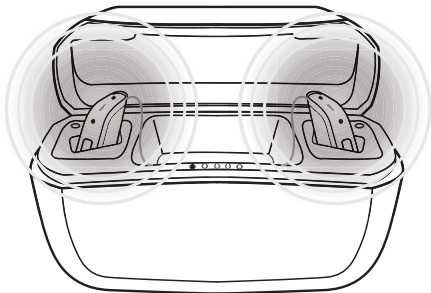

When fully charged, the hearing aid lights remain constant, not blinking.

The lights turn off when the hearing aids are removed from the charger.

Premium charger illustrated.
For more information, please see the individual user guide for each charger.

Worldwide Headquarters
GN Hearing A/S
Lautrupbjerg 7
DK-2750 Ballerup, Denmark
Tel.: +45 45 75 11 11
gnhearing.com
CVR no. 55082715

Australia
GN Hearing Australia Pty Ltd
Gate C, 19-25 Khartoum Road
Macquarie Technology Park
Macquarie Park NSW 2113
Tel.: (free) 1800 658 955
gnhearing.com

New Zealand
GN Hearing New Zealand Limited Ground
Ground Floor, North Entrance
4 Fred Thomas Drive
Takapuna, Auckland, 0622
Tel.: (free) 0800 900 126
gnhearing.com