

For professionals

A collection of hearing solutions **tailored to** **your patients**

Beltone **Portfolio**



Welcome more patients with the most intelligent Beltone portfolio yet

Our Organic Hearing philosophy

The human brain is our most powerful sound processor, naturally skilled at tuning in to sounds and filtering out distractions. Hearing loss can disrupt this intuitive ability.

That's why we develop technology with people in mind to emulate the natural hearing process, so the brain can hear as nature intended, with solutions designed to sound natural, feel natural and connect you naturally to the world.

Our philosophy has evolved over many years to be the heart of our hearing innovations, including supporting the human brain with Artificial Intelligence (AI). We have further refined our approach to introduce Intelligence Augmented – augmenting what people already are naturally good at, so when extra help is needed, their brain can continue its natural processing.



Sounds natural

- Top-rated for hearing in noise¹
- Industry-best access to surrounding sound²
- Natural own voice perception³



Feels natural

- Comfortable all-day wear in small and sleek designs
- All-weatherproof design
- Rechargeable for all-day use



Connects you naturally

- Crystal clear phone calls
- Hands-free calls for iPhone, iPad and compatible Android™ devices.*
- Made for Bluetooth® Auracast™

Intelligent hearing for everyone with two new families

Beltone Envision

Illuminate life with our latest Premium family of RIEs. Offer the world's best for hearing in noise and the smallest AI hearing aids, without compromising on battery, comfort or connectivity.¹⁻⁴



Beltone Commence

Introduce hearing clarity and connectivity for all with our new best-in-class Essential family, which combines our proven technologies to bring greater clarity and ease to everyday connections.



Hearing solutions to delight your patients

Premium	RIE styles Mild to profound hearing loss  microRIE 62S Rechargeable RIE 63 RIE 64 NEW	Beltone Envision
Premium	RIE, BTE and custom styles Mild to profound hearing loss  CIC Non-wireless ITC Wireless ITC Rechargeable ITE Rechargeable microRIE 62S Rechargeable RIE 63 RIE 64 BTE 76 Rechargeable Power BTE 86 Rechargeable	Beltone Serene
Advanced	RIE, BTE and custom styles Mild to profound hearing loss  CIC Non-wireless ITC Rechargeable ITE Rechargeable miniRIE 62 Rechargeable RIE 63 RIE 64 BTE 76 Rechargeable Power BTE 86 Rechargeable	Beltone Achieve
Advanced	RIE, BTE and custom styles Mild to profound hearing loss  ITC Wireless ITE Wireless miniBTE 66 BTE 76 Power BTE 86 High Power BTE 86* Super Power BTE 95*	Beltone Amaze Beltone Boost Ultra
Essential	RIE styles Mild to profound hearing loss  miniRIE 62 Rechargeable RIE 63 RIE 64 NEW	Beltone Commence
Essential	RIE, BTE and custom styles Mild to profound hearing loss  CIC Non-wireless ITC Wireless ITE Wireless RIE 63 Rechargeable RIE 63 RIE 64 miniBTE 66 BTE 76 High Power BTE 86 Super Power BTE 95	Beltone Rely



Receivers, thin tubes and domes overview



Auracast and streaming



Connectivity and accessories



Feature overview

Bring a world of sound **into focus**

NEW

Beltone **Envision™**

Beltone Envision gives clarity to the sounds of life, with intelligent hearing technology. Support your patients to hear with renewed focus and experience sound like never before.



World's best for
hearing in noise¹



World's smallest
AI hearing aid²



All-day battery* even
when streaming

Colors



Sparkling
Silver

Sand

Gold

Bronze

Warm
Grey

Graphite

Espresso

Deep
Black

Navy
Blue

Red

Charging options



Premium charger

Portable with powerbank
Compatible with the microRIE (CFF 8)



Standard charger

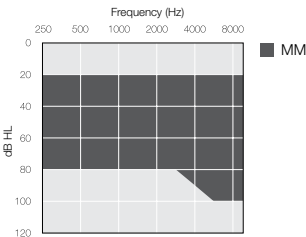
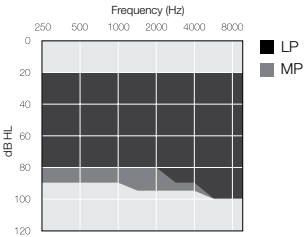
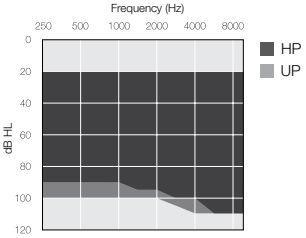
Portable
Compatible with the microRIE (CFF 8)



Desktop charger

Easy-to-use
Compatible with the microRIE (CFF 8)

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain [*]
 microRIE 62S-DRWC Rechargeable Available as CROS transmitter (ENV162S-DRWC)	• TapControl with microRIE • All-day battery on one charge[*] • Receiver options: MM, LP, MP, HP and UP • Exchangeable mic filters • Programmable push button or toggle • IP68 certified • Telecoil in 64		MM: 115/61 LP: 113/56 MP: 115/60 HP: 120/60 UP: 129/71
 RIE 63-DRW 312 battery			
 RIE 64-DRW 13 battery			

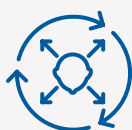
^{*}ANSI S3.22-2014, IEC 60118-0:2015, JIS C 5512: 2015



Experience more of life

Beltone **Serene**™

Beltone Serene hearing aids connect with new Bluetooth Low Energy (LE) Audio and Auracast broadcast audio, so your patients can feel more connected, included and social, wherever they are. All while enjoying an incredible sound experience, even in noisy situations, with tiny and comfortable styles.



Industry-best
access to surrounding
sound¹



Smallest
rechargeable style
with microRIE²



World's first hearing aids
with Bluetooth LE Audio &
Auracast broadcast audio²

RIE and BTE colors



Custom colors



Charging options



Premium charger
Portable with powerbank
Compatible with the microRIE (CFF 8)



Standard charger
Portable
Compatible with the microRIE (CFF 8)

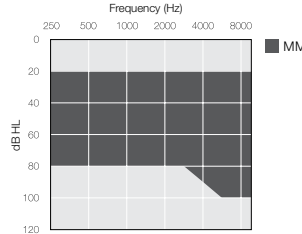
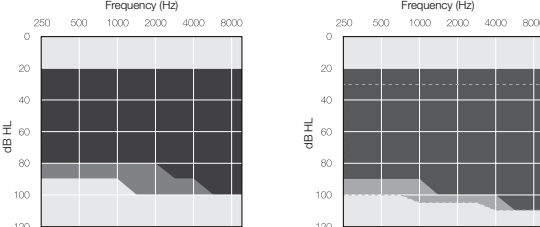
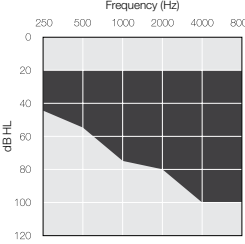
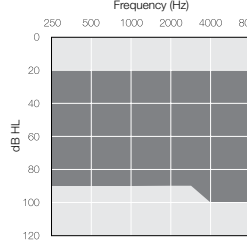
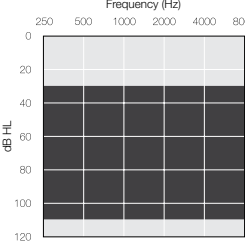
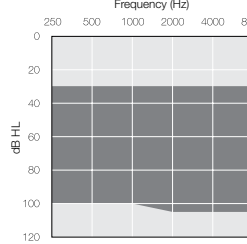
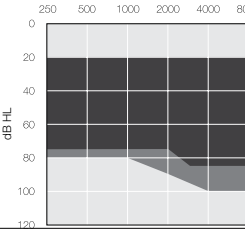
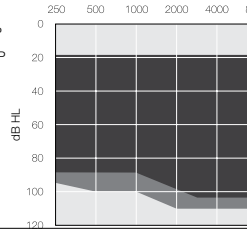
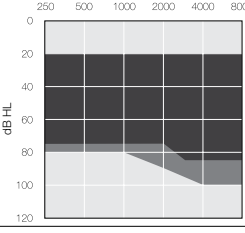
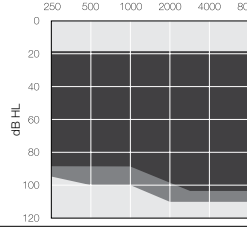
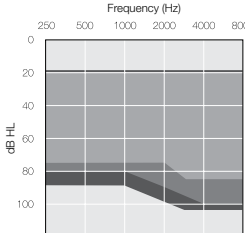


Desktop charger
Easy-to-use
Compatible with microRIE (CFF 8),
76 BTE (CFF 70), 86 BTE (CFF 80)



Custom charger
Portable for ITC/ITE
(CFF C4)

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain ¹
 microRIE 62S-DRWC Rechargeable Available as CROS Transmitter (CSE162S-DRWC)	- TapControl with microRIE - All-day battery on one charge[*] - Receiver options: MM, LP, MP, HP and UP - Programmable push button or toggle - IP68 certified - Telecoil in 64	 	MM: 115/53 LP: 114/54 MP: 116/59 HP: 120/63 UP: 130/73
 RIE 63-DRW 312 battery			
 RIE 64-DRW 13 battery			
 BTE 76-DWC Rechargeable Available as CROS Transmitter** (CSE176-DWC)	- All-day battery on one charge[*] - Programmable push button and/or toggle - Optional metal hook for 86-DWC - IP68 certified - Telecoil in BTE 76 and 86 - DAI in BTE 88 - Exchangeable mic filters	   	BTE (open): 127/53 BTE (closed): 128/60 Power BTE (plastic hook): 133/67 Power BTE (metal hook): 133/73
 Power BTE 86-DWC Rechargeable			
 ITC-DWC / ITE-DWC Rechargeable	- All-day battery on one charge[*] - Receiver options: LP[†], MP, HP and UP - Shell options: ITC, Half-shell^{††} and Full-shell^{†††} - Programmable push button - IP68 certified	 	LP: 114/40 ^{***} MP: 118/50 HP: 121/60 UP: 130/70
 ITC-DW 312 battery	- Receiver options: LP, MP, HP and UP - Shell options: ITC, Half-shell and Full-shell - Programmable push-button and volume control - IP68 certified	 	LP: 114/40 MP: 117/50 HP: 120/60 UP: 130/68
 CIC non-wireless 10A battery	- Cabled fitting required - Receiver options: LP, MP and HP - IP68 certified		LP: 114/40 MP: 116/50 HP: 121/60

Hear through noise without effort

Beltone **Achieve**™

Offer your patients outstanding speech understanding in noise¹, without effort, with the full family of Beltone Achieve hearing aids.



150% improvement
to speech understanding in
noise with Front Focus²



Great wearing
comfort



Robust connectivity
for uninterrupted
streaming

RIE and BTE colors



Custom colors



Charging options



Premium charger
Portable with powerbank
Compatible with the miniRIE (CFF 6)



Standard charger
Portable
Compatible with the miniRIE (CFF 6)

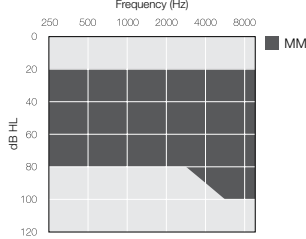
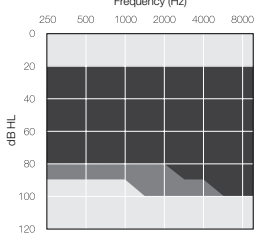
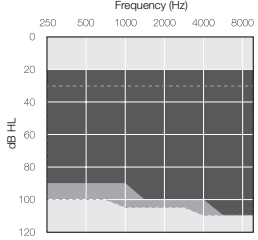
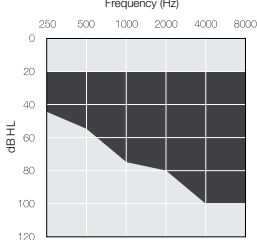
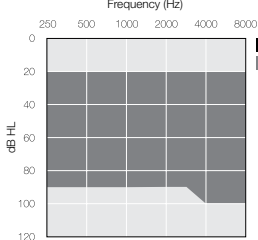
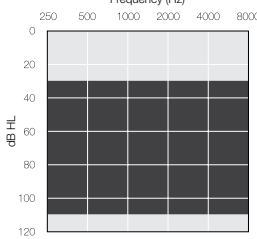
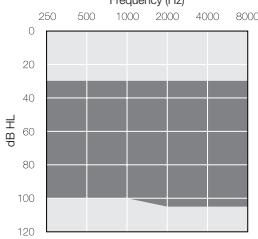
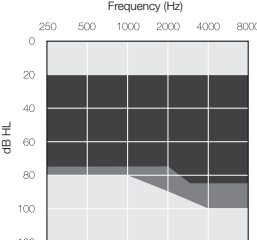
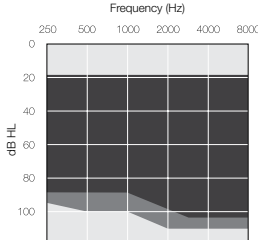
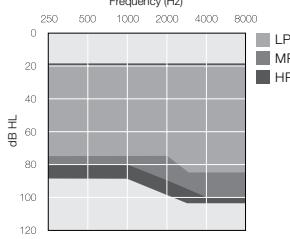


Desktop charger
Easy-to-use
Compatible with miniRIE (CFF 6),
76 BTE (CFF 70), 86 BTE (CFF 80)



Custom charger
Portable for ITC/ITE
(CFF C4)

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain†
 miniRIE 62-DRWC Rechargeable  RIE 63-DRW 312 battery  RIE 64-DRW 13 battery	- All-day battery on one charge* - Receiver options: MM, LP, MP, HP and UP - Programmable push button or toggle - Telecoil in 64 - IP68 certified	  	MM: 115/53 LP: 113/53 MP: 116/58 HP: 122/65 UP: 130/75
 BTE 76-DWC Rechargeable  Power BTE 86-DWC Rechargeable	- All-day battery on one charge* - Programmable push button/programmable push button and toggle - Optional metal hook for 86-DWC - IP68 certified - Telecoil in BTE 76 and 86 - DAI in BTE 86	   	BTE (open): 127/53 BTE (closed): 128/60 Power BTE (plastic hook): 133/67 Power BTE (metal hook): 133/73
 ITC-DWC / ITE-DWC Rechargeable	- All-day battery on one charge* - Receiver options: LP**, MP, HP and UP - Shell options: ITC, Half-shell*** and Full-shell**** - Programmable push button - IP68 certified	 	LP: 114/40*** MP: 118/50 HP: 121/60 UP: 130/70
 CIC non-wireless 10A battery	- Cabled fitting required - Receiver options: LP, MP and HP - IP68 certified - Optional push button		LP: 114/40 MP: 116/50 HP: 121/60

Make everyday amazing

Beltone **Boost Ultra**[™]
Beltone **Amaze**[™]

Every day is filled with special moments that make our lives truly amazing. Beltone Amaze ensures your patients won't miss a single one of those moments.



Distortion-free sound
with an input dynamic range
of up to 116 dB SPL



Designed for
everyday use



Crystal clear phone calls
and streaming from iPhone,
iPad or Android devices

BTE colors



Beige



Grey



Brown



Charcoal



Sterling
Grey¹

Custom colors

ITC and ITE



Beige



Khaki



Tan

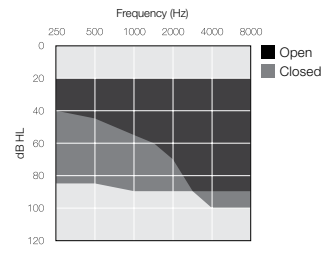
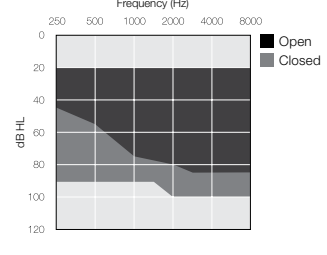
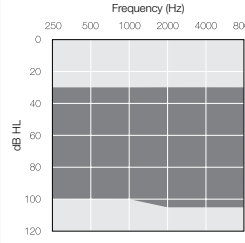
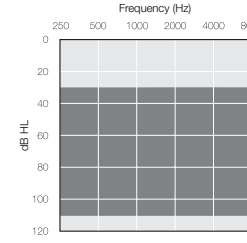
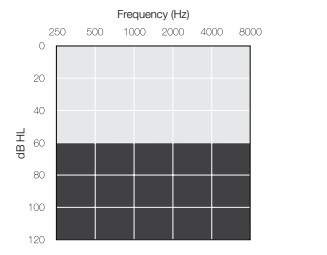
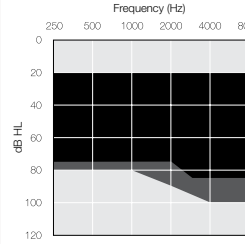
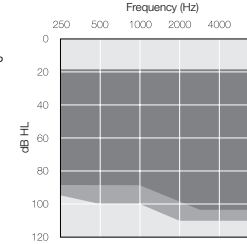


Brown



Espresso

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain†
 miniBTE 66-DWT 312 battery	• Programmable push button • Telecoil • IP68 certified		BTE (open): 123/52 BTE (closed): 121/56
 BTE 76-DWT 13 battery	• Push button and toggle • Telecoil and DAI • IP68 certified		BTE (open): 128/52 BTE (closed): 126/57
 Power BTE 86-DWT 13 battery  High Power BTE 86-DWHT** 13 battery	• Push button and toggle • Telecoil and DAI • IP68 certified	 	Power BTE: 134/68 High Power BTE: 134/73
 Super Power BTE 95-DWT* 675 battery	• Push button and toggle • Telecoil and DAI • IP68 certified		Super Power BTE: 141/83
 ITC-DW / ITE-DW 312 / 13 battery	• Receiver options: LP*, MP, HP and UP • Shell options: ITC, Half-shell**** and Full-shell***** • Programmable push button • IP68 certified • Telecoil (optional)	 	LP: 114/40** MP: 118/50 HP: 120/60 UP: 129/70

Beltone Amaze
Beltone Boost Ultra

Introducing hearing clarity and connectivity for all

Hearing aid adaptation is a well-known challenge for nearly all users.¹ Beltone Commence makes this process more effortless by combining our proven technologies to bring greater clarity and ease to everyday connections.



Best-in-class sound quality²⁻⁵



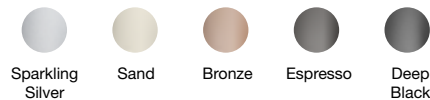
Best hearing comfort in tiny designs⁶



Clearest Bluetooth experience ever

Colors

Shimmering miniRIE colors^{**}



Matte RIE colors^{***}



Charging options



Premium charger

Portable with powerbank
Compatible with the miniRIE (CFF 9)



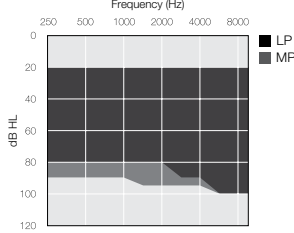
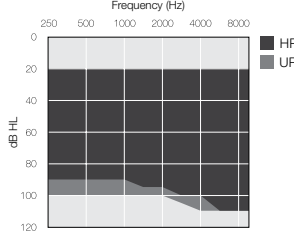
Desktop charger

Easy-to-use
Compatible with the miniRIE (CFF 9)

1 MarkeTrak (2022)
2 Groth (2016)
3 Groth & Cui (2023)
4 Haastrup (2012)
5 Sjolander et al. (2019)
6 GN data on file

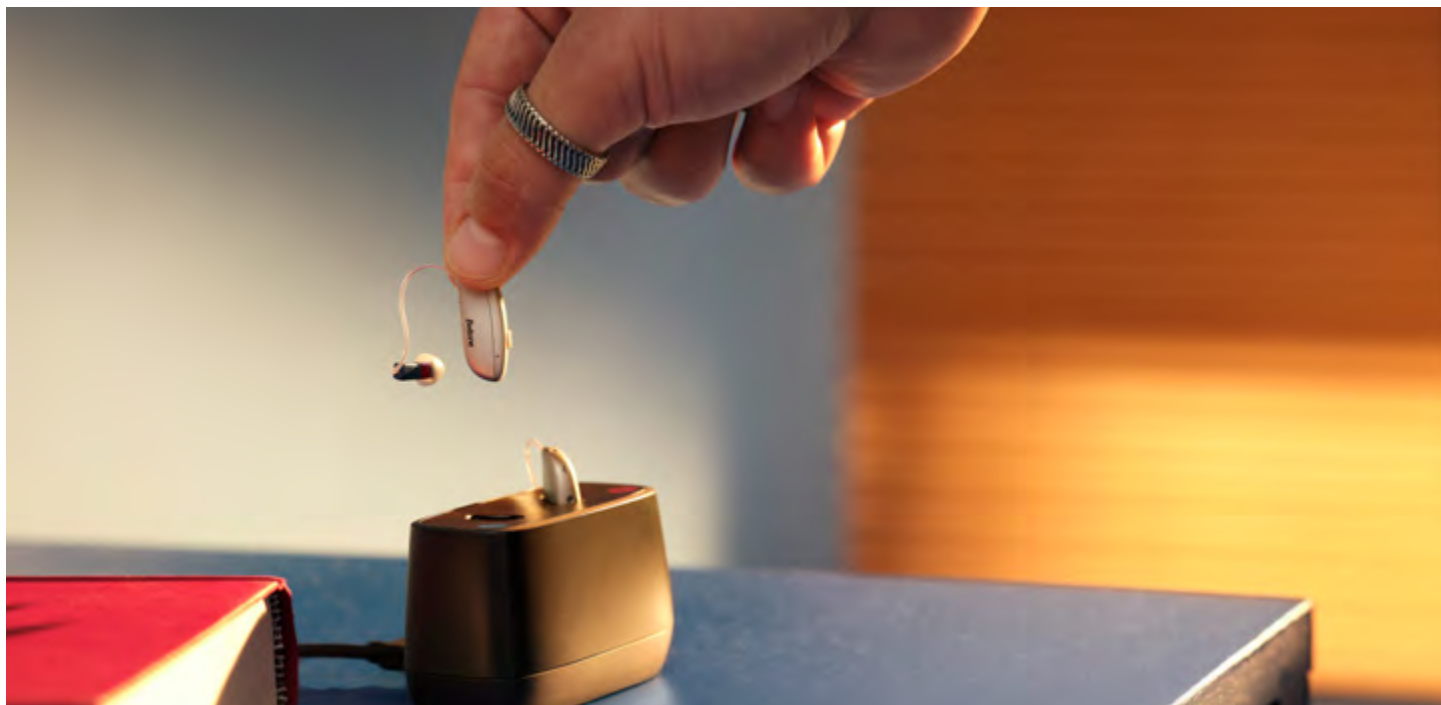
* Hands-free calls are compatible with iPhone 11 or later, iPad Pro 12.9-inch (5th generation), iPad Pro 11-inch (3rd generation), iPad Air (4th generation), iPad mini (6th generation), iPad (10th generation) or later, with software updates iOS 15.3 and iPadOS 15.3 or later.
** Available for miniRIE
*** Available for RIE 63 & RIE 64
**** All-day battery on one charge is based on typical use time of up to 24 hours with one charge or up to 20 hours when streaming 50% of the time across all styles. Battery life can vary by use.

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain [†]
 miniRIE 62-DRWC Rechargeable	• All-day battery on one charge**** • Receiver options: LP, MP, HP and UP • Programmable push button or toggle • IP68 certified • Telecoil in 64	 	LP: 113/56 MP: 115/60 HP: 120/60 UP: 129/71
 RIE 63-DRW 312 battery			
 RIE 64-DRW 13 battery			

Beltone Commence

[†]ANSI S3.22-2014, IEC 60118-0:2015, JIS C 5512: 2015



Great hearing for everyone

Beltone Rely™

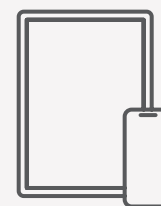
Say hello to a solution you can always rely on. Beltone Rely is designed to deliver a hearing solution with great sound quality, long-lasting rechargeability, and a comfortable design that fits your patient.



Great audibility all-around
and a distortion
free sound experience¹⁻³



Easy hearing
control on the
hearing aids



Streaming from
iPhone, iPad or
Android devices*

RIE and BTE colors



Custom colors

CIC Non-wireless



ITC and ITE



Charging options

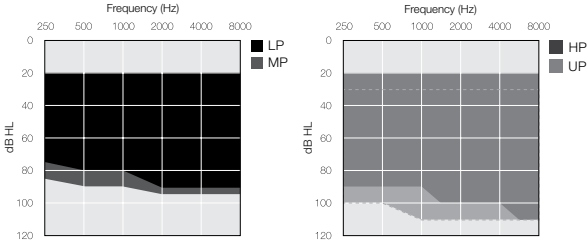
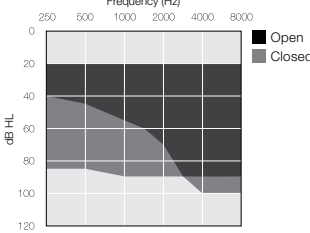
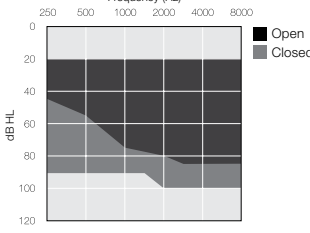
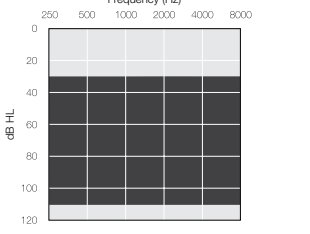
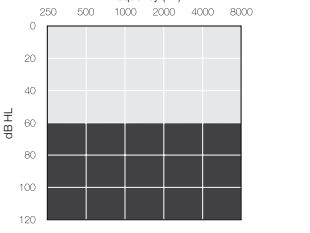
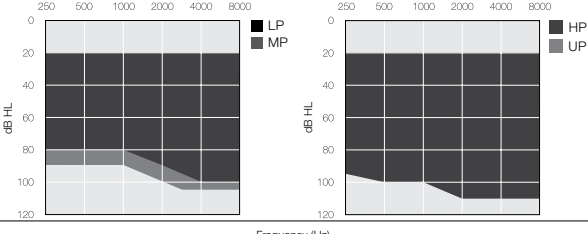
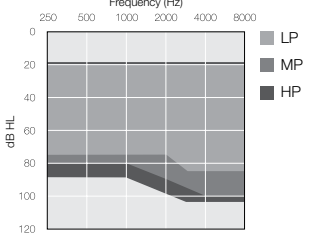


Premium charger
Portable with powerbank
Compatible with RIE 63-DRWC

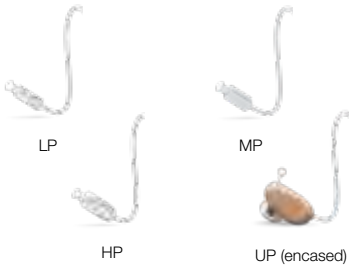


Desktop charger
Easy-to-use
Compatible with RIE 63-DRWC

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain†
 RIE 63-DRWC Rechargeable RIE 63-DRW 312 battery RIE 64-DRW 13 battery	- All-day battery on one charge* - Receiver options: LP, MP, HP and UP - Programmable push button or toggle - Telecoil and DAI in 64 - IP68 certified		LP: 113/52 MP: 116/58 HP: 120/65 UP: 128/75
 miniBTE 66-DW 312 battery	- Programmable push button - Telecoil - IP68 certified		BTE (open): 123/52 BTE (closed): 121/56
 BTE 76-DW 13 battery	- Push button and toggle - Telecoil and DAI - IP68 certified		BTE (open): 128/52 BTE (closed): 126/57
 Power BTE 86-DWH 13 battery	- Push button and toggle - Telecoil and DAI - IP68 certified		Power BTE: 134/73
 Super Power BTE 95-DW 675 battery	- Push button and toggle - Telecoil and DAI - IP68 certified		Super Power BTE: 141/83
 ITC-DW / ITE-DW 312 / 13 battery	- Receiver options: MP, HP and UP - Shell options: ITC, Half-shell™ and Full-shell™ - Programmable push button - IP68 certified - Telecoil (optional)		MP: 118/50 HP: 120/60 UP: 129/70
 CIC non-wireless 10A battery	- Cabled fitting required - Receiver options: LP, MP and HP - IP68 certified - Optional push button		LP: 114/40 MP: 116/50 HP: 121/60

Receiver, thin tubes and domes overview

	Hearing aids	Receivers and thin tubes	Domes
RIE Styles	Beltone Envision, Beltone Commence, Beltone Serene and Beltone Achieve  Beltone Envision and Beltone Serene microRIE 62S Rechargeable Beltone Commence miniRIE 62 Rechargeable Beltone Achieve miniRIE 62 Rechargeable  RIE 63 312 battery RIE 64 13 battery	SureFit 3 Receivers Available in 5 lengths: 0 to 4  MM* (M&RIE) LP MP HP UP (encased) * MM is only available for Beltone Envision, Beltone Serene & Beltone Achieve	SureFit 3 Domes*  Open (S / M / L)  Closed (S / M / L)  Power (S / M / L)  Tulip
	Beltone Rely  RIE 63 Rechargeable RIE 63 312 battery RIE 64 13 battery	SureFit 2C Receivers Available in 5 lengths: 0 to 4  LP MP HP UP (encased)	
BTE Styles	Beltone Serene and Beltone Achieve  BTE 76 Rechargeable	SureFit 3 thin tubes Available in 5 lengths: 0 to 4 	
	Beltone Amaze and Beltone Rely  BTE 66 BTE 76	SureFit 2C thin tubes Available in 5 lengths: -1 to 3 	*SureFit 3 Domes can be used for SureFit2C receivers and thin tubes



Enjoy the Beltone Solus Max™ 2.1 experience:

- 74% cleaner look for a more intuitive experience¹
- Compare and demo different tech-levels²
- Automated preferences based on your way of working



¹ Beltone Solus Max 2.0 compared to Beltone Solus Max 1.17, technical data on file
² Beltone Solus Max 2.1



Made for Auracast



With Auracast broadcast audio, you can let your patients experience a new world of connectivity in public and at home.



Auracast Assistant

With the world's first Auracast Assistant for hearing aids*, every phone that is compatible with the latest version of the Beltone HearMax™ app can connect Auracast-compatible hearing aids to streams at train stations, airports, museums, places of worship, and more.



Multi-Mic+

Give your patients extra support in situations where they struggle to hear – over distance or in environments with unfavourable acoustics. Offering a 16 dB SNR improvement¹.



TV-Streamer+

The TV-Streamer+ gives an improvement of 56% in speech recognition when listening to the TV compared to listening via the TV speakers¹. The TV-Streamer+ connects easily to help your patients listen at the volume they prefer and can transmit an Auracast stream.



Connectivity and accessories

Smartphone app and Remote Control

Empower your patients to take control



Beltone HearMax app



Beltone
Remote Control

Beltone HearMax app

Access to settings at all times
using a mobile device

Beltone Remote Control

Control at their fingertips without
a smartphone

Phone calls

Hands-free calls and streamed audio



Bluetooth Low Energy Audio

Connecting hands-free to Bluetooth
5.3 compatible devices*

Hands-free calls for iPhone and iPad

Connecting hands-free to iPhone and
iPad devices**

Direct audio streaming

From iPhone, iPad or Android devices

Multi-Mic+

Help your patients hear through the noise



Beltone
Multi-Mic+

Beltone Multi-Mic+

Cut through the noise now with the
world's first remote mic with Auracast*

TV

Improve TV time for your patients



Beltone
TV-Streamer+



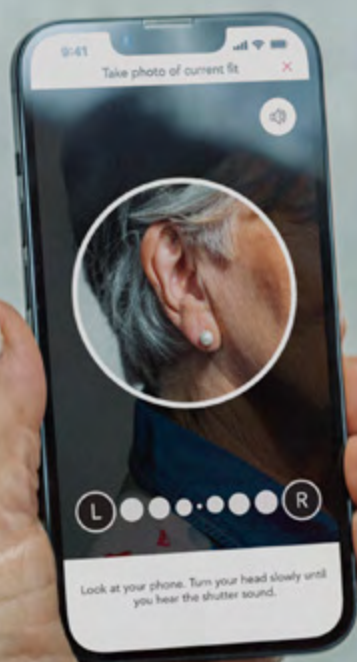
Beltone
TV Streamer 2

Beltone TV-Streamer+

For immediate TV speech clarity at
their preferred volume. Supports
Auracast broadcast audio*

Beltone TV Link 2

Enjoy crystal clear TV sound



Help your patients with correct hearing aid wear

Check My Fit helps users to wear their hearing aids properly and get the most out of their Beltone style each day, which can lead to fewer service visits. Wearing hearing aids correctly can deliver +11 dB more feedback-free gain¹. Using Check My Fit in the Beltone HearMax app, 70% of users wore their hearing aids better².

70%

users wore their hearing aids better²

+11 dB

more feedback-free gain¹

¹ Yang et al (2022)

² Jespersen et al (2022)

Feature overview

Features	Description	
Sound Quality		
 WDRC bands	Replicates the way the ear divides sound into distinct pitches to create a natural and clear sound picture.	
 Environment recognition	Automatically adjusts the volume and the amount of noise reduction based on the environment for a comfortable hearing experience.	
 Directionality	Binaural hearing strategy uses real time information about the listening environment to switch between different sound settings automatically for better hearing with less effort in all daily life situations.	
 Speech intelligibility in noise	Forms a stronger directional pattern using the sound from both hearing aids, helping to hear through the noise to enjoy the best 1:1 hearing experience.	
 Noise reduction	Reduces disturbing background noise without affecting speech audibility and listening comfort even when moving through rapidly changing sound environments.	
 Comfort	Reduces low level noise in very noisy environments. Quickly detects and reduces amplification for sudden, brief loud noises and reduces wind noise without lowering gain for other sounds.	
 Feedback management	Reduces unwanted feedback in nearly any situation without compromising gain, while ensuring that music is correctly identified and not lowered.	
 Low frequency boost	Optimizes loudness for users with severe and profound hearing loss. Boosts the listening experience with a louder, fuller sound quality when needed.	
 Sound shifter	Improves audibility of speech cues that would otherwise be lost while maintaining best possible sound quality.	
Connectivity		
 Direct audio streaming	Enjoy the convenience of streaming audio directly from an iPhone, iPad or Android device in everyday situations like listening to music or taking a call.	
 Public connectivity	Participate in public gatherings via built-in Telecoil or Auracast broadcast audio.	
 Accessories	Wireless accessories to support in challenging listening situations in 1:1 or group settings. Provide audio streaming from devices including TVs and PCs.	
Fitting Flexibility		
 Wireless CROS	CROS (Contralateral Routing of Signals) and BiCROS (Bilateral Contralateral Routing of Signals) transmit sounds from the CROS device to the other hearing aid for significant improvements in sound audibility all around.	
 Programs	Preset hearing profiles that allows for greater control of your patients' hearing in different situations.	
 Fitting options	Synchronized SATISFY, Tinnitus Breaker Pro, Datalogging, In-situ Audiometry.	

Premium			Advanced			
Beltone Serene			Beltone Achieve			
SER17	SER9	SER6	IMP17	IMP9	IMP6	
17	14	12	17	14	12	
Smart Gain Pro	Smart Gain	Smart Gain	Smart Gain Pro	Smart Gain	Smart Gain	
CrossLink Directionality 4	CrossLink Directionality 3	CrossLink Directionality 2	CrossLink Directionality 4	CrossLink Directionality 3	CrossLink Directionality 2	
Ultra Focus 2	Ultra Focus	Adaptive Directionality	Ultra Focus 2	Ultra Focus	Adaptive Directionality	
5 settings	3 settings	2 settings	5 settings	3 settings	2 settings	
3 settings	3 settings	On / Off	3 settings	3 settings	On / Off	
Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	
3 settings	2 settings	On / Off	3 settings	2 settings	On / Off	
Yes			Yes			
Hands-free calls and direct streaming for iPhone and iPad Connecting hands-free to compatible Bluetooth 5.3 or later devices, including select Android phones			Hands-free calls and direct streaming for iPhone and iPad. Direct streaming for Android devices			
Supports Auracast for new broadcast opportunities, Telecoil built into select styles, as well as myPAL Pro and Multi-Mic+			Telecoil built into select styles and in myPAL Pro			
TV-Streamer+, TV Link 2, Multi-Mic+, myPAL Pro, myPAL Micro, Remote Control, Beltone HearMax App			TV Link 2, myPAL Pro, myPAL Micro, Remote Control, Beltone HearMax App			
(Only compatible with RIE and BTE styles)			N/A			
4			4			
Yes			Yes			

	Advanced			Essential		
	Beltone Amaze & Beltone Boost Ultra			Beltone Rely		
	AMZ17/BBU17	AMZ9/BBU9	AMZ6/BBU6	RLY4	RLY3	RLY2
	17	14	12	12	8	6
	Smart Gain Pro	Smart Gain				
	CrossLink Directionality 2	CrossLink Directionality	Spatial Directionality	Spatial Directionality	SmartBeam	Adaptive Directionality
	Adaptive Directionality	Adaptive Directionality	Adaptive Directionality	Adaptive Directionality	Adaptive Directionality	Adaptive Directionality
	5 settings	3 settings	2 settings	On / Off	On / Off	On / Off
	3 settings	3 settings	On / Off	On / Off	On / Off	
	Feedback Eraser	Feedback Eraser	Feedback Eraser	Feedback Eraser	Feedback Eraser	Feedback Eraser
	3 settings	2 settings	On / Off	On / Off	On / Off	On / Off
	Yes			N/A		
	Direct streaming for iPhone, iPad and Android devices			Direct streaming for iPhone, iPad and Android devices		
	Telecoil built into select styles and in myPAL Pro			Telecoil built into select styles and in myPAL Pro		
	TV Link 2, myPAL Pro, myPAL Micro, Remote Control, Beltone HearMax App			TV Link 2, myPAL Pro, myPAL Micro, Remote Control, Beltone HearMax App		
	4			4		
	Yes			Yes		

NEW

Premium

Beltone Envision

NEW

Essential

Beltone Commence

ENV17

ENV9

ENV6

ENV4

COM3

COM2

17

14

12

12

12

10

Smart Gain Pro

Smart Gain

Smart Gain

Smart Gain

CrossLink
Directionality 4CrossLink
Directionality 3CrossLink
Directionality 3CrossLink
Directionality 2

Speech Spotter

Speech Spotter

True Focus*/
Clear Focus

Ultra Focus 2

Ultra Focus

Smart Beam

Smart Beam

Adaptive Directionality

5 settings

3 settings

3 settings

2 settings

1 setting

1 setting

3 settings

3 settings

2 settings

2 settings

1 setting

Feedback Eraser 2
w/ Music ModeFeedback Eraser 2
w/ Music ModeFeedback Eraser 2
w/ Music ModeFeedback Eraser 2
w/ Music Mode

Feedback Eraser 2

Feedback Eraser 2

3 settings

3 settings

3 settings

3 settings

3 settings

3 settings

Yes

Hands-free calls and direct streaming for iPhone and iPad. Connecting hands-free to compatible
Bluetooth 5.3 or later devices, including select Android phones

Supports Auracast for new broadcast opportunities, Telecoil built into select styles, as well as Multi-Mic+

TV-Streamer+, Multi-Mic+, Remote Control & Beltone HearMax App

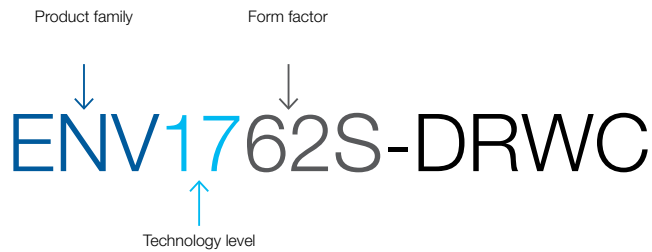
(Compatible with all RIE styles)

4

Yes

* True Focus is only available with the microRIE

Nomenclature



US Headquarters
Manufacturer According to FDA:
Beltone
8001 E Bloomington Fwy
Bloomington, MN 55420
U.S.A.
1-800-BELTONE
Beltone.com

Beltone product family

ENV* Envision
COM Commence
SER Serene
CSE Serene CROS transmitter
IMP Achieve
AMZ Amaze
BBU Boost Ultra
RLY Rely

Technology level

17 Premium level technology
9 Advanced level technology
6 Select level technology
4 Essential plus level technology
3 Essential level technology
2 Entry level technology

Form factors

Receiver-in-ear
62S microRIE (Lithium-ion)
62 miniRIE (Lithium-ion)
63 Standard RIE (312)
64 Standard RIE (13)

Customs

CIC Completely-in-Canal
ITC In-the-Canal
ITE In-the-Ear

Behind-the-ear

66 MiniBTE
76 Standard BTE
86 Power BTE
86 High Power BTE
95 Super Power BTE

Additional

D Dual microphones
R Receiver-in-Ear
W Wireless
C Rechargeable
T Telecoil
H High Power BTE
(with metal hook)

Receivers

LP Low Power
MP Medium Power
HP High Power
UP Ultra Power (encased)
MM M&RIE
(Microphone and receiver-in-ear)

Shells

FS Full-shell
HS Half-shell

* Available as microRIE CROS transmitter (ENV162S-DRWC)

Ready to experience Beltone?

Reach out to your sales rep for a demo or visit beltone.com to explore the portfolio and find out more.
We'll be right beside you every step of the way.

US Headquarters

Manufacturer According to FDA:
Beltone
8001 E Bloomington Fwy
Bloomington, MN 55420
U.S.A.
1-800-BELTONE
Beltone.com



© 2025 GN Hearing A/S. All rights reserved. Beltone is a trademark of GN Hearing A/S. Apple, the Apple logo, iPhone and iPad and are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The Auracast word mark and logos are trademarks owned by Bluetooth SIG, Inc.