

Grand Rounds by ReSound: Accessories – Something Old, Something New!

Valerie Kedem, Au.D.
Field Training Audiologist

Webinar FAQs

- How to ask a question:
 - Click Q&A, type your question and Send.
- Where to get the handout:
 - Resources in the classroom and under Pending Courses in your account
- How to view captions:
 - Click Live Transcript CC button
 - Click Show Subtitle
- How to earn CEUs:
 - After the webinar, go to your Pending Courses and pass the multiple-choice exam.
- For assistance or tech support:
 - Call 800-753-2160
 - customerservice@audiologyonline.com
 - Use the Q&A window

Learning Outcomes

1) After this course, participants will be able to identify ReSound wireless accessory options.

2) After this course, participants will be able to discuss purpose of each accessory.

- 3) After this course, participants will be able to compare/contrast new Multi-Mic+ & TV-Streamer+ with the Multi Mic and TV Streamer.



Risks/Benefits

- The applicability and accuracy of the course content may vary by the clinical, geographical, and cultural context; you should evaluate the course accordingly.
- As healthcare is rapidly evolving, new findings may change the validity of the information presented in this course. Providers should always consult the latest research and guidelines from professional associations.
- The interventions discussed in this course may be limited in generalizability, requiring practitioners to consider specific individual and population factors.
- The course may not cover all possible risk factors, diagnostic methods, or treatment options, and complex cases may require additional considerations.
- Healthcare providers should apply cultural competence and sensitivity to ensure effective and respectful care based on their patients' diverse needs.
- Providers should always evaluate the limitations of diagnostic and screening tools, considering their potential for false results as well as any ethical implications.
- It is the responsibility of course participants to critically evaluate the evidence from multiple sources to guide professional practice.



For further information please contact:

ReSound Customer Care at 1-800-248-4327

Additional resources:

Territory Sales Manager

Inside Sales Manager

Field Training Audiologist

Remote Training Audiologist


Accessories

ReSound GN



ReSound Phone Clip+

ReSound GN

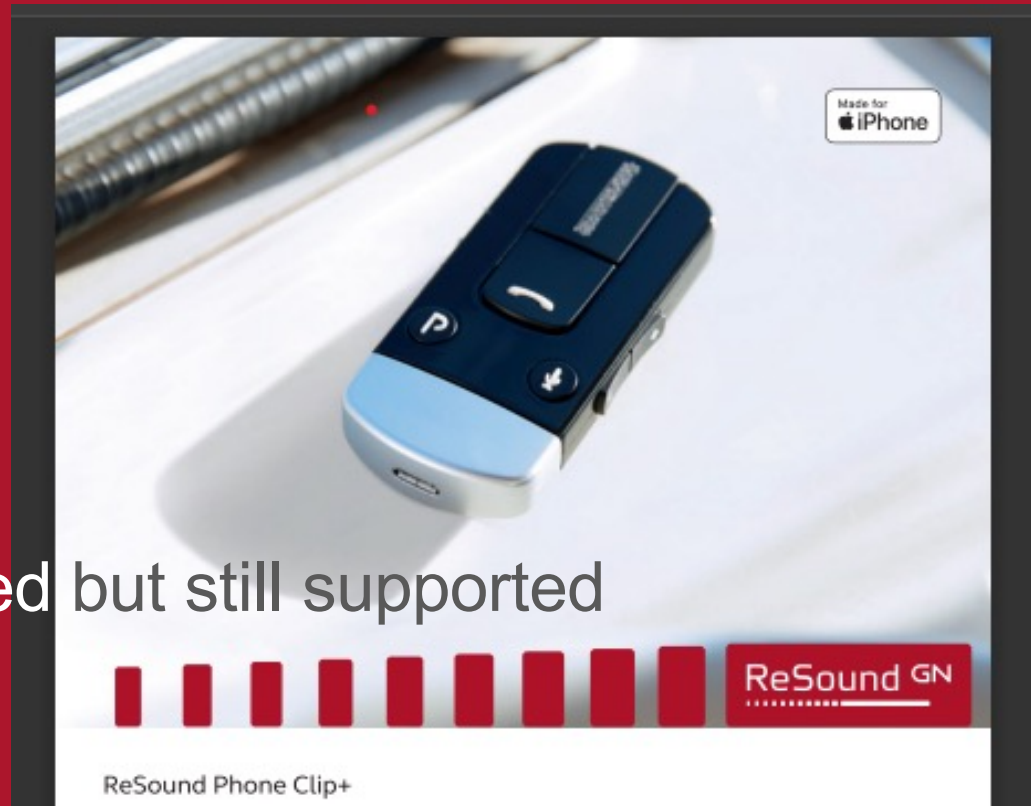

Discontinued but still supported

ReSound Remote Control 2



Discontinued but still supported

ReSound Remote Control

- ✓ Easily and discreetly allows patients to change programs and adjust volume on their hearing aids
- ✓ Features large, easy-to-use buttons for patients with dexterity concerns
- ✓ Ability to lock the device to avoid accidental button activation when put into a purse or pocket





ReSound Micro Mic

- ✓ Discreet clip-on microphone that streams the wearer's voice directly to your patient's hearing aids
- ✓ Significantly improves the signal-to-noise ratio in challenging listening situations for exceptional speech understanding
- ✓ Extends the hearing range by up to 80 feet in a clear line of sight
- ✓ Ability to adjust volume and mute the streamed sound
- ✓ Drop control automatically mutes the sound signal momentarily when dropped above 3 feet

ReSound Multi Mic

- ✓ Clip-on microphone that streams the wearer's voice directly to your patient's hearing aids
- ✓ Automatically switches into a table microphone when placed horizontally, so patients can hear voices from multiple speakers
- ✓ Significantly improves the signal-to-noise ratio in challenging listening situations for exceptional speech understanding
- ✓ Ability to connect with loop and FM systems, plus a mini-jack input to stream sound from virtually any source
- ✓ Extends the hearing range by up to 80 feet in a clear line of sight
- ✓ Drop control automatically mutes the sound signal momentarily when dropped above 3 feet




New Multi-Mic+

ReSound GN

World's first remote microphone with Auracast

New design for
ease-of-use

Longer battery life
up to 15 –20 hours

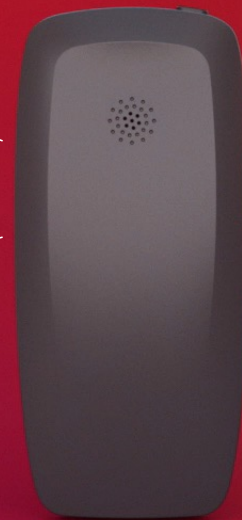

82 feet
streaming range

OneTouch pairing



Presenter mode

One-to-one/One-to-many when placed vertically



Table mode


To hear group conversations when placed horizontally



Audio mode

Connect to audio devices in private and public places

Auracast will transform assistive listening devices

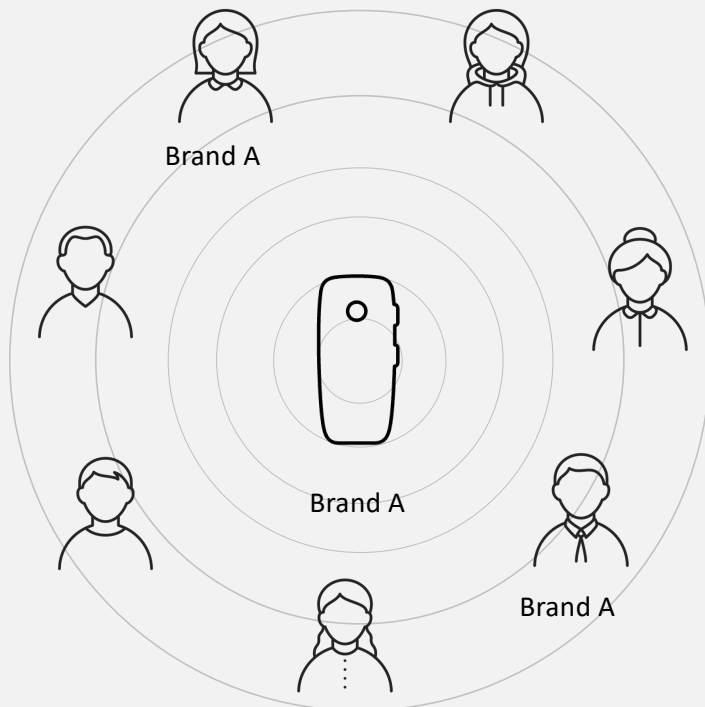

Traditional technology can only stream audio to hearing instruments of the same brand



Auracast will allow remote microphones to broadcast audio simultaneously to an unlimited number of nearby audio devices with Bluetooth LE Audio

Compare and contrast:

Where does the Multi Mic+ offer additional benefits over the Multi Mic?

Using Q&A, please write your thoughts.

Pairing Multi Mic



ReSound GN

1 No 1 Sec.

Pairing Accessories in Smart Fit

ReSound GN

ReSound Smart Fit 2.0

Home | File | View | Instrument | Fitting | Help

GN Online Services Login

Patient Information ^

Demo
Nexia
Experienced - Non-Linear

Instruments ^

Connect

Right
Connected

Left
Connected

NX9605-DRWC
RIE 2.1P Receiver

NX9605-DRWC
RIE 2.1P Receiver

Mute

Mute

Media Player v

Collapse Sidebar

ReSound GN

Patient | Instrument Set-Up | Gain Adjustments | Advanced Features | **Device Controls** | Finalize

Manual Controls | Standard Beeps | Accessories


Pair New Generation Accessory


Pair Previous Generation Accessory

START CONNECTING



Learn More

TV Streamers | Micro/ Multi/ Mini Mics | Phone Clip | Mobile Devices

Linked

TV - Streaming Bass Boost

Off | Mild | Moderate | Strong

Mic relative to TV Streamer

M=Access... | -3db | -6db | -9db | -12db | Mute

Off | Mild | Moderate | Strong

M=Access... | -3db | -6db | -9db | -12db | Mute

Save

17





When comparing the Multi Mic to the Multi Mic+:

What are some similarities with the pairing process?

How is the pairing process different?

What is a new use of the Multi Mic+ you could foresee?

ReSound TV Streamer 2

- ✓ Streams stereo sound from a TV, computer or music system directly to ReSound wireless hearing aids
- ✓ Reconnects automatically if the patient moves out of range and returns within 5 minutes
- ✓ Ability to adjust the balance between streamed sound and hearing aid volume
- ✓ Features a 23-foot range in a clear line of sight
- ✓ Works with both analog and digital audio devices





OneTouch Pairing

Press once and all hearing aids in proximity (<5 cm) will pair automatically


Volume Up

LED

Volume Down

Audio input

USB-C powered

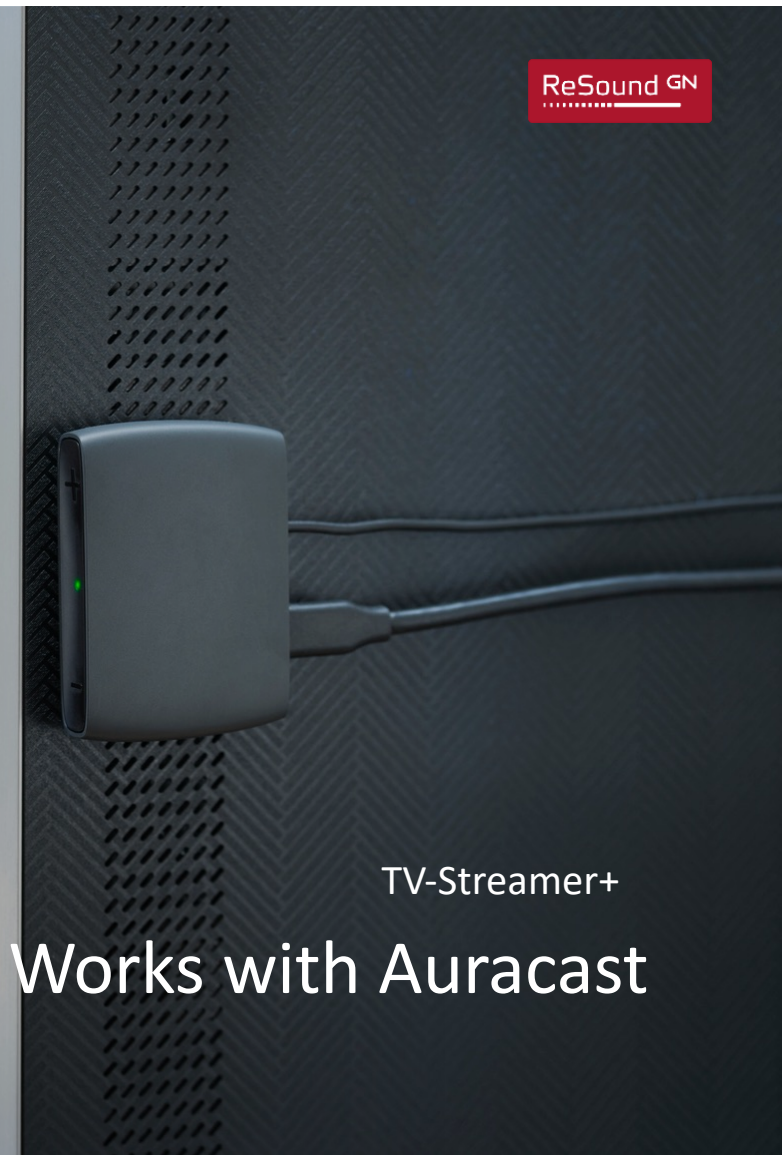

ReSound GN

TV-Streamer+

Works with Auracast



The ReSound GN logo is located in the top right corner. It consists of the brand name "ReSound" in a bold, sans-serif font, followed by "GN" in a smaller font. Below the brand name is a thin horizontal line.

ReSound GN

The background of the slide is a close-up photograph of a grey, curved electronic device, identified as the TV-Streamer+. The device has a small circular lens or sensor on its surface. The text is overlaid on this image.


How to pair TV-Streamer+ with ReSound Smart 3D™

The background of the entire image is a close-up, angled shot of a grey, rounded electronic device, identified as the TV-Streamer+. The device has a smooth, matte finish and a small circular port on its side. The lighting is soft, highlighting the contours of the device against a dark red background.

How to put your TV-Streamer+ in Auracast™ broadcast audio mode

New patient benefits

Describe a scenario in which the TV-Streamer+ can offer additional listening benefits to a patient and others in the household.

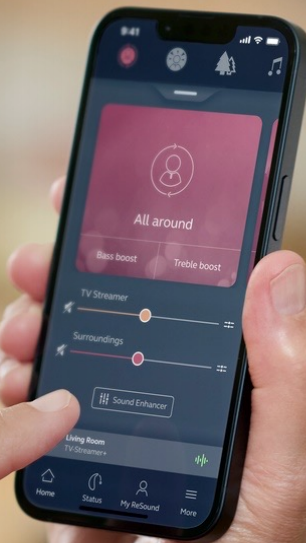

How many devices can be paired to ReSound Nexia?

Electronic devices

Via Bluetooth LE Audio, MFi and ASHA



Electronic devices

TV-Streamer+

Up to 8 devices

TV Streamer/Multi-mic

Previous generation of wireless accessories



Up to 3 in total

Phone Clip

Connects to phones with Classic Bluetooth®



1 unit

Remote Control



1 unit

Tips for pairing

OneTouch! No need to reboot the hearing aids.

Make a VERY short press on the pairing button.

Ensure the hearing aids are close by the accessory.

Look for the solid light to confirm pairing.

Accessory Recap

8 accessories in Total

6 accessories are backwards and forwards compatible

- Multi Mic
- Micro Mic
- TV Streamer
- Remote Control
- Phone Clip +: Discontinued
- Remote Control 2: Discontinued

2 accessories that are only compatible with NEXIA.

- TV Streamer +
- Multi Mic +

Thank you!

ReSound GN

