

Charging options for ReSound hearing aids

ReSound GN

How to match rechargeable hearing aids with charger

Each charger type is engineered to hold the hearing aid in the optimal position for charging reliability and efficiency. The charging form factor (CFF) relates to a specific ReSound hearing aid collection or style.

The CFF is indicated with a number (see chart below). The number is shown inside the charging bay and on the charger packaging*.

*CFF numbers 1 and 3 are not shown on the packaging. Refer only to the number inside the charging bay.



ReSound Vivia™ and ReSound Nexia™ Micro RIE 60S	8 22978800	8 23366400	
ReSound OMNIA™ Mini RIE 60	06 22405700	06 23366300	-
ReSound OMNIA™ and ReSound ONE™ RIE 61	1 21569800	1 23366200	-
ReSound OMNIA™ and ReSound ONE™ BTE	-	70 22816900	-
ReSound Nexia™, ReSound OMNIA™ and ReSound ONE™ Power/High Power BTE 88	-	80 22817000	-
ReSound Nexia™ Customs, ReSound OMNIA™ Customs and Customs by ReSound	-	-	C 22465400

Custom charger includes custom-made inserts. **We recommend reminding patients to bring their custom-made charger along with their custom hearing aids to all follow-up appointments.

Charger features



	ReSound Viva, ReSound Nexia & ReSound OMNIA			
	Premium charger	Standard charger	Desktop charger	Custom charger
	microRIE/miniRIE/RIE		microRIE/miniRIE/RIE BTE/Power BTE	ITE/ITC
Inductive charging	•	•	•	•
Portable with onboard power bank	•			
Portable without power bank		•		•
USB-C connector and cable	•	•	•	•
LED on the back	Shows the power bank battery status	Shows when the charger is connected to a power supply	Shows when the charger is connected to a power supply	Shows when the charger is connected to a power supply
LED on the front	Shows battery level of the hearing aids*			Shows the hearing aids charging

*LED lights are visible when lid is open

How to tell if the hearing aids are charging

Place the hearing aids in the charger and let the light indicator on the hearing aids guide you. For rechargeable Customs, the LED on the front of the charger will be your guidance.

You know that charging has started when the hearing aid lights blink slowly.

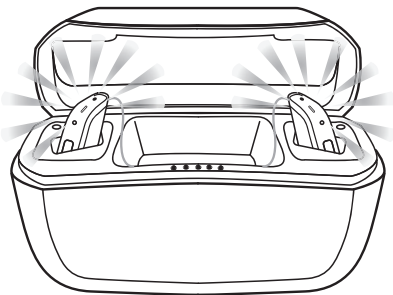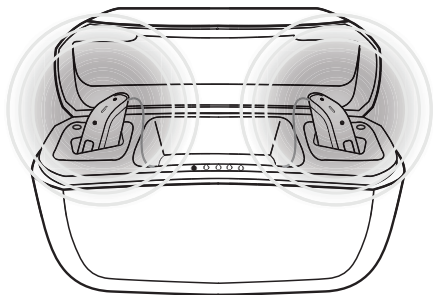

When fully charged, the hearing aid lights remain constant, not blinking.

The lights turn off when the hearing aids are removed from the charger.

Premium charger illustrated. The process is the same for the Desktop charger. For more information, please see the individual user guide for each charger.

Manufacturer according to FDA:

ReSound Government Services
 8001 E Bloomington Freeway
 Bloomington, MN 55420
 USA
 1-800-392-9932
 gs.resound.com