



Signia Government Services May 2025 Connexx Fitting Software Tour

Lisa A. Perhacs, Au.D.

Government Services May 2025 Connexx Fitting Software Tour

Lisa A, Perhacs, Au.D.



Webinar FAQs

- **Where to get the handout:**
 - Resources in the classroom and under Pending Courses in your account
- **How to earn CEUs:**
 - After the webinar, go to your Pending Courses and pass the multiple-choice exam.
 - Must have a paid membership to earn CEUs
- **For assistance or tech support:**
 - Call 800-753-2160
 - customerservice@audiologyonline.com

Learning Outcomes

Upon completion of this course attendees will be able to:

- 1) Complete the simple first fit process in the Connexx fitting software.
- 2) Discuss how the flexibility of the Dynamic Soundscape Processing 3.0 can enrich the Real Time Conversation Enhancement adjustments.
- 3) List the steps and benefits of running Own Voice Processing 2.0.



Risks/Benefits

- The applicability and accuracy of the course content may vary by the clinical, geographical, and cultural context; you should evaluate the course accordingly.
- As healthcare is rapidly evolving, new findings may change the validity of the information presented in this course. Providers should always consult the latest research and guidelines from professional associations.
- The interventions discussed in this course may be limited in generalizability, requiring practitioners to consider specific individual and population factors.
- The course may not cover all possible risk factors, diagnostic methods, or treatment options, and complex cases may require additional considerations.
- Healthcare providers should apply cultural competence and sensitivity to ensure effective and respectful care based on their patients' diverse needs.
- Providers should always evaluate the limitations of diagnostic and screening tools, considering their potential for false results as well as any ethical implications.
- It is the responsibility of course participants to critically evaluate the evidence from multiple sources to guide professional practice.

Signia BCT IX – THE PERFECT CHOICE

- **Bluetooth® Classic** - provides universal connectivity
- **T-coil** – better hearing in challenging environments
- **Exceptional battery life** – 36hr with 5 hrs streaming = All day wear time
- **LED light** – confirms power on status
- **Serial number viewing window**
– easy identification
- **Optional on-the-go charging case**
– portability & convenience
- **Full audiological power of IX** – keeps you in the conversation




GS Signia Portfolio



Connexx Agenda


- Easy First Fit
- OVP
- Dynamic Soundscape Processing (DSP)
- Sound Settings
- Audio Streaming
- Service Tab
- Connect/Disconnect button
- SCIF compliance
- Acclimatization manager
- Tinnitus

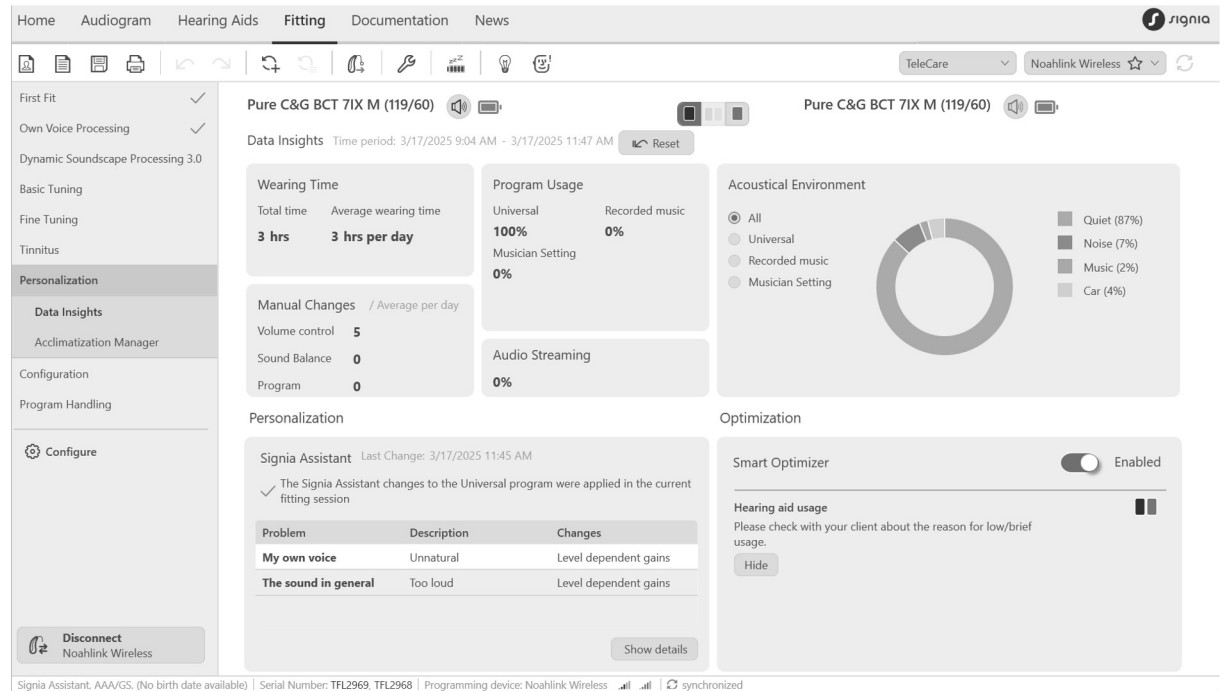

Connexx 9.13 – May 2025



Connexx 9.13 – May 2025

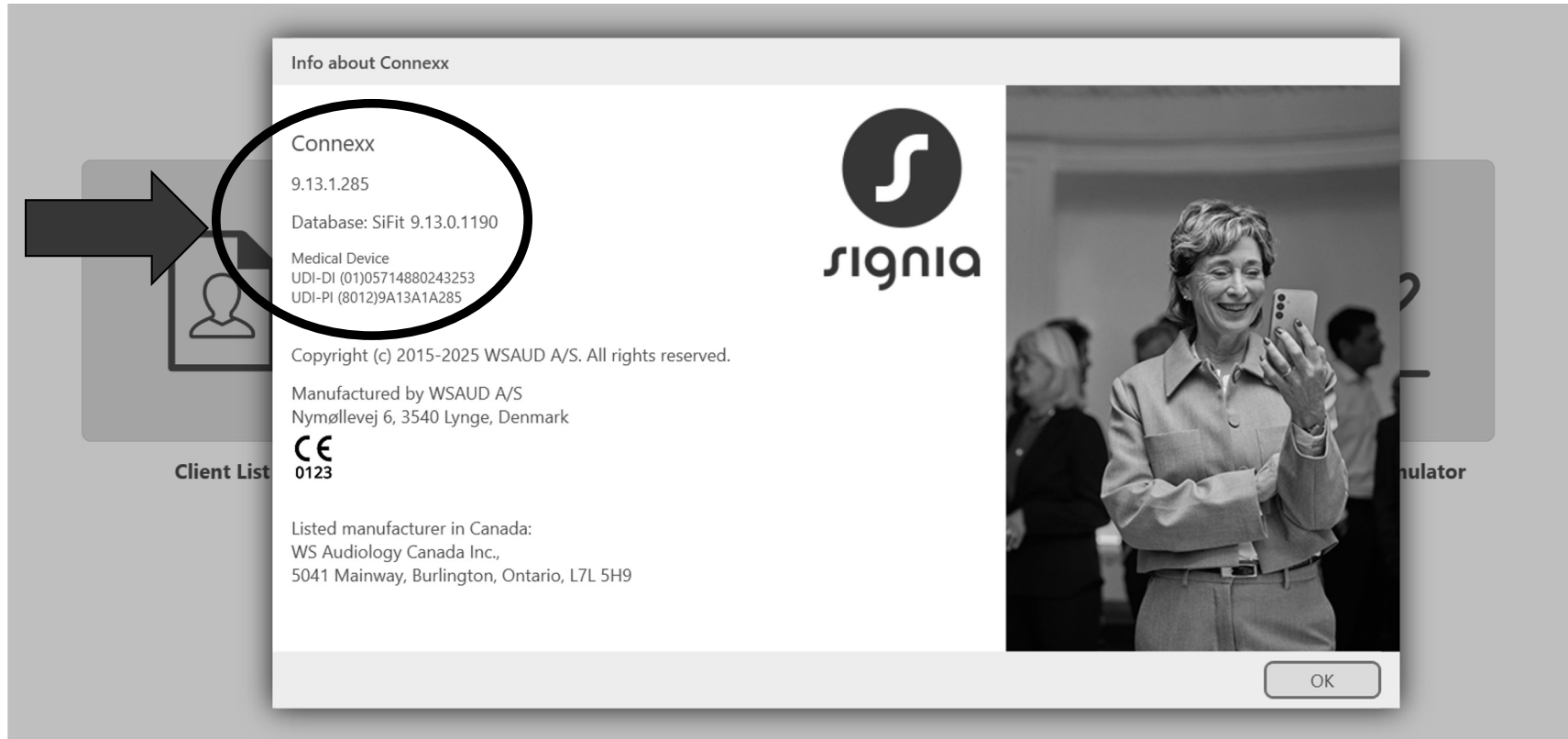



Thank You

veterans.signiausa.com
(800) 955-4327