


Genie 2 Fitting Focus Series:

Oticon Jet PX First Fit

Trenton England, Au.D.
Regional Audiologist

oticon
life-changing technology

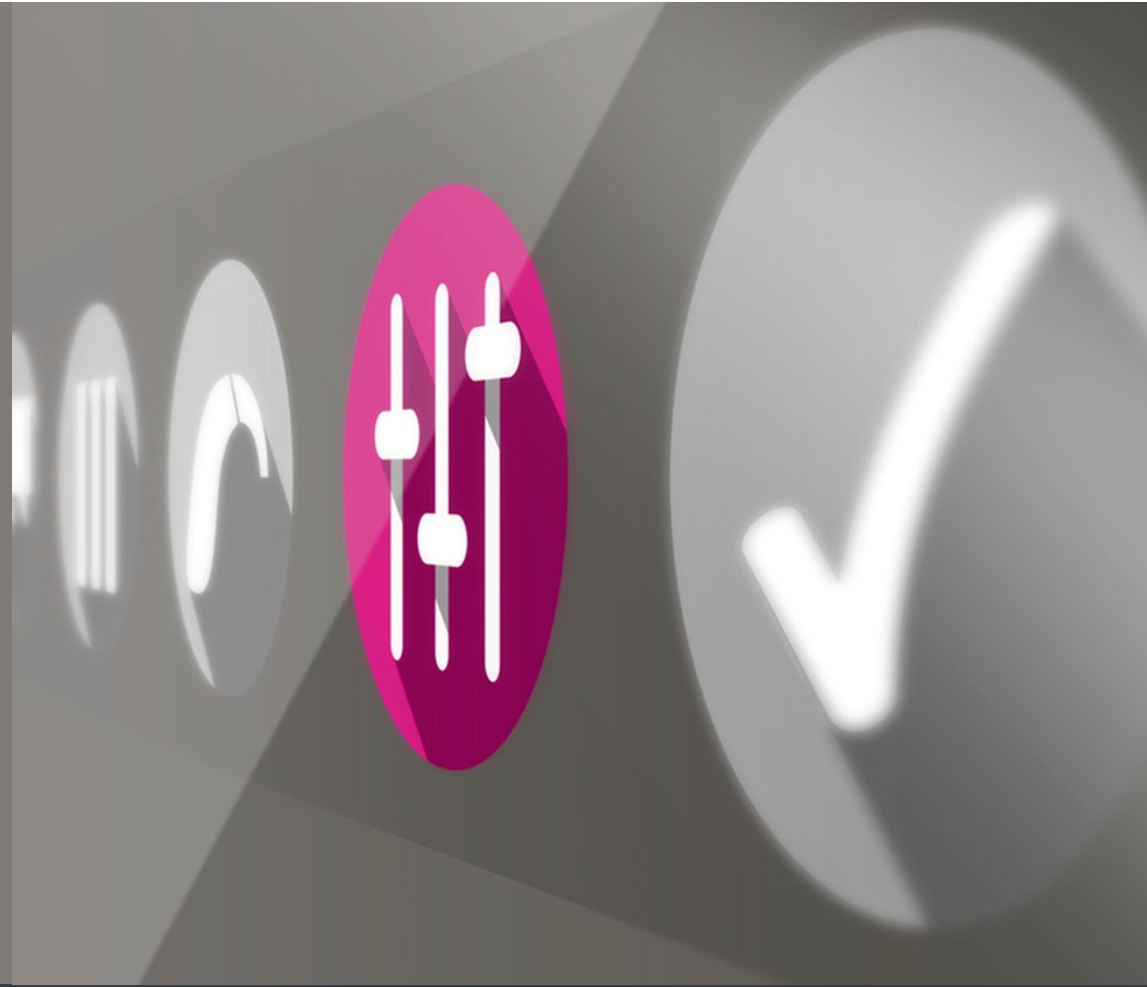

Learner Outcomes:


After this course, participants will be able to:

- . . . explain the process of pairing and connecting Oticon Jet PX instruments using Genie 2 software.
- . . . list the required initial fitting steps for Oticon Jet PX.
- . . . find the OpenSound Navigator tool in the Genie 2 fit screen and describe the controls available.

Risks / benefits

- The applicability and accuracy of the course content may vary by the clinical, geographical, and cultural context; you should evaluate the course accordingly.
- As healthcare is rapidly evolving, new findings may change the validity of the information presented in this course. Providers should always consult the latest research and guidelines from professional associations.
- The interventions discussed in this course may be limited in generalizability, requiring practitioners to consider specific individual and population factors.
- The course may not cover all possible risk factors, diagnostic methods, or treatment options, and complex cases may require additional considerations.
- Healthcare providers should apply cultural competence and sensitivity to ensure effective and respectful care based on their patients' diverse needs.
- Providers should always evaluate the limitations of diagnostic and screening tools, considering their potential for false results as well as any ethical implications.
- It is the responsibility of course participants to critically evaluate the evidence from multiple sources to guide professional practice.

Available in four styles and five colors

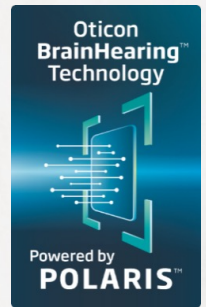

miniRITE R

miniRITE T

miniBTE R

miniBTE T



C091
Silver Grey



C092
Steel Grey



C063
Diamond Black



C093
Chestnut Brown



C090
Chroma Beige

Fitting Oticon Jet PX

Programming devices



Noahlink Wireless/Noahlink Wireless 2

are the only programming devices for programming and firmware updates

First steps

Attach speaker units to hearing instruments

"Wake up" hearing aids
to deactivate battery protection mode



Wireless programming
with Noahlink Wireless



Rechargeability



SmartCharger



Desktop Charger

Genie 2 Fitting Focus Series:

Oticon Jet PX First Fit

Trenton England, Au.D.
Regional Audiologist

oticon
life-changing technology