


Genie 2 Fitting Focus Series:

Oticon Own SI first fit

Keri Salvatore, Au.D.
Regional Audiologist

oticon
life-changing technology

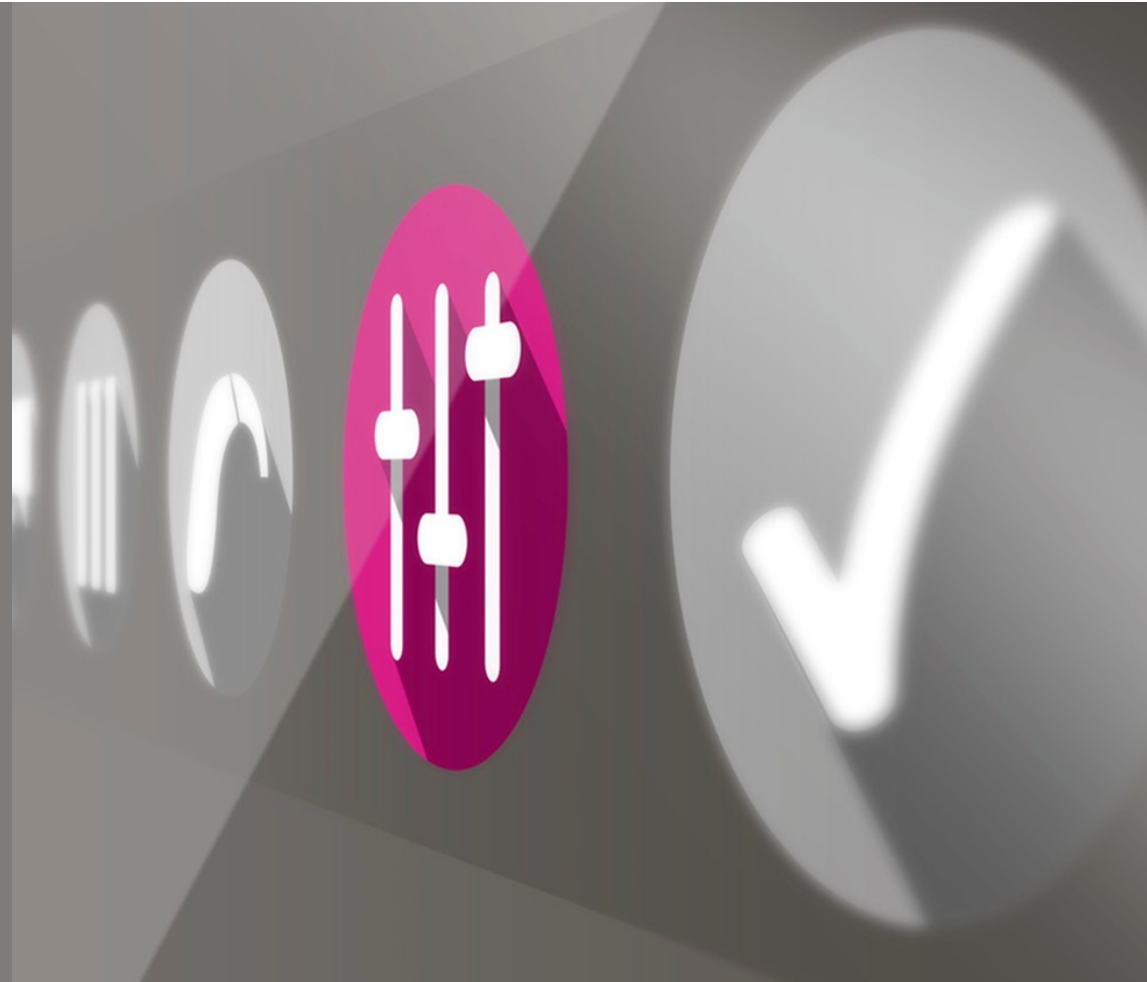

Learner Outcomes:

After this course, participants will be able to:

- . . . explain the process of pairing and connecting Oticon Own SI instruments using the Genie 2 fitting software.
- . . . list the required initial fitting steps for Oticon Own SI.
- . . . find the MoreSound Intelligence and Sudden Sound Stabilizer tools in the Genie 2 fit screen and describe the controls available on these screens.

Risks / benefits


- The applicability and accuracy of the course content may vary by the clinical, geographical, and cultural context; you should evaluate the course accordingly.
- As healthcare is rapidly evolving, new findings may change the validity of the information presented in this course. Providers should always consult the latest research and guidelines from professional associations.
- The interventions discussed in this course may be limited in generalizability, requiring practitioners to consider specific individual and population factors.
- The course may not cover all possible risk factors, diagnostic methods, or treatment options, and complex cases may require additional considerations.
- Healthcare providers should apply cultural competence and sensitivity to ensure effective and respectful care based on their patients' diverse needs.
- Providers should always evaluate the limitations of diagnostic and screening tools, considering their potential for false results as well as any ethical implications.
- It is the responsibility of course participants to critically evaluate the evidence from multiple sources to guide professional practice.

Two discreet styles

Made to match individual preferences




C001
Beige

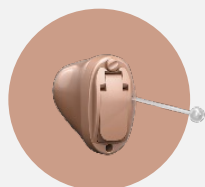


C002
Light Brown




C003
Medium Brown

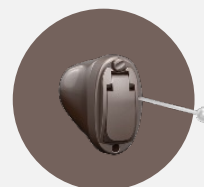


C004
Dark Brown

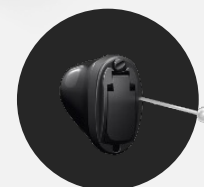


C005
Black

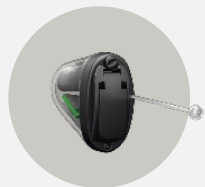

Transparent

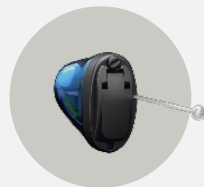

Transparent
Blue

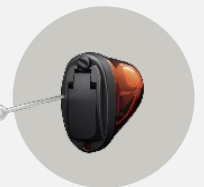

Transparent
Red

Programming devices

Wireless or wired



Noahlink Wireless 2





HiPro®




ExpressLink

Programming Adapter Mini

IIC and CLC

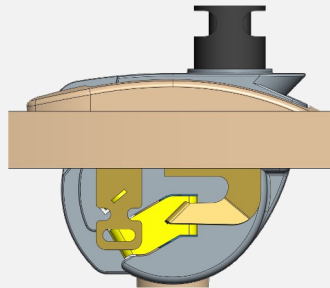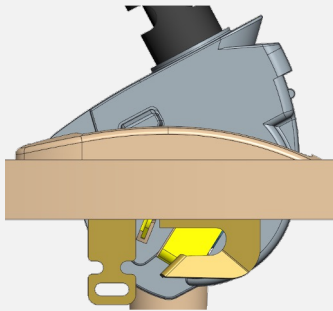

oticon

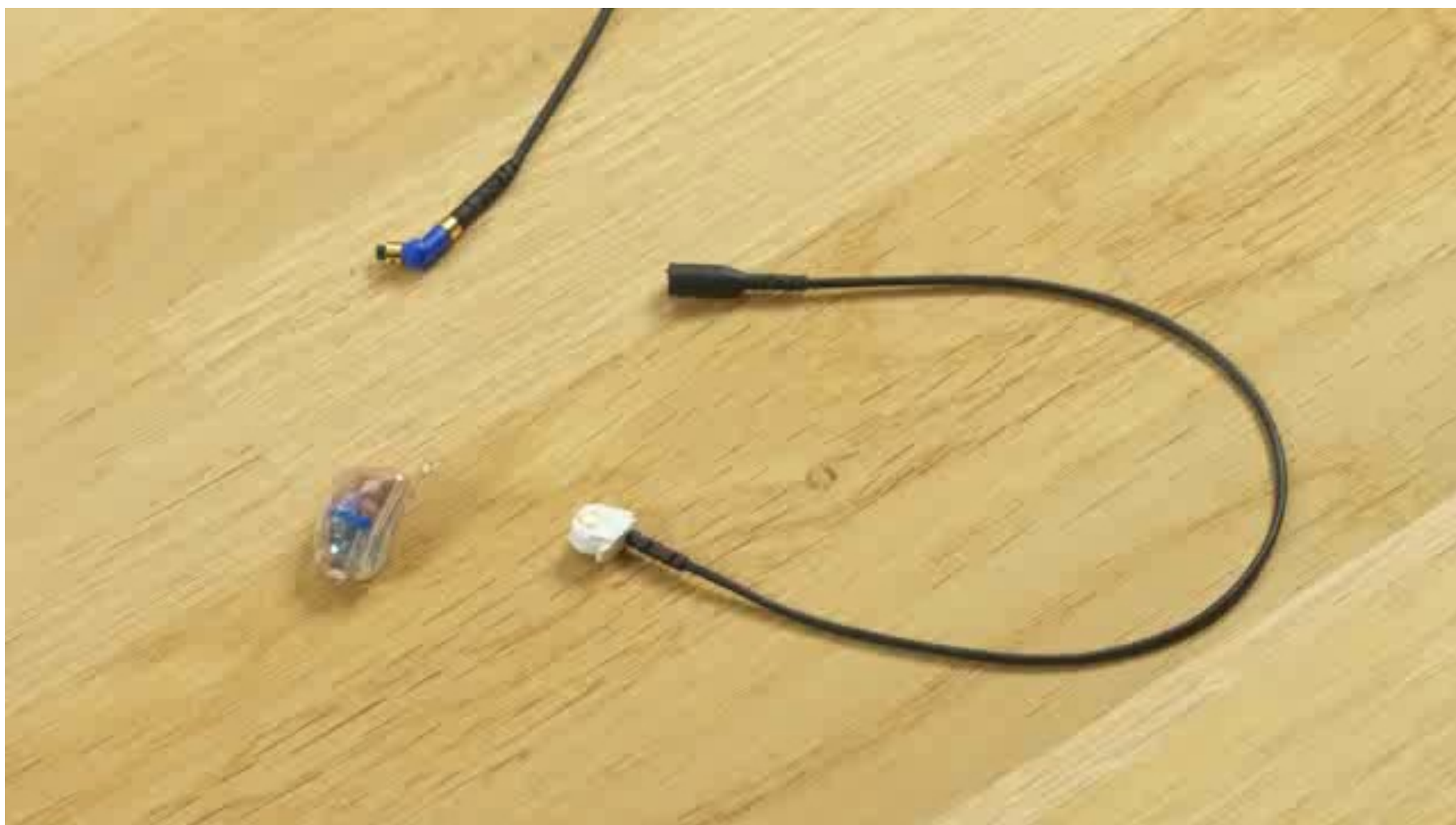

Genie 2 Fitting Focus Series:

Oticon Own SI first fit

Keri Salvatore, Au.D.
Regional Audiologist

oticon
life-changing technology

