

1. This document is created for accessibility and is intended to be a supplemental resource. If you would like to print a hard copy of this document, please follow the general instructions below to print multiple slides on a single page or in black and white.
2. This handout is for reference only. Non-essential images have been removed for your convenience. Any links included in the handout are current at the time of the live webinar, but are subject to change and may not be current at a later date.
3. Copyright: Images used in this course are used in compliance with copyright laws and where required, permission has been secured to use the images in this course. All use of these images outside of this course may be in violation of copyright laws and is strictly prohibited.
4. Social Workers: For additional information regarding standards and indicators for cultural competence, please review the NASW resource: Standards and Indicators for [Cultural Competence in Social Work Practice](#)
5. Need Help? Select the “Help” option in the member dashboard to access FAQs or contact us.

How to print Handouts

On a Mac

- Open PDF in Preview
- Click File
- Click Print
- Click dropdown menu on the right “preview”
- Click layout
- Choose # of pages per sheet from dropdown menu
- Checkmark Black & White if wanted.

On a PC

- Open PDF
- Click Print
- Choose # of pages per sheet from dropdown menu
- Choose Black and White from “Color” dropdown

No part of the materials available through the continued.com site may be copied, photocopied, reproduced, translated or reduced to any electronic medium or machine-readable form, in whole or in part, without prior written consent of continued.com, LLC. Any other reproduction in any form without such written permission is prohibited. All materials contained on this site are protected by United States copyright law and may not be reproduced, distributed, transmitted, displayed, published or broadcast without the prior written permission of continued.com, LLC. Users must not access or use for any commercial purposes any part of the site or any services or materials available through the site.



ReSound Enzo IA: More Sound, More Power

Tammara Stender, Au.D.

Director of External Relations, Global Audiology
GN Hearing

Webinar FAQs

- How to ask a question:
 - Use the Q&A
- Resources Tab:
 - Download the course handout
 - Link to course information page
 - View presenter information
- How to view captions:
 - Click the CC button to show/hide captions
- How to earn CEUs:
 - After the webinar, go to your Pending Courses and pass the multiple-choice exam
- For assistance or tech support:
 - Call 800-753-2160
 - customerexperience@continued.com
 - Use the Q&A window

Disclosures

- **Presenter Disclosure:** Financial: Presenter is employed by GN Hearing. Non-financial: Presenter has no relevant non-financial relationships to disclose.
- **Content Disclosure:** This learning focuses exclusively on a specific product or service.
- **Sponsor Disclosure:** GN Hearing is the external sponsor for this course.

Limitations/Risks

- The applicability and accuracy of the course content may vary by the clinical, geographical, and cultural context; you should evaluate the course accordingly.
- As healthcare is rapidly evolving, new findings may change the validity of the information presented in this course. Providers should always consult the latest research and guidelines from professional associations.
- The interventions discussed in this course may be limited in generalizability, requiring practitioners to consider specific individual and population factors.
- The course may not cover all possible risk factors, diagnostic methods, or treatment options, and complex cases may require additional considerations.
- Healthcare providers should apply cultural competence and sensitivity to ensure effective and respectful care based on their patients' diverse needs.
- Providers should always evaluate the limitations of diagnostic and screening tools, considering their potential for false results as well as any ethical implications.
- It is the responsibility of course participants to critically evaluate the evidence from multiple sources to guide professional practice.

Learning Outcomes

After this course, participants will be able to:

- List the hardware and design improvements in ReSound Enzo IA.
- Identify the technology improvements in ReSound Enzo IA.
- Explain the patient benefits of rechargeability and connectivity.

More hearing in noise

To empower people
with severe to
profound hearing loss



Living with severe to profound hearing loss

Coping to connect and communicate every day

ReSound GN

11%

Of hearing loss cases are severe to profound¹

Up to **69%**

Adoption rate for severe to profound hearing loss globally²

All ages

No other hearing loss spans as many ages²

Lowest
quality of life score

The more severe the hearing loss, the greater the loss in quality of life²

1. WHO – World Report on Hearing (2021), 2. Hear It – Key Facts on Hearing Loss (2022)

Self-aware and resilient

A community that refuses to be overlooked

ReSound GN

"It's not that the hearing aids I use help me hear better; they help me hear," – James

"[...] I'm not going to hear the same sounds as other people, but I hear them differently," – Helen

"I tend to be positive because it makes me unique," – Helen

"People say to me now that I'm not the person I was when I first lost my hearing," – Sophie


"Because I have such a severe hearing loss, the tiniest improvement in sound or clarity, [...] is a big thing for me," – James

Quotes from interviews with users with severe-to-profound hearing loss, GN Hearing Interviews (2025)

Most important factors when upgrading hearing aids

ReSound GN

Every upgrade makes a difference

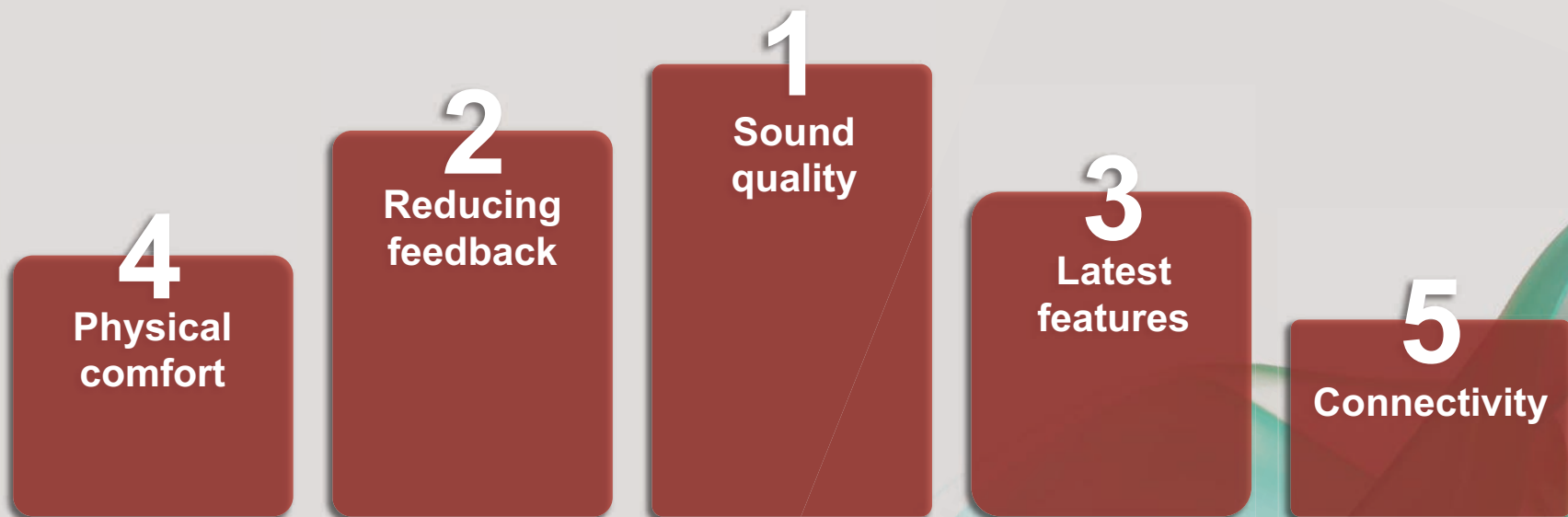

MarkeTrak 22 (2022) & Insights emphasized by users with severe-to-profound hearing loss in interviews conducted by GN Hearing (2025)

Sound quality and reducing feedback

Gain without compromise

ReSound GN



Hearing speech in noise is the biggest challenge - determines whether users stay engaged or withdraw from the situation¹⁻²



Users need powerful amplification and cannot accept any compromise on gain¹⁻²



Feedback is still a challenge - especially when it comes with restrictions to gain¹⁻²

*"Getting the confidence to go where I felt I couldn't go or didn't enjoy going before"*³

- James

1. MarkeTrak 22 (2022), 3. Quote from user with severe-to-profound hearing loss, GN Hearing Interviews (2025)
2. Interviews with users with severe-to-profound hearing loss conducted by GN Hearing (2025),

Latest features

Power that lasts the day

ReSound GN



Rechargeability means freedom, removing the hassle of carrying batteries²

91%

Of users rate their hearing aids easier to use with rechargeability¹

90%

Of those with rechargeable option are happy with it¹

35%

Increase in binaural rates with rechargeability¹

1. MarkeTrak 22 (2022); 2. Interviews with users with severe-to-profound hearing loss conducted by GN Hearing (2025)

Physical comfort

Comfort is what counts

ReSound GN



Rely on their hearing aid every day to support their hearing¹⁻²



Size is not everything, but wearing comfort is¹⁻²

#1

Top reason for not getting hearing aids is comfort³

1. MarkeTrak 22 (2022), 2. Interviews with users with severe-to-profound hearing loss conducted by GN Hearing (2025), 3. EuroTrak UK (2022)

Connectivity

Connected and included



Ability to stream calls is **crucial** when communicating via phone¹⁻²



Improvements to streamed sound quality and stable connections is most important¹



Auracast is known in the severe to profound hearing community²



People with severe to profound hearing loss use their accessories on a **daily basis**¹⁻²

ReSound GN


"To be able to enjoy a concert experience is something I have dreamt of"³

1. MarkeTrak 22 (2022), 2. Interviews with users with severe-to-profound hearing loss conducted by GN Hearing (2025), 3. Quote from hearing aid user at Sydney Opera House event

Most important factors when upgrading hearing aids



Built around what people actually need

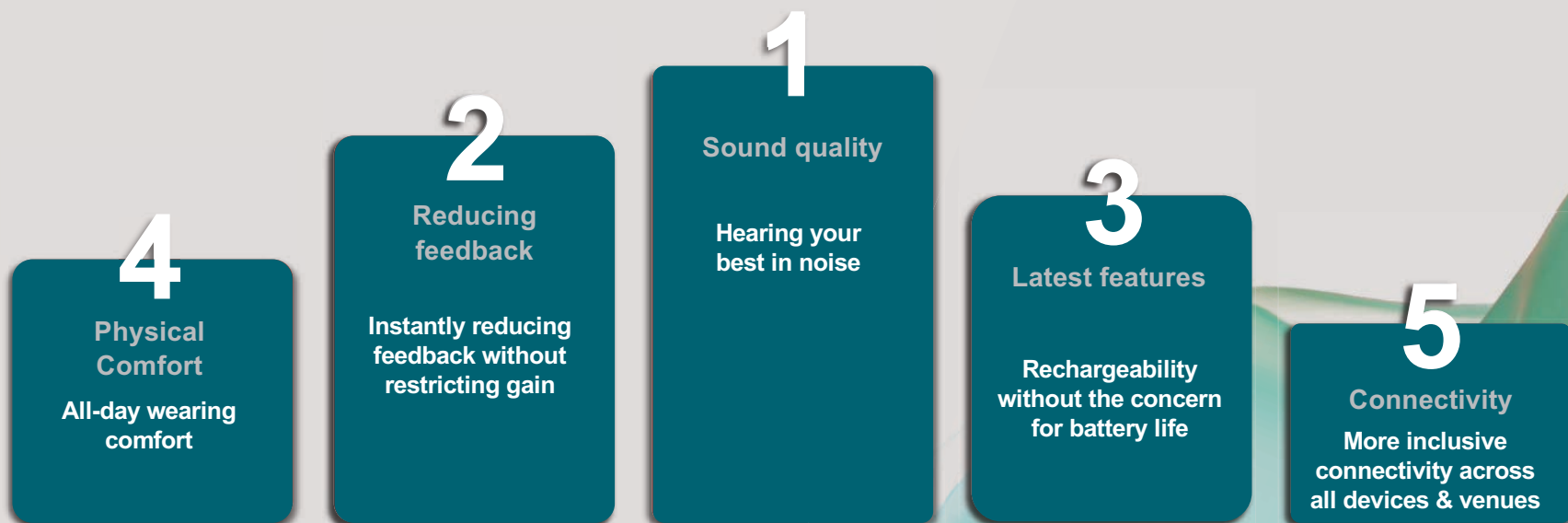

ReSound GN

ReSound Enzo IA

**More sound. More power.
Now in the smallest rechargeable
Super Power hearing aids.**



Made for hearing in noise

Enjoy real gain while spotlighting speech and eliminating distraction – without cutting you off from the world¹

World's best feedback management

Get the fastest elimination of feedback without compromising on gain²

World's smallest rechargeable Super Power³

with all-day comfort and all-day battery even when streaming³

World first Super Power with Bluetooth LE Audio & Auracast³

Clearest Bluetooth streaming at the volume you prefer in every situation with TV-Streamer+ & Multi-Mic+⁴

1. Jespersen & Skovlund (2025), Groth et al (2023) & Jespersen et al. (2024); 2. Cui & Groth (2025);
3. GN Proprietary data on file (2025); 4. Jespersen & Rubachandran (2025)

Organic hearing

Sounds natural

Emulating the natural hearing process and the natural ways we listen as closely as possible, helping the brain to hear as nature intended



Intelligence Augmented
- a human approach to AI

How we use AI across
ALL our solutions

Feels natural

Adapts organically to your patients
for best wearing comfort all day long
and ease of use

ReSound GN

Connects naturally

Inspired by the way we intuitively
interact with people and devices in
our daily lives

Intelligence Augmented

How we use AI across all our solutions

360 Chip

Machine learning

Allows computers to make predictions or recommendations by learning data over time



Environmental
Classifier



Feedback
Manager

Deep learning

Advanced type of machine learning that analyzes large, complex amounts of data in ways that are inspired by the human brain



Check My Fit

ReSound Enzo IA: Made for hearing in noise

ReSound GN




World's smallest rechargeable
Super Power hearing aid¹



Fast charging



All-day battery⁴

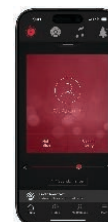

World's first
Super Power
hearing aids
with Bluetooth
LE Audio &
Auracast⁴



All-weatherproof
IP68 rated

 **AURACAST**

93%

preference for hearing
in noise with Clear
Focus over legacy²

replaceable
mic filters



Multi-Mic+
TV-Streamer+



Clear Focus

New premium colors



World's best
feedback
manager³



1. Compared to competing Super Power solutions. GN proprietary data on file (2025); 2. Groth & Cui (2025); 3. Jespersen & Skovlund (2025); 4. GN proprietary data on file (2025)

More hearing in noise

Giving access to the
sounds that matter



ReSound Enzo IA

More sound

**Industry's most accurate
real-time environmental
classification¹**



Environmental Classifier

For clear and comfortable
hearing that automatically
adapts to changing
environments.

**Industry-leading access
to surrounding sounds²**



360 All-Around

Better hearing all-
around with less effort
in daily situations and
noisy environments.

**Industry's most narrow
beamformer³**



Clear Focus

Spotlight speech and
eliminate distractions,
without cutting you off
from the world.

ReSound GN

1. Groth & Cui (2023); 2. Groth et al (2023) & Jespersen et al. (2024); 3. Cui & Groth (2024)

Environmental Classifier

More awareness

ReSound GN



Uses AI to scan the environment in real time and classify sound into seven distinct categories



Activates appropriate features based on the amount of noise and the presence of speech



Helps users to hear their best automatically

91%

Accuracy in Environmental classification¹

¹ Groth et al (2023)

360 All-Around

More access

Industry best at providing access to surrounding sound¹

Compared to legacy and leading competing solutions with 4-mic beamforming


90%

Preference for listening in 360 All-Around over legacy²



ReSound GN

1. Groth et al (2023) + Jespersen et al. (2024); 2. Jespersen & Skovlund (2025)

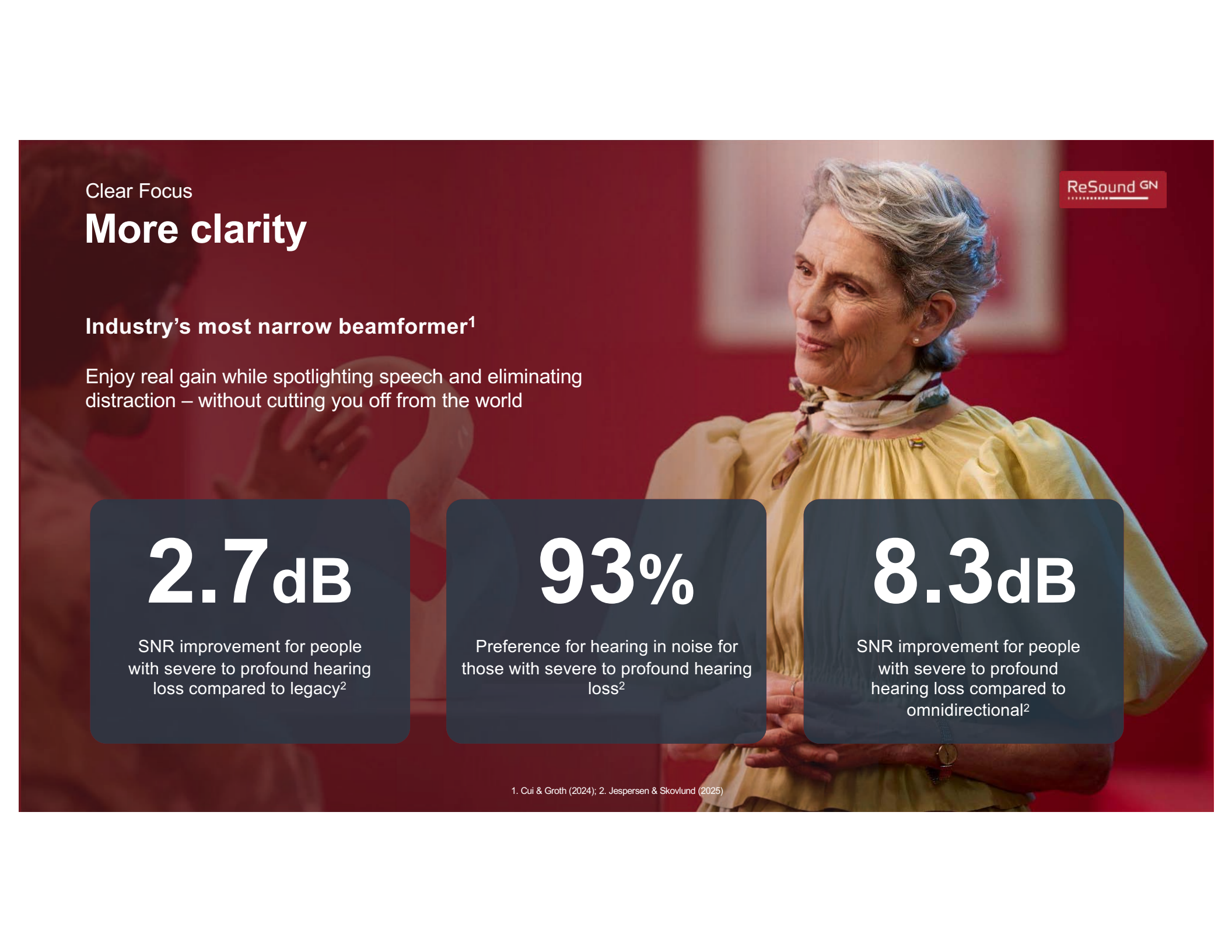

Clear Focus

More clarity

ReSound GN

Industry's most narrow beamformer¹

Enjoy real gain while spotlighting speech and eliminating distraction – without cutting you off from the world

2.7dB

SNR improvement for people with severe to profound hearing loss compared to legacy²

93%

Preference for hearing in noise for those with severe to profound hearing loss²

8.3dB

SNR improvement for people with severe to profound hearing loss compared to omnidirectional²

1. Cui & Groth (2024); 2. Jespersen & Skovlund (2025)

More sound. More power. Now in the smallest rechargeable Super Power hearing aids

ReSound GN



ReSound ENZO Q™



ReSound Enzo™ IA



DSF Ultra III


More relief from feedback

ReSound GN

World's best feedback management¹

Our advanced technology works tirelessly to provide the fastest elimination of feedback without compromising on gain.

Added stable gain (dB)

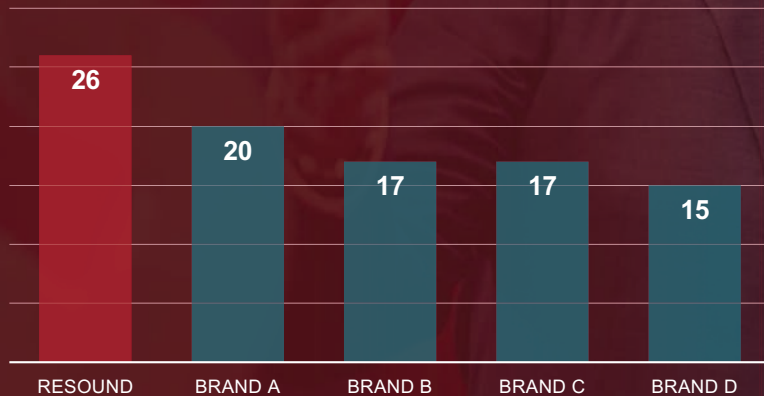

Up to **6dB**

More added stable gain
compared to other leading
Super Power solutions¹

1. Cui & Groth (2025)


DSF Ultra III

ReSound GN

More relief from feedback


ReSound Enzo IA

Before
calibration:
AutoDFS

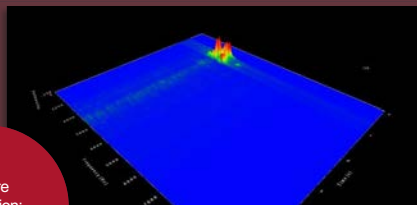

Response time: 0.46s


After
calibration:
Mild
setting

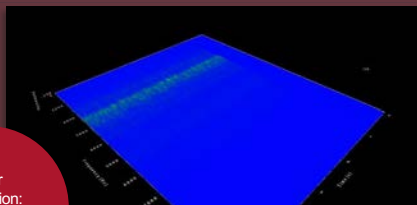

No reduced gain


Brand A

Without
calibration:
Moderate
setting

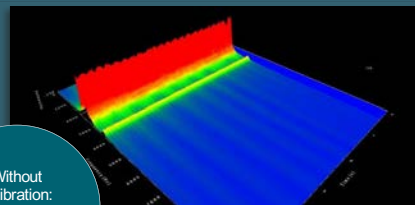

Whistles the whole time


After
calibration:
Gain
reduced

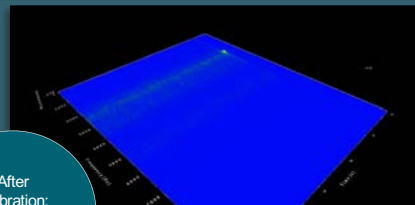

Gain reduced by 7 dB


Brand B

Without
calibration:
1.5dB
less gain
than
ReSound

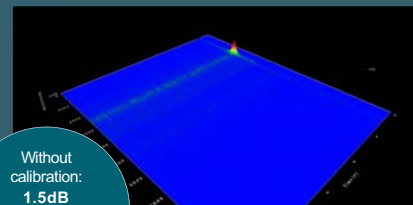

Response time: 0.18s


After
calibration:
Gain
reduced

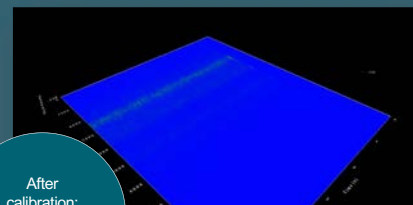

Gain reduced by 4 dB


Brand C

Without
calibration:
Slight
feedback
all the
time

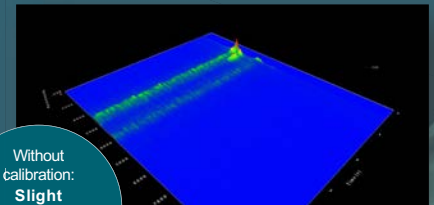


Response time: 0.4s

After
calibration:
Gain
reduced

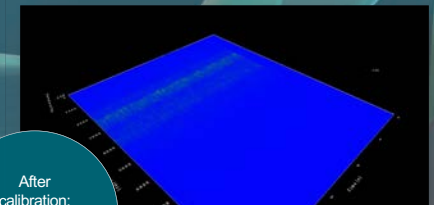

Gain reduced by 2 dB

Intelligence Augmented

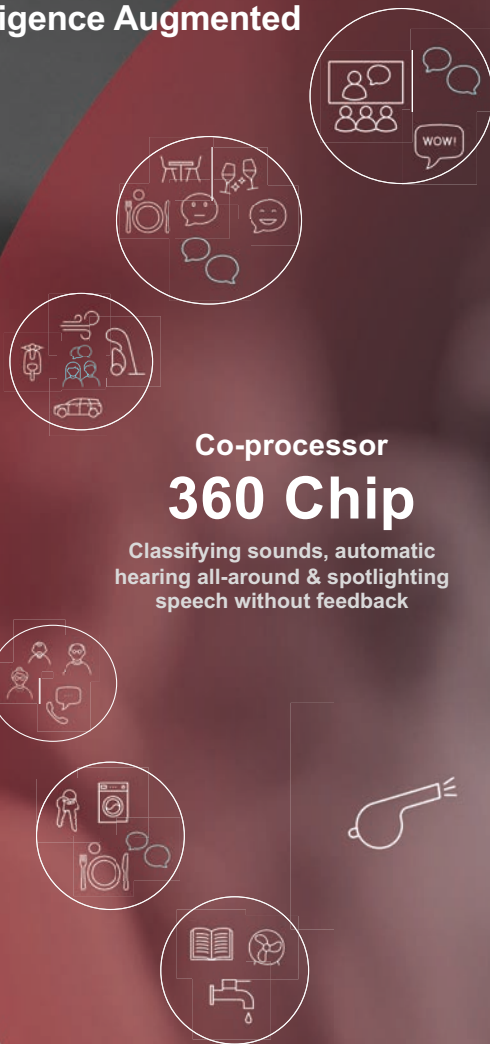

Co-processor 360 Chip

Classifying sounds, automatic
hearing all-around & spotlighting
speech without feedback



Core processor Human brain

Makes the final decision
on what to listen to

ReSound GN



More robust

Designed for comfort.
Made to last.
Powered to go.

ReSound GN



World's smallest rechargeable Super Power



EI998-DWHC

Super Power BTE

Rechargeable

Tech levels 9, 7, 5 and 4

All-day wear

More comfort

ReSound GN



Smaller than our previous
Super Power hearing aid



More organic design
that naturally conforms
to the shape of the ear



One-point touch on ear
for increased comfort



12%

Smaller than competing
rechargeable Super
Power solutions¹

1. Based on 3D scans – GN Proprietary data on file (2025)

More premium colors



Sparkling
Silver

Sand

Gold

Bronze

Warm
Grey

Graphite

Espresso

Deep
Black

Navy
Blue

Red



All-weatherproof IP68-rated and
protective nanocoating in and out

More power

Inductive charging
prevents debris and
moisture build-up

Hours of use¹



Max



with 2hrs
streaming



with 50%
streaming

1. GN Proprietary data on file (2025)

Reasons to choose rechargeability

More convenience

ReSound GN

1

Reliable Power

Full-day use on a single charge
– even when streaming

2

Easy to use

Eliminates the need to handle
batteries or battery doors– ideal for
patients with limited dexterity

3

Peace of mind

Check battery status anytime in the
ReSound Smart 3D™ app.

4


Fast charging

Fast-charge hearing aids for a
quick power boost

ReSound GN

More moments

Hear your best in
every situation

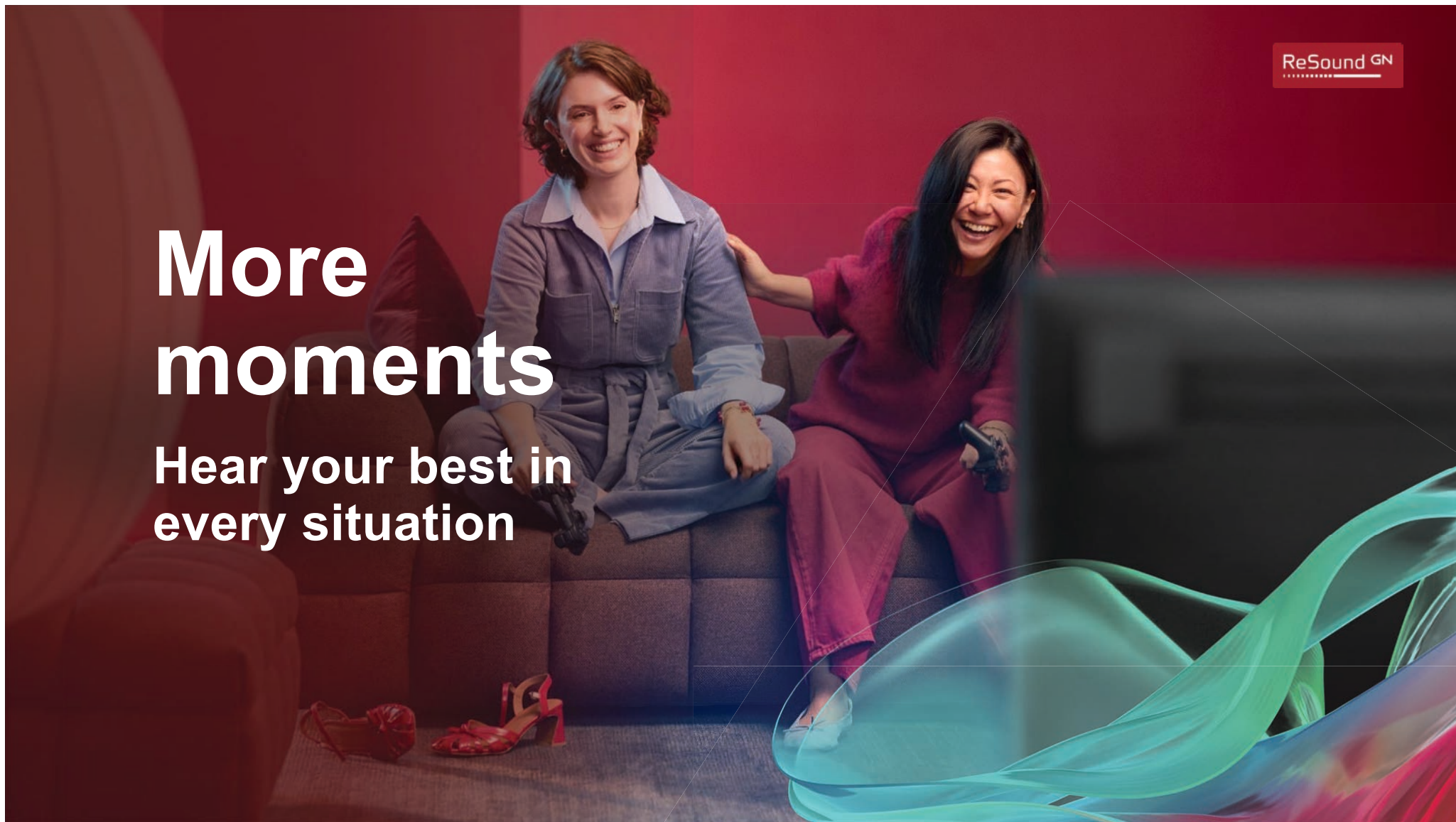

TV-Streamer+

More entertainment

ReSound GN



Easily pair the hearing aids to the TV-streamer+ with OneTouch Pairing



AutoStreaming makes streaming effortless – just turn on the TV-Streamer+ to send sound directly to the hearing aids



Adjust the TV volume and surrounding sounds with the ReSound Smart 3D app



Supports Auracast, allowing an unlimited number of Auracast-enabled devices to connect for shared listening experiences



Multi-Mic+

More tuning in

Capture every word up to 82 ft away*

Whether you're in a bustling restaurant or at a crowded event, using the Auracast-enabled Multi-Mic+ results in greater speech understanding in noise.

9.9dB

SNR improvement compared to omni for Super Power hearing aid users¹

Clip the Multi-Mic+ onto a conversation partner or place it on the table for group conversations

+6.0dB

Additional SNR improvement than with Clear Focus with severe to profound hearing loss¹

ReSound GN



Accessibility
and AgeTech

1. Jespersen & Rubachandran (2025)



2+1


Connect to two smart
devices and the FSW
simultaneously

MultiConnect

ReSound GN

More seamless use

Seamlessly switch between devices with MultiConnect.

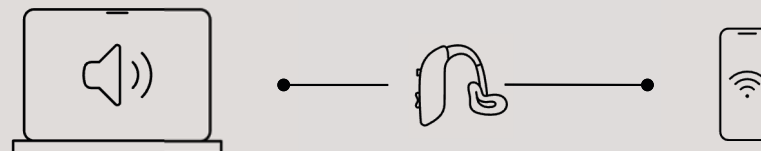

Pair ReSound Enzo IA with two smart devices simultaneously for uninterrupted streaming – such as taking a phone call while streaming music or movie audio from a laptop.

More connections

Join in like
never before



Bluetooth LE Audio and Auracast

More accessibility

World's first Super Power hearing aid made for Bluetooth Low Energy (LE) Audio and Auracast¹



1. GN Proprietary data on file (2025)

Made for Bluetooth LE Audio and Auracast

More places and devices

ReSound GN



Movie theaters



Televisions



Conference venues



Electronic devices



Public transportation



Airports

Auracast Assistant

More streams


World's first Auracast Assistant for hearing aids¹

The latest inclusive connectivity is more than just streaming and listening anywhere, it also lets you join in on the same terms as everyone else.

How it works in the ReSound Smart 3D™ app:

- 1 Click on "Find a broadcast at the bottom of the screen"
- 2 Choose an Auracast broadcast from the list
- 3 Enjoy sound directly streamed to your hearing aids

ReSound GN

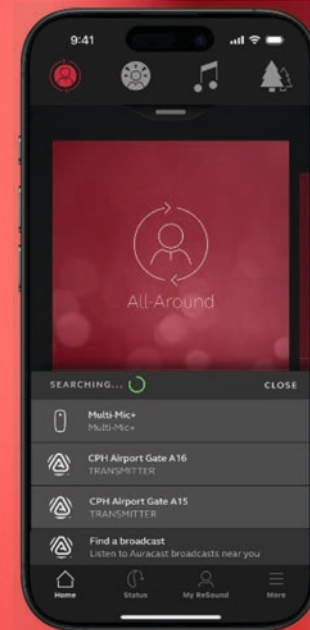


ReSound Smart 3D with
Auracast Assistant



1. GN Proprietary data on file (2025)


ReSound Smart 3D app with Auracast Assistant

ReSound GN

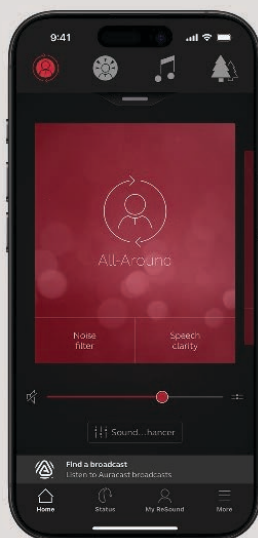


Available for all users

With iOS and Android™ devices compatible with ReSound Smart 3D

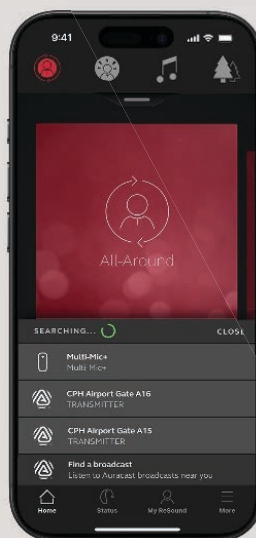

Access both Auracast and accessory streams in one list


Available right at your fingertips

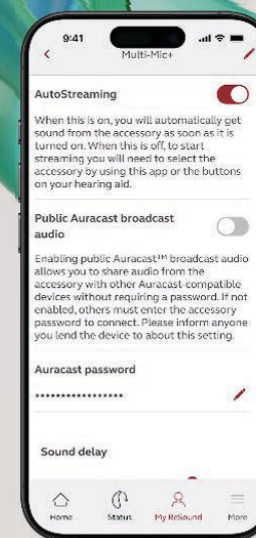

Easily share Auracast streams with family and friends

Share and modify passwords to protect your privacy when using the Multi-Mic+ and TV-Streamer+

Auracast is coming to more public venues

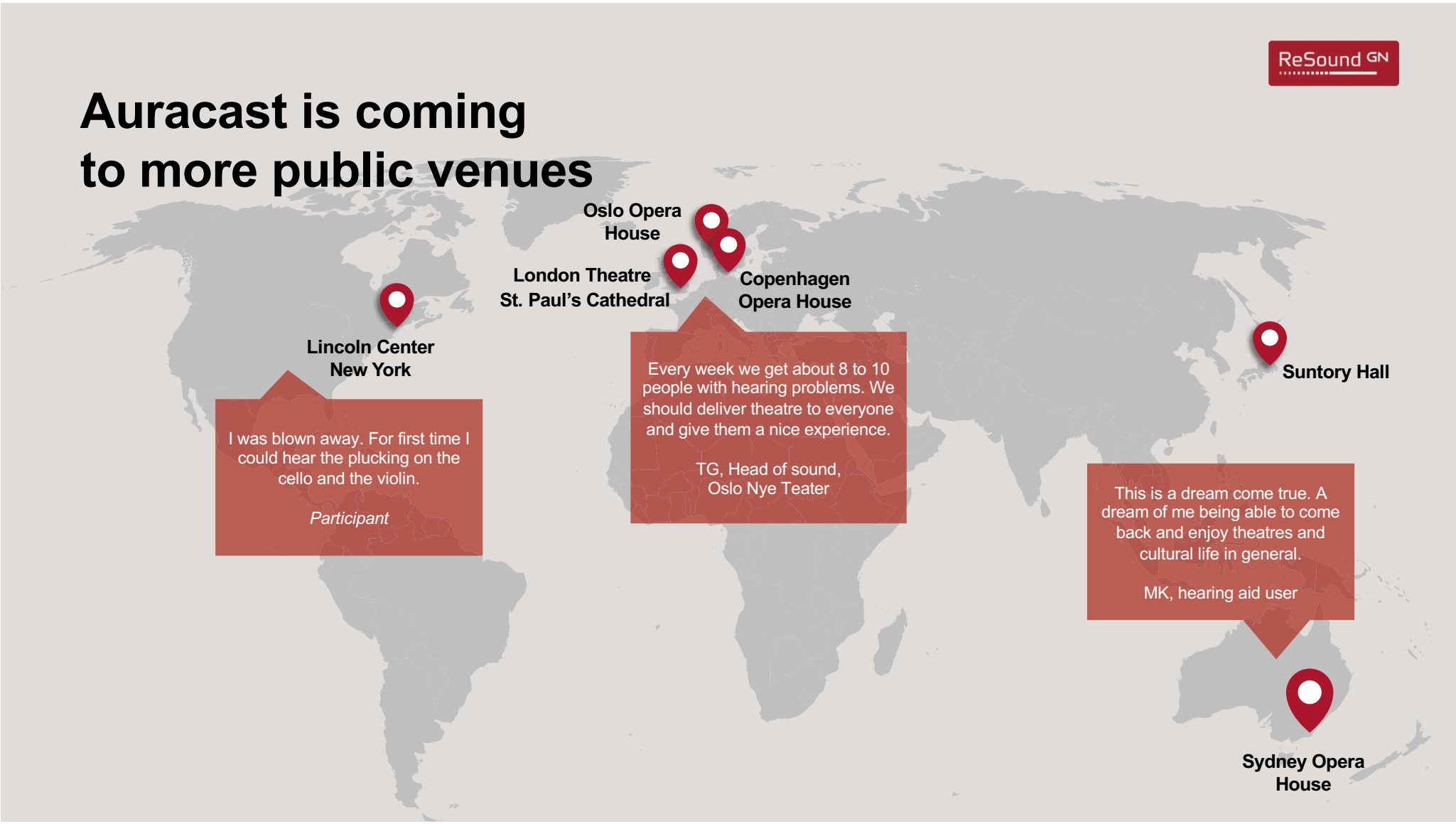

**Lincoln Center
New York**

I was blown away. For first time I could hear the plucking on the cello and the violin.

Participant

Oslo Opera House
London Theatre St. Paul's Cathedral
Copenhagen Opera House

Every week we get about 8 to 10 people with hearing problems. We should deliver theatre to everyone and give them a nice experience.

TG, Head of sound,
Oslo Nye Teater

Suntory Hall

This is a dream come true. A dream of me being able to come back and enjoy theatres and cultural life in general.

MK, hearing aid user

Sydney Opera House

If you had seen me inside,



Enhance your fitting experience

ReSound GN



Save time during your fitting sessions

Discovery screen

A new, more intuitive discovery screen



Firmware updates

30% faster firmware updates¹



Test box measurement

Our first directionality presets for test box testing with 50% time saving compared to legacy¹




1. GN Proprietary data on file (2025)

ReSound Enzo IA

Replaceable mic filters

Enhance durability and
extend your in-clinic service

What's new in ReSound Smart Fit 2.2

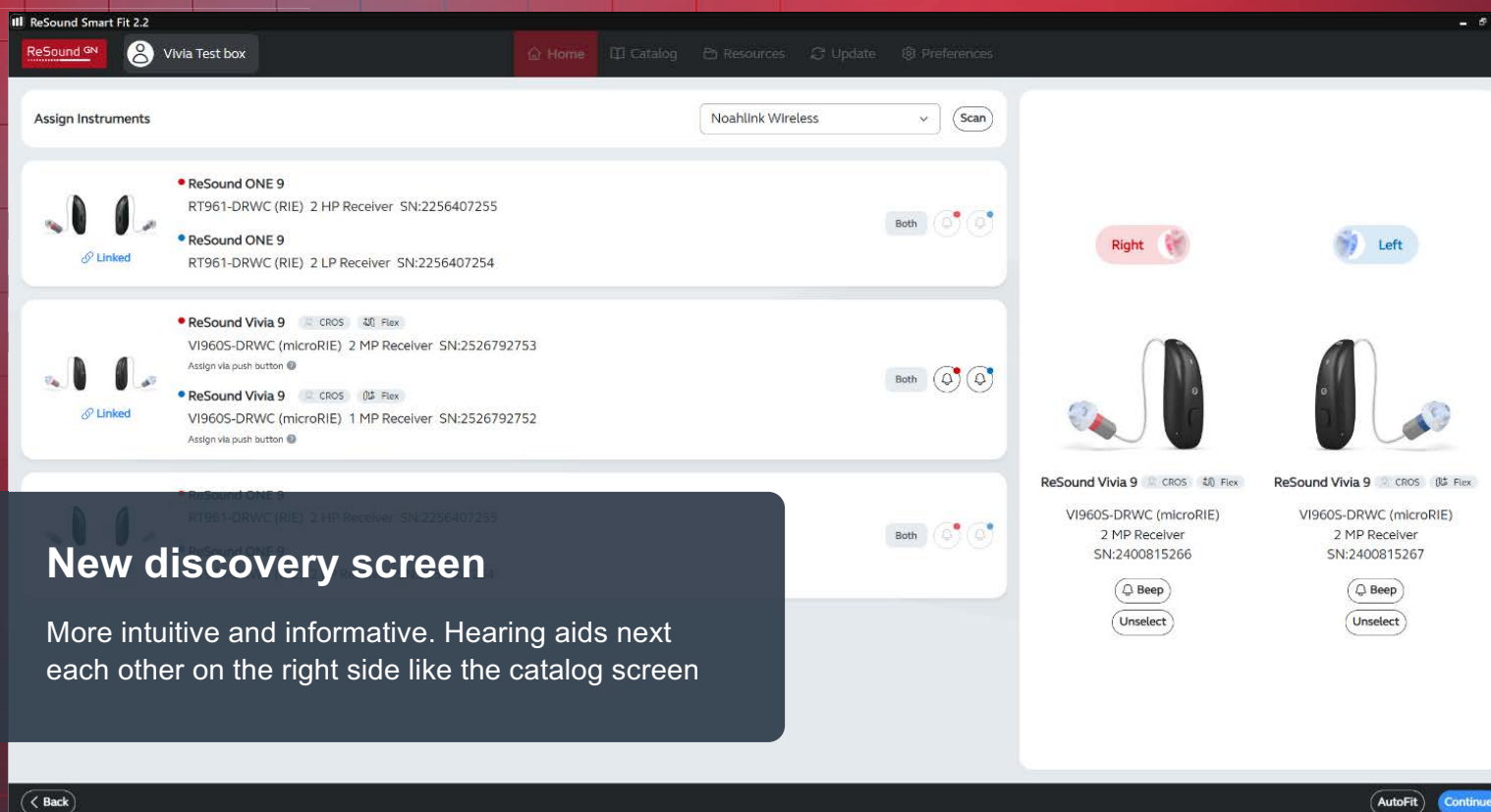


1

New discovery screen



More intuitive and informative. Hearing aids next each other on the right side like the catalog screen

2 Faster firmware updates

Previous versions: Sequential

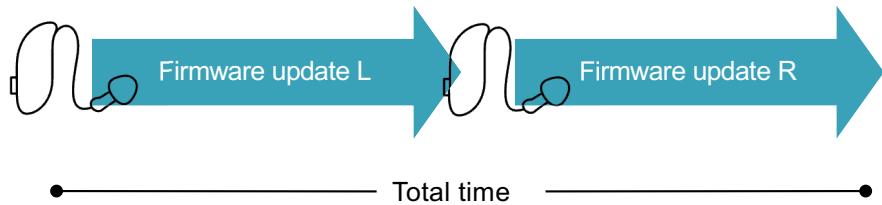


ReSound Smart Fit 2.2: simultaneous

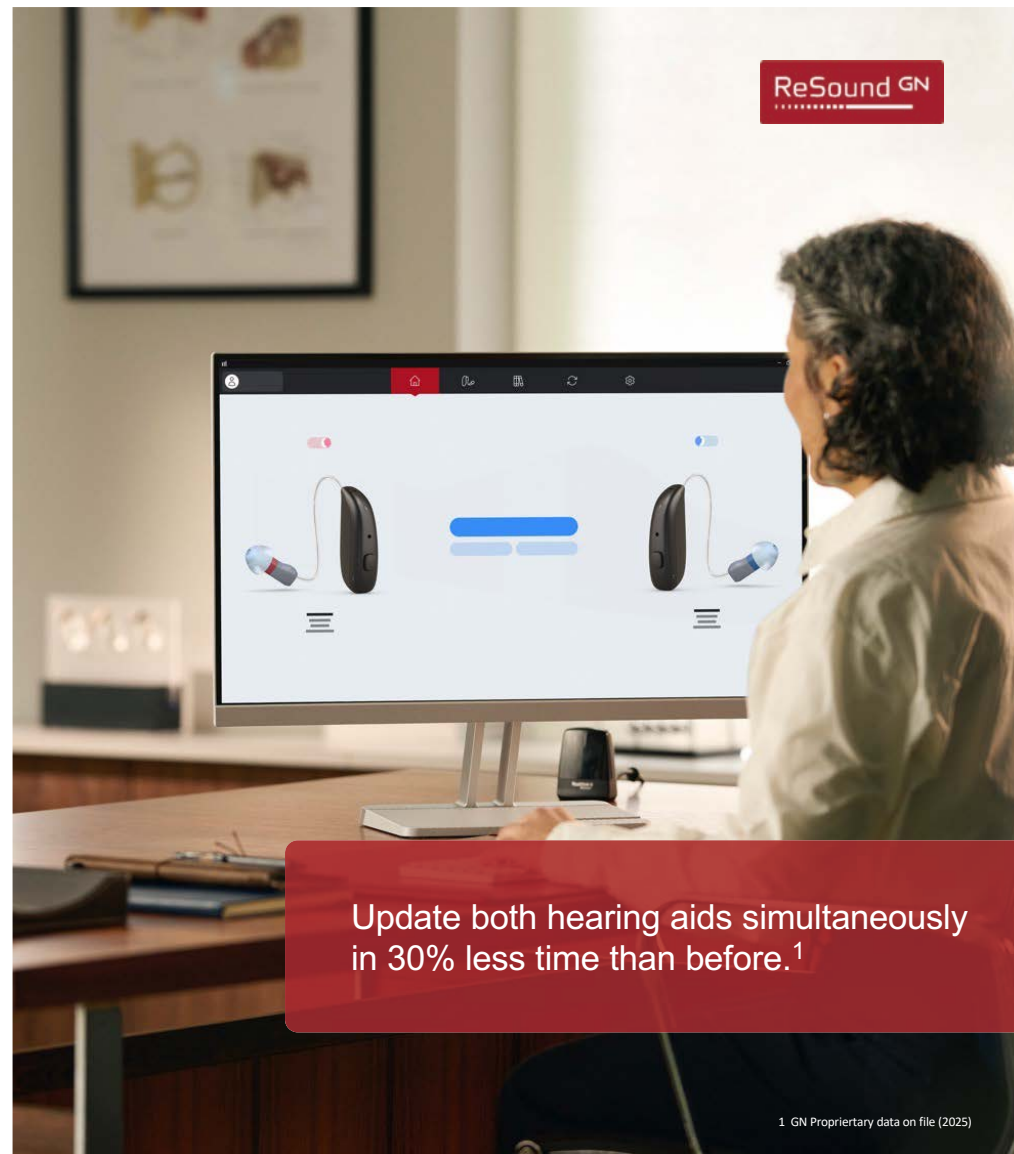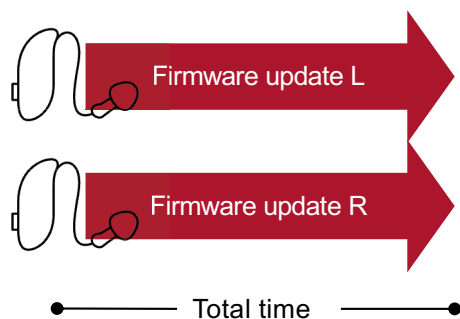

1 GN Proprietary data on file (2025)

3 New test device

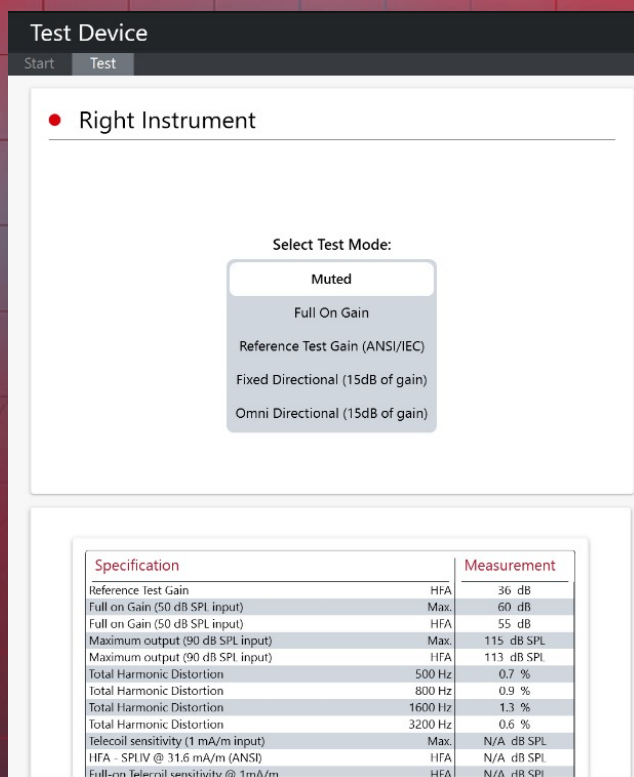

Hearing aids remain connected to the FSW throughout the test



Ability to mute hearing aids from FSW



Now with directionality tests from Test Mode menu



Specs viewable within Test Device

ReSound Enzo IA

ReSound GN

Gain so much more without compromise

Sounds natural



Made for hearing in noise to spotlight speech and eliminate distractions, without cutting you off from the world¹



World's best feedback management without compromising on gain²



Industry-leading access to surrounding sounds³ with 360 All-Around

Powered by Intelligence Augmented™

Feels natural



World's smallest rechargeable Super Power hearing aids⁴ with all-day comfort



All-weatherproof IP68 rating and replaceable mic filters to withstand everyday life



Rechargeable all-day battery⁴ with 28 hours' use – and 20 hours when streaming 50% of the time⁵

Connects you naturally



World's first Super Power hearing aid with Bluetooth LE Audio and Auracast⁶



Connect to two devices at once with MultiConnect



Clearest Bluetooth streaming at the volume you prefer in every situation with TV-Streamer+ & Multi-Mic⁷

1. Jespersen & Skovlund (2025), Groth et al (2023) & Jespersen et al. (2024); 2. Cui & Groth (2025); 3. Groth et al (2023) & Jespersen et al. (2024); 4. GN Proprietary data on file (2025); 5. GN Proprietary data on file (2025); 6. GN Proprietary data on file (2025); 7. Jespersen & Rubachandran (2025)

ReSound GN

ReSound Enzo IA

Tech level
differences



Sounds natural

Tech level

Hear all around

Spatial awareness

Spatial Sense

1:1 conversations

Hear in noise

Group conversations

Number of channels

Personalization

Adjustable Directional Mix

ReSound Enzo IA



9



7



5



Natural Dir II Binaural Dir III All-Access 360 All-Around



Natural Dir II Binaural Dir III All-Access 360 All-Around



Natural Dir II Binaural Dir III All-Access 360 All-Around



Fixed Dir Ultra Focus Front Focus Clear Focus



Fixed Dir Ultra Focus Front Focus Clear Focus



Fixed Dir Ultra Focus Front Focus Clear Focus



Adaptive Directionality MultiScope AutoScope



Adaptive Directionality MultiScope AutoScope



Adaptive Directionality MultiScope AutoScope



6 8 12 14 17



6 8 12 14 17



6 8 12 14 17



All models incorporate Tinnitus Sound Generator. See datasheet for complete overview of features.

Sounds natural

Tech level

Hear all around

Spatial awareness

Spatial Sense

1:1 conversations

Hear in noise

Group conversations

Number of channels

Personalization

Adjustable Directional Mix

ReSound Enzo IA



5




4



Natural Dir II Binaural Dir III All-Access 360 All-Around



Natural Dir II Binaural Dir III All-Access 360 All-Around

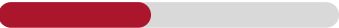

Fixed Dir Ultra Focus Front Focus Clear Focus



Fixed Dir Ultra Focus Front Focus Clear Focus



Adaptive Directionality MultiScope AutoScope



Adaptive Directionality MultiScope AutoScope



6 8 12 14 17



6 8 12 14 17



All models incorporate Tinnitus Sound Generator. See datasheet for complete overview of features.

Feels natural

Tech level

ReSound Enzo IA

ReSound GN

9



7



5



Comfort

Environmental noise reduction



Wind noise reduction



Impulse noise reduction



Feedback cancellation



Speech enhancement

Environmental Optimizer



Sound Shaper



CROS/BiCROS



Ease of use

TapControl



See datasheet for complete overview of features

Feels natural

Tech level

ReSound Enzo IA



5



4



Comfort

Environmental noise reduction



Wind noise reduction



Impulse noise reduction



Feedback cancellation



Speech enhancement

Environmental Optimizer



Sound Shaper



CROS/BiCROS



Ease of use

TapControl



See datasheet for complete overview of features

Connects naturally

ReSound Enzo IA

ReSound GN

Tech level



ReSound Smart 3D app



- Sound Enhancer
- Auracast Assistant
- Remote fittings: ReSound Assist / Assist Live



Direct audio streaming

- Bluetooth LE Audio/Auracast
- MultiConnect (FSW/new accessory and 2 Bluetooth devices)
- Made for iPhone
- Hands-free calls for iPhone and iPad
- Android Streaming for Hearing Aids



Wireless accessories



- 2.4 GHz Legacy accessories
- New accessories with BTLE Audio and Auracast



Telecoil



See datasheet for complete overview of features

ReSound Enzo IA



Super Power BTE
Rechargeable

ReSound ENZO Q



Super Power BTE
675 Battery

Form factors

Comparison based on Tech Level 9

360 All-Around



Clear Focus



TapControl



MFi/Android streaming



Bluetooth LE Audio / Auracast



Wireless accessories: Legacy TV Streamer & Remote Control



Wireless accessories: New TV-Streamer+ & Multi-Mic+



MultiConnect



Telecoil



DAI



ReSound GN

ReSound Enzo IA

More sound. More power.
Now in the smallest rechargeable
Super Power hearing aids.



© 2025 GN Hearing A/S. All rights reserved. ReSound is a trademark of GN Hearing A/S. Apple, the Apple logo, iPhone, iPad and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The Auracast word mark and logos are trademarks owned by Bluetooth SIG, Inc.