

Quick Resource: Verification

Experience Level Parameters

Experience Levels within the Patient Profile impact gain, compression, and select advanced features. When selecting the Experience Level, consider where verification falls within your fitting flow.

Verification at initial fit

Experienced Non-Linear

Standard gain prescription for the selected fitting rule and the best match for reaching target.

Verification at follow-up

First Time User

Reduced gain and increased noise reduction strategies for inexperienced users.

–OR–

First Time User - Onboarding

Gain is reduced to 70% of First Time User. Focus on comfort across all input levels and most comfort in noise.

All experience level calculations are influenced by individual audiograms. Therefore, gain and compression settings may vary based on degree and configuration of hearing loss.

Manufacturer According to FDA:

GN ReSound North America
8001 E Bloomington Freeway
Bloomington, MN 55420
USA
1-800-248-4327
pro.resound.com

ReSound Government Services
8001 E Bloomington Freeway
Bloomington, MN 55420
USA
1-800-392-9932
gs.resound.com

Manufacturer According to Health Canada:

ReSound Canada
2 East Beaver Creek Road, Building 3
Richmond Hill, ON L4B 2N3
Canada
1-888-737-6863
pro.resound.com