

For professionals

A collection of hearing solutions **tailored to** **your patients**



Beltone **Portfolio**

Hearing solutions to delight your patients

Premium	RIE styles Mild to profound hearing loss    microRIE 62S Rechargeable RIE 63 RIE 64	Beltone Envision
Premium	BTE styles Severe to profound hearing loss   Super Power BTE 95 Metal hook Rechargeable Super Power BTE 95 Plastic hook Rechargeable	Beltone Boost Max S
Premium	RIE, BTE and custom styles Mild to profound hearing loss          CIC Non-wireless ITC Wireless ITC Rechargeable ITE Rechargeable microRIE 62S Rechargeable RIE 63 RIE 64 BTE 76 Rechargeable Power BTE 86 Rechargeable	Beltone Serene
Advanced	RIE, BTE and custom styles Mild to profound hearing loss         CIC Non-wireless ITC Rechargeable ITE Rechargeable miniRIE 62 Rechargeable RIE 63 RIE 64 BTE 76 Rechargeable Power BTE 86 Rechargeable	Beltone Achieve
Advanced	BTE styles Severe to profound hearing loss   High Power BTE 86 Super Power BTE 95	Beltone Boost Ultra
Essential	RIE and custom styles Mild to profound hearing loss       CIC Non-wireless ITC Wireless NEW miniRIE 62 Rechargeable RIE 63 RIE 64	Beltone Commence
Essential	RIE, BTE and custom styles Mild to profound hearing loss           CIC Non-wireless ITC Wireless ITE Wireless RIE 63 Rechargeable RIE 63 RIE 64 miniBTE 66 BTE 76 High Power BTE 86 Super Power BTE 95	Beltone Rely



Receivers, thin tubes and domes overview



Auracast and streaming



Connectivity and accessories



Feature overview

Receiver, thin tubes and domes overview

	Hearing aids	Receivers and thin tubes	Domes
RIE Styles	Beltone Envision, Beltone Commence, Beltone Serene and Beltone Achieve  Beltone Envision and Beltone Serene microRIE 62S Rechargeable Beltone Commence miniRIE 62 Rechargeable Beltone Achieve miniRIE 62 Rechargeable  RIE 63 312 battery RIE 64 13 battery	SureFit™ 3 Receivers Available in 5 lengths: 0 to 4  MM* (M&RIE) LP MP  HP UP (encased) * MM is not available for Beltone Commence	AllFit and SureFit 3 Domes can be used for SureFit 3 and SureFit 2C receivers and thin tubes AllFit Domes  Open (XS, S, M, L, XL)  Vented (XS, S, M, L, XL)  Power (S, M, L, XL)  Tulip (S, M)  Sleeve* (S, M, L) *Recommended for MM (M&RIE). Sleeve dome is not compatible with thin tubes.
	Beltone Rely  RIE 63 Rechargeable RIE 63 312 battery RIE 64 13 battery	SureFit™ 2C Receivers Available in 5 lengths: 0 to 4  LP HP MP UP (encased)	SureFit™ 3 Domes  Open (S / M / L)  Closed (S / M / L)  Power (S / M / L)  Tulip
	Beltone Serene and Beltone Achieve  BTE 76 Rechargeable	SureFit 3 thin tubes Available in 5 lengths: 0 to 4 	
BTE Styles	Beltone Rely  miniBTE 66 312 battery BTE 76 13 battery	SureFit 2C thin tubes Available in 5 lengths: -1 to 3 	

Bring a world of sound **into focus**



World’s best for hearing in noise¹



World’s smallest AI hearing aid²



All-day battery* even when streaming

Colors



Charging options



Premium charger
Portable with powerbank
Compatible with the microRIE (CFF 8)



Standard charger
Portable
Compatible with the microRIE (CFF 8)

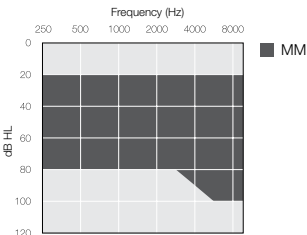
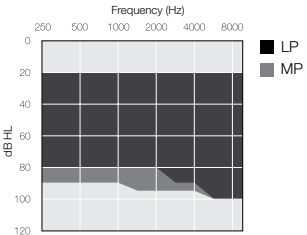
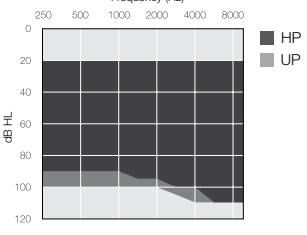


Desktop charger
Easy-to-use
Compatible with the microRIE (CFF 8)

¹ Groth & Cui (2023)
² GN data on file

* All-day battery on one charge is based on typical use time of up to 24 hours with one charge or up to 20 hours when streaming 50% of the time across all styles. Battery life can vary by use.

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain [*]
 microRIE 62S-DRWC Rechargeable Available as CROS transmitter (ENV162S-DRWC)	• TapControl with microRIE • All-day battery on one charge[*] • Receiver options: MM, LP, MP, HP and UP • Exchangeable mic filters • Programmable push button or toggle • IP68 certified • Telecoil in 64	 Frequency (Hz) dB HL MM  Frequency (Hz) dB HL LP MP  Frequency (Hz) dB HL HP UP	MM: 115/61 LP: 113/56 MP: 115/60 HP: 120/60 UP: 129/71

Beltone Envision

^{*}ANSI S3.22-2014, IEC 60118-0:2015, JIS C 5512: 2015

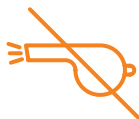


Boost their world of sound

Beltone **Boost™ Max S**



Made for hearing in noise:
2.7dB SNR improvement
with Clear Focus¹



World's best
feedback
management²



World's smallest
rechargeable super-power
hearing aid³

Colors



Sparkling
Silver



Sand



Gold



Bronze



Warm
Grey



Graphite



Espresso



Deep
Black



Navy
Blue



Red

Charging options



Desktop charger

Easy-to-use

Compatible with the super-power BTE (CFF 90)

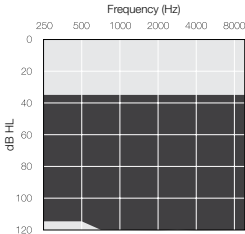
¹ For those with severe to profound hearing loss compared to legacy. Jespersen & Skovlund (2025)

² Compared to leading super-power solutions. Cui & Groth (2025)

³ Compared to leading rechargeable super-power solutions. GN Proprietary data on file (2025)

* All-day battery on one charge is based on typical use time of up to 24 hours with one charge or up to 20 hours when streaming 50% of the time across all styles. Battery life can vary by use.

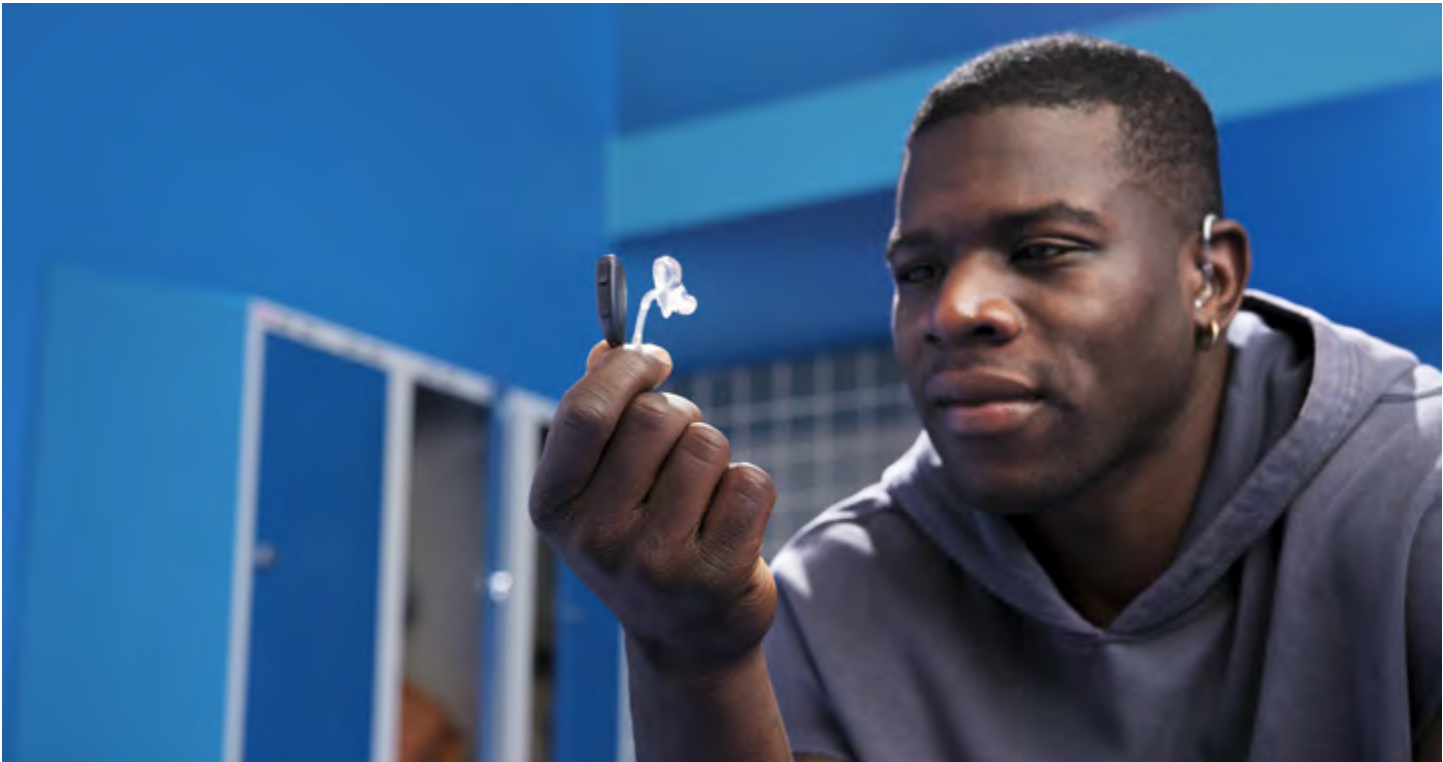
For severe to profound hearing loss

Style	Features	Fitting range	Max output/gain†
 Super Power 95-DWHC Metal hook Rechargeable	• All-day battery on one charge* • IP68 rated • TapControl • Programmable push button and volume control • Telecoil • Exchangeable mic filters • Optional plastic hook configurable in FSW		Metal hook: 140/84 Plastic hook: 140/79
 Super Power 95-DWHC Plastic hook Rechargeable			

Beltone

Boost Max S

†ANSI S3.22-2014, IEC 60118-0:2015, JIS C 5512: 2015



Exceptional clarity and effortless connectivity for all



Beltone **Commence**™



Best-in-class
sound quality¹⁻²



Best hearing comfort
in tiny designs³



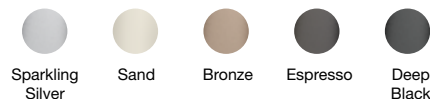
Clearest Bluetooth
experience ever*

Colors

Custom colors



Shimmering miniRIE colors**



Matte RIE colors***



Charging options



Premium charger

Portable with powerbank
Compatible with the miniRIE (CFF 9)



Desktop charger

Easy-to-use
Compatible with the miniRIE (CFF 9)

* Hands-free calls are compatible with iPhone 11 or later, iPad Pro 12.9-inch (5th generation), iPad Pro 11-inch (3rd generation), iPad Air (4th generation), iPad mini (6th generation), iPad (10th generation) or later, with software updates iOS 15.3 and iPadOS 15.3 or later and compatible Android mobile devices. Hands-free calls not available for ITC or CIC styles.

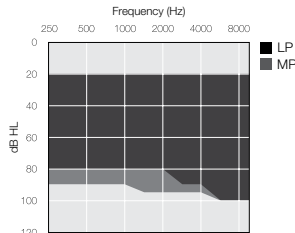
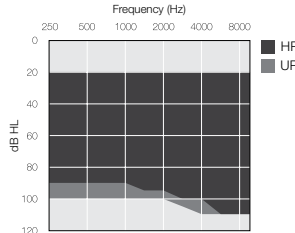
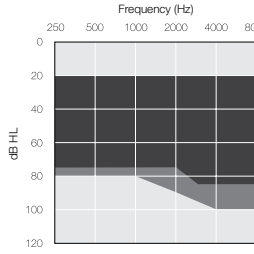
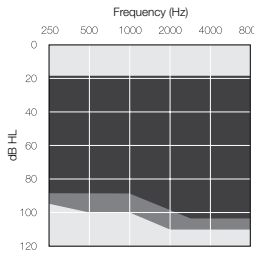
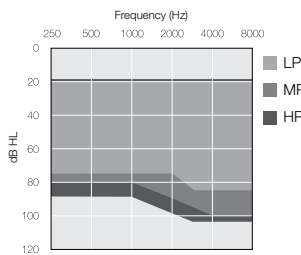
** Available for miniRIE

*** Available for RIE 63 & RIE 64

**** All-day battery on one charge is based on typical use time of up to 24 hours with one charge or up to 20 hours when streaming 50% of the time across all styles. Battery life can vary by use.

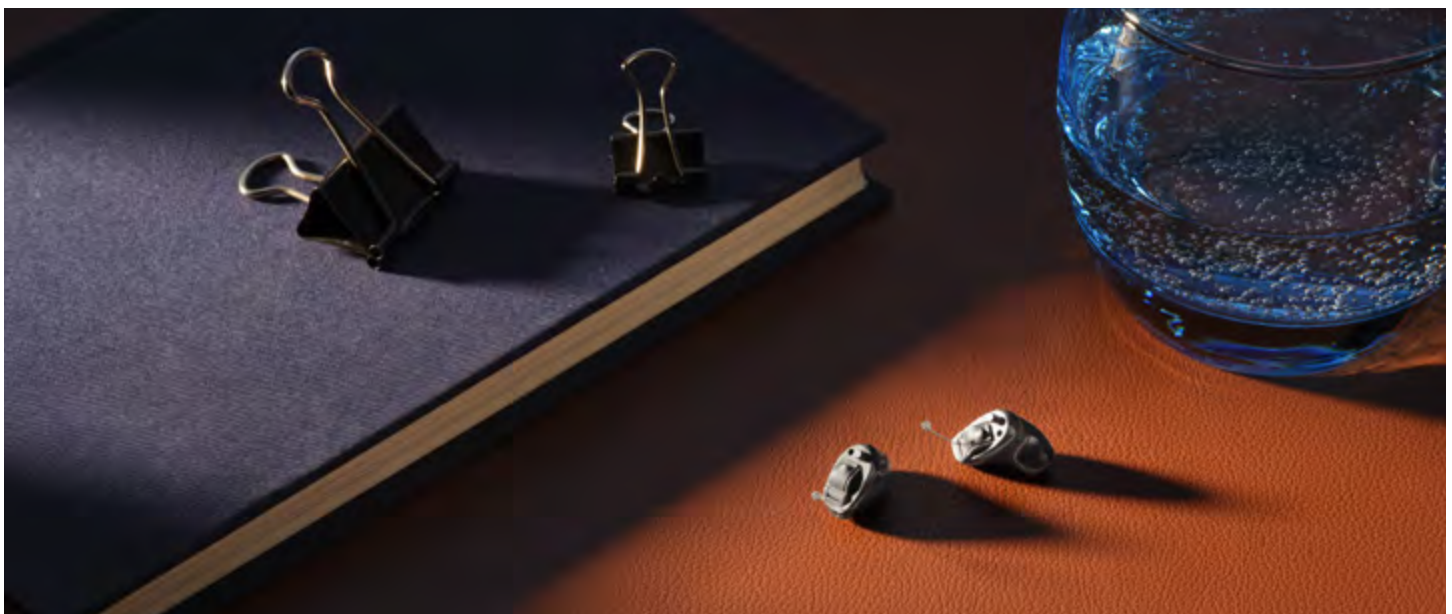
1 Haastrup (2012)
2 Sjolander et al. (2019)
3 GN data on file

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain†
 miniRIE 62-DRWC Rechargeable  RIE 63-DRW 312 battery  RIE 64-DRW 13 battery	• All-day battery on one charge**** • Receiver options: LP, MP, HP & UP • Programmable push button or toggle • IP68 certified • Telecoil in 64	 	LP: 113/56 MP: 115/60 HP: 120/60 UP: 129/71
 ITC Wireless	• Receiver options: LP, MP, HP & UP • Shell options: ITC, Half-shell & Full-shell • Programmable push-button & optional volume control • IP68 rated • Bluetooth LE Audio & Auracast	 	LP: 113/40 MP: 115/50 HP: 120/60 UP: 128/68
 CIC Non-wireless	• Cabled fitting required • Receiver options: LP, MP & HP • Optional push button • IP68 rated		LP: 113/40 MP: 115/50 HP: 120/60

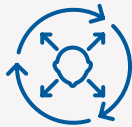
[†]ANSI S3.22-2014, IEC 60118-0:2015, JIS C 5512: 2015

Beltone Commence



Experience more of life

Beltone **Serene**™



Industry-best
access to surrounding
sound¹



Smallest
rechargeable style
with microRIE²

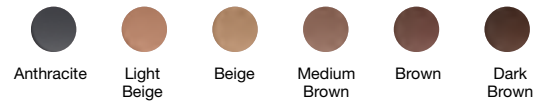


World's first hearing aids
with Bluetooth LE Audio &
Auracast broadcast audio²

RIE and BTE colors



Custom colors



Charging options



Premium charger
Portable with powerbank
Compatible with the microRIE (CFF 8)



Standard charger
Portable
Compatible with the microRIE (CFF 8)



Desktop charger
Easy-to-use
Compatible with microRIE (CFF 8),
76 BTE (CFF 70), 86 BTE (CFF 80)



Custom charger
Portable for ITC/ITE
(CFF C4)

* All-day battery on one charge is based on typical use time of up to 24 hours with one charge or up to 20 hours when streaming 50% of the time across all styles. Battery life can vary by use.

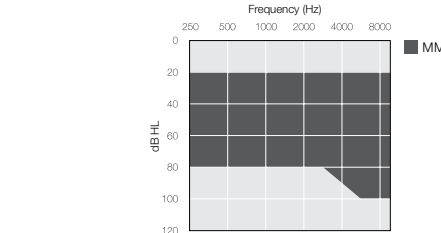
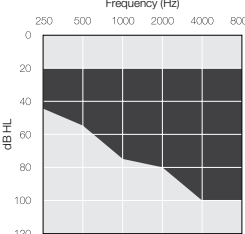
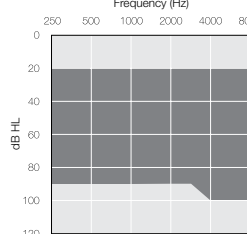
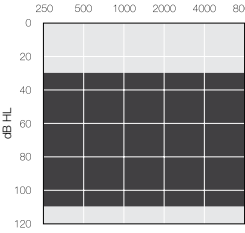
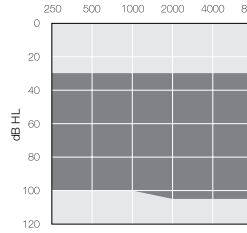
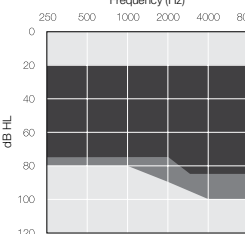
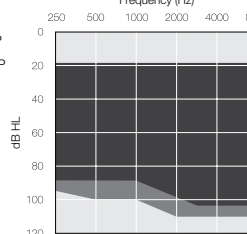
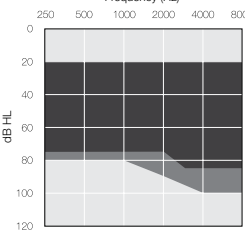
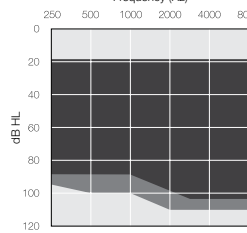
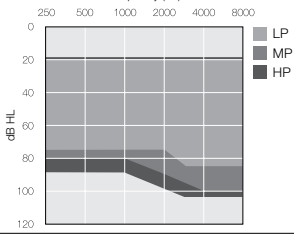
** Also compatible with Beltone Serene rechargeable ITC/ITE

*** Only available for ITC

**** Only available for ITE

1 Groth & Cui (2023)
2 GN data on file

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain†
 microRIE 62S-DRWC Rechargeable Available as CROS Transmitter (CSE162S-DRWC)	• TapControl with microRIE • All-day battery on one charge* • Receiver options: MM, LP, MP, HP and UP • Programmable push button or toggle • IP68 certified • Telecoil in 64		MM: 115/53 LP: 114/54 MP: 116/59 HP: 120/63 UP: 130/73
 RIE 63-DRW 312 battery			
 RIE 64-DRW 13 battery			
 BTE 76-DWC Rechargeable Available as CROS Transmitter** (CSE176-DWC)	• All-day battery on one charge* • Programmable push button and/or toggle • Optional metal hook for 86-DWC • IP68 certified • Telecoil in BTE 76 and 86 • DAI in BTE 86 • Exchangeable mic filters		BTE (open): 127/53
 Power BTE 86-DWC Rechargeable			BTE (closed): 128/60
			Power BTE (plastic hook): 133/67
			Power BTE (metal hook): 133/73
 ITC-DWC / ITE-DWC Rechargeable	• All-day battery on one charge* • Receiver options: LP**, MP, HP and UP • Shell options: ITC, Half-shell*** and Full-shell**** • Programmable push button • IP68 certified		LP: 114/40*** MP: 118/50 HP: 121/60 UP: 130/70
			
 ITC-DW 312 battery	• Receiver options: LP, MP, HP and UP • Shell options: ITC, Half-shell and Full-shell • Programmable push-button and volume control • IP68 certified		LP: 114/40 MP: 117/50 HP: 120/60 UP: 130/68
			
 CIC non-wireless 10A battery	• Cabled fitting required • Receiver options: LP, MP and HP • IP68 certified		LP: 114/40 MP: 116/50 HP: 121/60

†ANSI S3.22-2014, IEC 60118-0:2015, JIS C 5512: 2015

Hear through noise without effort

Beltone **Achieve™**



150% improvement
to speech understanding in
noise with Ultra Focus 2¹



Great wearing
comfort

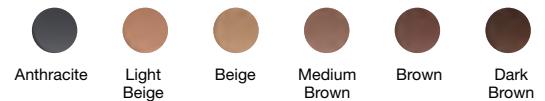


Robust connectivity
for uninterrupted
streaming

RIE and BTE colors



Custom colors



Charging options



Premium charger
Portable with powerbank
Compatible with the miniRIE (CFF 6)



Standard charger
Portable
Compatible with the miniRIE (CFF 6)



Desktop charger
Easy-to-use
Compatible with miniRIE (CFF 6),
76 BTE (CFF 70), 86 BTE (CFF 80)



Custom charger
Portable for ITC/ITE
(CFF C4)

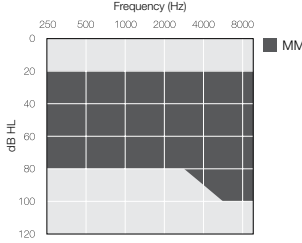
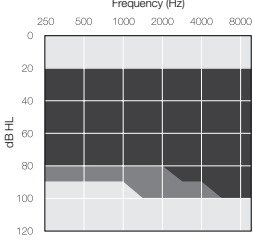
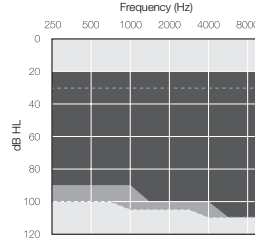
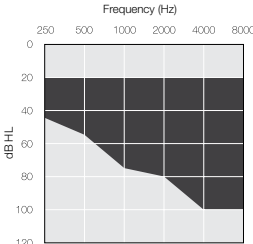
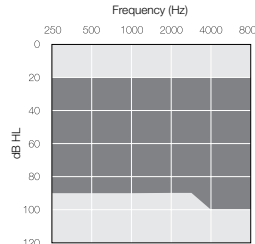
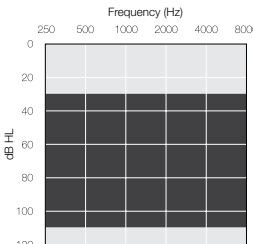
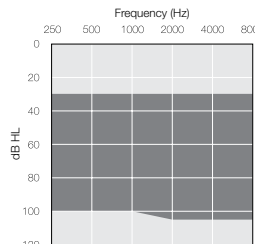
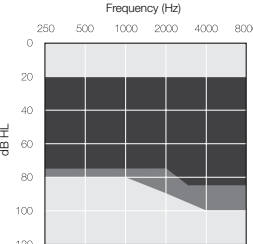
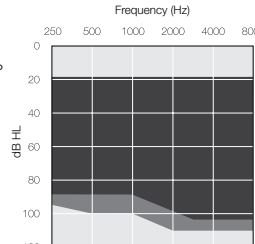
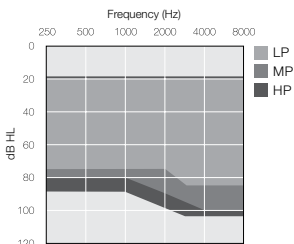
* All-day battery on one charge is based on typical use time of up to 24 hours with one charge or up to 20 hours when streaming 50% of the time across all styles. Battery life can vary by use.

** Only available for ITC

*** Only available for ITE

¹ 4.36 dB SNR improvement compared to legacy products. Jespersen et al (2022)

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain†
 miniRIE 62-DRWC Rechargeable  RIE 63-DRW 312 battery  RIE 64-DRW 13 battery	- All-day battery on one charge* - Receiver options: MM, LP, MP, HP and UP - Programmable push button or toggle - Telecoil in 64 - IP68 certified	  	MM: 115/53 LP: 113/53 MP: 116/58 HP: 122/65 UP: 130/75
 BTE 76-DWC Rechargeable  Power BTE 86-DWC Rechargeable	- All-day battery on one charge* - Programmable push button/programmable push button and toggle - Optional metal hook for 86-DWC - IP68 certified - Telecoil in BTE 76 and 86 - DAI in BTE 86	   	BTE (open): 127/53 BTE (closed): 128/60 Power BTE (plastic hook): 133/67 Power BTE (metal hook): 133/73
 ITC-DWC / ITE-DWC Rechargeable	- All-day battery on one charge* - Receiver options: LP*, MP, HP and UP - Shell options: ITC, Half-shell*** and Full-shell**** - Programmable push button - IP68 certified	 	LP: 114/40** MP: 118/50 HP: 121/60 UP: 130/70
 CIC non-wireless 10A battery	- Cabled fitting required - Receiver options: LP, MP and HP - IP68 certified - Optional push button		LP: 114/40 MP: 116/50 HP: 121/60

*ANSI S3.22-2014, IEC 60118-0:2015, JIS C 5512: 2015

Boost patients' hearing

Beltone **Boost Ultra™**



Distortion-free sound
with an input dynamic range
of up to 116 dB SPL



Designed for
everyday use



Crystal clear phone calls
and streaming from iPhone,
iPad or Android devices

BTE colors



Beige



Grey

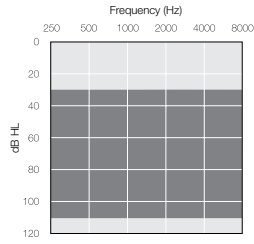
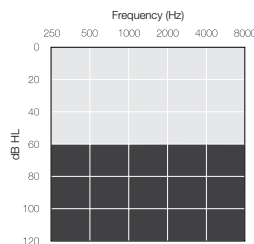


Brown



Charcoal

For severe to profound hearing loss

Style	Features	Fitting range	Max output/gain†
 High Power BTE 86-DWHT* 13 battery	• Push button and toggle • Telecoil and DAI • IP68 certified		High Power BTE: 134/73
 Super Power BTE 95-DWT 675 battery	• Push button and toggle • Telecoil and DAI • IP68 certified		Super Power BTE: 141/83

Beltone Boost Ultra

†ANSI S3.22-2014, IEC 60118-0:2015, JIS C 5512: 2015

Great hearing for everyone

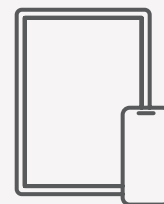
Beltone **Rely™**



Great audibility all-around
and a distortion
free sound experience¹⁻³



Easy hearing
control on the
hearing aids



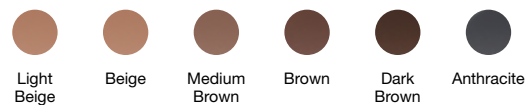
Streaming from
iPhone, iPad or
Android devices*

RIE and BTE colors



Custom colors

CIC Non-wireless



ITC and ITE



Charging options



Premium charger
Portable with powerbank
Compatible with RIE 63-DRWC

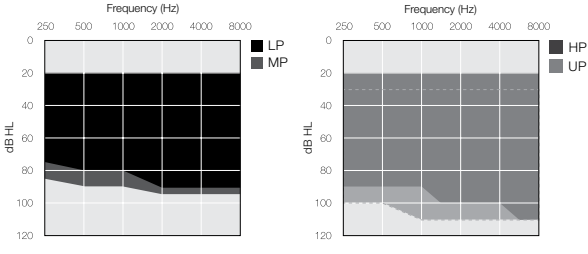
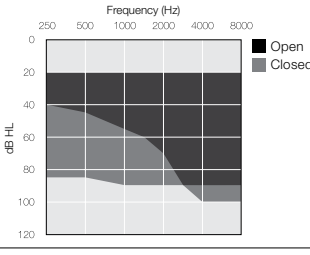
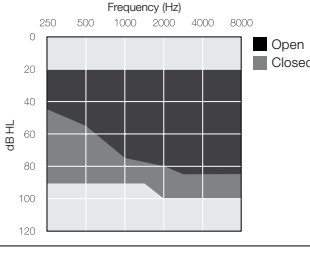
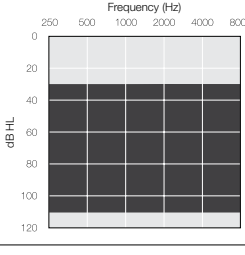
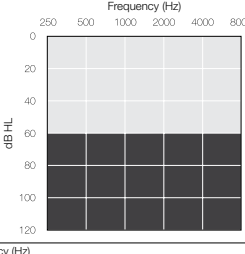
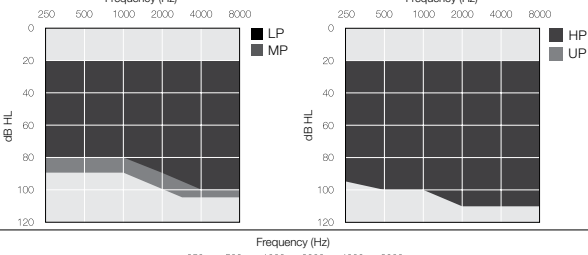
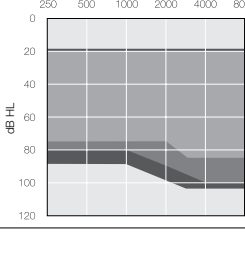


Desktop charger
Easy-to-use
Compatible with RIE 63-DRWC

1 Jespersen et al. (2017)
2 Sjolander et al. (2019)
3 Groth (2018)

* All-day battery on one charge is based on typical use time of up to 24 hours with one charge or up to 20 hours when streaming 50% of the time across all styles.
Battery life can vary by use.
** Only available for ITC
*** Only available for ITE

For mild to profound hearing loss

Style	Features	Fitting range	Max output/gain†
 RIE 63-DRWC Rechargeable  RIE 63-DRW 312 battery  RIE 64-DRW 13 battery	- All-day battery on one charge* - Receiver options: LP, MP, HP and UP - Programmable push button or toggle - Telecoil and DAI in 64 - IP68 certified		LP: 113/52 MP: 116/58 HP: 120/65 UP: 128/75
 miniBTE 66-DW 312 battery	- Programmable push button - Telecoil - IP68 certified		BTE (open): 123/52 BTE (closed): 121/56
 BTE 76-DW 13 battery	- Push button and toggle - Telecoil and DAI - IP68 certified		BTE (open): 128/52 BTE (closed): 126/57
 Power BTE 86-DWH 13 battery	- Push button and toggle - Telecoil and DAI - IP68 certified		Power BTE: 134/73
 Super Power BTE 95-DW 675 battery	- Push button and toggle - Telecoil and DAI - IP68 certified		Super Power BTE: 141/83
 ITC-DW / ITE-DW 312 / 13 battery	- Receiver options: MP, HP and UP - Shell options: ITC, Half-shell™ and Full-shell™ - Programmable push button - IP68 certified - Telecoil (optional)		MP: 118/50 HP: 120/60 UP: 129/70
 CIC non-wireless 10A battery	- Cabled fitting required - Receiver options: LP, MP and HP - IP68 certified - Optional push button		LP: 114/40 MP: 116/50 HP: 121/60

Connectivity and accessories

Smartphone app and Remote Control

Empower your patients to take control



Belton HearMax app



Belton
Remote Control

Belton HearMax™ app

Access to settings at all times using a mobile device

Belton Remote Control

Control at their fingertips without a smartphone

Auracast Assistant

With the world's first Auracast Assistant for hearing aids*, every phone that is compatible with the latest version of the Belton HearMax™ app can connect Auracast-compatible hearing aids to broadcasts.

Phone calls

Hands-free calls and streamed audio



Bluetooth Low Energy Audio

Connecting hands-free to Bluetooth 5.3 compatible devices*

Hands-free calls for iPhone and iPad

Connecting hands-free to iPhone and iPad devices**

Direct audio streaming

From iPhone, iPad or Android devices

Multi-Mic+

Help your patients hear through the noise



Belton
Multi-Mic+

Belton Multi-Mic+

Cut through the noise now with the world's first remote mic with Auracast*

TV

Improve TV time for your patients



Belton
TV-Streamer+



Belton
TV Link 2

Belton TV-Streamer+

For immediate TV speech clarity at their preferred volume. Supports Auracast broadcast audio*

Belton TV Link 2

Enjoy crystal clear TV sound

* Compatible with Belton Envision, Belton Commence, Belton Boost Max S and Belton Serene

** Compatible with Belton Envision, Belton Commence, Belton Serene, Belton Boost Max S and Belton Achieve

Feature overview

Features	Description
Sound Quality	
 WDRC bands	Replicates the way the ear divides sound into distinct pitches to create a natural and clear sound picture.
 Environment recognition	Automatically adjusts the volume and the amount of noise reduction based on the environment for a comfortable hearing experience.
 Directionality	Binaural hearing strategy uses real time information about the listening environment to switch between different sound settings automatically for better hearing with less effort in all daily life situations.
 Speech intelligibility in noise	Forms a stronger directional pattern using the sound from both hearing aids, helping to hear through the noise to enjoy the best 1:1 hearing experience.
 Noise reduction	Reduces disturbing background noise without affecting speech audibility and listening comfort even when moving through rapidly changing sound environments.
 Comfort	Reduces low level noise in very noisy environments. Quickly detects and reduces amplification for sudden, brief loud noises and reduces wind noise without lowering gain for other sounds.
 Feedback management	Reduces unwanted feedback in nearly any situation without compromising gain, while ensuring that music is correctly identified and not lowered.
 Low frequency boost	Optimizes loudness for users with severe and profound hearing loss. Boosts the listening experience with a louder, fuller sound quality when needed.
 Sound Shifter	Improves audibility of speech cues that would otherwise be lost while maintaining best possible sound quality.
Connectivity	
 Direct audio streaming	Enjoy the convenience of streaming audio directly from an iPhone, iPad or Android device in everyday situations like listening to music or taking a call.
 Public connectivity	Participate in public gatherings via built-in Telecoil or Auracast broadcast audio.
 Accessories	Wireless accessories to support in challenging listening situations in 1:1 or group settings. Provide audio streaming from devices including TVs and PCs.
MultiConnect	Seamlessly switch between two paired smart devices for uninterrupted streaming.
Fitting Flexibility	
 Wireless CROS	CROS (Contralateral Routing of Signals) and BiCROS (Bilateral Contralateral Routing of Signals) transmit sounds from the CROS device to the other hearing aid for significant improvements in sound audibility all around.
 Programs	Preset hearing profiles that allows for greater control of your patients' hearing in different situations.
 Fitting options	Synchronized SATISFY, Tinnitus Breaker Pro, Datalogging, In-situ Audiometry.

For a detailed feature overview across styles, please refer to the data sheets.

Premium				Premium			
Beltone Envision				Beltone Boost Max S			
ENV17	ENV9	ENV6	ENV4	BMS17	BMS9	BMS6	BMS4
17	14	12	12	17	14	12	12
Smart Gain Pro	Smart Gain	Smart Gain	Smart Gain	Smart Gain Pro	Smart Gain	Smart Gain	Smart Gain
CrossLink Directionality 4	CrossLink Directionality 3	CrossLink Directionality 3	CrossLink Directionality 2	CrossLink Directionality 4	CrossLink Directionality 3	CrossLink Directionality 3	CrossLink Directionality 2
True Focus*/ Clear Focus	Ultra Focus 2	Ultra Focus	Smart Beam	Clear Focus	Ultra Focus 2	Ultra Focus	Adaptive Directionality
5 settings	3 settings	3 settings	2 settings	5 settings	4 settings	3 settings	2 settings
3 settings	3 settings	2 settings	2 settings	3 settings	3 settings	2 settings	1 settings
Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2
3 settings	3 settings	3 settings	3 settings	3 settings	3 settings	3 settings	3 settings
Yes				Yes			
Hands-free calls and direct streaming for iPhone and iPad. Connecting hands-free to compatible Bluetooth 5.3 or later devices, including compatible Android phones.”							
Supports Auracast for new broadcast opportunities, Telecoil built into select styles, as well as Multi-Mic+							
TV-Streamer+, Multi-Mic+, Remote Control & Beltone HearMax App							
microRIE only				Yes			
(Compatible with all RIE styles)				N/A			
4				4			
Yes				Yes			

*True Focus is only available with the microRIE

**Not available with the CIC



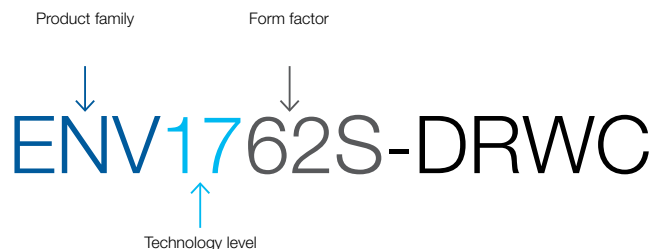
Essential		Premium		
Beltone Commence		Beltone Serene		
COM3	COM2	SER17	SER9	SER6
12	10	17	14	12
–	–	Smart Gain Pro	Smart Gain	Smart Gain
Speech Spotter Pro	Speech Spotter	CrossLink Directionality 4	CrossLink Directionality 3	CrossLink Directionality 2
Smart Beam Steering	Adaptive Directionality	Ultra Focus 2	Ultra Focus	Adaptive Directionality
1 setting	1 setting	5 settings	3 settings	2 settings
1 setting	–	3 settings	3 settings	On / Off
Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2
3 settings	3 settings	3 settings	2 settings	On / Off
Yes		Yes		
Hands-free calls and direct streaming for iPhone and iPad. Connecting hands-free to compatible Bluetooth 5.3 or later devices, including compatible Android phones		Hands-free calls and direct streaming for iPhone and iPad. Connecting hands-free to compatible Bluetooth 5.3 or later devices, including compatible Android phones		
Supports Auracast for new broadcast opportunities, Telecoil built into select styles, as well as Multi-Mic+		Supports Auracast for new broadcast opportunities, Telecoil built into select styles, as well as Multi-Mic+		
TV-Streamer+, Multi-Mic+, Remote Control, & Beltone HearMax App		TV-Streamer+, TV Link 2, Multi-Mic+, Remote Control, & Beltone HearMax App		
N/A		N/A		
N/A		(Only compatible with RIE, BTE, and rechargeable custom styles)		
4		4		
Yes		Yes		

Advanced			Advanced		
Beltone Achieve			Beltone Boost Ultra		
IMP17	IMP9	IMP6	BBU17	BBU9	BBU6
17	14	12	17	14	12
Smart Gain Pro	Smart Gain	Smart Gain	Smart Gain Pro	Smart Gain	–
CrossLink Directionality 4	CrossLink Directionality 3	CrossLink Directionality 2	CrossLink Directionality 2	CrossLink Directionality	Spatial Directionality
Ultra Focus 2	Ultra Focus	Adaptive Directionality	Adaptive Directionality	Adaptive Directionality	Adaptive Directionality
5 settings	3 settings	2 settings	5 settings	3 settings	2 settings
3 settings	3 settings	On / Off	3 settings	3 settings	On / Off
Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser 2	Feedback Eraser	Feedback Eraser	Feedback Eraser
3 settings	2 settings	On / Off	3 settings	2 settings	On / Off
Yes			Yes		
Hands-free calls and direct streaming for iPhone and iPad. Direct streaming for Android devices			Direct streaming for iPhone, iPad and Android devices		
Telecoil built into select styles			Telecoil built into select styles		
TV Link 2, Remote Control, & Beltone HearMax App			TV Link 2, Remote Control, Beltone HearMax App		
N/A			N/A		
N/A			N/A		
4			4		
Yes			Yes		



Essential		
Beltone Rely		
RLY4	RLY3	RLY2
12	8	6
–	–	–
Spatial Directionality	SmartBeam	Adaptive Directionality
Adaptive Directionality	Adaptive Directionality	Adaptive Directionality
On / Off	On / Off	On / Off
On / Off	On / Off	–
Feedback Eraser	Feedback Eraser	Feedback Eraser
On / Off	On / Off	On / Off
N/A		
Direct streaming for iPhone, iPad and Android devices		–
Telecoil built into select styles		
TV Link 2, Remote Control, & Beltone HearMax App		
N/A		
N/A		
4		
Yes		

Nomenclature



Beltone product family

ENV*	Envision
BMS	Boost Max S
COM	Commence
SER	Serene
CSE**	Serene CROS transmitter
IMP	Achieve
BBU	Boost Ultra
RLY	Rely

Technology level

17	Premium level technology
9	Advanced level technology
6	Select level technology
4	Essential plus level technology
3	Essential level technology
2	Entry level technology

Form factors

Receiver-in-ear	
62S	microRIE (Lithium-ion)
62	miniRIE (Lithium-ion)
63	Standard RIE (312)
64	Standard RIE (13)

Customs

CIC	Completely-in-Canal
ITC	In-the-Canal
ITE	In-the-Ear

Behind-the-ear

66	MiniBTE
76	Standard BTE
86	Power BTE
86	High Power BTE
95	Super Power BTE

Additional

D	Dual microphones
R	Receiver-in-Ear
W	Wireless
C	Rechargeable
T	Telecoil
H	High Power BTE (with metal hook)

Receivers

LP	Low Power
MP	Medium Power
HP	High Power
UP	Ultra Power (encased)
MM	M&RIE (Microphone and receiver-in-ear)

Shells

FS	Full-shell
HS	Half-shell

* Available as microRIE CROS transmitter (ENV162S-DRWC)

** Available as microRIE (CSE162S-DRWC) and BTE (CSE176-DWC)

US Headquarters

Manufacturer According to FDA:

Beltone

5005 Dean Lakes Blvd.
Shakopee, MN 55379
U.S.A.
1-800-BELTONE
Beltone.com

Beltone Envision™, Beltone Commence™, Beltone Boost™ Max S, Beltone Serene™ or Beltone Achieve wireless models paired with iPhone, iPad and compatible Android™ mobile devices, your hearing aids will become a hands-free headset. Hands-free calls are not available with Beltone Commence and Beltone Serene non-rechargeable ITC style.

© 2026 GN Hearing A/S. All rights reserved. Beltone is a trademark of GN Hearing A/S. Apple, the Apple logo, iPhone and iPad and are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The Auracast word mark and logos are trademarks owned by Bluetooth SIG, Inc.

