

ReSound Government Services

ReSound Smart Fit 2.3 Navigation & Updates

Webinar FAQs

- How to ask a question:
 - Click Q&A, type your question and Send.
- Where to get the handout:
 - Download from Pending Courses
 - Or Click on link from Chat
- How to earn CEUs:
 - After the course, go to your Pending Courses on AudiologyOnline and pass the multiple-choice exam. Must have paid membership to earn CEUs.
- For assistance or tech support:
 - Call 800-753-2160
 - customerexperience@continued.com
 - Use the Q&A window



Learning objectives

After this course, participants will be able to:

- 1 Describe the initial fit connection flow.
- 2 List the updated features in ReSound Smart Fit™ 2.3 fitting software.
- 3 Identify and utilize common software fine tuning programming features.




ReSound Government Services

ReSound Smart Fit 2.3

Navigation and Updates



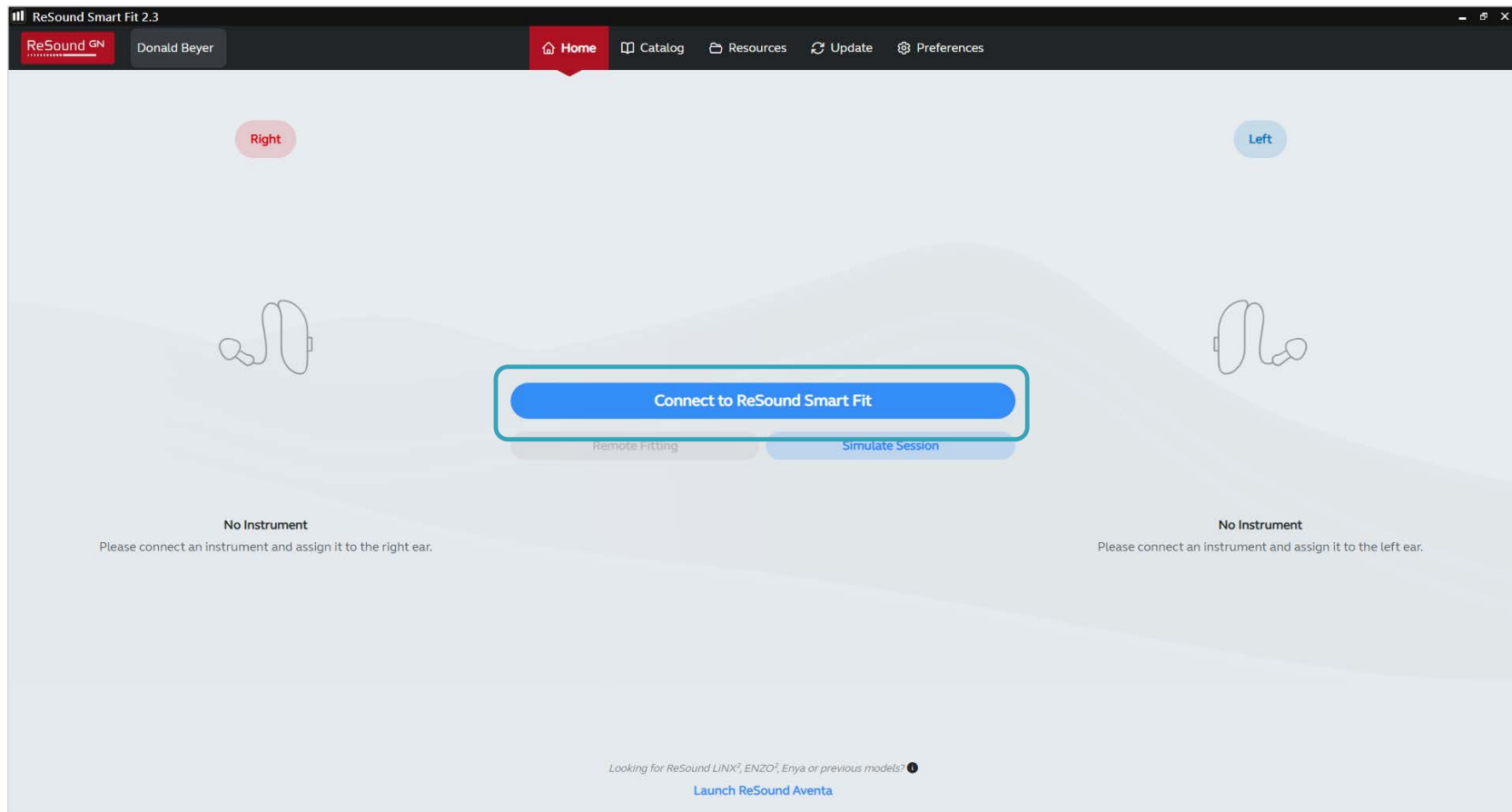

ReSound Smart Fit 2.3

ReSound GN

Donald Beyer

Home

Catalog

Resources

Update

Preferences

Assign Instruments

Noahlink Wireless

Stop

Scanning for instruments



Linked

ReSound Vivia 9

VI960S-DRWC (microRIE) 2 MP Receiver SN:2526911296

Assign via push button

ReSound Vivia 9

VI960S-DRWC (microRIE) 2 MP Receiver SN:2526911297

Assign via push button

Both





Right



No Instrument

Assign Hearing Instrument to this side

Left



No Instrument

Assign Hearing Instrument to this side

Back

AutoFit

Continue

ReSound Smart Fit 2.3

ReSound GN

Donald Beyer

Home

Catalog

Resources

Update

Preferences

Assign Instruments

Noahlink Wireless

Scan

Right

Left



ReSound Vivlia 9
VI960S-DRWC (microRIE)
2 MP Receiver
SN:2526911296

Beep

Unassign



ReSound Vivlia 9
VI960S-DRWC (microRIE)
2 MP Receiver
SN:2526911297

Beep

Unassign

< Back

AutoFit

Continue

ReSound Smart Fit 2.3
Physical Properties

Right Instrument

ReSound Vivia 9 VI960S-DRWC (microRIE) 2 MP Receiver

Earpiece Type: Domes Earmolds

Earpiece: Open AllFit

Tube: 2

Dome Size: S

Use Recommended

Left Instrument

ReSound Vivia 9 VI960S-DRWC (microRIE) 2 MP Receiver

Earpiece Type: Domes Earmolds

Earpiece: Open AllFit

Tube: 2

Dome Size: S

Use Recommended

Copy

Back
Continue

ReSound Smart Fit 2.3

Calibration

● Right Instrument

ReSound Vivia 9 VI960S-DRWC (microRIE) 2 MP Receiver

Unmute

Unmute

● Both Ears

○ Right Ear

○ Left Ear

Target Curves

☒ 50
 ☒ 65
 ☒ 80

Legend

Calibration

✓ Right Instrument

● Left Instrument

ReSound Vivia 9 VI960S-DRWC (microRIE) 2 MP Receiver

Unmute

Unmute

● Both Ears

○ Right Ear

○ Left Ear

Target Curves

☒ 50
 ☒ 65
 ☒ 80

Legend

Calibration

✓ Left Instrument

⚠

Please inform the patient that the calibration tone may be loud. Ensure the environment is as quiet as possible during calibration, including surrounding noise and talking. Please note if you continue without calibration, the patient may experience feedback.

Update Domes

Calibrate

Continue

ReSound Smart Fit 2.3

[Home](#) | [File](#) | [View](#) | [Instrument](#) | [Fitting](#) | [Help](#)

Kara Sinner

Patient Information

Donald Beyer

First Time User

Instruments

Connect

Right

Connected

Left

Connected

VIBROS-DRWC (microRIE)

2 MR Receiver

VIBROS-DRWC (microRIE)

2 MR Receiver

Mute

Mute

Media Player

Collapse Sidebar

Patient

Instrument Set-Up

Gain Adjustments

Advanced Features

Device Controls

Finalize

Fine Tuning

Environmental Optimizer II

AutoREM

Acceptance Manager

100

90

80

70

60

50

40

30

20

10

0

-10

125

250

500

1K

2K

4K

8K

Frequency (Hz)

Mute

Mute

Display Options

Linked

Gain % 100%

Feedback Guard

Above FOG

100

90

80

70

60

50

40

30

20

10

0

-10

125

250

500

1K

2K

4K

8K

Frequency (Hz)

All-Around

Hear in Noise

Program Options

All	250	500	750	1K	1.5K	2K	3K	4K	6K	8K
50	2	5	9	12	14	14	15	10	10	8
65	0	2	4	6	8	7	9	4	4	2
80	0	0	0	1	1	1	3	-1	0	0
CR	1	1.2	1.4	1.7	1.8	1.8	1.7	1.6	1.6	1.4
MPO	96	101	103	105	108	111	110	112	111	110

Handles: 6 10

1

2

3

Use Instrument MPO

All	250	500	750	1K	1.5K	2K	3K	4K	6K	8K
50	2	5	9	12	14	14	15	10	10	8
65	0	2	4	6	8	7	9	4	4	2
80	0	0	0	1	1	1	3	-1	0	0
CR	1	1.2	1.4	1.7	1.8	1.8	1.7	1.6	1.6	1.4
MPO	96	101	103	105	108	111	110	112	111	110

Handles: 6 10

Save

ReSound Smart Fit 2.3

Save

Right Instrument

ReSound Vivia 9 VI960S-DRWC (microRIE) 2 MP Receiver



Right Instrument

☒ Save to Database
 ☐ Save to Instrument

Left Instrument

ReSound Vivia 9 VI960S-DRWC (microRIE) 2 MP Receiver



Left Instrument

☒ Save to Database
 ☐ Save to Instrument

The current fitting session is being saved.

Exit ReSound Smart Fit

Continue Fitting

ReSound Smart Fit 2.3
Save

Right Instrument

ReSound Vivia 9 VI960S-DRWC (microRIE) 2 MP Receiver



Right Instrument

✓ Saved to Database

✓ Saved to Instrument

Left Instrument

ReSound Vivia 9 VI960S-DRWC (microRIE) 2 MP Receiver



Left Instrument

✓ Saved to Database

✓ Saved to Instrument

✓ The current fitting session has been saved.

It is OK to disconnect the instrument(s). After successfully saving, reboot instruments

Exit ReSound Smart Fit

Continue Fitting

Designed with you. For your patients.

ReSound GN


ReSound Smart Fit 2.3



ReSound Smart Fit 2.3

Audiogram selection improvements

ReSound GN

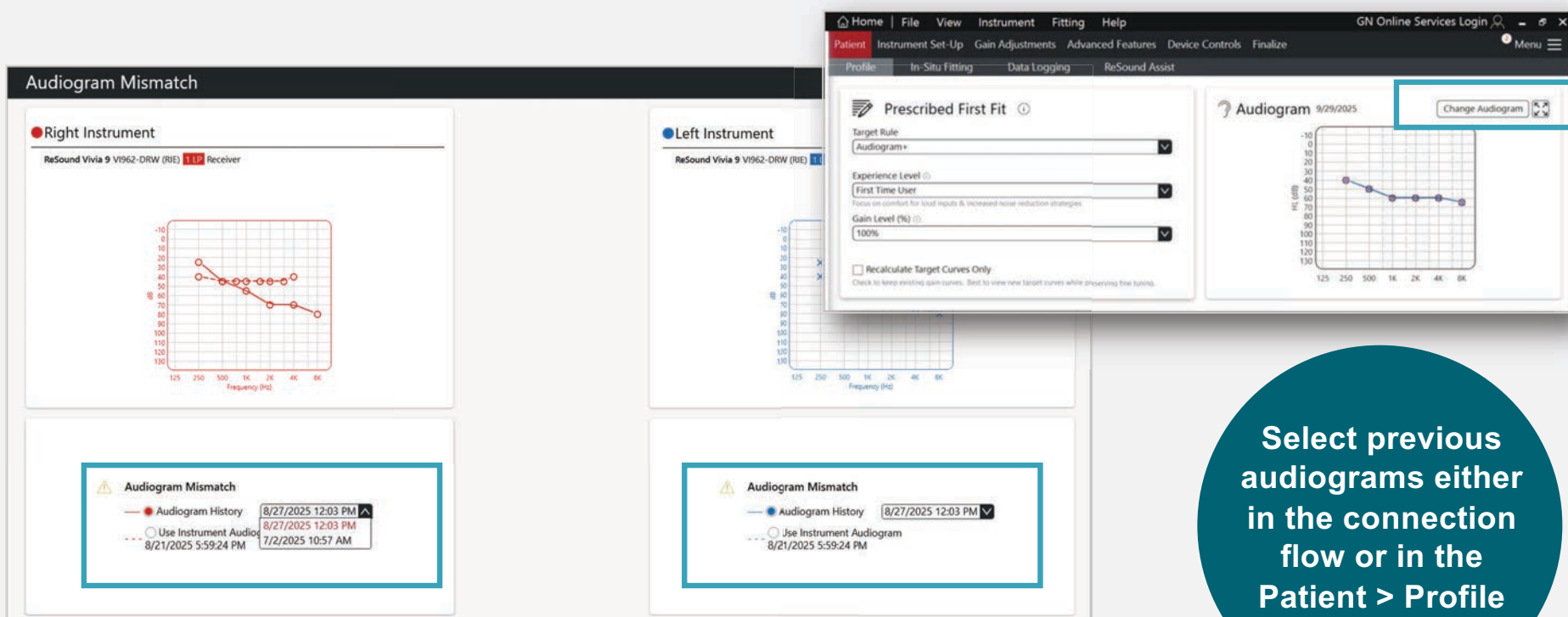

Select previous audiograms either in the connection flow or in the Patient > Profile screen

ReSound Smart Fit 2.3

AutoREM enhancements

ReSound GN




New REAR option

To align with the latest clinical best practice, we have replaced REAG with REAR in the AutoREM feature.

ReSound Smart Fit 2.3

Firmware updates improvements

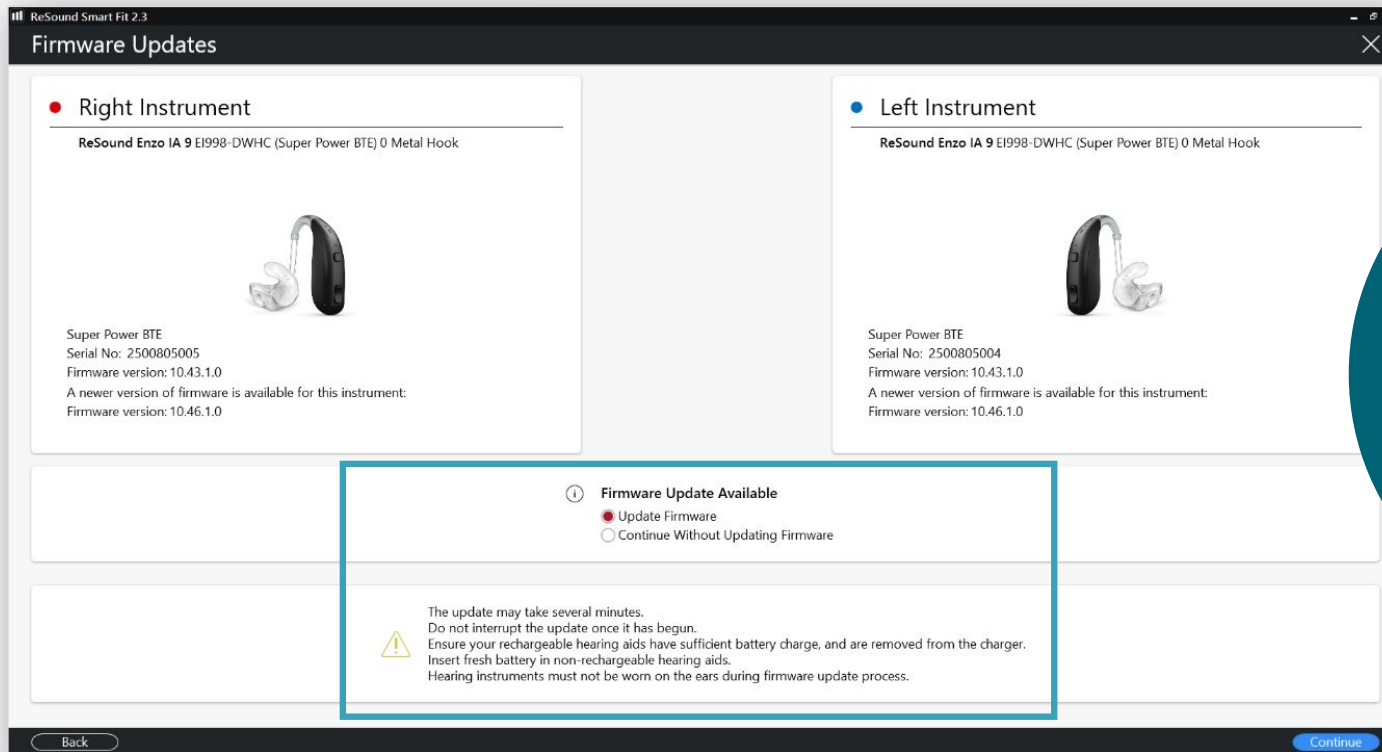


**Firmware update
step added to the
connection flow**

See firmware update
notifications in the
connection flow

ReSound Smart Fit 2.3

ReSound GN



Firmware updates binaural fitting mismatch

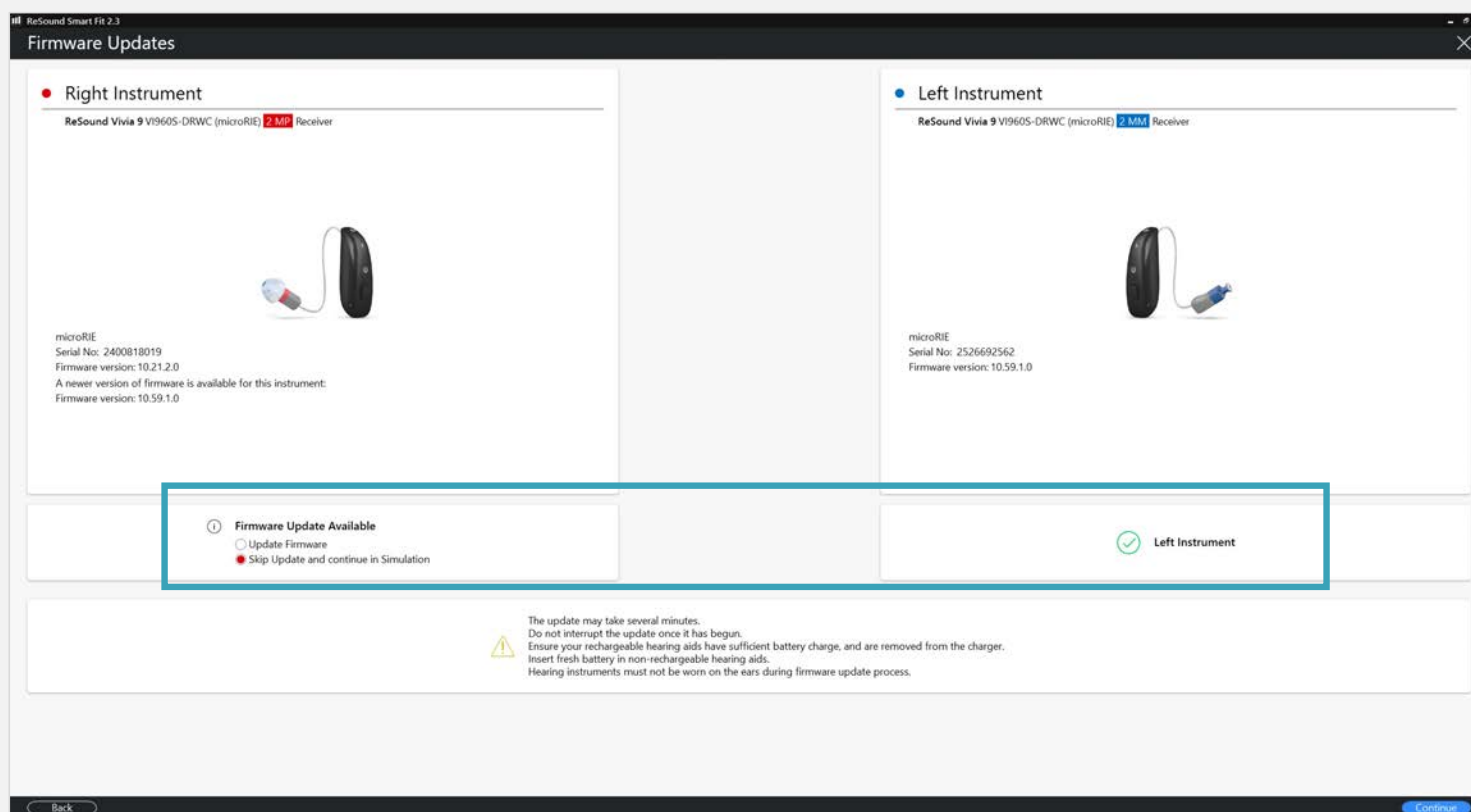

ReSound Smart Fit 2.3

Physical Properties updates

ReSound GN

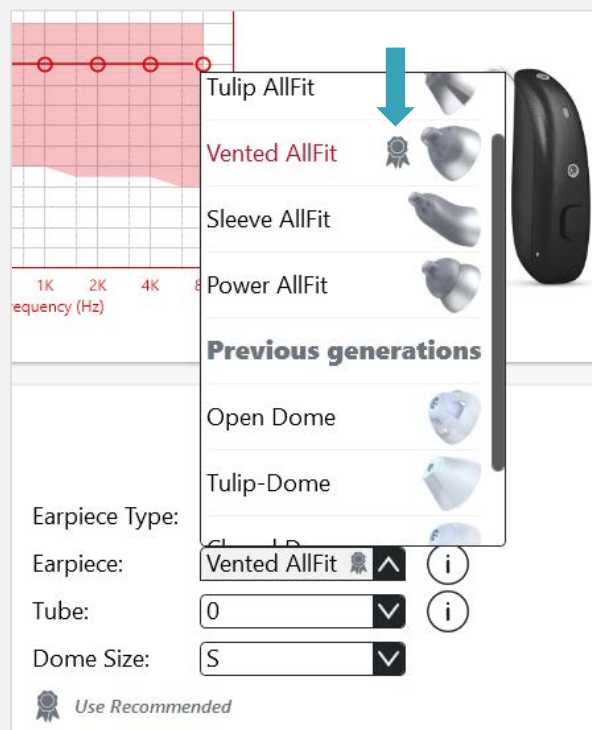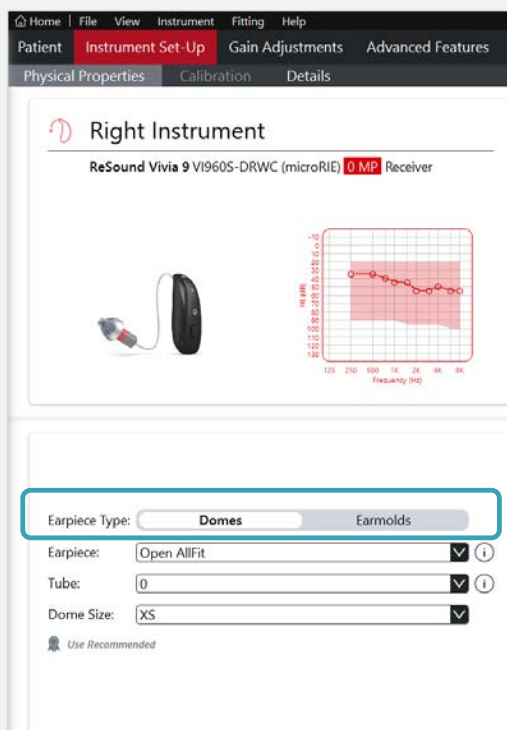

ReSound GN

World's first fitting software with NAL-NL3¹

alongside Audiogram+ to enable you to fit your way

NAL-NL3

The next generation fitting system developed by the National Acoustic Laboratories (NAL) available with ReSound Smart Fit 2.3

*"...built on a foundation of **millions of real-world hearing aid fittings**, enhanced by **machine learning-driven analysis**, **real-ear measurements**, and **in-the-moment user feedback**."*²

1) NAL (2026); 2) [NAL-NL3: The Next Generation Fitting System - NAL](#)




ReSound Smart Fit 2.3


NAL-NL3




Prescribed First Fit > Target Rule

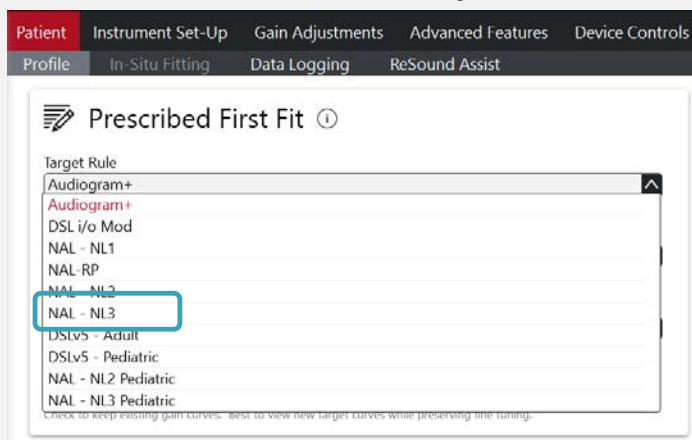

Fitting > Target Rule

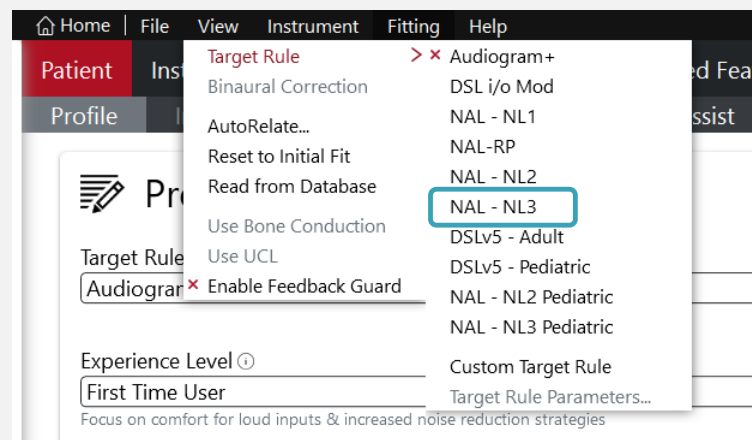

Preferences > Machine Preferences > Default Target Rule

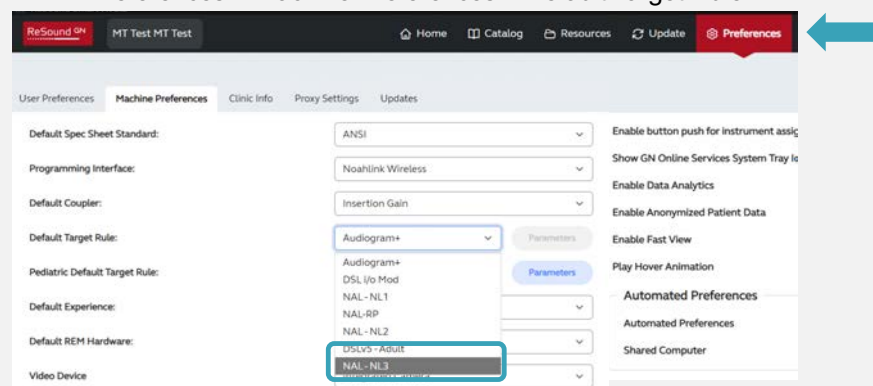

ReSound Smart Fit 2.3



ReSound GN

ReSound Smart Fit 2.3

ReSound Government Services
1-800-392-9932
gs.resound.com

© 2026 GN Hearing A/S. All rights reserved. ReSound is a trademark of GN Hearing A/S.

