


7 reasons why

ReSound Smart Fit™ will enrich your fitting experience

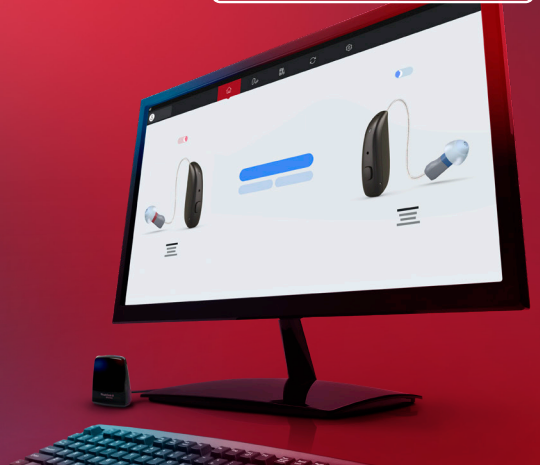

As a hearing care professional, you spend a lot of your day in fitting software. You deserve the best experience. Enjoy smarter and more intuitive software, every time you fit.



Spend less time clicking

Enjoy a more simple experience when navigating the home screen and preferences.



Enjoy a more intuitive experience

With a significant simplification of the user interface, there's now 74% less clutter¹ on the home screen. Your journey through our software has never been cleaner and simpler.



Improved features screen

More easily review the features catalog with new sectioned categories.



Complete verification testing

Use directionality presets in our fitting software to verify directional microphone performance in the test box and complete all verification measurements while staying connected to the hearing aids – in 50% less time than before.²

ReSound Smart Fit is the world's first fitting software supporting NAL-NL3™, the next generation prescriptive fitting system by National Acoustics Laboratory.



Personalized onboarding for your patients

Provide your patients with personalized guidance by printing onboarding reports with all the features and guidance they need.



Revamped home screen

Stay up to date on software, accessory firmware, and all our latest materials, now directly accessible from the home screen.



Adapts to your way of working


Select your favorite settings to become part of your standard fitting flow. For your convenience, ReSound Smart Fit will also keep track of your fittings and learn from them. If it recognizes that you do the same thing consistently, it will automatically suggest that preference be set for your fitting flow moving forward.

¹Compared to ReSound Smart Fit 1.17, technical data on file


²50% time saving compared to ReSound Smart Fit 2.1 (from 5.36 mins to 2.54 mins)

6 easy steps to fit ReSound

- 1 Launch ReSound Smart Fit and put the hearing aids in discovery mode by removing them from the charger or closing the battery door for non-rechargeable models. Click **Connect to ReSound Smart Fit**.
- 2 The hearing aids will appear in the **Assign Instruments** section. Use the **Push to Assign** feature to assign to the respective right and left ear placement. Click **Continue** in the lower right of the screen. Select appropriate physical properties and click **Continue** to proceed through connection process.
- 3 Calibrate the hearing aids on the patient's ears to activate DFS Ultra III. Click **Continue** in the lower right of the screen. **Note:** Hearing aids will automatically unmute once calibrated.

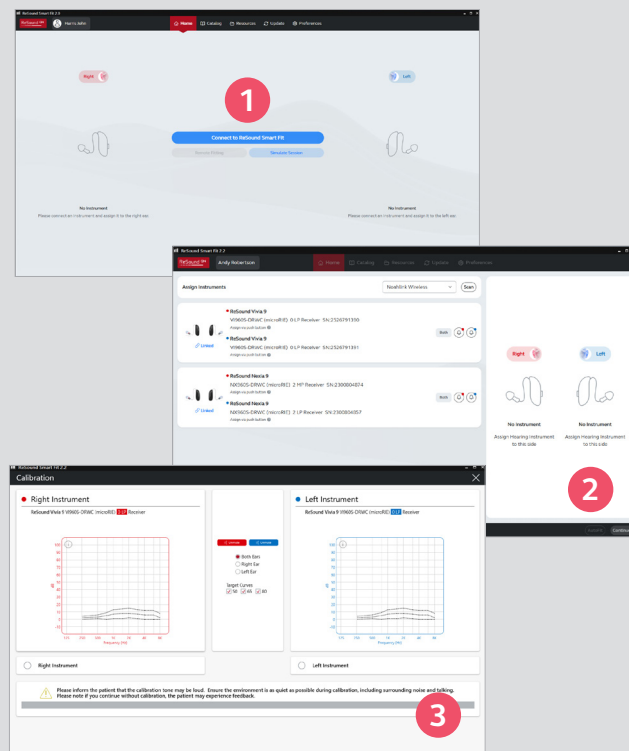

- 4 Navigate to **Patient > Profile** screen, customize the Prescribed First Fit selections as appropriate:
 - a Select Patient **Experience Level** based on previous experience with amplification and to maximize acceptance of hearing aids.
 - b Click **Fit Patient** in the lower right, or navigate to **Gain Adjustments**.
- 5 Fine-tune hearing aids using **Gain Adjustments**:
 - a Add or remove programs from this screen.
 - b Once programming is set, navigate to **Device Controls** to customize the manual push button, demonstrate beeps or pair accessories.
- 6 Click **Save** at bottom right of screen. Devices will reboot and play the start-up melody upon exiting the software.

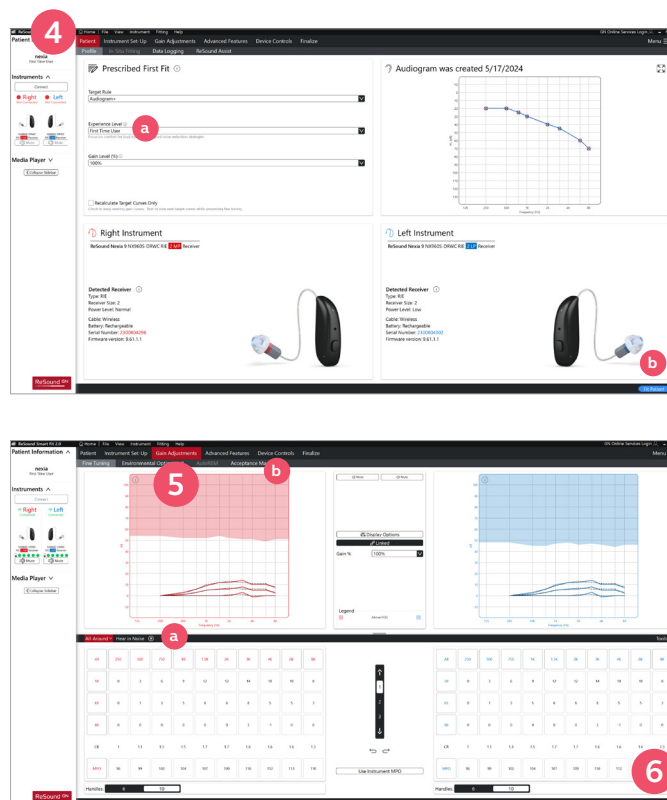

Manufacturer according to FDA:

ReSound Government Services
 5005 Dean Lakes Blvd.
 Shakopee, MN 55379
 USA
 1-800-392-9932
 gs.resound.com