



signia

Signia Government Services May 2026 Connexx Fitting Software Tour

Caitlin Turriff, Au.D.

Government Services May 2026 Connexx Fitting Software Tour

Caitlin Turriff, Au.D.



Webinar FAQs

- Where to get the handout:
 - Resources in the classroom and under Pending Courses in your account
- How to earn CEUs:
 - After the webinar, go to your Pending Courses and pass the multiple-choice exam.
 - Must have a paid membership to earn CEUs
- For assistance or tech support:
 - Call 800-753-2160
 - customerservice@audiologyonline.com

Learning Outcomes

Upon completion of this course attendees will be able to:

- 1) Describe and complete the simple first fit process to fit any Signia hearing aid.
- 2) Discuss how Dynamic Soundscape Processing 3.0 can be used to enrich the Real Time Conversation Enhancement adjustments, especially during follow up appointments.
- 3) List the steps and benefits of running Own Voice Processing 2.0.



Risks/Benefits

- The applicability and accuracy of the course content may vary by the clinical, geographical, and cultural context; you should evaluate the course accordingly.
- As healthcare is rapidly evolving, new findings may change the validity of the information presented in this course. Providers should always consult the latest research and guidelines from professional associations.
- The interventions discussed in this course may be limited in generalizability, requiring practitioners to consider specific individual and population factors.
- The course may not cover all possible risk factors, diagnostic methods, or treatment options, and complex cases may require additional considerations.
- Healthcare providers should apply cultural competence and sensitivity to ensure effective and respectful care based on their patients' diverse needs.
- Providers should always evaluate the limitations of diagnostic and screening tools, considering their potential for false results as well as any ethical implications.
- It is the responsibility of course participants to critically evaluate the evidence from multiple sources to guide professional practice.

One Complete Portfolio. Every Veteran. Zero Compromise.



Pure CG T IX RIC-R
CROS Pure CG IX RIC-R



Pure CG BCT IX RIC-R
CROS CG BCT IX RIC-R



Styletto IX RIC-R
CROS Styletto IX RIC-R



Motion CG M IX BTE-R
Motion CG P IX BTE-R
Motion CG SP IX BTE-R



Insio IX IIC
Insio IX CIC



Insio IX CA
Insio IX HS
Insio IX ITE

NEW



Active Pro IX HS-R
CROS Active Pro IX-R



Silk CG IX CIC-R
CROS Silk CG IX CIC-R




Insio CG IX CIC-R

Restricted



Introducing Insio Charge&Go IX CIC

- The worlds first rechargeable custom CIC with directional speech enhancement in noise

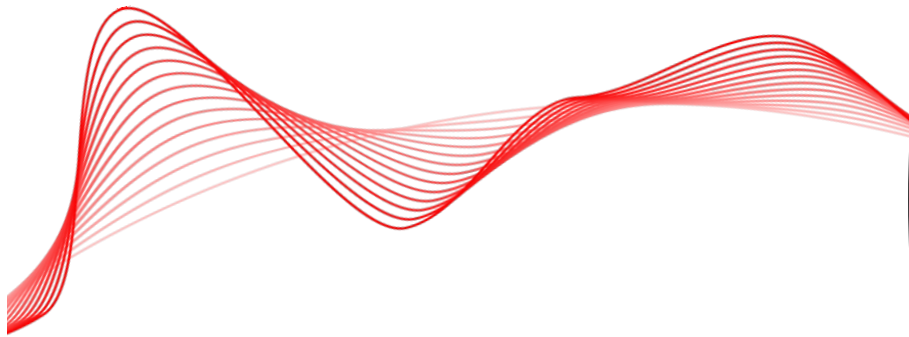

Making the impossible possible with Insio Charge&Go IX CIC

Discreet customized comfort •



With the ease of rechargeability •



**With 5x the speech
enhancement in noise*** •


*Jensen, N. S., Samra, B., Hain, J., & Branda, E. (2025). Binaural OneMic Directionality 2.0 delivers 5 times the speech enhancement in noise versus key competitors. Signia White Paper.

Restricted



All-day power

- Up to 35 hours runtime
- Up to 9 hours runtime after a 30 minute charge
- The pocket-sized portable charging case holds up to 3 additional charges
- Compatible with the CROS Silk Charge&Go IX

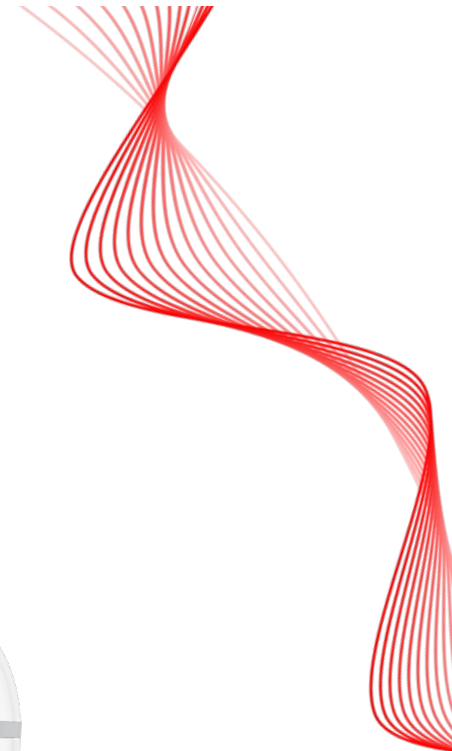

Restricted





Silk Charge&Go IX



signia

Be
Brilliant™



Rechargeability Without Compromise


Silk Charge&Go IX



Fits mild to moderate hearing loss
50dB/114 dB SPL (2ccm coupler)

Power to hear all day
Up to 28 hours on a single charge
30-min fast charge provides additional 7 hours

Binaural OneMic Directionality 2.0
Focus on the speaker in a noisy environment

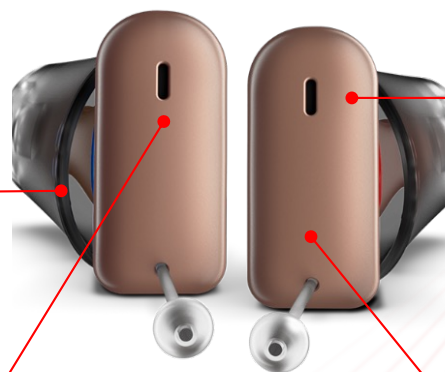

Unmatched Discretion
The soft, textured faceplate surface avoids reflections for added discretion

Convenient Control
Signia App or miniPocket via acoustic pairing



Thoughtful Design for Ease of Use

Silk Charge&Go IX



Active Pro IX
It's a **MODERN** Hearing
Aid



A game changing **modern** prescription hearing aid



Reconnect with the conversation

RealTime Conversation Enhancement – a breakthrough technology that enhances what people are saying even when they move



Modern earbud design

Custom or Instant fit iconic earbud design with premium technology makes Active Pro IX a hearing aid unlike any other



All-day power

Full day wear time up to 34 h and convenient portable charging on the go with Qi wireless charging case



Bluetooth connectivity

Bluetooth streaming for iPhone and Android - Hands-free calling with CallControl to answer/end calls with a simple tap



Water & Dust resistant


IP68 certified



Restricted



Active Pro IX is ready for a full day

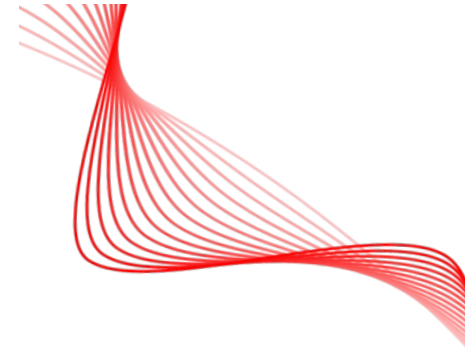

Up to 34 h of wear time*
Up to 29 h wear time with 5 h streaming

Charger holds up to 3
full charges

Quick 30-min charge provides up
to 8 h of wear time

Charge on the go with
Qi wireless charging

**Li-ion cell after 3 years of use under normal operating condition = up to 27 h of runtime*



One Complete Portfolio. Every Veteran. Zero Compromise.



Pure CG T IX RIC-R
CROS Pure CG IX RIC-R



Pure CG BCT IX RIC-R
CROS CG BCT IX RIC-R



Styletto IX RIC-R
CROS Styletto IX RIC-R



Pure 312 AX RIC
CROS Pure 312 AX RIC



Motion CG M IX BTE-R
Motion CG P IX BTE-R
Motion CG SP IX BTE-R



Intuis M 4 BTE
Intuis P 4 BTE
Intuis SP 4 BTE



Active Pro IX HS-R
CROS Active Pro IX-R



Silk CG IX CIC-R
CROS Silk CG IX CIC-R



Insio IX IIC
Insio IX CIC



Insio CG IX CIC-R



Insio CG AX ITE-R
Insio CG AX HS-R
Insio CG AX CA-R



Insio IX CA
Insio IX HS




Insio IX ITE

Restricted




Connexx Agenda


- Easy First Fit
- OVP
- Dynamic Soundscape Processing (DSP)
- Sound Settings
- Audio Streaming
- Connect/Disconnect button
- SCIF compliance
- Acclimatization manager
- Tinnitus

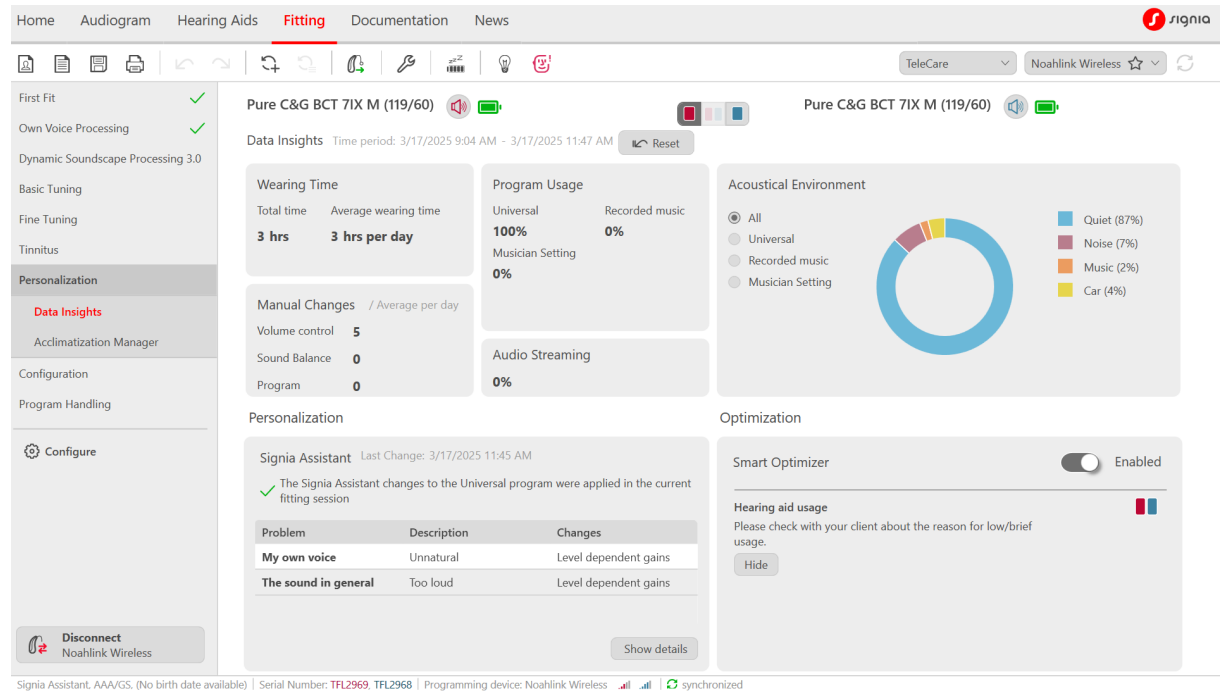

Connexx 9.14.6 – May 2026

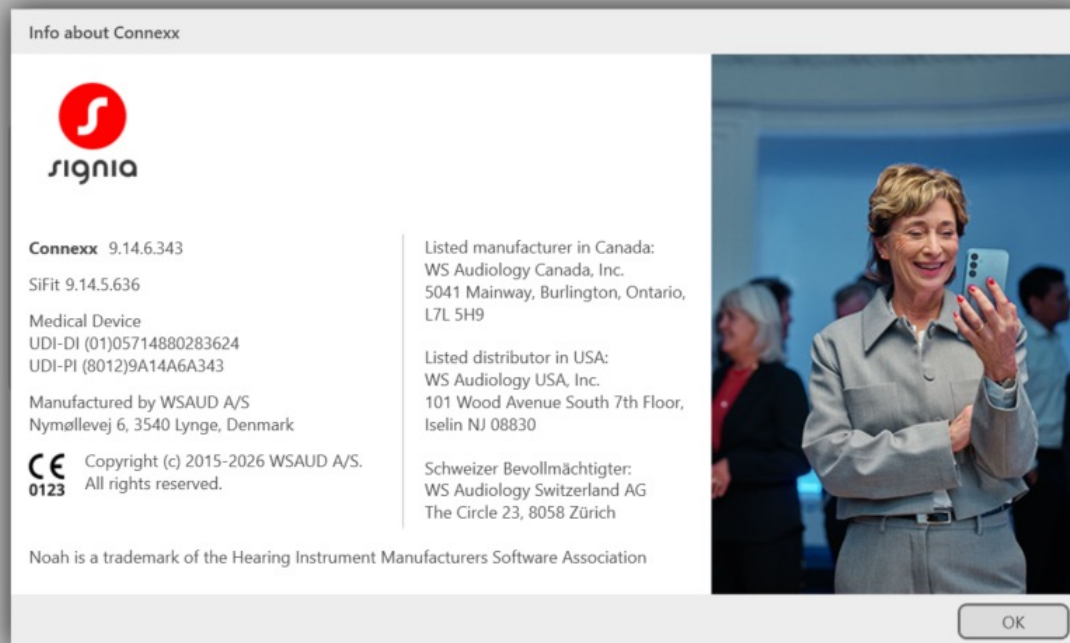

Connexx 9.14.6 – May 2026

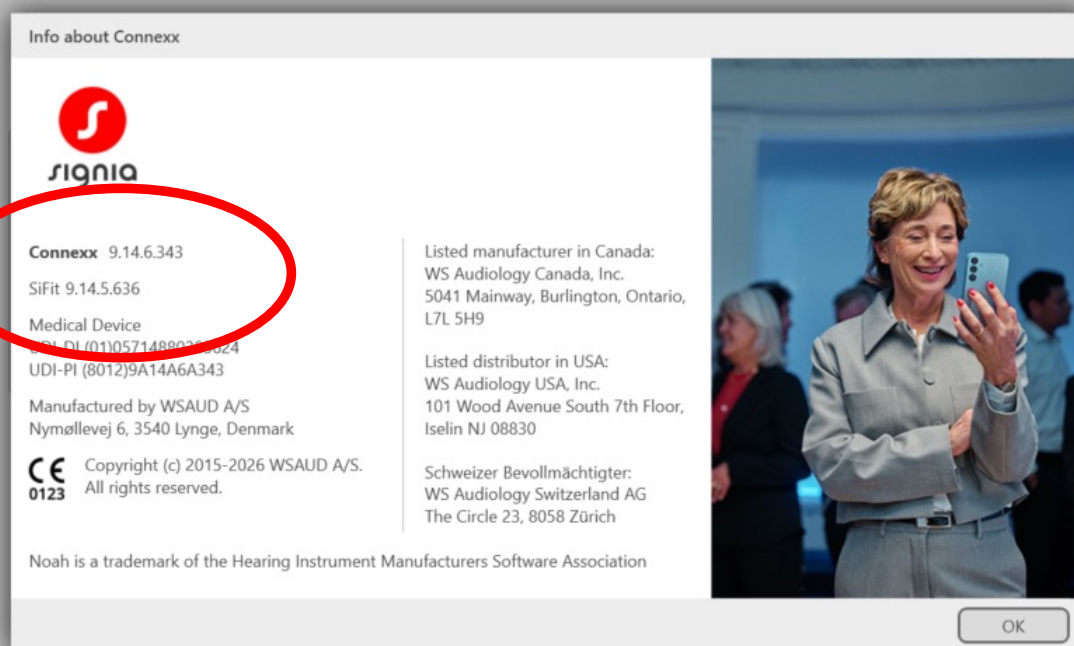



signia

Thank You

veterans.signiausa.com
(800) 955-4327