

Moxi S-RX: Bring on the noise






U Smile,

we all


Smile




Moxi S-R

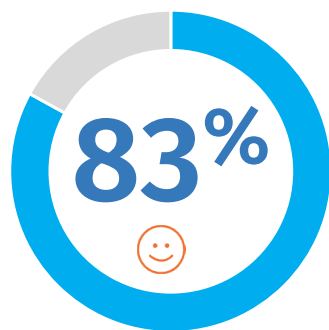

Overall satisfaction

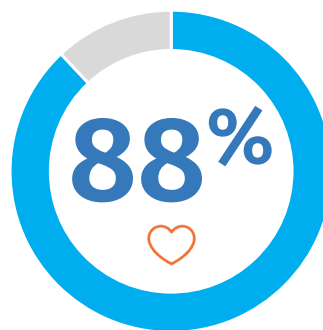

Likelihood to fit in future

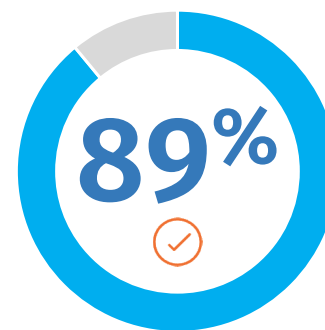

Easy to fit

Sonova proprietary research. (2025). Project ID 4791.
Please contact insights@unitron.com if you are interested in further information.

unitron[™]



U Smile,

we all

Smile



Introducing **Moxi™ S-RX**

on the **Smile** platform



u

Let's Talk about AI

Is AI new to hearing aids?

audiologyonline

Log In

Join Now

[How It Works](#)

[Continuing Education](#) ▾

[Career Center](#) ▾

[Journal](#) ▾

[Partners](#) ▾

[Our Experts](#)

[Group Learning](#)

Artificial Intelligence: The New Advanced Technology in Hearing Aids


Donald J. Schum, PhD

June 7, 2004

[Articles](#) / [Hearing Aids - Adults](#) / Artificial Intelligence: The New Advanced Technology in Hearing Aids


audiotronics™

Unitron has been using AI for over two decades

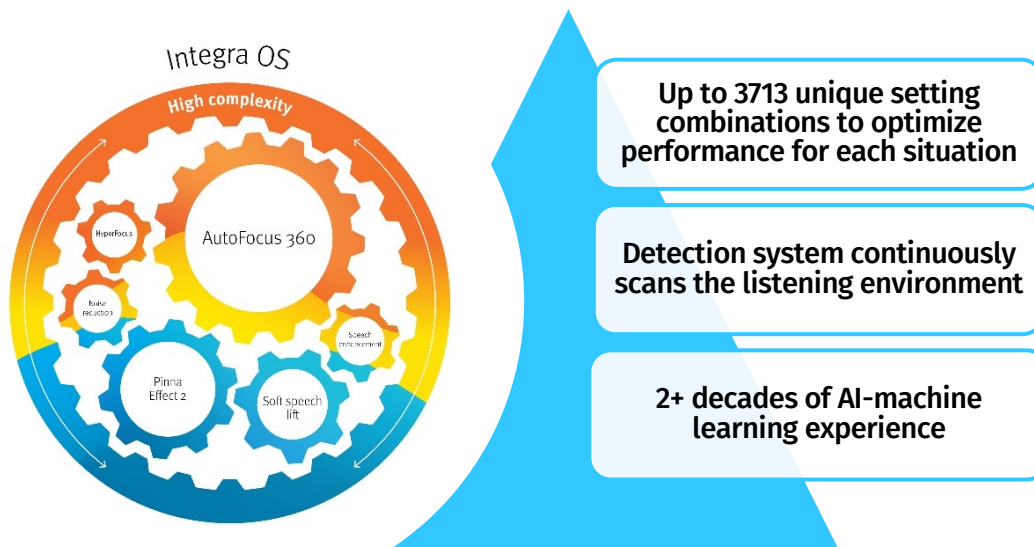

unitron™


Understanding Artificial Intelligence

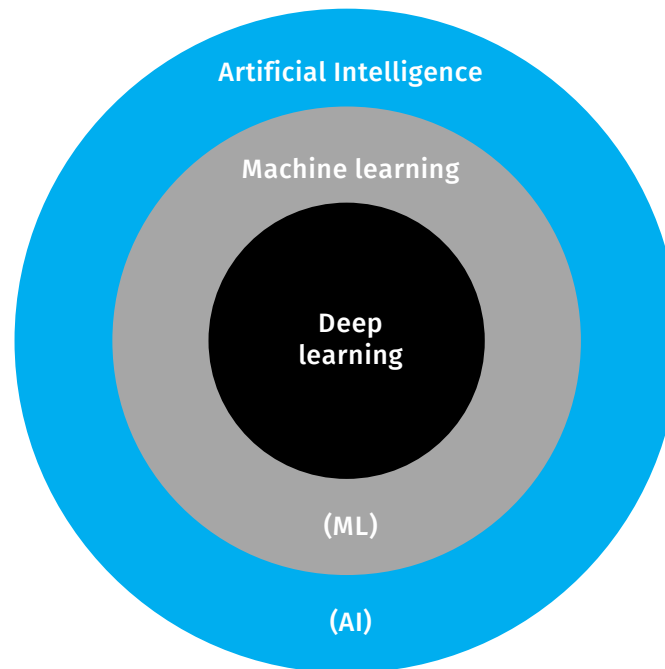

<https://nix-united.com/blog/artificial-intelligence-vs-machine-learning-vs-deep-learning-explaining-the-difference/>

unitron™

What is a Deep Neural Network (DNN)?

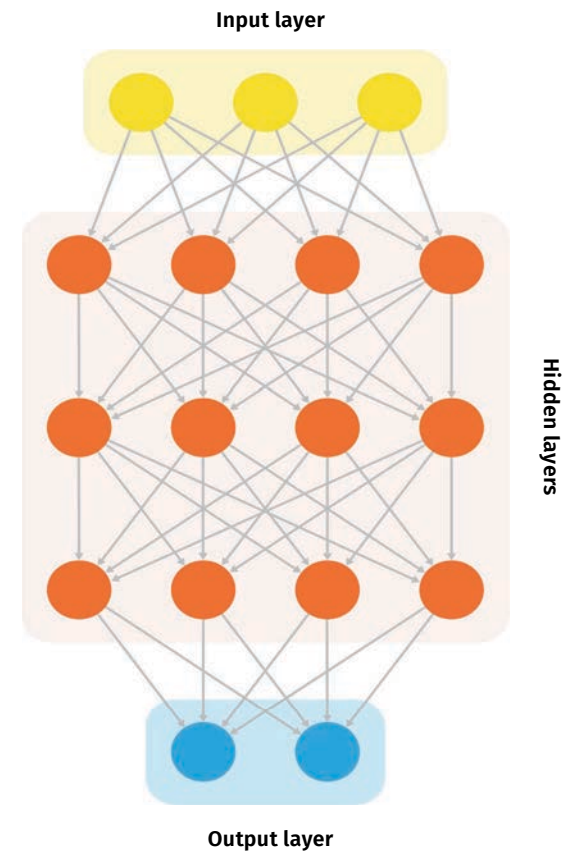

u


**AI and DNNs.
Part of the world we live in.**








Specific situations

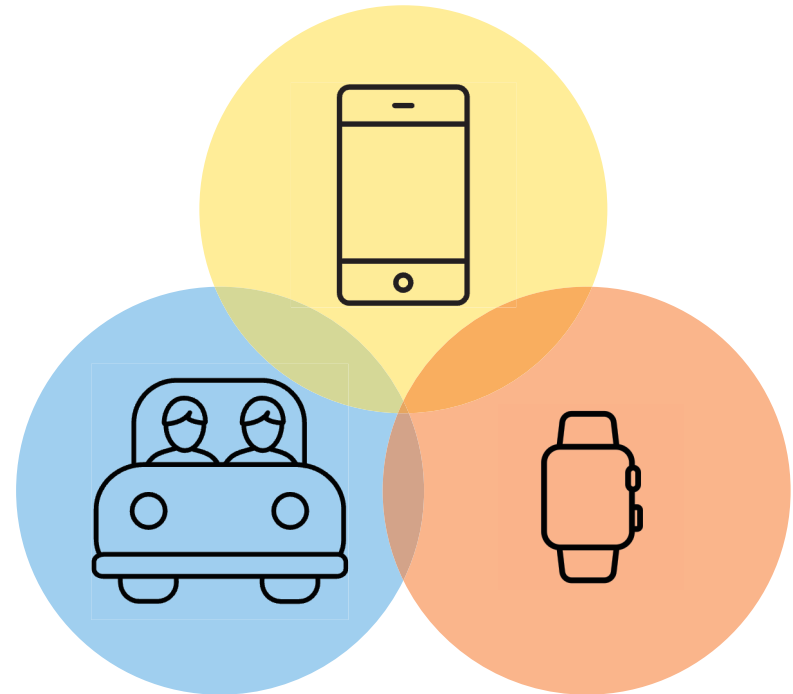

Foundational technology

unitron™

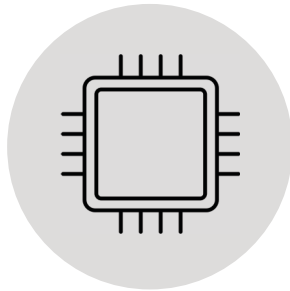


Feature




unitron™

Needing the right tools for the job

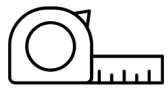

Analyze the situation or task

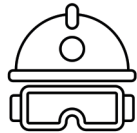

Consider the environment



Reflect on our skill set



Select from available tools

Easy to hear

- A deep-dive on SoundSonic 360
- Proven efficacy for listening in noise
- Universal Connectivity





SoundSonic360

AutoFocus 360

Noise
Reduction

Speech
Enhancement

High complexity

High complexity environments

unitron™

Easy to hear

Easy to wear

Easy for you

Two really is better than one...

Sonova's newest innovation in sound performance, brought to you by DEEPSONIC™



The Sonova DEEPSONIC chip is the world's first dedicated AI chip for all-around speech from noise separation.



The DNN denoising feature, SoundSonic360, can improve speech understanding by reducing noise.



It distinguishes speech and noise and then reduces the noise by up to 10 dB (up to 10 dB SNR improvement).



u

Easy to hear

Easy to wear

Easy for you

Training the Chip

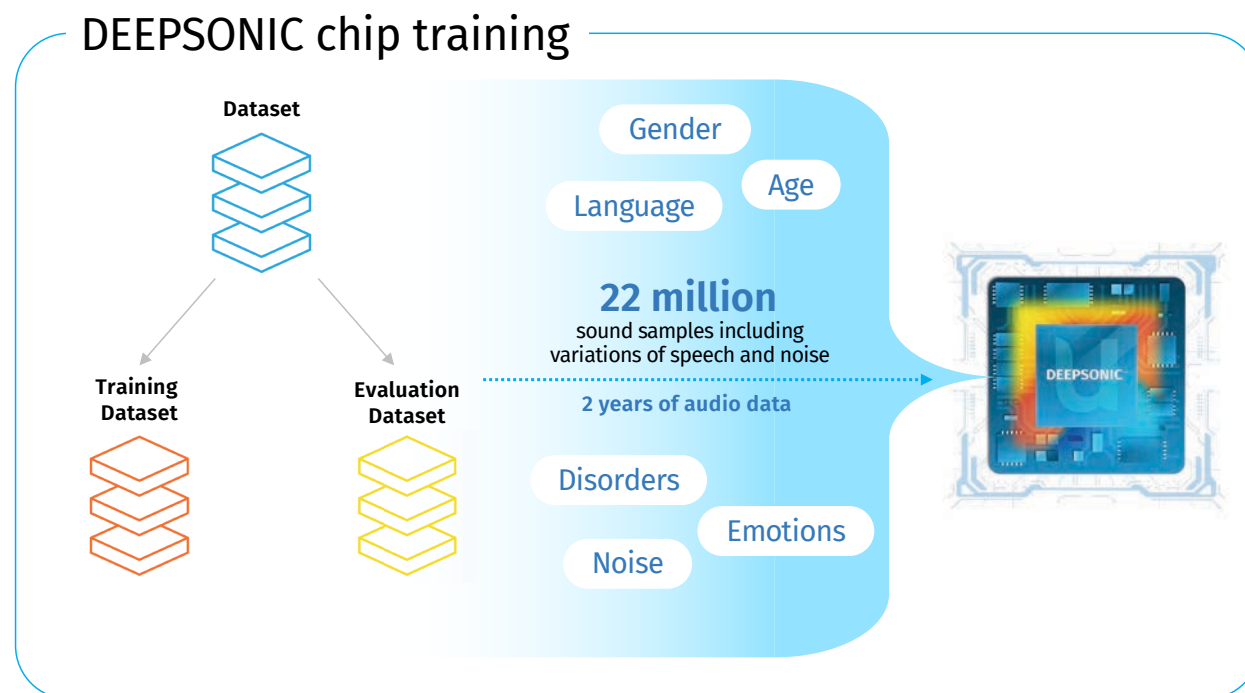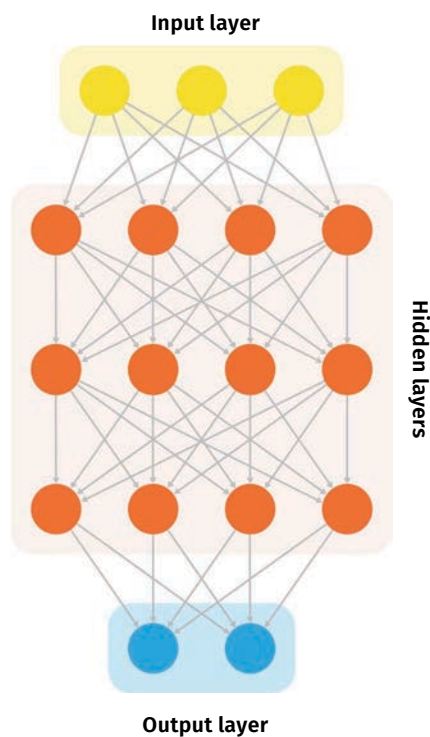

unitron™


Easy to hear

Easy to wear

Easy for you

Confidence in the chaos

Our Deep Neural Network, SoundSonic360, improves listening in loud noise

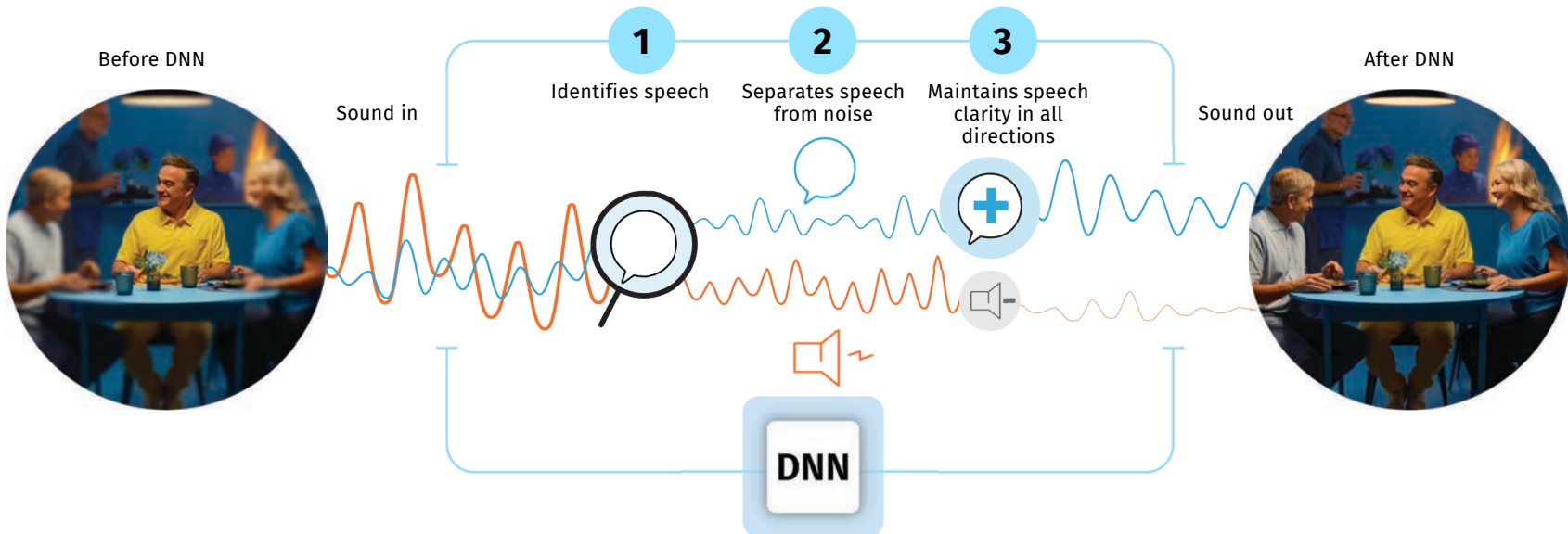

unitron™



Easy to hear

Easy to wear

Easy for you





Making it easier to stay in the game

- SoundSonic360 works within the Conversation in Loud Noise environment
- Conversation in Loud Noise is activated when:
 - Listener is stationary
 - Speech occurs in very loud noise
- Available in monaural fittings **NEW!**

unitron™

Easy to hear

Easy to wear

Easy for you

Who's behind the wheel?

HyperFocus



SoundSonic360



unitron™

Easy to hear

Easy to wear

Easy for you

Easy to hear

Easy to wear

Easy for you

3





Dynamic, just like life

3x

more likely to **understand speech from any direction** in complex listening environments with SoundSonic360¹

Easy to hear

Easy to wear

Easy for you

1. Unitron Research Insights: Better Hearing in Noise with DNN (2026).

unitron™

The positive effect on Listening Effort

Reduces listening effort by up to

37%

compared to binaural beamformer¹




Easy to hear

Easy to wear

Easy for you

1. Unitron Research Insights: Better Hearing in Noise with DNN (2026).

We put it to the test

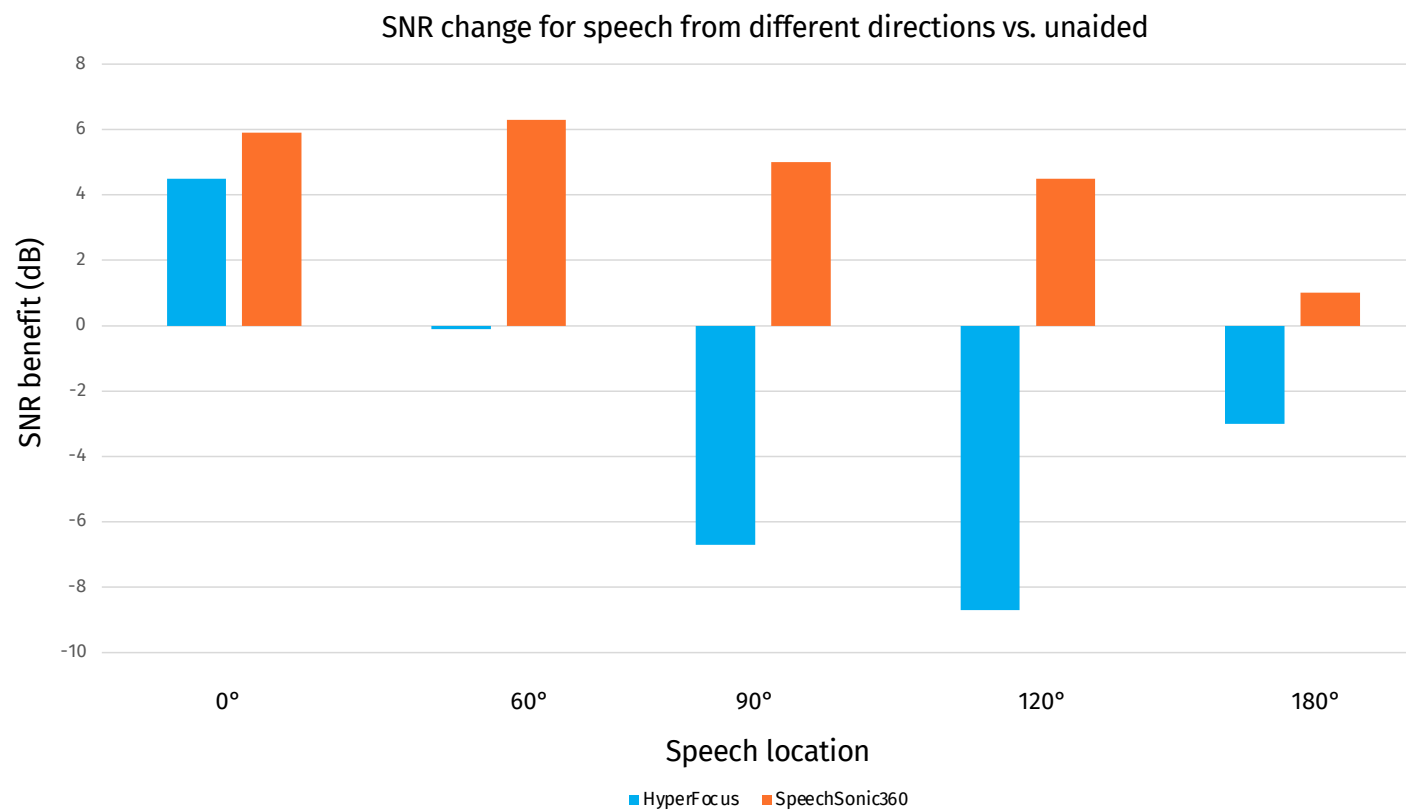

unitron™

Easy to hear

Easy to wear

Easy for you

Noise is no longer a barrier

Conversations at family gatherings, a restaurant, or on a busy street – understanding in noise is the client's number one need.²

10 dB

Unprecedented SNR improvement*

44%

Instant boost to speech understanding

3.7 dB

More SNR benefit compared to key competitors

*At maximum settings

unitron™

Unitron Research Insights: Better hearing in noise with AI

A one-stop-shop for a summary of all the benefits of SoundSonic360

Research Insights:

Better hearing in noise with AI

Pushing the boundaries of what AI can do for you; SoundSonic360 is powered by the DEEPSONIC™ AI chip, isolating voices from noise from any direction.

Harvesting the power of AI
Unitron's SoundSonic360 is powered by the world's first dedicated artificial intelligence chip, DEEPSONIC, to prioritize speech over noise from any direction. It operates in real time by extracting, enhancing, and integrating the speech signal.¹

Noise is no longer a barrier
Conversations at family gatherings, a restaurant, or on a busy street – understanding in noise is the client's number one need.²



Dynamic, just like life

SoundSonic360 enables you to understand conversations from multiple speakers and any angle simultaneously, giving you the freedom to choose which speaker to focus on.²

Clients are 2x more likely to understand speech from any direction in challenging listening situations with SoundSonic360 vs. the off condition, based on statistical modeling.³

The importance of a positive SNR

SoundSonic360 at maximum strength provides instant speech separation from noise all around - with an unprecedented 10 dB SNR improvement compared to omni at 0 degrees.²

SoundSonic360 provides up to 3.7 dB more SNR benefit compared to key competitors.²

SoundSonic360 boosts speech understanding by 44% – instantly – compared to HyperFocus, for subjects with moderate-severe hearing loss.³

Cognitive load & fatigue

SoundSonic360 makes listening easier by reducing the effort by up to 37% compared to HyperFocus – as measured on subjects with moderate-severe hearing loss.³


With SoundSonic360, clients can navigate even noisier environments (by as much as 2.9 dB SNR) without an increase in listening effort – compared to without, for moderate to moderately-severe hearing losses.³

¹ Sonova AG. Data on file. PDL-22112. Please contact insights@unitron.com if you are interested in further information.

² Sonova AG. Data on file. PDL-30520. Please contact insights@unitron.com if you are interested in further information.

³ Sonova AG. Data on file. PDL-22338. Please contact insights@unitron.com if you are interested in further information.

What's in Level 9

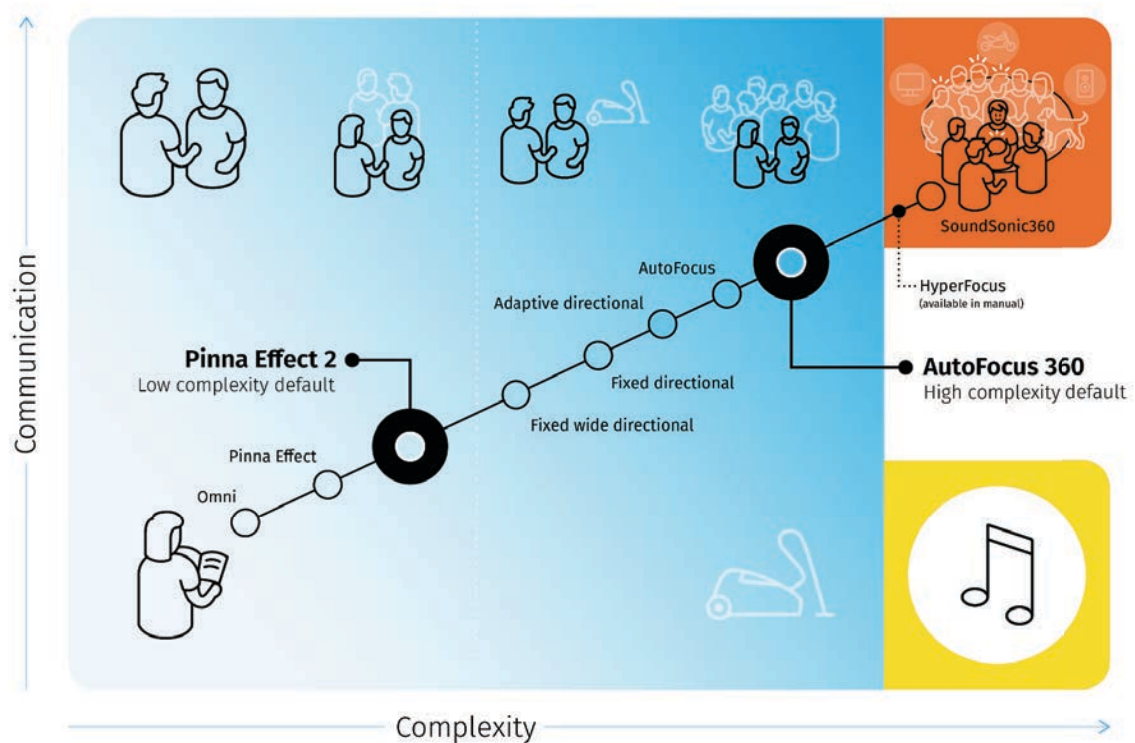

Highlights:

- SoundSonic360 in Integra OS
- Autofocus 360
- HyperFocus as a manual program
- Two new manual programs
 - SpeechSonic360
 - NoiseSonic360


Easy to hear

Easy to wear

Easy for you

What's in Level 7

Conversation in Loud noise – now available for level 7 technology!

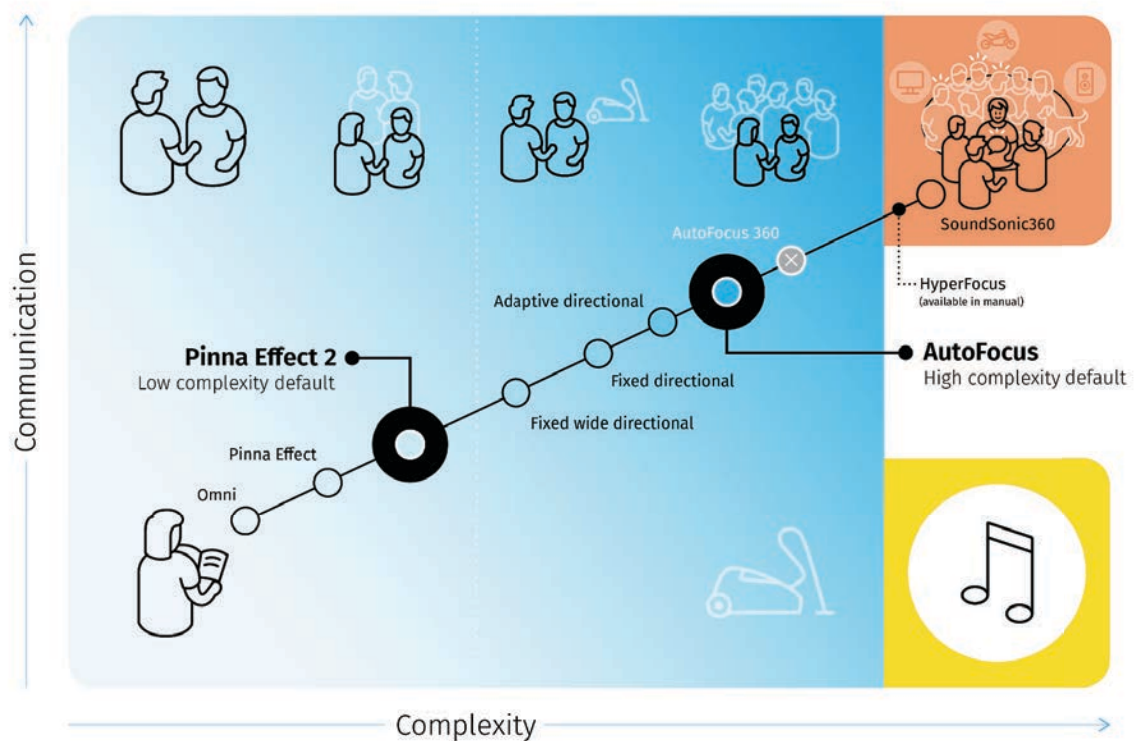

Highlights:

- Conversation in Loud Noise in Integra OS
- Autofocus
- SoundSonic360 in Integra OS (at a lower strength)
 - Motion Detector in Level 7
- HyperFocus as a manual program
- Two new manual programs
 - SpeechSonic360
 - NoiseSonic360

Easy to hear

Easy to wear

Easy for you

Personalize your Fittings

With two new manual programs



SpeechSonic360



NoiseSonic360

unitron™

Easy to hear

Easy to wear

Easy for you

For Level 7 and Level 9 – Two DNN-based OAPs

Optional App Programs (OAPs) for further personalization


Easy to hear

Easy to wear

Easy for you

Slider available to adjust DNN strength

- The AI Noise Filter allows for patient adjustments of the DNN

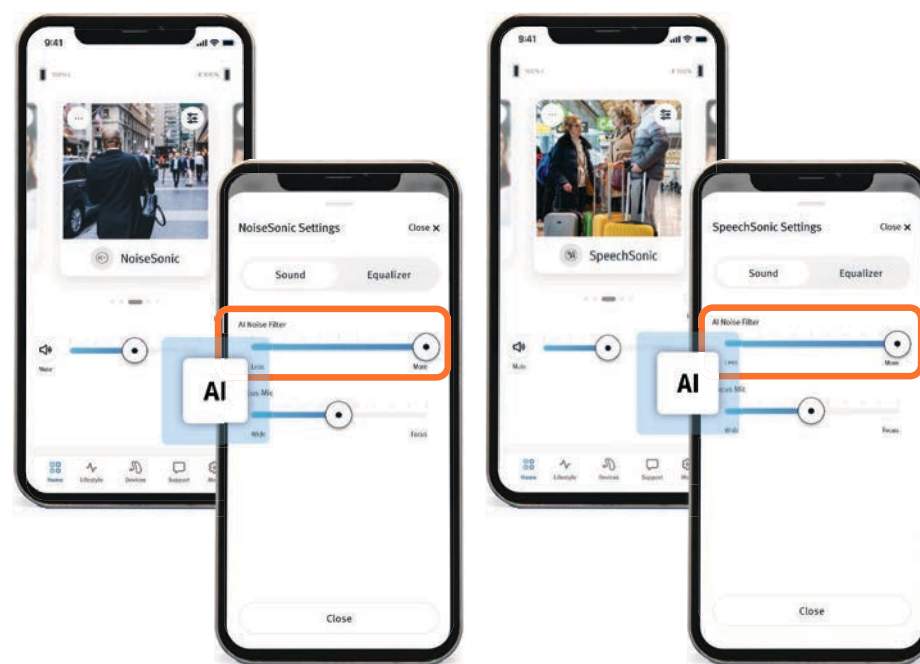

unitron™

u

Any questions?



Easy to wear

- Battery life
- New Hardware
- Charging on the go



Moxi S-RX – New Hardware



- New housing
- New Integrated rocker switch design
- 4 microphone ports
- Water resistant coatings

unitron™

Easy to hear

Easy to wear

Easy for you

Available colors



Amber



Charcoal



Espresso



Pewter



Platinum



Sand



unitron™

Easy to hear

Easy to wear

Easy for you

Moxi S-RX

Battery runtime values



Real world up to

21
hours

Integra OS, including 4 hours of DNN
8 hours of streaming
media/calls/TV



Maximum up to

56
hours

Maximum with Integra OS
in quiet listening situation,
no streaming, no DNN

The battery runtime is determined by battery conditions, hearing loss, environmental classification, streaming usage and activated feature set

unitron[™]

ChargerGo RIC S-RX



unitron™

Easy to hear

Easy to wear


Easy for you

u

Questions?



Universal Connectivity

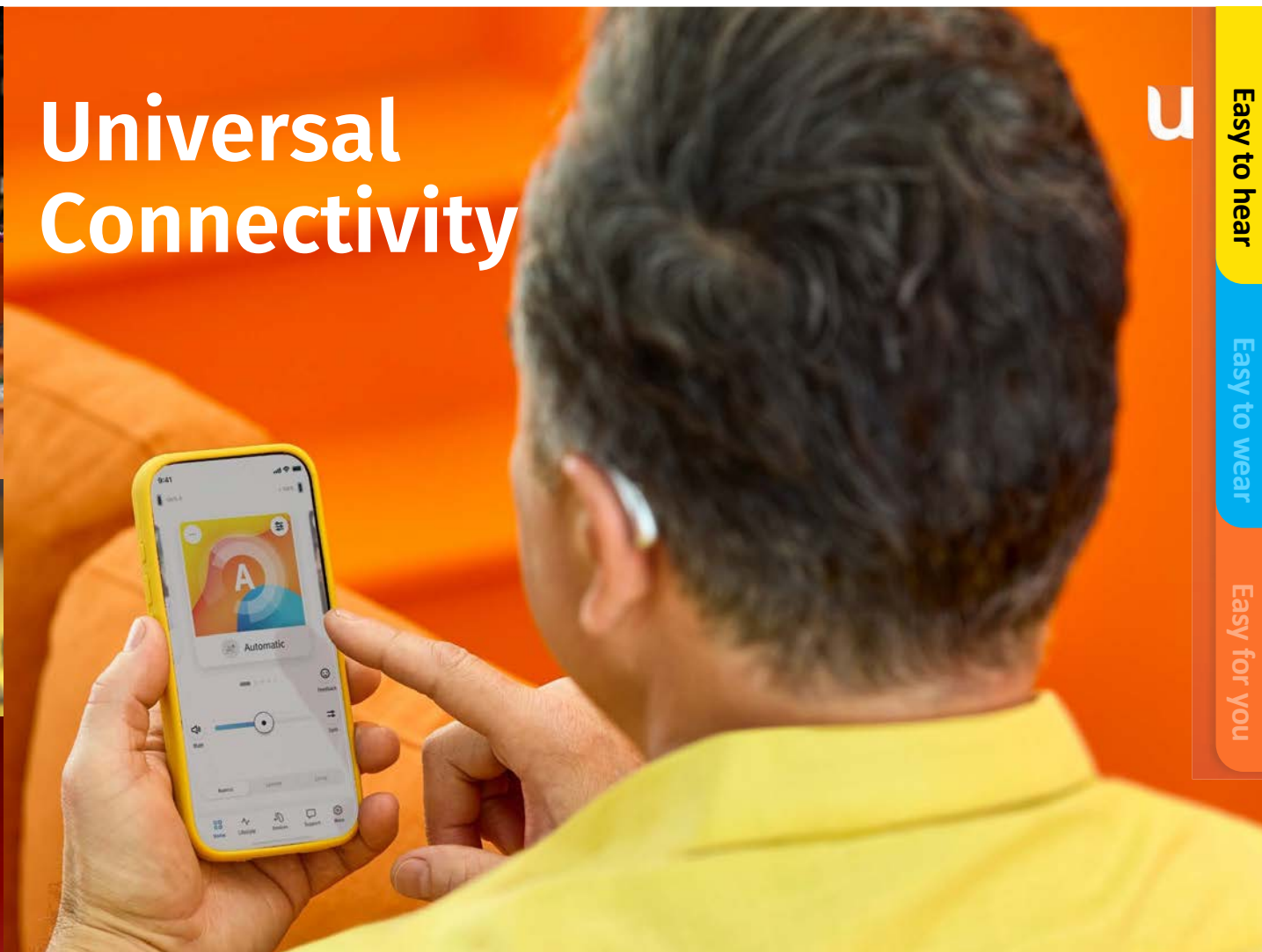


3

Easy to hear

Easy to wear

Easy for you

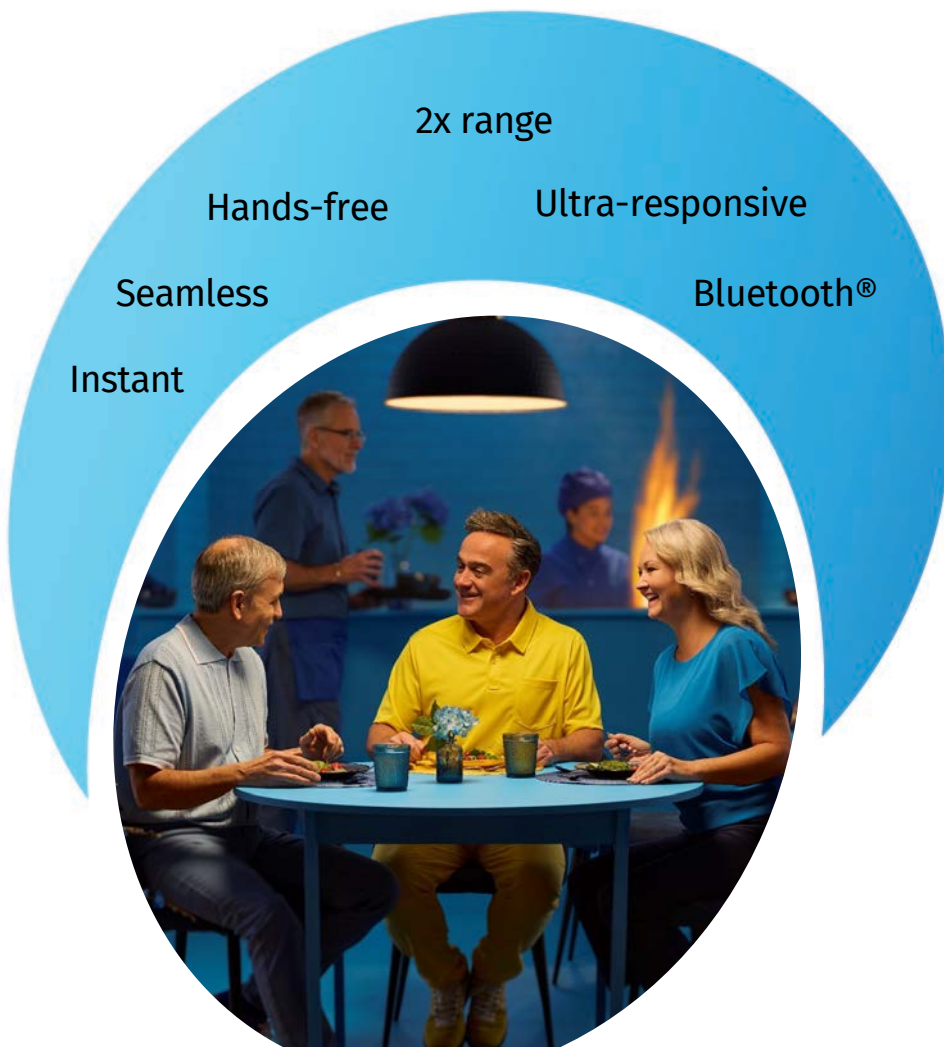

Universal Connectivity

unitron™

Easy to hear

Easy to wear

Easy for you

Universal and hands-free

android 

 iOS



2 active BT device connections


Smartphones and traditional mobile phones with a compatible Bluetooth Hands-Free Profile



One-step pairing for Moxi S-RX

Remote Plus app and streaming together in one pairing process

Remote Plus app and
Bluetooth now paired in
one easy step

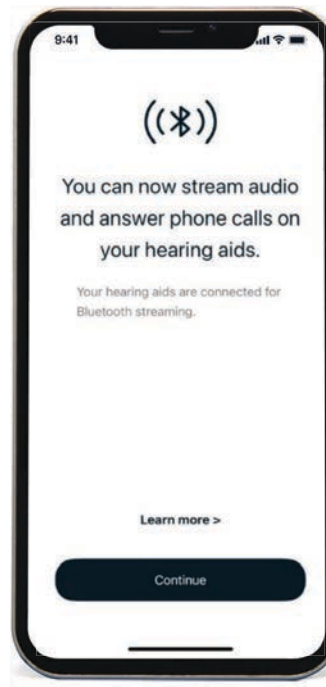


unitron™

Easy to hear

Easy to wear

Easy for you

Pairing Workflow – A Snapshot

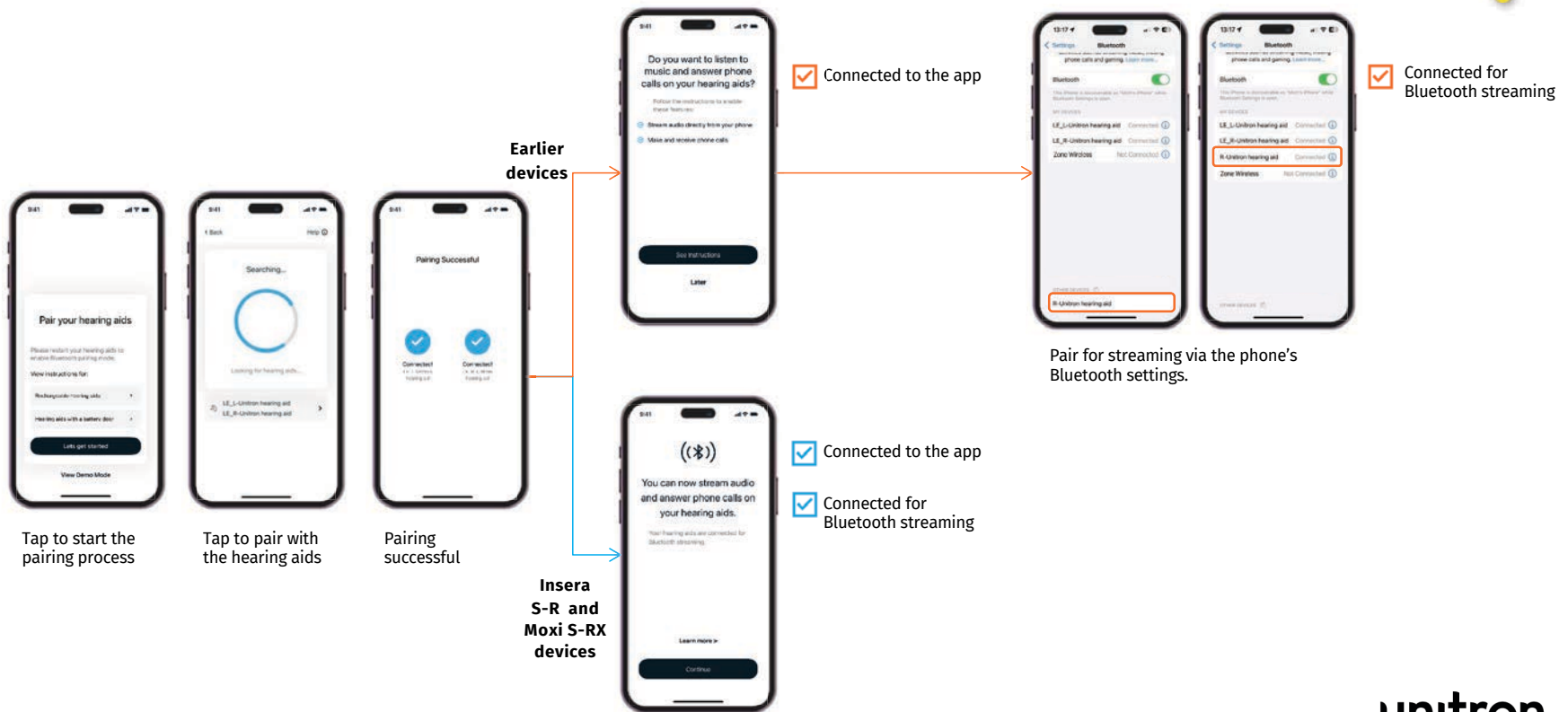

unitron™

Classification of streamed signals

Integra OS	+
------------	---

Media	-
-------	---

Media speech	
--------------	--

Media music	
-------------	--

Wireless	+
----------	---

Optional App	+
--------------	---



RogerDirect™



Easy to hear

Easy to wear

Easy for you

unitron™

Convenient accessories



TV Connector



PartnerMic



Remote Control

unitron™

Easy to hear

Easy to wear

Easy for you



Easy to hear

Easy to wear

Easy for you

u

Questions?

EasyGuard Domes



“Sometimes, little things make a big difference.”

– Nino Varsimashvili



The challenge with earwax

57%

Wax buildup – among the top reasons for **service appointments**¹

33%

of **service cases** can be prevented by patients with a simple wax filter replacement²

30%

Of **patients struggle** with wax guard and dome changes³

!

Wax-related problems result in **lower satisfaction** with hearing aids³

Easy to hear

Easy to wear

Easy for you


unitron™

EasyGuard, for easy wax management

EasyGuard Power



EasyGuard Vented

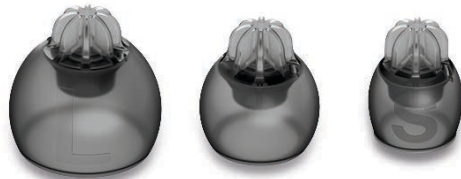

EasyGuard Open



unitron™

Easy to hear

Easy to wear

Easy for you

Easy for you

- TrueFit 5.10 Updates
- Updates to Log It All
- Extended EasyGuard support
- The Remote Plus App



Unitron TrueFit™ fitting software




3

Easy to hear

Easy to wear

Easy for you

Adjusting SoundSonic360

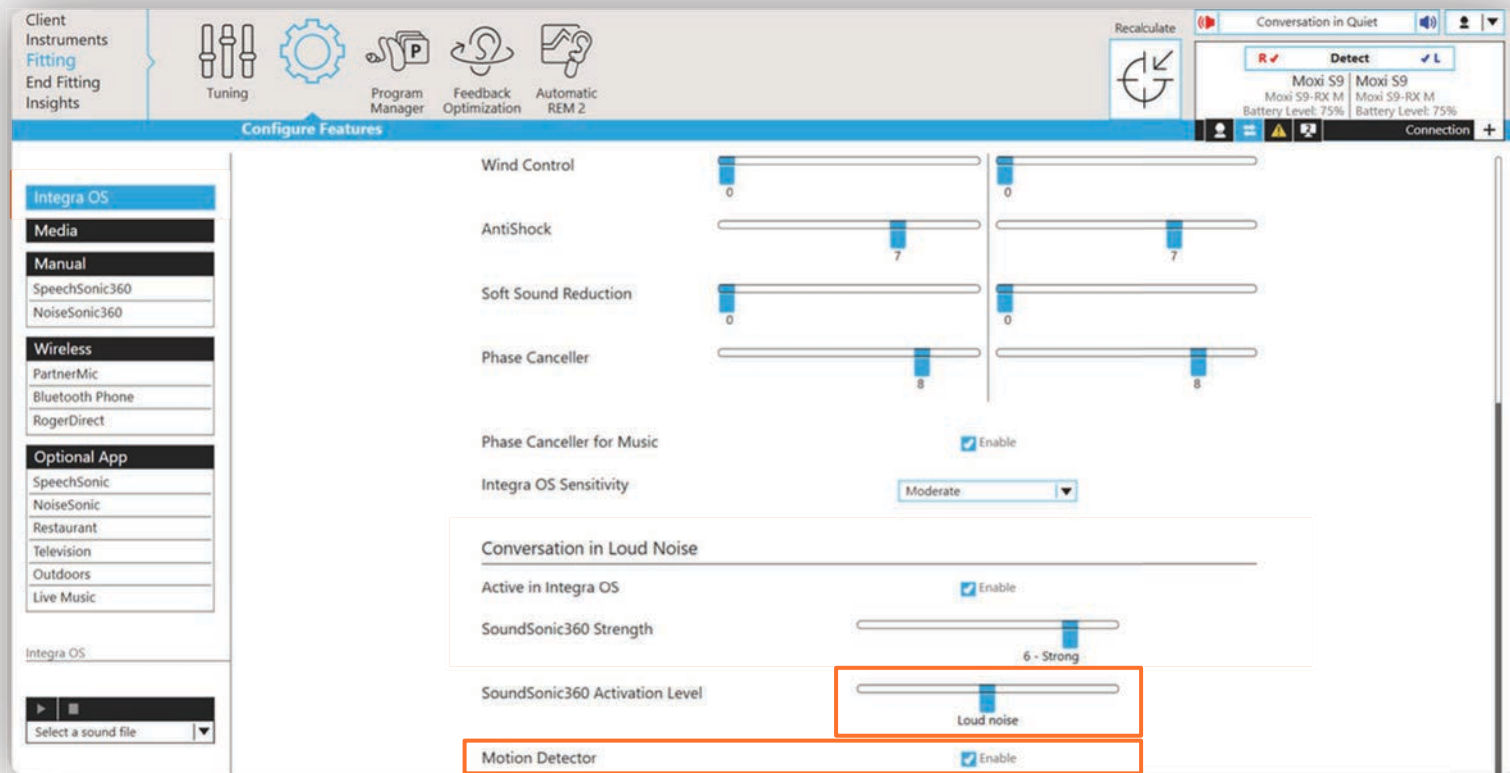


unitron™

Easy to hear

Easy to wear

Easy for you

Adjustments to Manual DNN programs

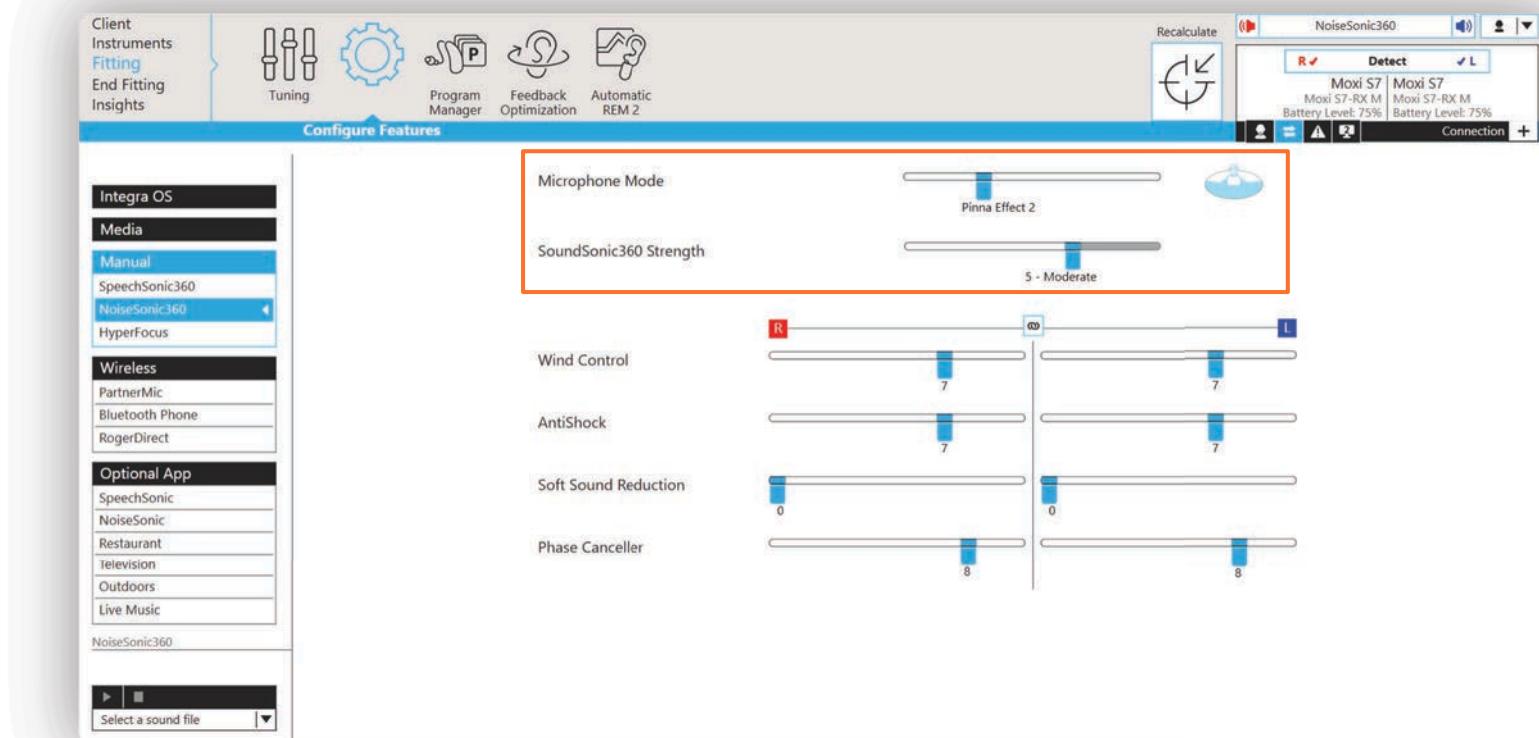


unitron™

Easy to hear

Easy to wear

Easy for you

Enhanced Log It All Performance screen

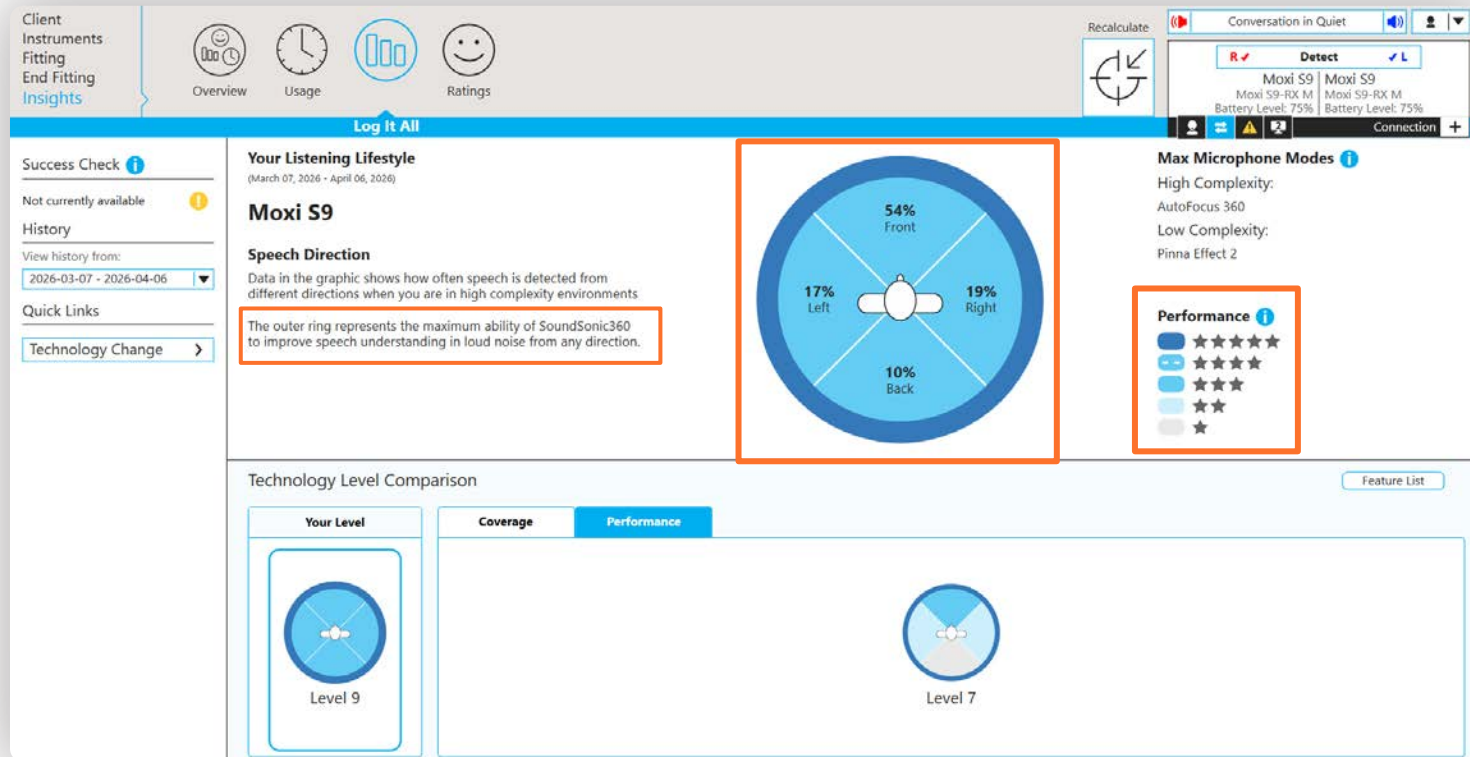


unitron™

Easy to hear

Easy to wear

Easy for you

Enhanced Log It All Performance screen

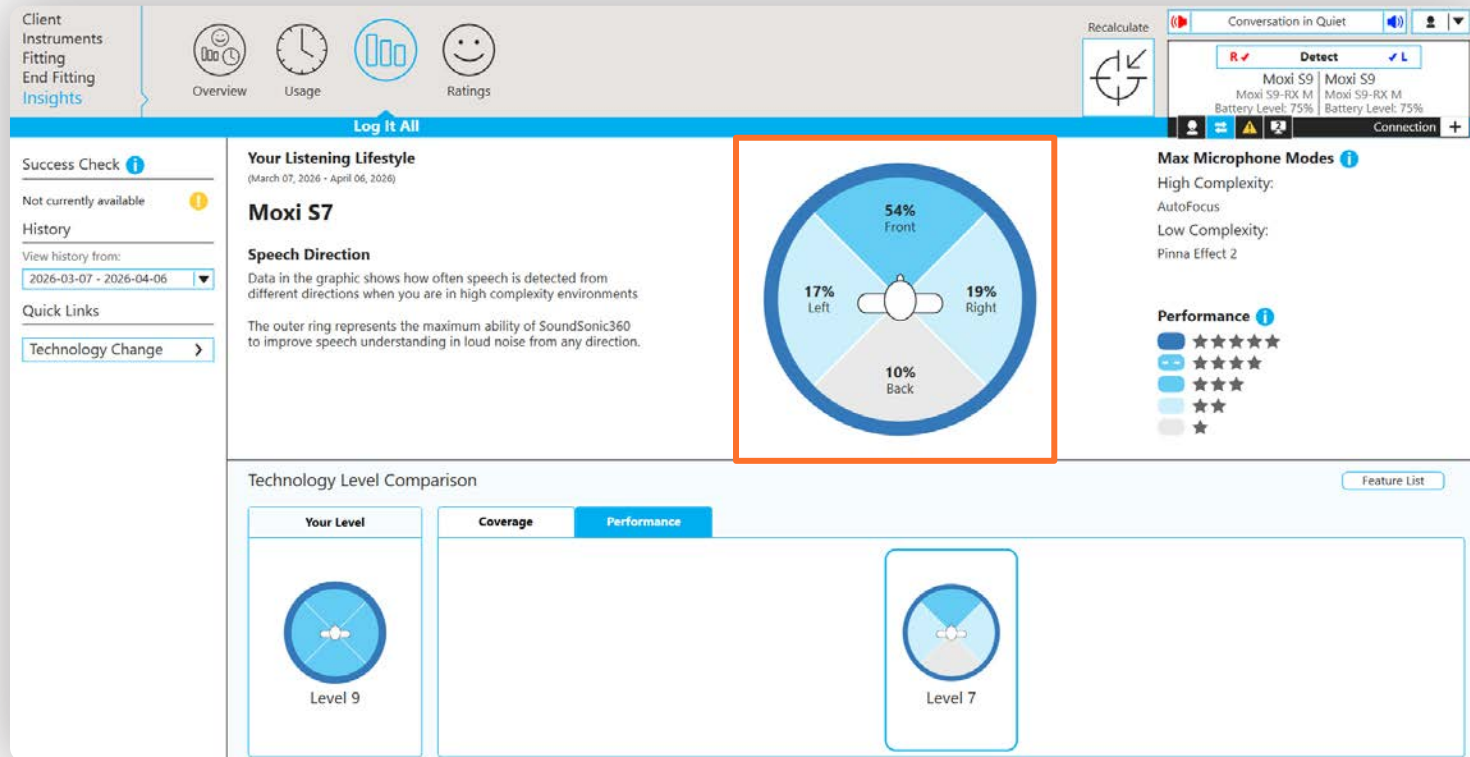


unitron™

Easy to hear

Easy to wear

Easy for you

Mic modes in Performance screen

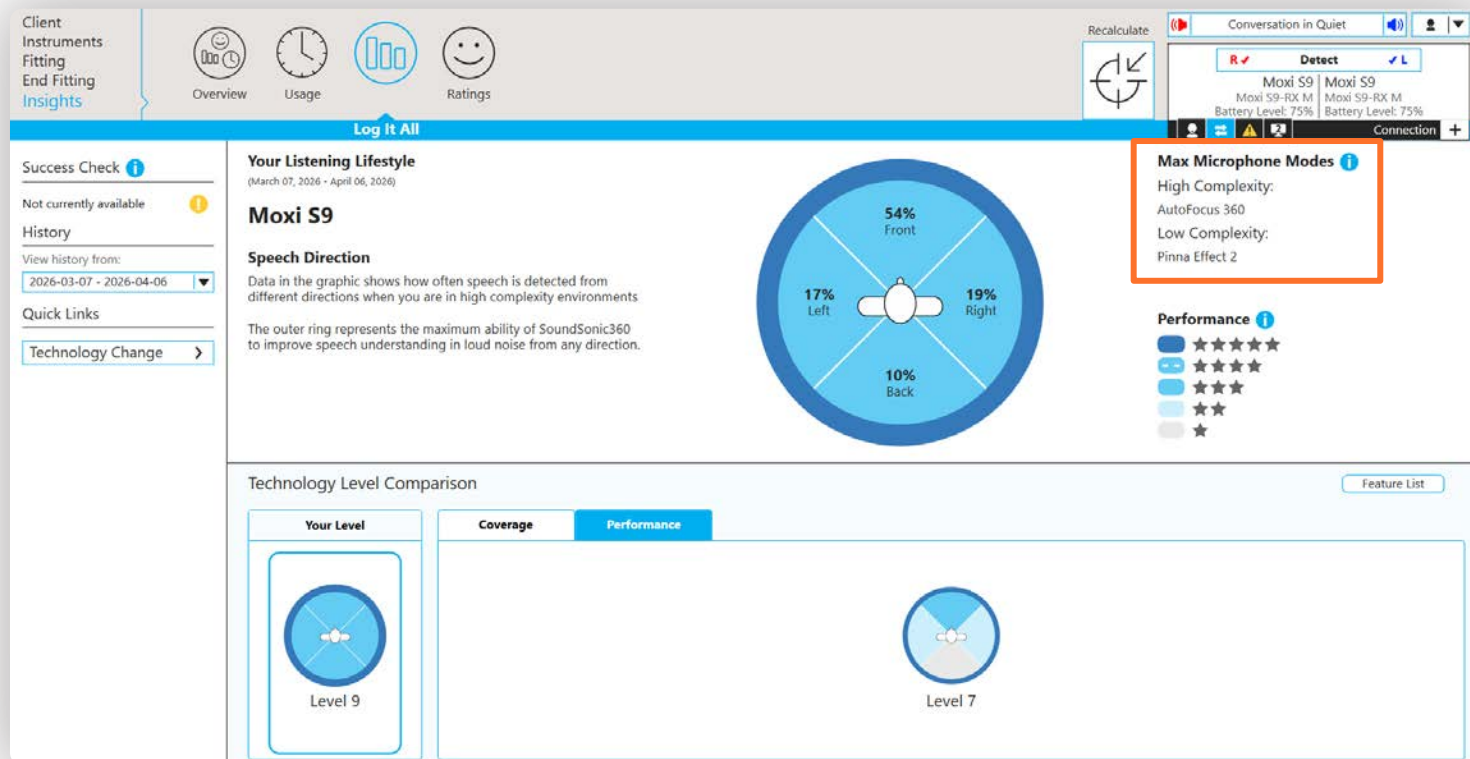

unitron™

Easy to hear

Easy to wear

Easy for you

Extended EasyGuard support

- TrueFit 5.10 supports EasyGuard Open and EasyGuard Power domes
- AI dome proposer now recommends EasyGuard domes by default
- Continued support for the EasyGuard Vented (introduced with TrueFit 5.9)



unitron™

Easy to hear

Easy to wear

Easy for you

New Feedback Test – 72% Faster!


U

Easy to hear

Easy to wear

Easy for you


Pre-S-RX devices – Open dome

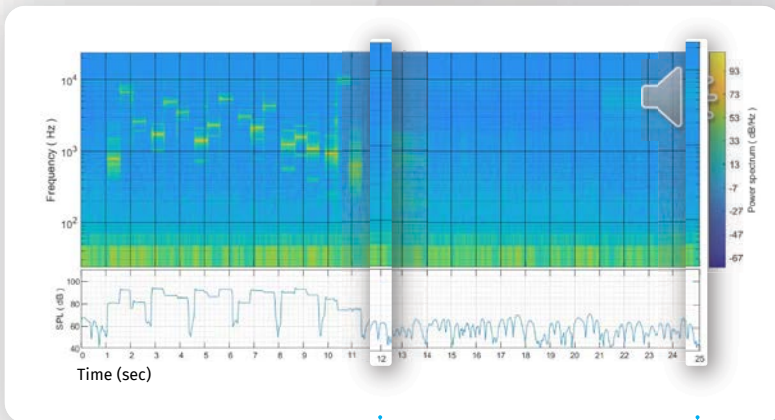

End of test
signal

End
of test

Moxi S-RX – Open dome

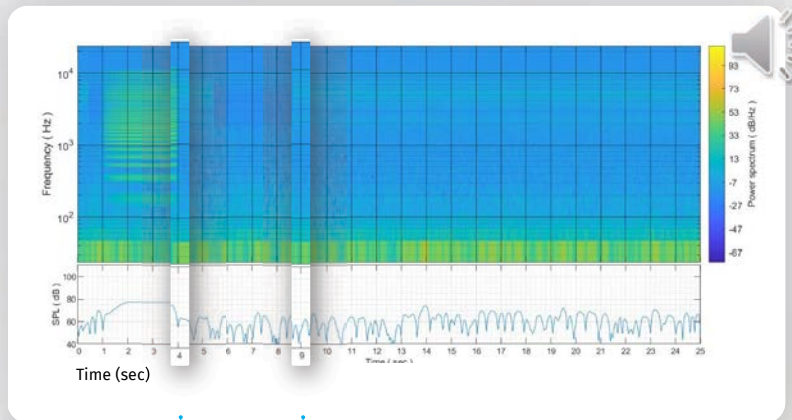

End of test
signal


End
of test

Faster firmware updates

Firmware updates are now up to

46%

faster, giving you more time for what truly matters!

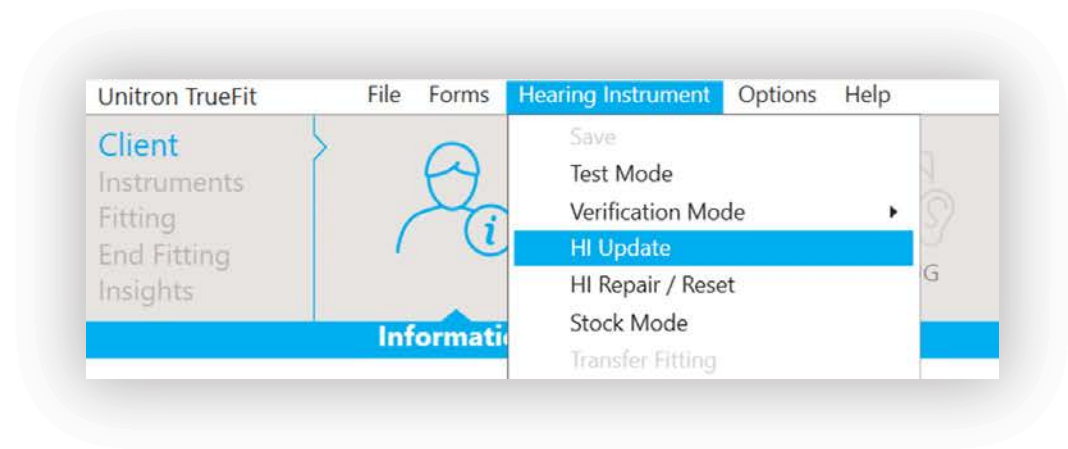

unitron™

Easy to hear

Easy to wear

Easy for you

Remote Plus app



Easy to hear





Easy to wear

Easy for you

New for the Remote Plus app

New features included with the v5.5 app release

- Two new DNN Optional App Programs (OAPs)
- Enhancements to Ratings
- Remaining battery life indicators and low battery alerts
*all programs except Automatic
- End-user feedback options

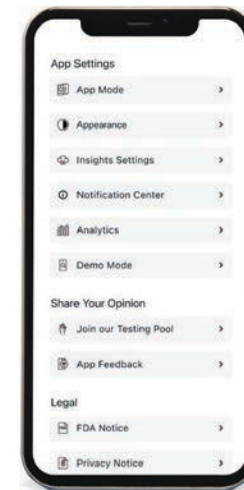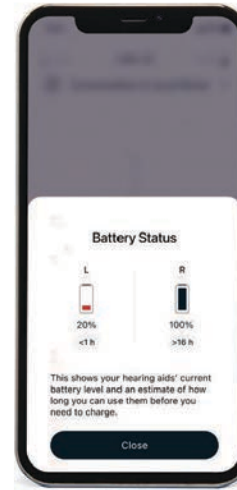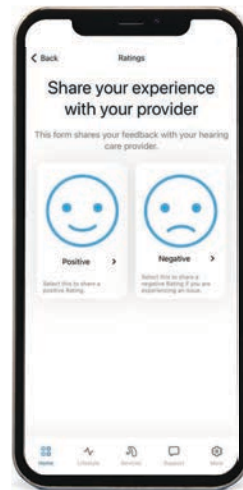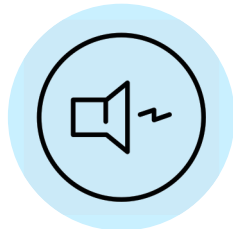

unitron™

Optional App Programs, translated



SpeechSonic

NoiseSonic

unitron™

Easy to hear

Easy to wear


Easy for you

Ratings workflow & expanded responses


Easy to hear

Easy to wear


Easy for you

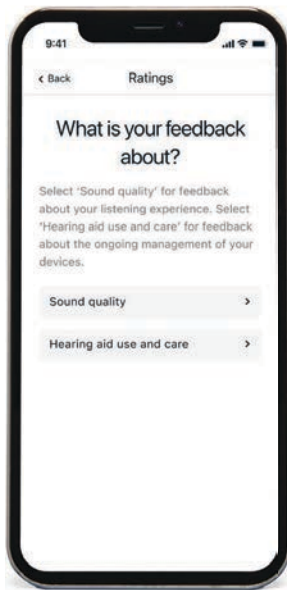

Optimized Ratings categorization

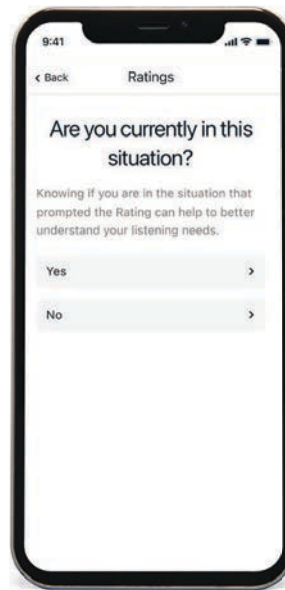

To help determine if HI data capture is relevant to Rating*

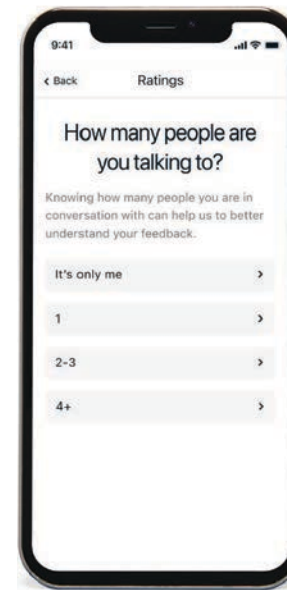


Additional data to help HCP better understand the hearing situation*

*These screens will only appear if Sound Quality was selected

unitron™

Ratings workflow

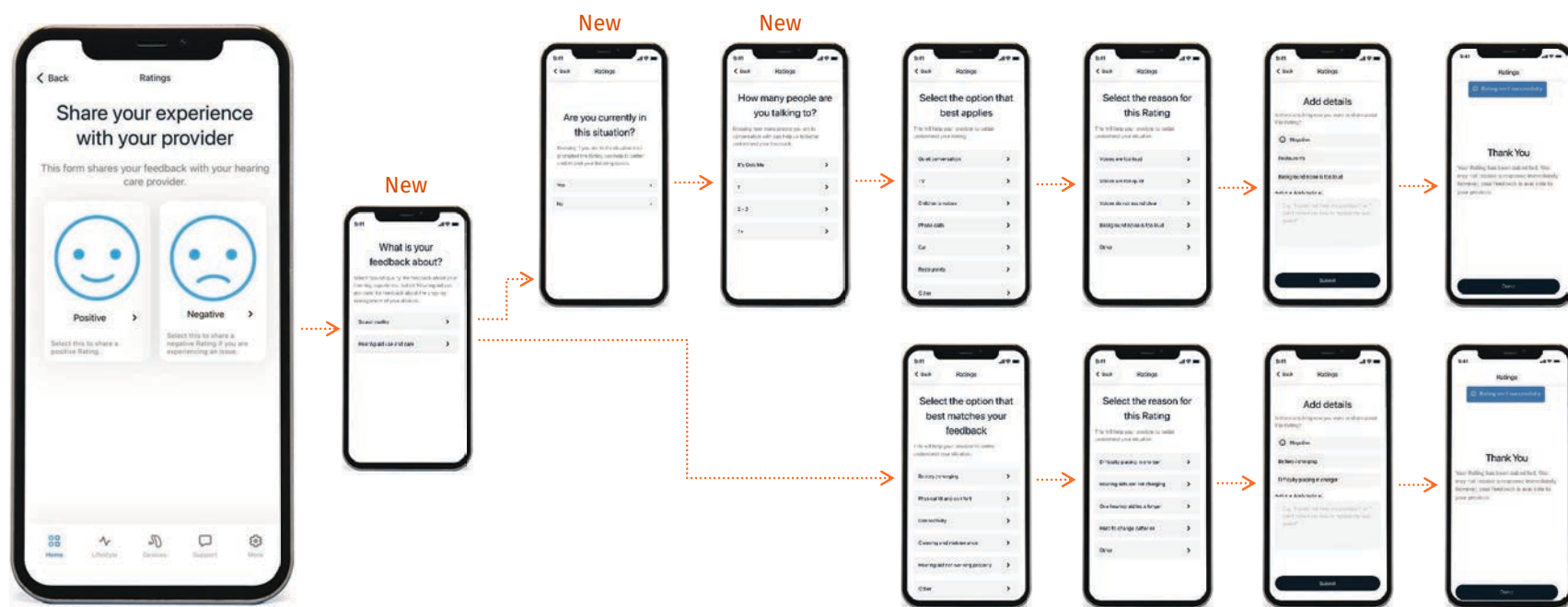

Note: The "reason for Rating" screen doesn't appear if the patient is submitting a positive Rating

unitron™


Easy to hear

Easy to wear

Easy for you

Comfort and Clarity controls in the Automatic Program

- Comfort and Clarity controls in the Automatic Program
 - Dynamically adjust speech enhancement and noise reduction features including SoundSonic360, when activation criteria are met

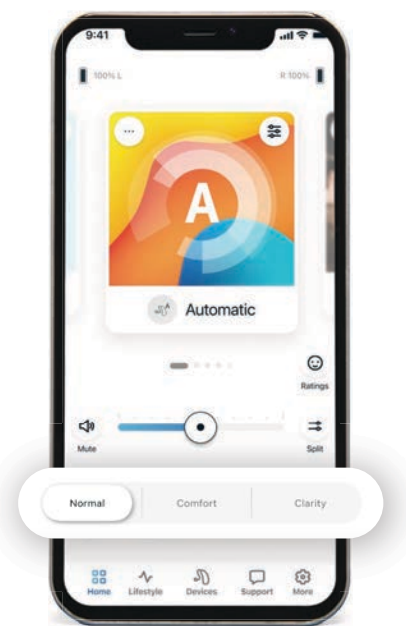


unitron™

Easy to hear

Easy to wear

Easy for you

Easy Line™ Pro app v1.2 release - August 4, 2026

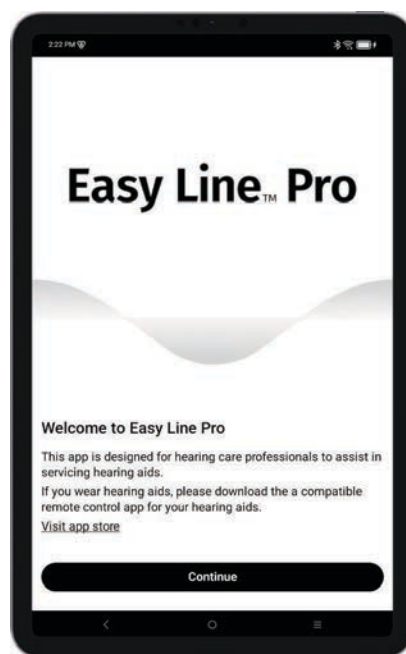


unitron™

Let's Summarize the Moxi S-RX




Easy to hear

Better speech understanding,
reduced listening effort and less
fatigue at the end of the day

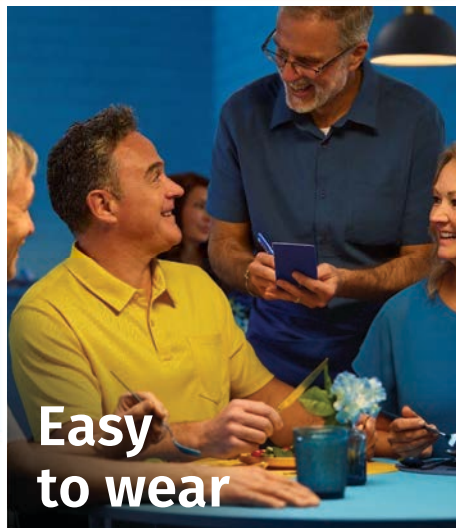

Easy to wear

Devices that look great, feel good
and are easy to use

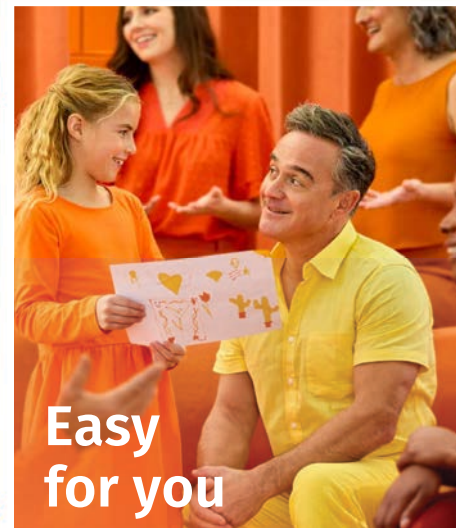

Easy for you

Offer the client:

- More flexibility over their hearing experience
- Increased personalization

unitron[™]



U Smile,

we all

Smile



