

Quick Resource: AutoREM Features and Benefits

ReSound Smart Fit™ fitting software offers automatic real ear measurement. AutoREM is an integrated feature within ReSound Smart Fit so verifying hearing aid fittings with real ear equipment can be done within the software.

Targets

- 1 Targets automatically adjust based on selected fitting formula, e.g. Audiogram+ or NAL-NL3, allow for ease of programming and increasing accuracy of adjustments.

Probe tube placement indicator

- 2 A visual feature during Unaided Gain measurement confirming when the ideal probe tube insertion depth has been reached.

Open fitting

- 3 Turn on *Vent Accommodations* to ensure the measurement accounts for low frequency gain escaping the ear canal when using an open dome.

Streamline adjustments

- 4 Measure in program of choice and apply gain changes to other programs as needed using AutoRelate in Program Options. Quickly adjust all input levels and apply changes to all input levels.

Measures and adjusts in compression channels

- 5 Rather than adjusting only the gain handles available in fine tuning, AutoREM will adjust all available *compression channels* for increased precision.

Find compatibility inside software.
Gain Adjustments > AutoREM > Hover to see compatibility

Manufacturer According to FDA:

GN ReSound North America

5005 Dean Lakes Blvd.
Shakopee, MN 55379
USA
1-800-248-4327
pro.resound.com

ReSound Government Services

5005 Dean Lakes Blvd.
Shakopee, MN 55379
USA
1-800-392-9932
gs.resound.com

Manufacturer According to Health Canada:

ReSound Canada

2 East Beaver Creek Road, Building 3
Richmond Hill, ON L4B 2N3
Canada
1-888-737-6863
pro.resound.com