


SONNET 3

Features, Benefits, Connectivity, and Accessories

Andrea Hedley-Williams, AuD, MED-EL Clinical Education Manager

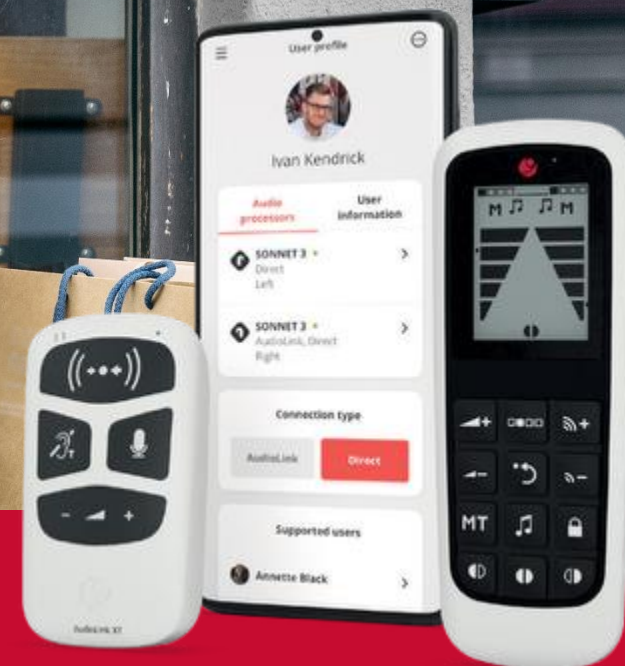

Learning Objectives

1

Learner will be able to state three key features of the MED-EL SONNET 3 audio processor

2

Learner will be able to identify compatible accessories and their clinical applications

3

Learner will be able to list the steps needed to successfully pair a SONNET 3 audio processor to a smartphone



SONNET 3

Features



Built-in Direct Streaming

- Bluetooth low energy connectivity
- Android (ASHA) and iPhone compatible (MFi)



Smaller and Lighter

- 18% shorter
- 2g lighter




New & Improved Design

- Convenient touch-key
- Aerodynamic robust housing with a flexible ear hook



Waterproof Design

- IP68 Rating
- Submersion in 1 meter of water for 60 minutes

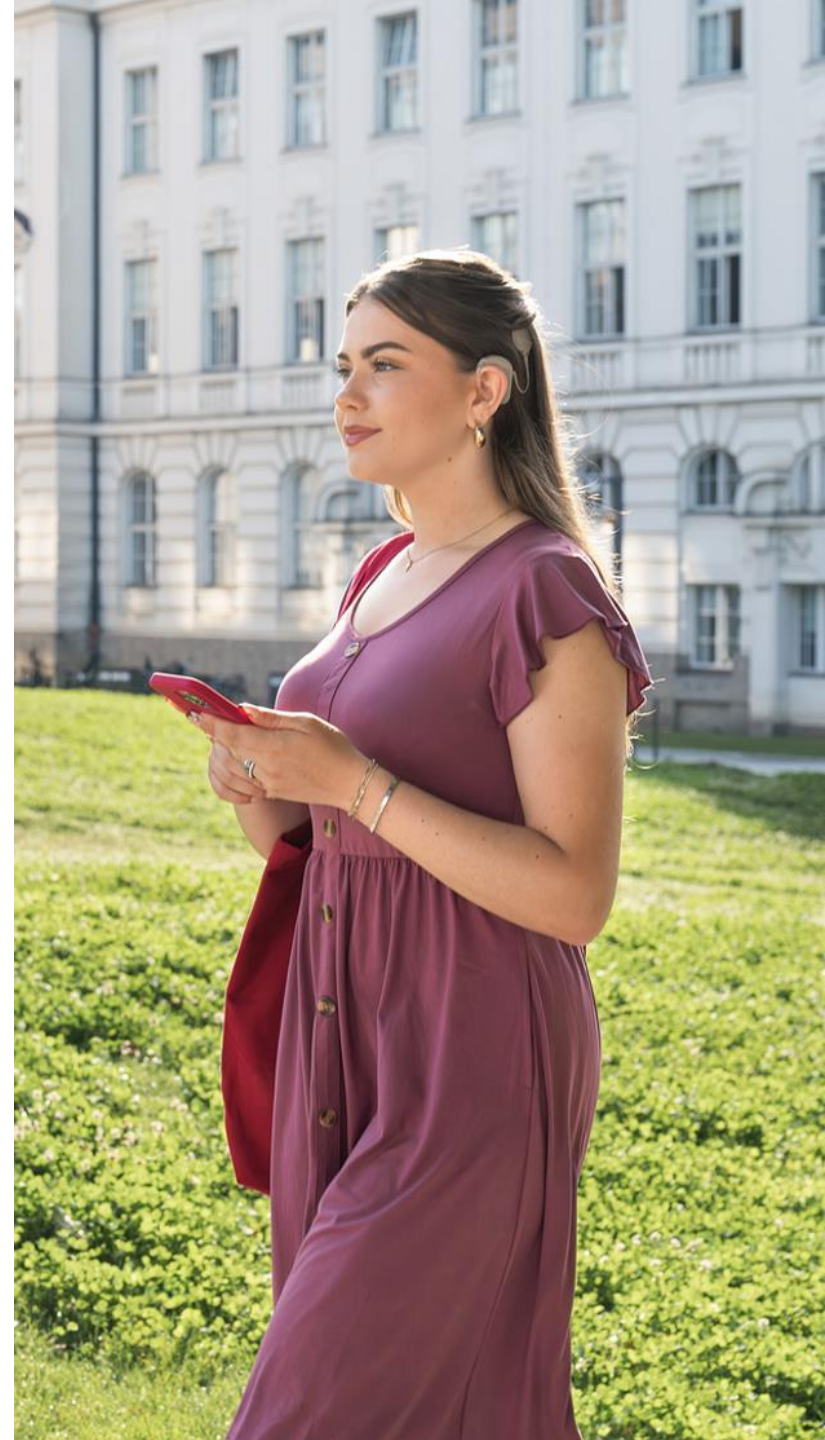

New Design



MED-EL's Smallest BTE



-11 %



Supercharged

Added power with our new rechargeable battery

- SONNET 3 with Rechargeable Battery Standard Pro
 - Up to 23 hours
- SONNET 3 with Rechargeable Battery Micro Pro
 - Up to 12 hours
- Our most powerful processor yet

SONNET 3

supercharged

SMALLER SIZE

Easy-to-Use Charging Unit

Battery slides right in with magnetic attraction

Works with or without the processor attached


USB-C port for flexible charging

Charging status lights



New Mechanical Design

Enhanced Housing

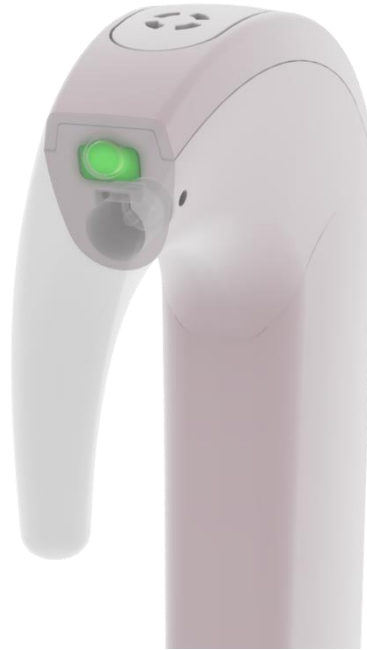

Sonnet 3: Wearing Comfort

Flexible Silicone Earhook





Touch Key

- Ear level control
- Disabled by default
- Easy to sense/locate touch key
- Simple finger tap
- Two available options
 - Program change
 - Stand-by mode

Energy Management




- Standby mode is entered if the external device cannot detect the internal implant for 5 minutes
- Powers down the processor and coil by 90%
- Re-activates the processor and the coil automatically when the coil is placed back over the implant

SONNET 3 EAS vs SONNET 3 System



A woman with blonde hair, wearing a light green raincoat and a black backpack, is walking on a train platform. She is carrying two young children. One child, wearing a pink shirt and blue polka-dot pants, is being held in her arms. The other child, wearing a grey hoodie and blue jeans, is walking alongside her, looking up at her. The background shows a train platform with tracks, a yellow safety line, and a modern building with large windows.

Perfect for
Pediatric Patients

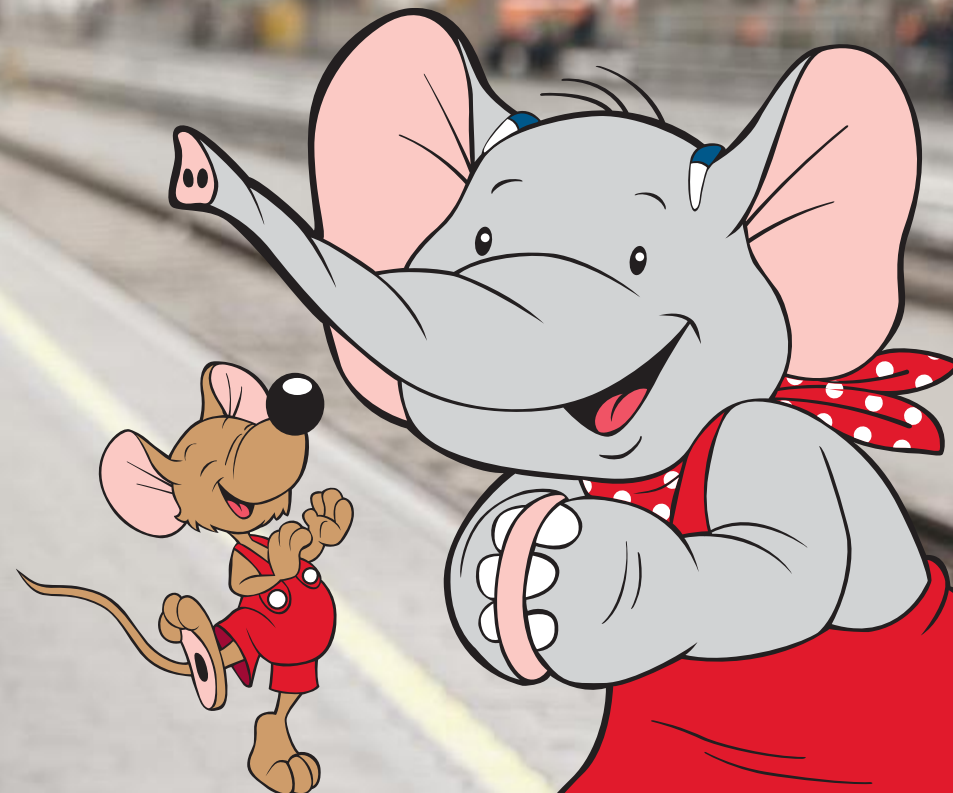

SONNET 3

Pediatric-Friendly Design



Comfortable and Durable

- Small and light weight
- Durable housing
- Flexible ear hook for improved retention



Safe and Secure

- Tamper-proof
- Locked battery cover



Peace of Mind for Parents and Caregivers

- Real-time monitoring
- All day battery life with RCB Pro
- Smartphone Apps—AudioKey and HearCare
- Flexible wearing options





SONNET 3

Stream Big With Our Smallest Processor

SONNET 3

Easy Wireless Streaming


- Direct wireless streaming
- Android™ ASHA or Apple MFi LEA audio streaming protocol
- **2.4 GHz Bluetooth® 6 Low Energy** wireless technology
- Wireless connection to the FineTuner Echo, AudioLink XT, compatible smartphones/tablets etc., AudioKey mobile app
- Binaural streaming for bilateral SONNET 3 users and SONNET 3 + another audio processor with AudioStream



SONNET 3


Easy Pairing

1.

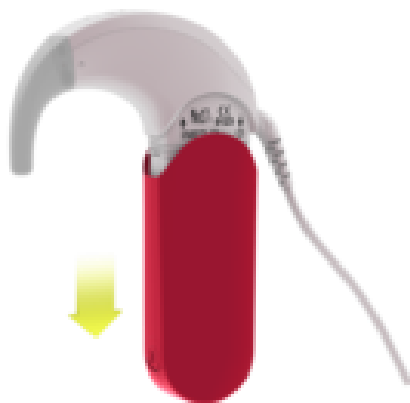

Turn audio processor OFF

2.

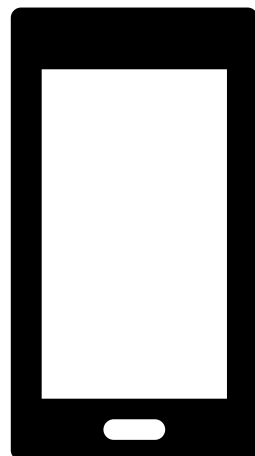

Place audio processor close to the mobile device (closer than 3cm)

3.




Turn audio processor ON

4.



Switch on the Bluetooth functionality on mobile device for Android phones and pair in Accessibility for iPhones.

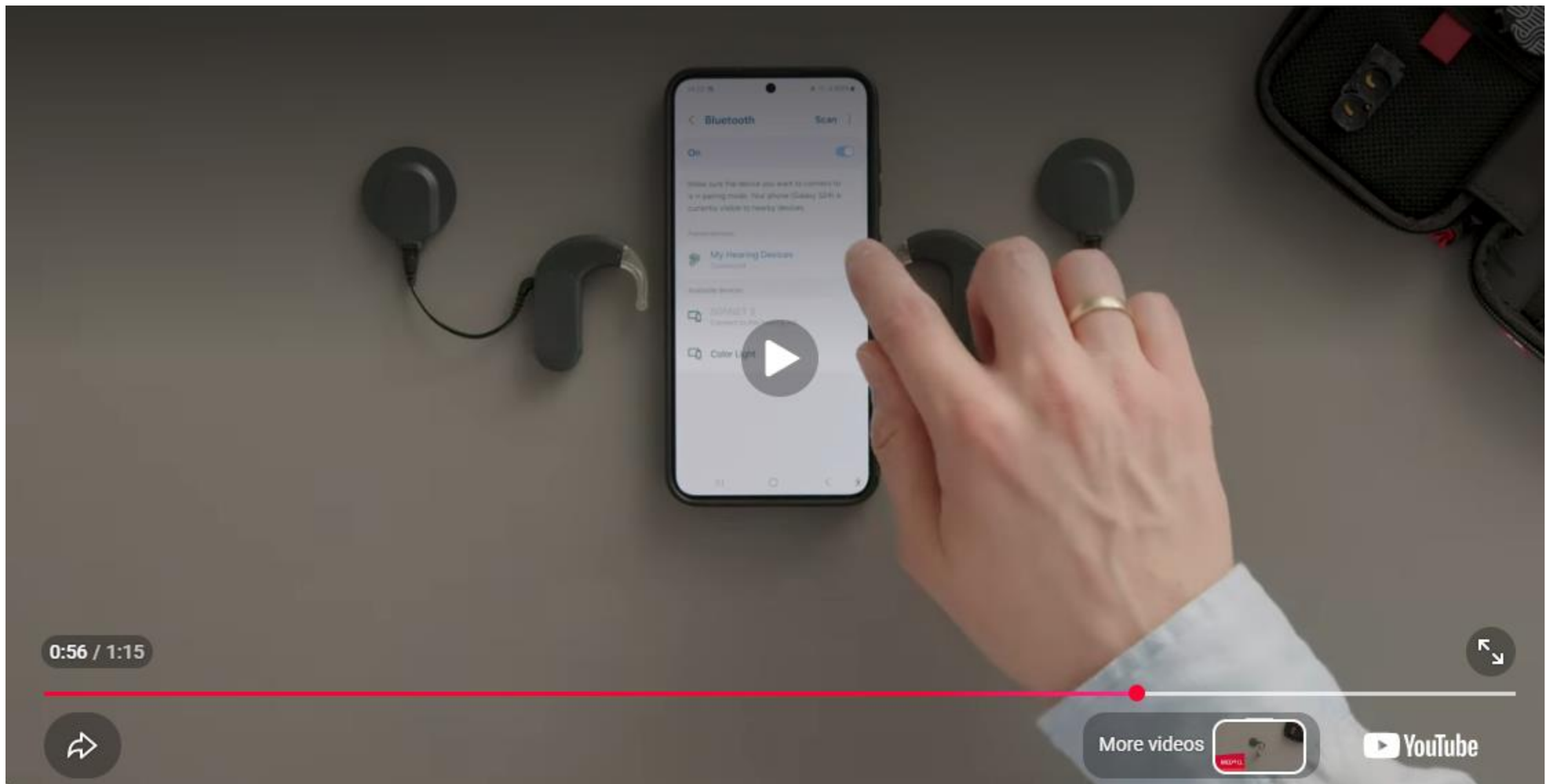


SONNET 3

Bimodal Listening and Streaming




Bimodal Synchronization

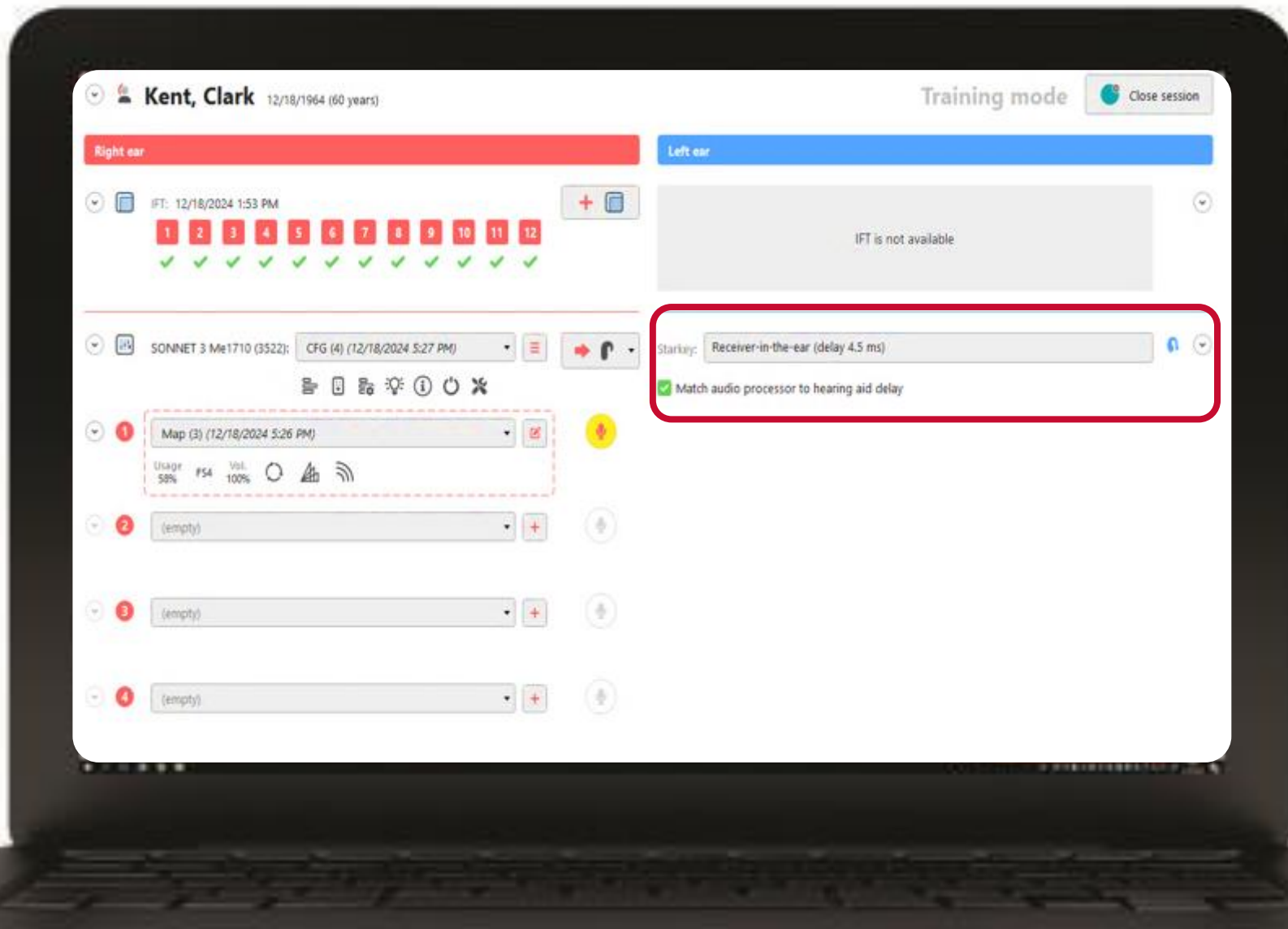


Bimodal Synchronization in MAESTRO

1. MED-EL's CI processing is very fast – close to normal hearing
2. Hearing Aids have a processing delay longer than a CI or normal hearing – which can be noticeable to the brain
3. Introducing a delay to the CI processor can align processing – potentially improving localization for the bimodal user

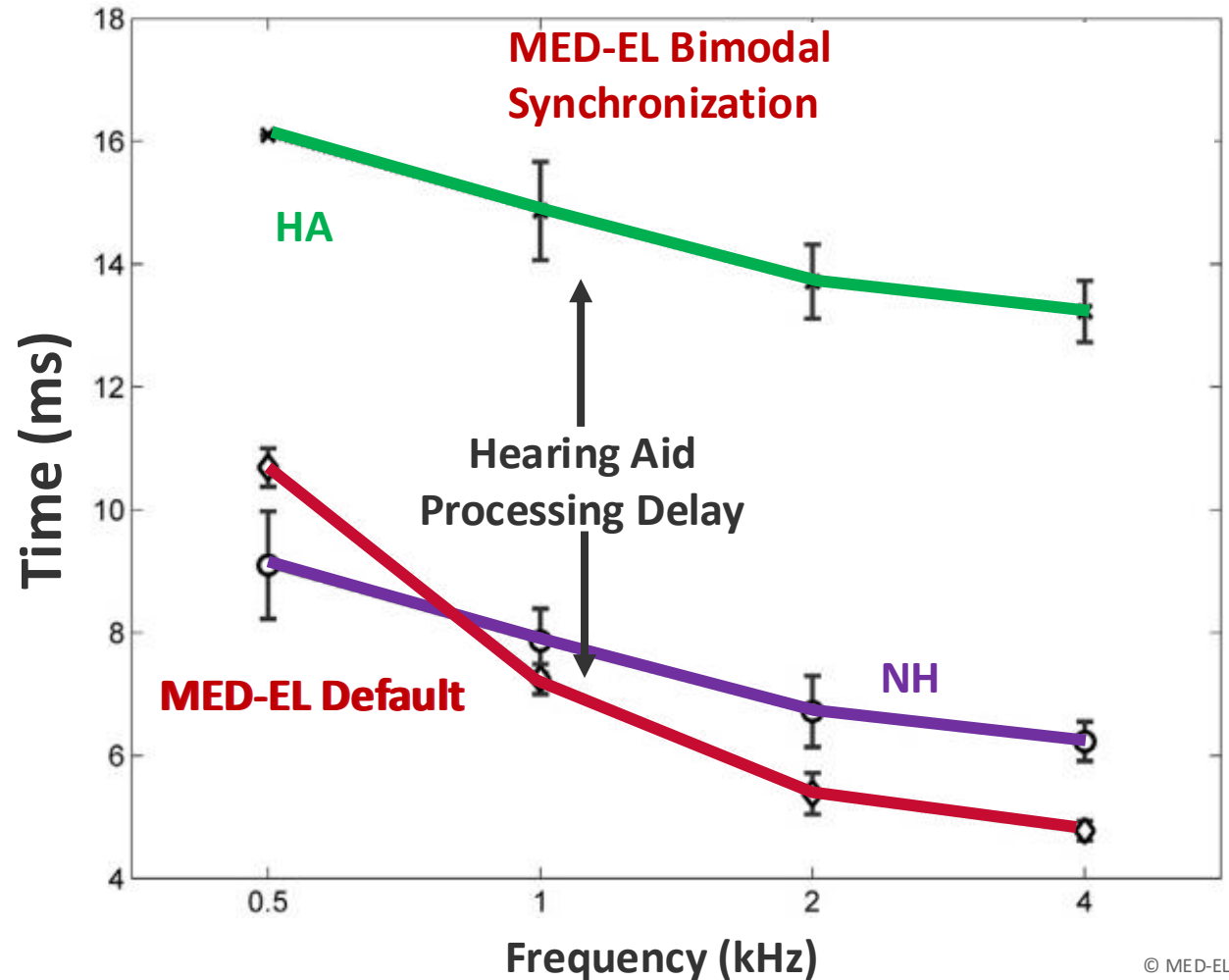

How Does DualSync Work?

AudioKey App

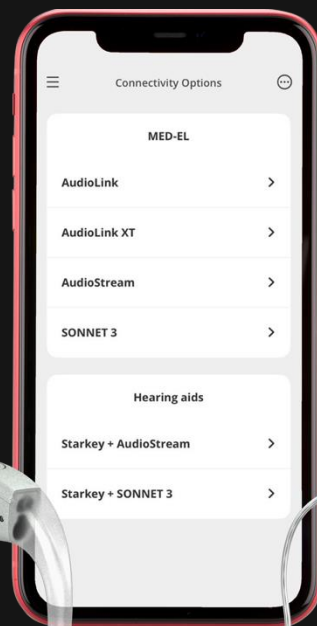

MED-EL SONNET 3
OR OTHER PROCESSOR
WITH AUDIOSTREAM



APPLE IPHONE

STARKEY HEARING AID





SONNET 3 + ASHA Compatible Hearing Aid

Bimodal Streaming for All



Android
12 or higher



AudioLink XT &
iPhones/Tablets



SONNET 3

Accessories



SONNET 3

Compatible Accessories



AudioLink XT



AudioKey 3 and HearCare App




FineTuner Echo



MED⁹EL

AudioLink XT

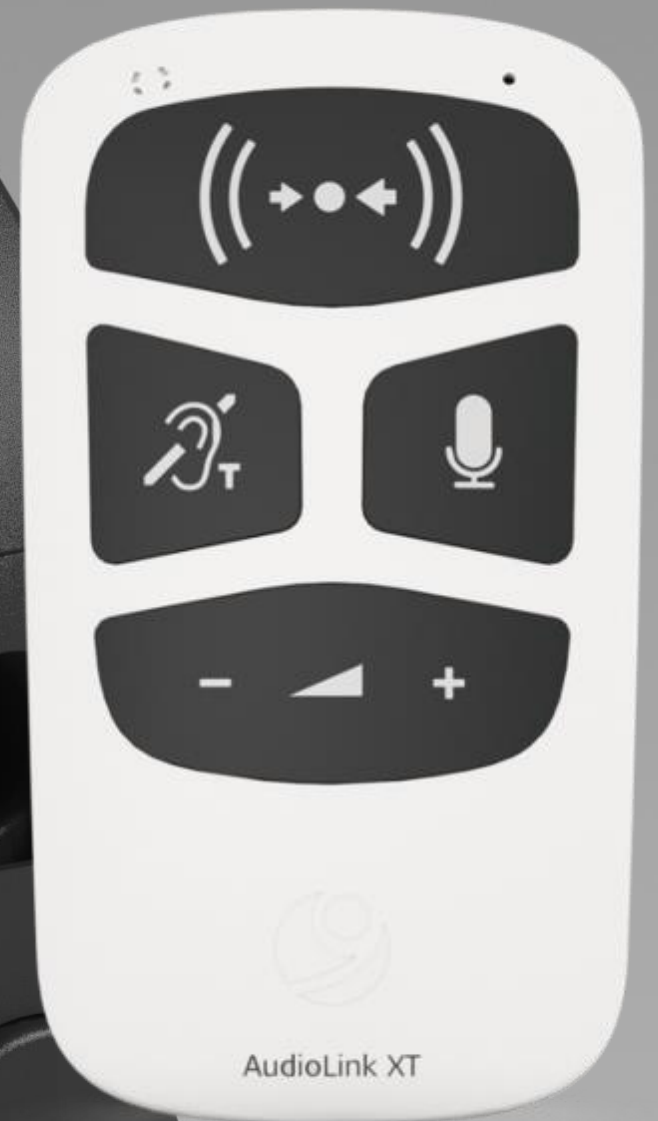

AudioLink XT

Accessories

- New Keypad Layout
- SONNET 3 Connection via ASHA Protocol
- Functions Include:



Remote Microphone



TV Streamer



Integrated Telecoil






FM Adaptor

Note: extended battery duration (> 10 h streaming)




AudioLink XT

Turning On/OFF

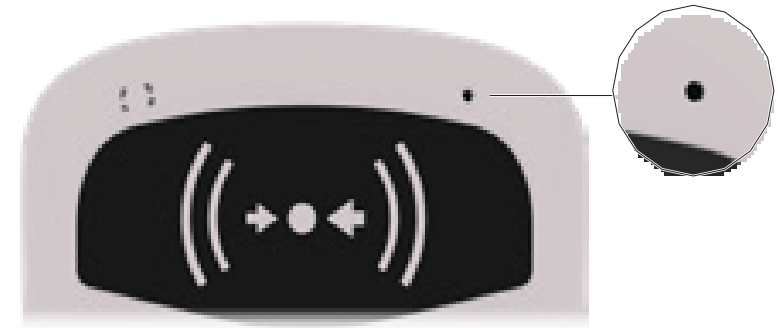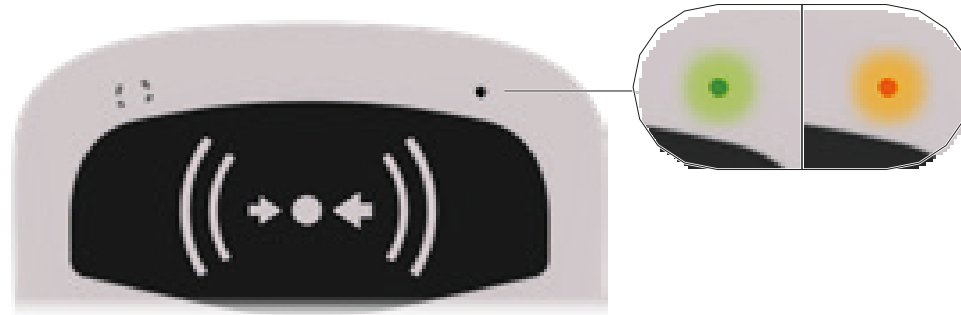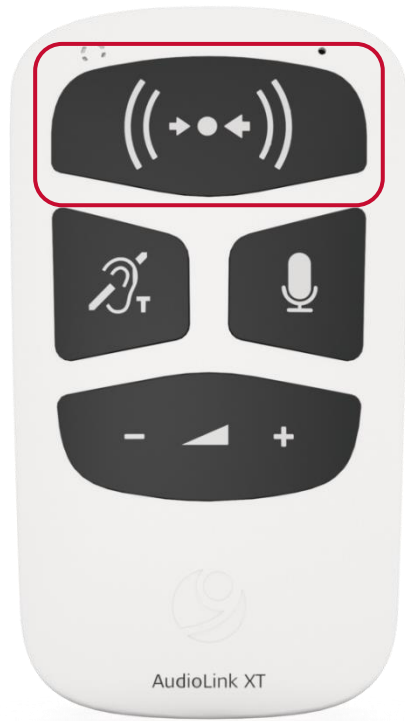


AudioLink XT


Pairing

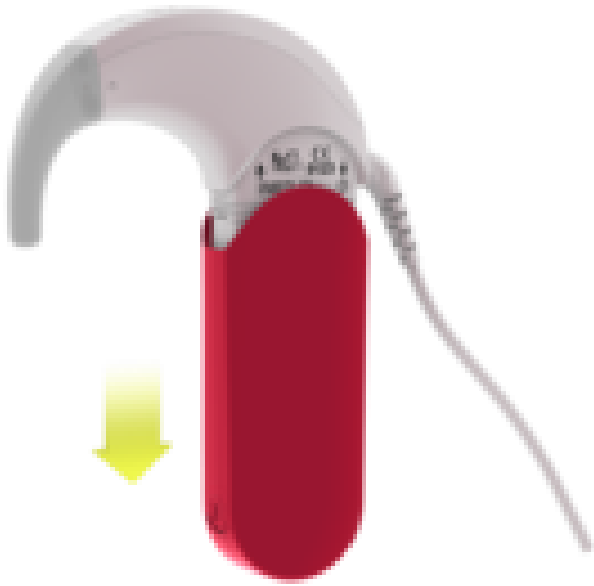

Step 1
Turn audio
processor OFF

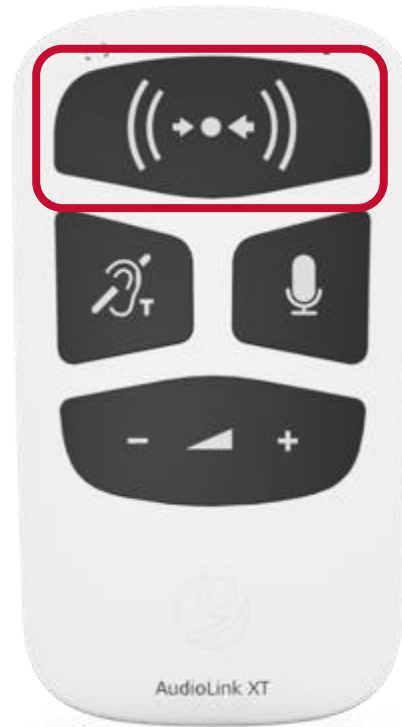


Step 2
Turn on

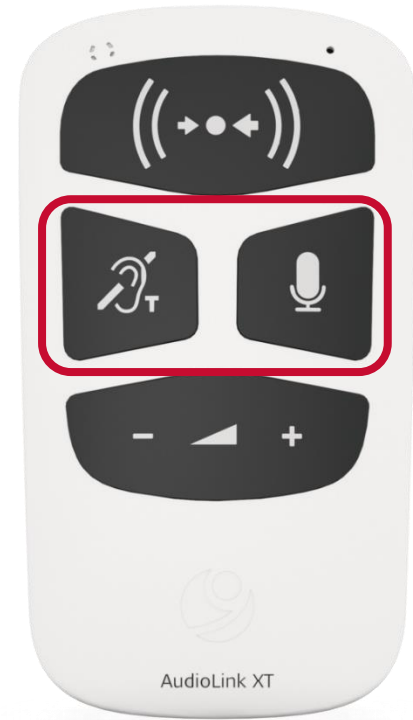

Step 3
Pairing Mode

AudioLink XT

Pairing

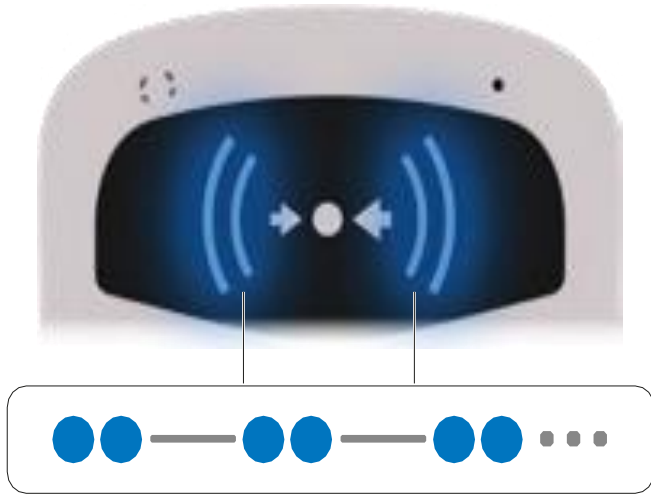

Step 4
BT searching



Step 5
Turn audio
processor ON




Step 6
Connection


SONNET 3

Compatible Applications

AudioKey App

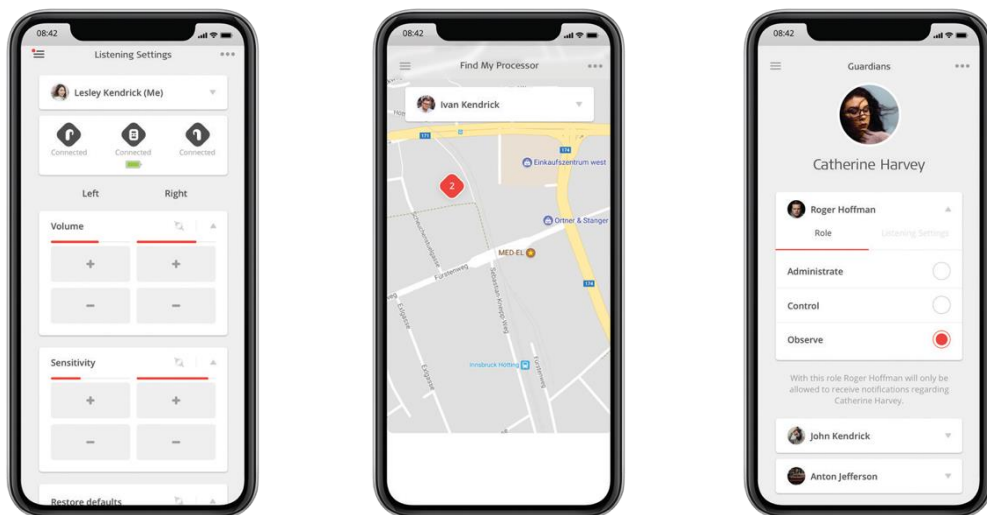


HearCare App

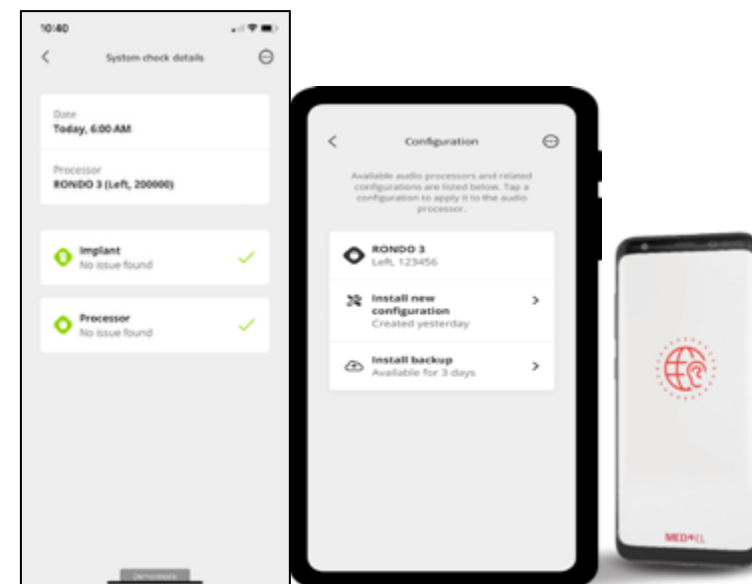

AudioKey 3 App

Remote Control & More

- Android & iOS
- Remote control
 - Listening settings
- Guardian Roles
- Find My Processor
- Bilateral streaming pairing
- My Stats
 - Check usage statistics

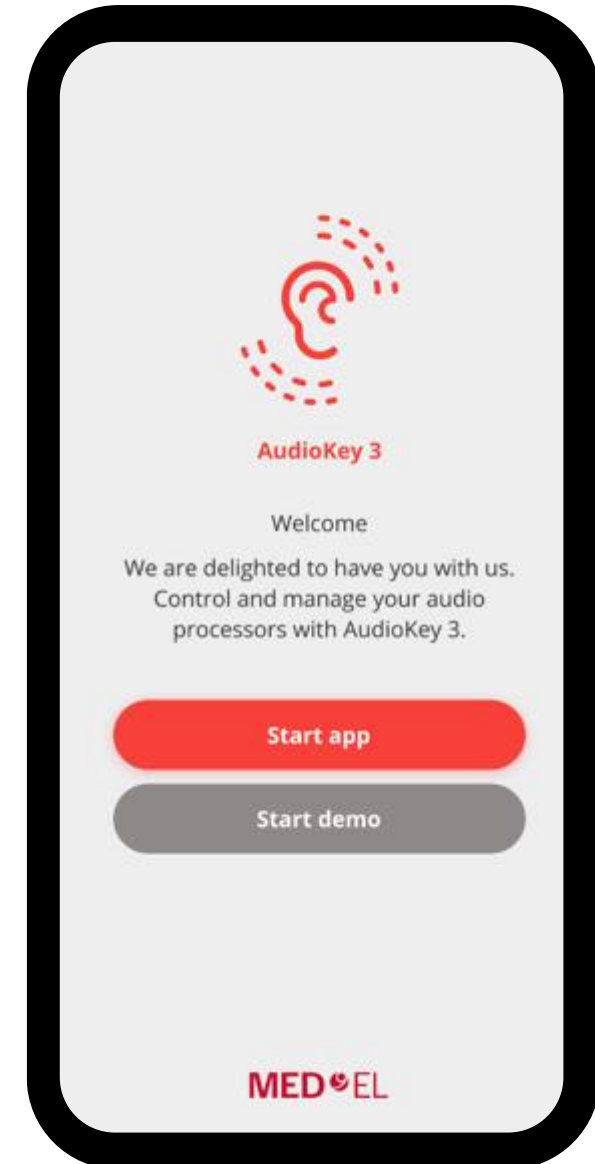

HearCare App

Features



System Checks

- Implant Check
- Audio Processor Checks



Configuration Backups

- Backup and Install Configurations




Guardian Roles

- Caregiver Management



Manage Clinic Sharing/Remote Care


- Clinic Sharing Information
- Clinicians can Adapt or Create New Configurations

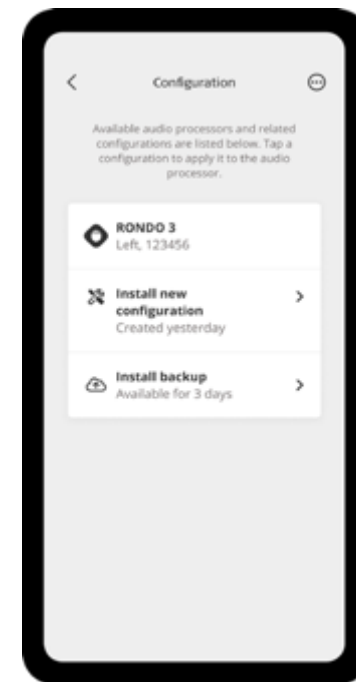

FineTuner Echo

With BLE Support

- Upgraded Firmware to support BTLE
- Improved range, stability, and reliability
- Compatible with:
 - SONNET 2
 - SONNET 3
 - RONDO 3

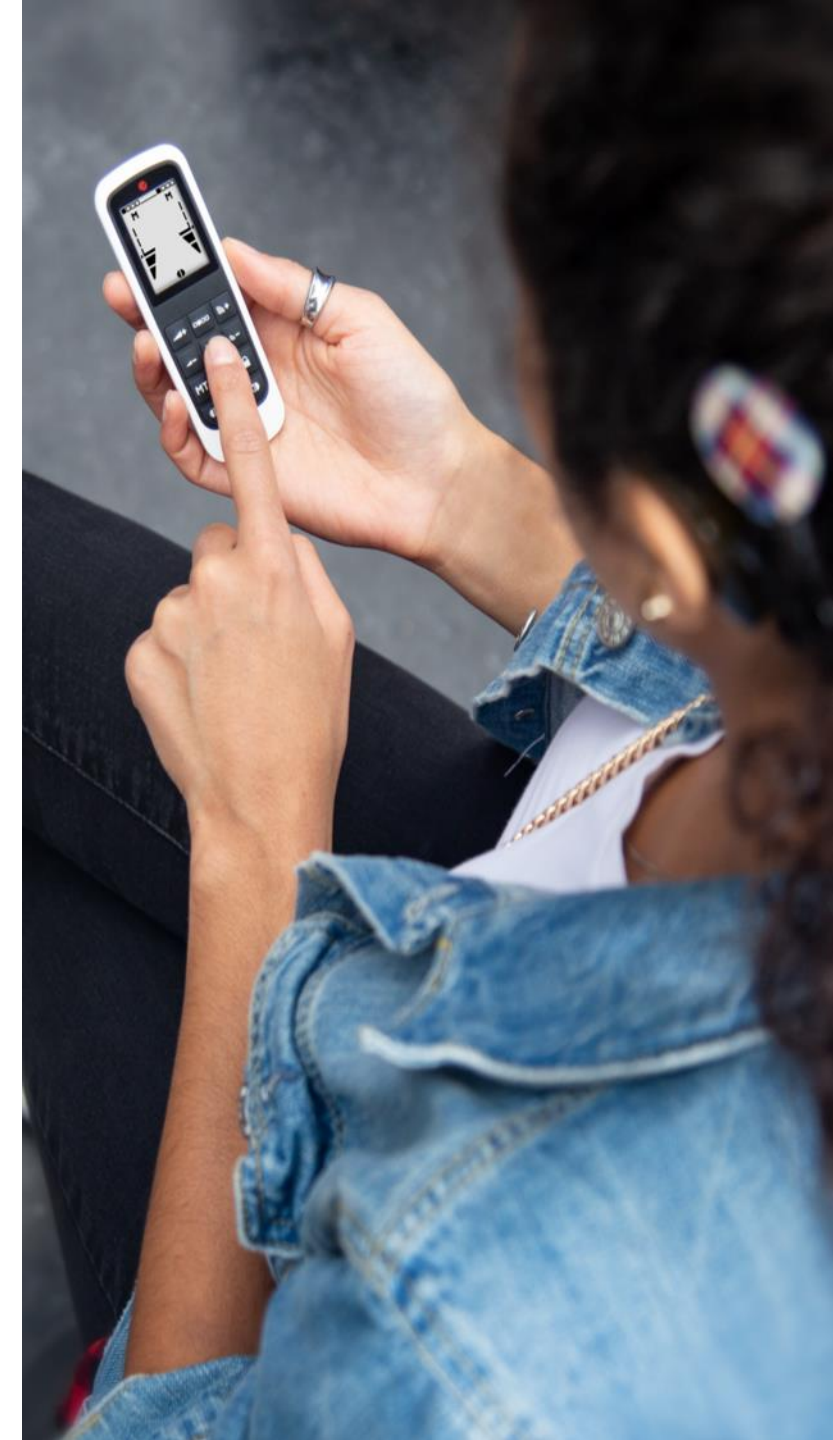

FineTuner Echo

With BLE Support

- New revision (r9) appears on the FineTuner Echo pairing display
- A Bluetooth® symbol is visible on the back
- The MT button does not function for SONNET 3



Resources

QUICK TIPS AudioLink XT

This universal connectivity device lets you connect to almost any device, including televisions, computers, wireless systems, and hearing loops. The included FM adapter can be used to connect to FM systems.



Turn On/Off Au
Press the main b
shows green/ora
XT turns off aut
5 minutes or wh

Pairing with Au

1. Turn off
2. Turn on / standby
3. Press the same tin
4. AudioLink mode for seconds, continuo connects
5. Place the and turn paired, it
6. If you un step 4 w

Note: You pre

QUICK TIPS DualSync iOS SETUP



Before pairing your devices:

Apply any available MED-EL device updates in the Audiotalk app.
Go to Connectivity Options > SONNET 3 (or AudioStream) > Updates.

If your SONNET 3 or AudioStream is already paired to your Apple device.
Go to Settings > Accessibility > Hearing Devices > Forget this device to re-pair it.

If the Starkey app is installed, connecting these apps is necessary for successful use. The app may be reinstalled or installed after configuration is complete.

Forget any "iCloud" connections.
"Pairing" screens under the Apple i Bluetooth can
Go to Settings > Bluetooth

Part 1: Starkey Hearing Aid Pairing

1. Select Settings > Accessibility > Hearing Devices.
2. Ensure Bluetooth® setting is on.
3. Power off, then power on the hearing aid to place in pairing mode.
4. Select the hearing aid when it appears on the screen.
5. When prompted, select Pair.
6. Ensure it says "Connected" after the hearing aid name.

Note: Do not repeat Part 1 with your MED-EL device. The MED-EL device should not have an active connection to any Bluetooth® device until after Part 2.



*Please visit medel.com/support for AudioStream iOS setup.

Note: If the Starkey hearing aid is rechargeable, the charge case and power connection may be needed to power the device.

Part 2: MED-EL AudioKey App Configuration

Download the latest version of the AudioKey app from the App Store.

1. Open the AudioKey app.
2. Select Connectivity Options.
3. Under Hearing aids, select the combination app(s) for you (Starkey + SONNET 3 or Starkey + AudioStream®).
4. Review the Important Information. Tap Next to continue.
5. Follow the instructions for the Starkey hearing aid and the MED-EL device.
6. Tap Configure to start the configuration process.
7. Upon successful pairing, a "Success" pop-up is shown. Click "OK."
8. Go to Settings > Accessibility > Hearing Devices (iOS) menu to confirm that both devices are listed under the combination app(s) you selected. If both hearing devices are not listed under the iOS menu, toggle Bluetooth OFF/ON or restart the Apple device. Restart both hearing devices and tap "Not Connected" to pair.
9. You should now be ready to start bimodal streaming.

QUICK TIPS AUDIOKEY 3 APP SETUP

Make the most of your hearing experience with the AudioKey 3 app. After checking your audio processor settings, check on your child's hearing even locate your lost audio processor—all without carrying a device.

GET STARTED

1. Turn your phone's Bluetooth®, Wi-Fi, and location services on.
2. Download AudioKey 3.
3. Register or log in to myMED-EL.
Note: If you don't have an account, please click Register Now and complete the registration process.

PAIR YOUR PROCESSOR

1. If you are using AudioKey 3 for the first time, the pairing screen will automatically appear.
2. Restart your audio processor.
3. Select the appropriate Connection type from the pop-up.
Original SONNET Recipients: Select AudioLink
SONNET 2, SONNET 3 & RONDO 3 Recipients: Select Direct or AudioLink/AudioLink XT.
4. Click Next.
5. Choose the audio processor you want to pair.
Note: Bilateral recipients will be prompted to pair one processor first and then the other.

SONNET 3

MED-EL

youtube.com

Create new user – If the processor belongs to someone you care for, select this setting and enter their name.

7. Success!
Note: Bilateral recipients will be asked to pair another processor. Repeat steps 4-9 to continue.



For further information about using AudioKey 3, AudioStream, and your audio processor, please refer to the product instructions for use.

The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc., and any use of such marks by MED-EL is under license. Other trademarks and trade names are those of their respective owners.

2645 Meridian Parkway | Suite 100 | Durham, NC 27713
<http://audiotalk.com/medel.com>

medel.com

© 2018 MED-EL Corporation. All rights reserved.

MED-EL SUPPORT

www.MEDEL.com or 888 633 3524



MED-EL Mondays+

- Virtual on demand and live classes for new recipients that cover after activation basics
- Led by MED-EL's experienced hearing health professionals



 medelevents.com



1-on-1 Virtual Appointments

- Helps your patients with technical information and product/upgrade support
- Led by our experienced audiologists



 medelevents.com



Hearpeers Community

- Community of MED-EL recipients, parents, and families ready to share experiences and navigate the hearing journey



 hearpeers.us
 hearpeers.us@medel.com



myMED-EL WebShop

- Allows patients to order equipment out-of-pocket at their convenience



 us.shop.medel.com



Customer Service & Audiology Technical Support

- Assists with warranty claims and purchases of spare parts/accessories
- Facilitates remote loading of replacement processors with maps provided by your clinic

 **888.633.3524**

 customerservice.us@medel.com



Rehabilitation Support

- Rehabilitation resources, including blogs, therapy materials & free online downloads
- Workshops to help with listening & spoken language strategies

 medelbridge.com

 rehab.us@medel.com

Our smallest and lightest behind-the-ear audio processor with integrated direct streaming is ready to seamlessly connect your patients with what they love the most. **SONNET 3 offers convenience, comfort and superior hearing performance.**

- Built-in direct streaming
- Compact design
- Convenient Touch Key
- Waterproof design
- Flexible earhook

SONNET 3

Stream Big

WITH OUR SMALLEST PROCESSOR



Delivering leading-edge technology
to restore hearing and empower connection

MED  EL

